

# *Spare Parts Catalogue*

---

Version: 2016/07/28b EN



KÜSCHALL COMPACT<sub>attract</sub>



**Yes, you can.®**

## **General Information**

*For any order of spare parts the following information is required:*

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

*Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.*

*The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.*



Yes, you can.®

## TABLE OF CONTENTS

---

### INFORMATION

INFORMATION .....	5
Abbreviations .....	6
Frame colours .....	7

### CHASSIS

CHASSIS .....	8
Front frames standard .....	9
Cross brace single, SB36-42 .....	12
Cross brace double, SB44-50 .....	14
Rear frames .....	16
Rear wheel fixation .....	17

### SEAT

SEAT .....	18
Seat cover with 1 Velcro band (soft band) .....	19
Seat cover with 1 Velcro band (soft band) for seat bag .....	21
Seat cushion standard .....	23

### BACKREST

BACKREST .....	25
Push handles for backrest standard .....	26
Push handles for backrest angle adjustable .....	27
Push handles for backrest foldable .....	28
Push handles for backrest foldable and angle adjustable .....	29
Push handles height adjustable .....	30
Push handles height adjustable for backrest angle adjustable .....	31
Küscharl backrest parts for Matrix backs .....	32
Push handles, inclined .....	33
Backrest stabilization bar .....	34
Back cover, standard .....	35
Bands to adjustable backrest .....	36
Cover to adjustable backrest .....	37

### LEGRESTS

LEGRESTS .....	38
Legrest hanger .....	39

### FOOTREST

FOOTREST .....	40
Footrest 2-piece .....	41
Footrest 2-piece, angle adjustable .....	43
Footbands .....	44

### SIDERESTS

SIDERESTS .....	45
Clothes-guards composite .....	46
Clothes-guards carbon .....	47



Yes, you can.®

## TABLE OF CONTENTS

---

Mudguards composite .....	48
Mudguards carbon .....	49
Mudguards composite removable .....	50
Mudguards carbon removable .....	51
Siderest height adjustable .....	52
Siderest with desk function .....	53
Tubular armrest .....	54

### FRONT WHEELS

FRONT WHEELS .....	55
Front wheel supporter .....	56
Front wheels .....	57

### REAR WHEELS

REAR WHEELS .....	58
Active wheel .....	59
24" Kuschall rear wheels complete .....	60
High Performance wheel .....	61
Spinergy LXX wheel .....	62
Drumbrake wheel .....	63
Handrim .....	64
Fixation handrim .....	65
Tyre .....	66
Inner tube and rim protector .....	67
Spoke guard .....	68
Quick release axle and bearing .....	69

### BRAKES

BRAKES .....	70
Brakes .....	71
Drum brake .....	72


### OPTIONS & ACCESSORIES

OPTIONS & ACCESSORIES .....	73
Tipper aid .....	74
Antitipper .....	75
Cane holder .....	76
Therapy table .....	77
Seat belt .....	78
Chair accessories .....	79
Fixation: screws, ... ..	80
Socket head cap screws .....	81
Countersunk screws .....	84
Button head screws .....	86
Hex-head screws, diverse, ... ..	88
Washers .....	90
Nuts .....	92
Pins, rivets, ... ..	94


**INFORMATION**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
1	-	Abbreviations		1
2	-	Frame colours		1

## Abbreviations

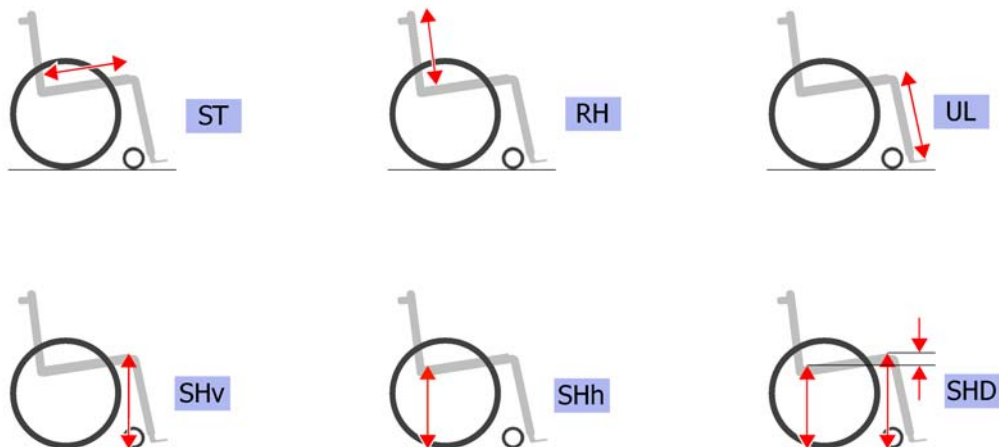


Pos.	Part number	Description	Valid from → until	Qty
L	-	Left side (sitting in the wheelchair)		0
R	-	Right side (sitting in the wheelchair)		0
L+R	-	Left an right sides (paire)		0
SB	-	Seat width (2 digits in cm or 3 digits in mm)		0
ST	-	Seat depth (2 digits in cm or 3 digits in mm)		0
RH	-	Backrest height (2 digits in cm or 3 digits in mm)		0
UL	-	Knee-to-heel length (2 digits in cm or 3 digits in mm)		0
SHv	-	Front seat-to-floor height (2 digits in cm or 3 digits in mm)		0
SHh	-	Rear seat-to-floor height (2 digits in cm or 3 digits in mm)		0
SHD	-	Front and rear seat-to-floor height difference (2 digits in cm or 3 digits in mm)		0

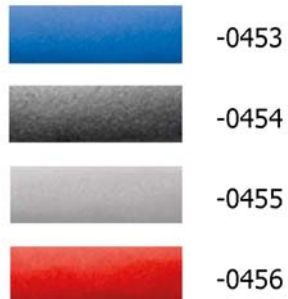**Frame colours**

Pos.	Part number	Description	Valid from → until	Qty
-0453	-	Color code for attract blue frames		0
-0454	-	Color code for attract black frames		0
-0455	-	Color code for attract grey frames		0
-0456	-	Color code for attract red frames		0


# CHASSIS

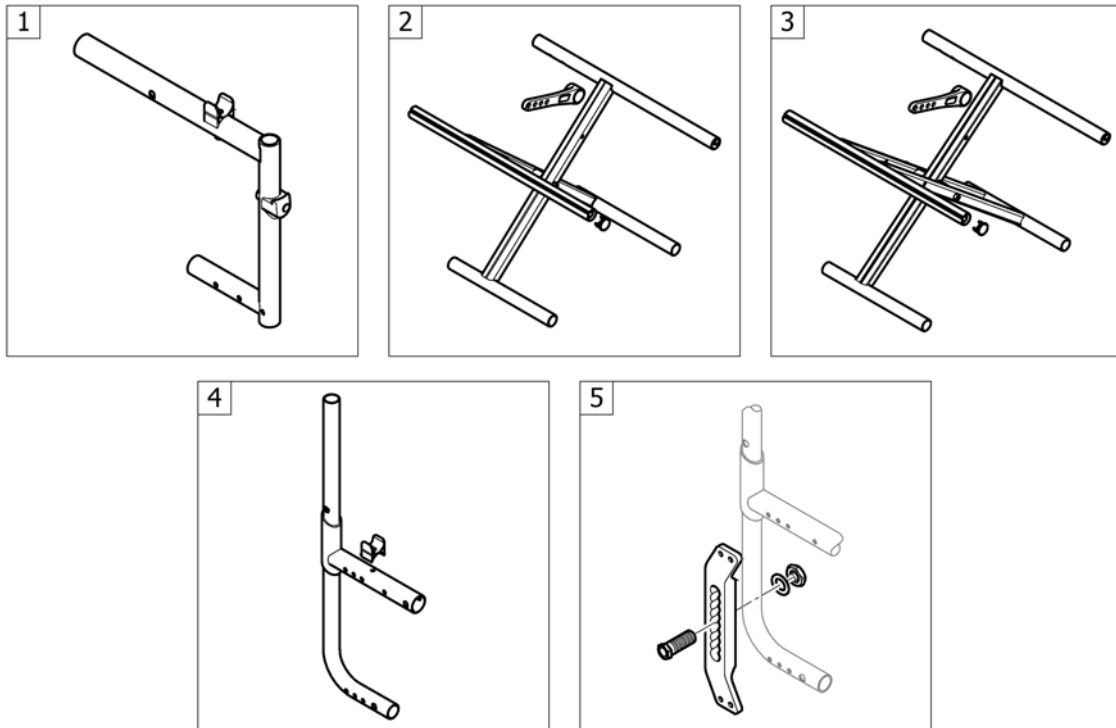

Pos.	Part number	Description	Valid from → until	Qty
1	-	Front frames standard		1
2	-	Cross brace single, SB36-42		1
3	-	Cross brace double, SB44-50		1
4	-	Rear frames		1
5	-	Rear wheel fixation		1



## Front frames standard

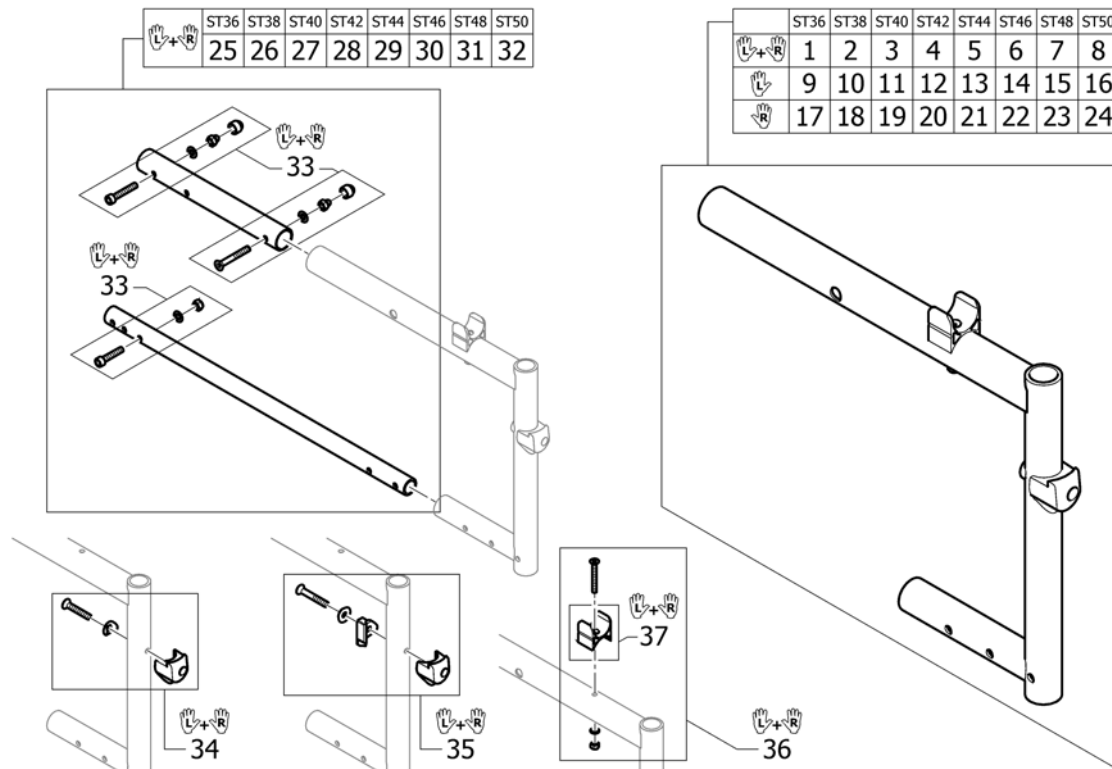

Pos.	Part number	Description	Valid from → until	Qty
1	SP1602872-0453	Front frame ST36, attract blue, pair		1
1	SP1602872-0454	Front frame ST36, attract black, pair		1
1	SP1602872-0455	Front frame ST36, attract grey, pair		1
1	SP1602872-0456	Front frame ST36, attract red, pair		1
2	SP1602873-0453	Front frame ST38, attract blue, pair		1
2	SP1602873-0454	Front frame ST38, attract black, pair		1
2	SP1602873-0455	Front frame ST38, attract grey, pair		1
2	SP1602873-0456	Front frame ST38, attract red, pair		1
3	SP1602942-0453	Front frame ST40, attract blue, pair		1
3	SP1602942-0454	Front frame ST40, attract black, pair		1
3	SP1602942-0455	Front frame ST40, attract grey, pair		1
3	SP1602942-0456	Front frame ST40, attract red, pair		1
4	SP1602964-0453	Front frame ST42, attract blue, pair		1
4	SP1602964-0454	Front frame ST42, attract black, pair		1
4	SP1602964-0455	Front frame ST42, attract grey, pair		1
4	SP1602964-0456	Front frame ST42, attract red, pair		1
5	SP1602965-0453	Front frame ST44, attract blue, pair		1
5	SP1602965-0454	Front frame ST44, attract black, pair		1
5	SP1602965-0455	Front frame ST44, attract grey, pair		1
5	SP1602965-0456	Front frame ST44, attract red, pair		1
6	SP1602966-0453	Front frame ST46, attract blue, pair		1
6	SP1602966-0454	Front frame ST46, attract black, pair		1
6	SP1602966-0455	Front frame ST46, attract grey, pair		1
6	SP1602966-0456	Front frame ST46, attract red, pair		1
7	SP1602993-0453	Front frame ST48, attract blue, pair		1
7	SP1602993-0454	Front frame ST48, attract black, pair		1
7	SP1602993-0455	Front frame ST48, attract grey, pair		1
7	SP1602993-0456	Front frame ST48, attract red, pair		1
8	SP1602994-0453	Front frame ST50, attract blue, pair		1
8	SP1602994-0454	Front frame ST50, attract black, pair		1
8	SP1602994-0455	Front frame ST50, attract grey, pair		1


**Front frames standard**

Pos.	Part number	Description	Valid from → until	Qty
8	SP1602994-0456	Front frame ST50, attract red, pair		1
9	SP1602995-0453	Front frame left ST36, attract blue		1
9	SP1602995-0454	Front frame left ST36, attract black		1
9	SP1602995-0455	Front frame left ST36, attract grey		1
9	SP1602995-0456	Front frame left ST36, attract red		1
10	SP1602996-0453	Front frame left ST38, attract blue		1
10	SP1602996-0454	Front frame left ST38, attract black		1
10	SP1602996-0455	Front frame left ST38, attract grey		1
10	SP1602996-0456	Front frame left ST38, attract red		1
11	SP1602997-0453	Front frame left ST40, attract blue		1
11	SP1602997-0454	Front frame left ST40, attract black		1
11	SP1602997-0455	Front frame left ST40, attract grey		1
11	SP1602997-0456	Front frame left ST40, attract red		1
12	SP1603008-0453	Front frame left ST42, attract blue		1
12	SP1603008-0454	Front frame left ST42, attract black		1
12	SP1603008-0455	Front frame left ST42, attract grey		1
12	SP1603008-0456	Front frame left ST42, attract red		1
13	SP1603009-0453	Front frame left ST44, attract blue		1
13	SP1603009-0454	Front frame left ST44, attract black		1
13	SP1603009-0455	Front frame left ST44, attract grey		1
13	SP1603009-0456	Front frame left ST44, attract red		1
14	SP1603010-0453	Front frame left ST46, attract blue		1
14	SP1603010-0454	Front frame left ST46, attract black		1
14	SP1603010-0455	Front frame left ST46, attract grey		1
14	SP1603010-0456	Front frame left ST46, attract red		1
15	SP1603011-0453	Front frame left ST48, attract blue		1
15	SP1603011-0454	Front frame left ST48, attract black		1
15	SP1603011-0455	Front frame left ST48, attract grey		1
15	SP1603011-0456	Front frame left ST48, attract red		1
16	SP1603012-0453	Front frame left ST50, attract blue		1
16	SP1603012-0454	Front frame left ST50, attract black		1
16	SP1603012-0455	Front frame left ST50, attract grey		1
16	SP1603012-0456	Front frame left ST50, attract red		1
17	SP1603013-0453	Front frame right ST36, attract blue		1
17	SP1603013-0454	Front frame right ST36, attract black		1
17	SP1603013-0455	Front frame right ST36, attract grey		1
17	SP1603013-0456	Front frame right ST36, attract red		1
18	SP1603014-0453	Front frame right ST38, attract blue		1
18	SP1603014-0454	Front frame right ST38, attract black		1
18	SP1603014-0455	Front frame right ST38, attract grey		1
18	SP1603014-0456	Front frame right ST38, attract red		1
19	SP1603015-0453	Front frame right ST40, attract blue		1
19	SP1603015-0454	Front frame right ST40, attract black		1
19	SP1603015-0455	Front frame right ST40, attract grey		1
19	SP1603015-0456	Front frame right ST40, attract red		1
20	SP1603016-0453	Front frame right ST42, attract blue		1
20	SP1603016-0454	Front frame right ST42, attract black		1
20	SP1603016-0455	Front frame right ST42, attract grey		1
20	SP1603016-0456	Front frame right ST42, attract red		1
21	SP1603017-0453	Front frame right ST44, attract blue		1
21	SP1603017-0454	Front frame right ST44, attract black		1
21	SP1603017-0455	Front frame right ST44, attract grey		1
21	SP1603017-0456	Front frame right ST44, attract red		1
22	SP1603018-0453	Front frame right ST46, attract blue		1
22	SP1603018-0454	Front frame right ST46, attract black		1

**Front frames standard**

Pos.	Part number	Description	Valid from → until	Qty
22	SP1603018-0455	Front frame right ST46, attract grey		1
22	SP1603018-0456	Front frame right ST46, attract red		1
23	SP1603019-0453	Front frame right ST48, attract blue		1
23	SP1603019-0454	Front frame right ST48, attract black		1
23	SP1603019-0455	Front frame right ST48, attract grey		1
23	SP1603019-0456	Front frame right ST48, attract red		1
24	SP1603020-0453	Front frame right ST50, attract blue		1
24	SP1603020-0454	Front frame right ST50, attract black		1
24	SP1603020-0455	Front frame right ST50, attract grey		1
24	SP1603020-0456	Front frame right ST50, attract red		1
25	SP1603021	Frames connecting tubes kit, ST36		1
26	SP1603022	Frames connecting tubes kit, ST38		1
27	SP1603023	Frames connecting tubes kit, ST40		1
28	SP1603386	Frames connecting tubes kit, ST42		1
29	SP1603387	Frames connecting tubes kit, ST44		1
30	SP1603388	Frames connecting tubes kit, ST46		1
31	SP1603389	Frames connecting tubes kit, ST48		1
32	SP1603390	Frames connecting tubes kit, ST50		1
33	SP1603391	Screws kit to frames connecting tubes		1
34	SP1603392	Position mechanism for swing-away, kit, pair		1
35	SP1603777	Rotating limiter for swing-away, kit, pair		1
36	SP1603774	Seat lock kit, complete		1
37	SP1603393	Seat lock kit		1

# Cross brace single, SB36-42

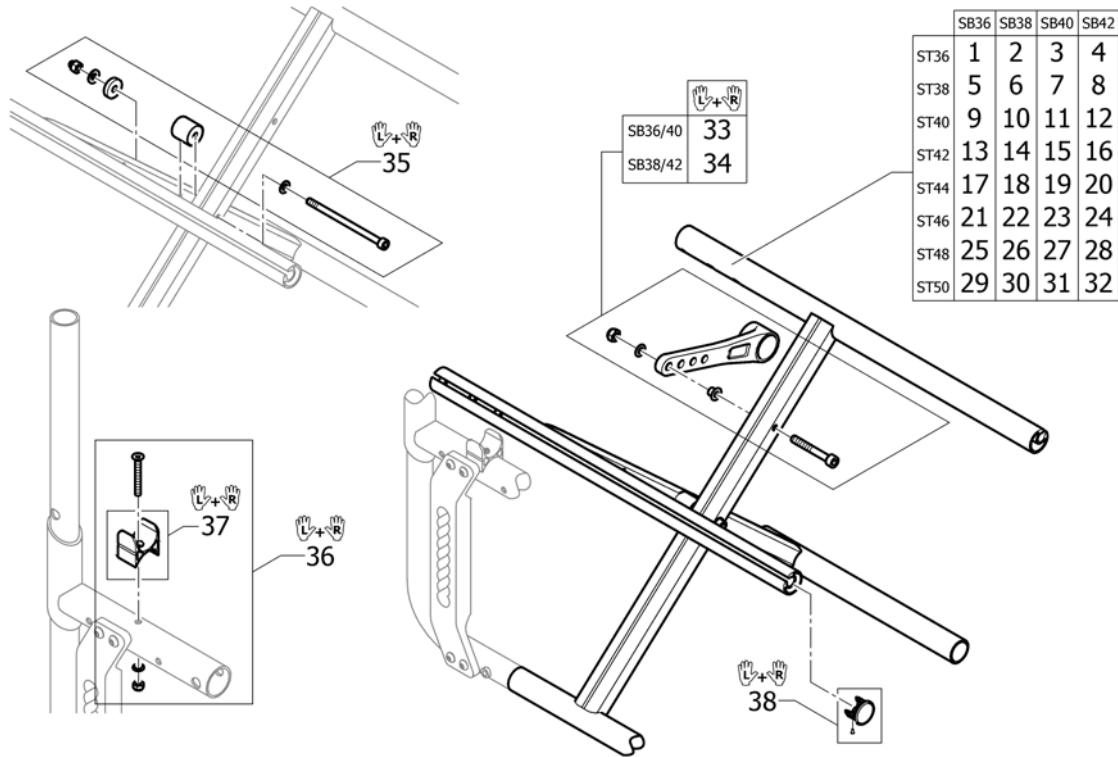


Pos.	Part number	Description	Valid from → until	Qty
1	SP1603394-0454	Cross-brace single, SB36 ST36, attract black		1
2	SP1603402-0454	Cross-brace single, SB38 ST36, attract black		1
3	SP1603410-0454	Cross-brace single, SB40 ST36, attract black		1
4	SP1603512-0454	Cross-brace single, SB42 ST36, attract black		1
5	SP1603395-0454	Cross-brace single, SB36 ST38, attract black		1
6	SP1603403-0454	Cross-brace single, SB38 ST38, attract black		1
7	SP1603411-0454	Cross-brace single, SB40 ST38, attract black		1
8	SP1603545-0454	Cross-brace single, SB42 ST38, attract black		1
9	SP1603396-0454	Cross-brace single, SB36 ST40, attract black		1
10	SP1603404-0454	Cross-brace single, SB38 ST40, attract black		1
11	SP1603412-0454	Cross-brace single, SB40 ST40, attract black		1
12	SP1603554-0454	Cross-brace single, SB42 ST40, attract black		1
13	SP1603397-0454	Cross-brace single, SB36 ST42, attract black		1
14	SP1603405-0454	Cross-brace single, SB38 ST42, attract black		1
15	SP1603413-0454	Cross-brace single, SB40 ST42, attract black		1
16	SP1603555-0454	Cross-brace single, SB42 ST42, attract black		1
17	SP1603398-0454	Cross-brace single, SB36 ST44, attract black		1
18	SP1603406-0454	Cross-brace single, SB38 ST44, attract black		1
19	SP1603414-0454	Cross-brace single, SB40 ST44, attract black		1
20	SP1603556-0454	Cross-brace single, SB42 ST44, attract black		1
21	SP1603399-0454	Cross-brace single, SB36 ST46, attract black		1
22	SP1603407-0454	Cross-brace single, SB38 ST46, attract black		1
23	SP1603415-0454	Cross-brace single, SB40 ST46, attract black		1
24	SP1603557-0454	Cross-brace single, SB42 ST46, attract black		1
25	SP1603400-0454	Cross-brace single, SB36 ST48, attract black		1
26	SP1603408-0454	Cross-brace single, SB38 ST48, attract black		1
27	SP1603503-0454	Cross-brace single, SB40 ST48, attract black		1
28	SP1603401-0454	Cross-brace single, SB36 ST50, attract black		1
29	SP1603409-0454	Cross-brace single, SB38 ST50, attract black		1
30	SP1603504-0454	Cross-brace single, SB40 ST50, attract black		1
31	SP1603558-0454	Cross-brace single, SB42 ST48, attract black		1

**Cross brace single, SB36-42**

Pos.	Part number	Description	Valid from → until	Qty
32	SP1603559-0454	Cross-brace single, SB42 ST50, attract black		1
33	SP1603751	Retaining lever SB36/40/44/48, pair		1
34	SP1603752	Retaining lever SB38/42/46/50, pair		1
35	SP1603753	Screw kit to Single crossbrace		1
36	SP1603774	Seat lock kit, complete		1
37	SP1603393	Seat lock kit		1
38	SP1603765	Cross-brace plug kit, plastic		1

# Cross brace double, SB44-50

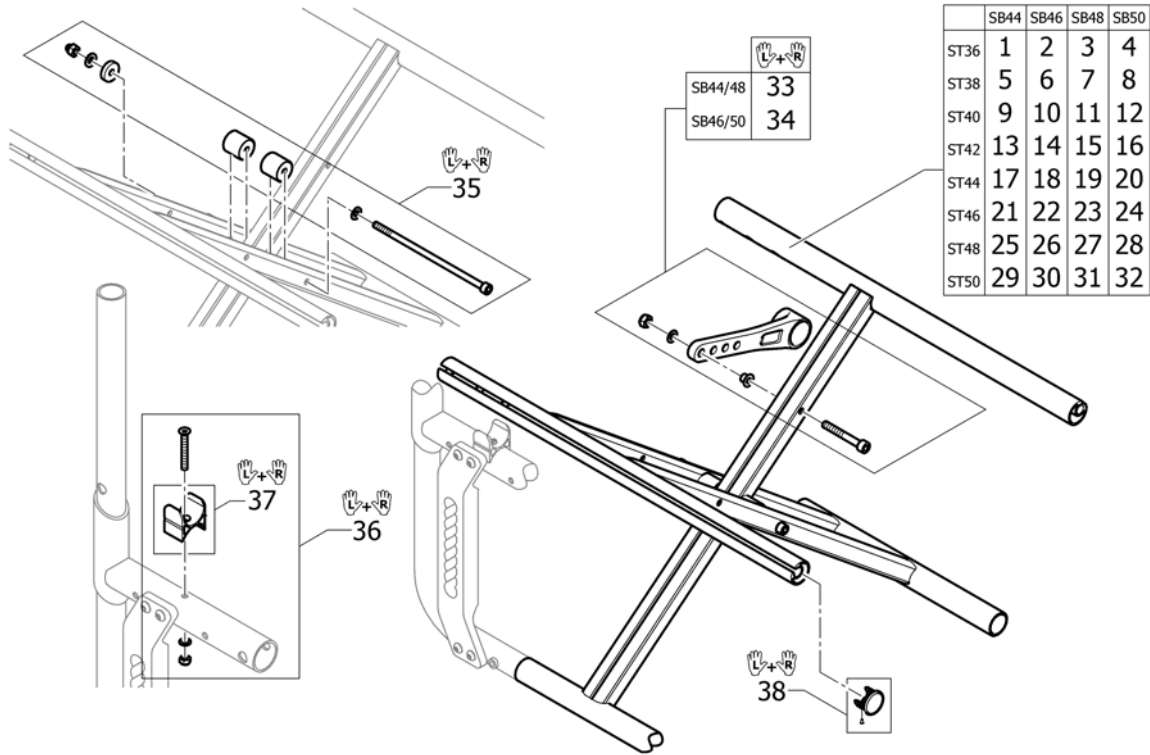


Pos.	Part number	Description	Valid from → until	Qty
1	SP1603560-0454	Cross-brace triplex, SB44 ST36, attract black		1
2	SP1603568-0454	Cross-brace triplex, SB46 ST36, attract black		1
3	SP1603625-0454	Cross-brace triplex, SB48 ST36, attract black		1
4	SP1603743-0454	Cross-brace triplex, SB50 ST36, attract black		1
5	SP1603561-0454	Cross-brace triplex, SB44 ST38, attract black		1
6	SP1603569-0454	Cross-brace triplex, SB46 ST38, attract black		1
7	SP1603626-0454	Cross-brace triplex, SB48 ST38, attract black		1
8	SP1603744-0454	Cross-brace triplex, SB50 ST38, attract black		1
9	SP1603562-0454	Cross-brace triplex, SB44 ST40, attract black		1
10	SP1603619-0454	Cross-brace triplex, SB46 ST40, attract black		1
11	SP1603627-0454	Cross-brace triplex, SB48 ST40, attract black		1
12	SP1603745-0454	Cross-brace triplex, SB50 ST40, attract black		1
13	SP1603563-0454	Cross-brace triplex, SB44 ST42, attract black		1
14	SP1603620-0454	Cross-brace triplex, SB46 ST42, attract black		1
15	SP1603628-0454	Cross-brace triplex, SB48 ST42, attract black		1
16	SP1603746-0454	Cross-brace triplex, SB50 ST42, attract black		1
17	SP1603564-0454	Cross-brace triplex, SB44 ST44, attract black		1
18	SP1603621-0454	Cross-brace triplex, SB46 ST44, attract black		1
19	SP1603629-0454	Cross-brace triplex, SB48 ST44, attract black		1
20	SP1603747-0454	Cross-brace triplex, SB50 ST44, attract black		1
21	SP1603565-0454	Cross-brace triplex, SB44 ST46, attract black		1
22	SP1603622-0454	Cross-brace triplex, SB46 ST46, attract black		1
23	SP1603740-0454	Cross-brace triplex, SB48 ST46, attract black		1
24	SP1603748-0454	Cross-brace triplex, SB50 ST46, attract black		1
25	SP1603566-0454	Cross-brace triplex, SB44 ST48, attract black		1
26	SP1603623-0454	Cross-brace triplex, SB46 ST48, attract black		1
27	SP1603741-0454	Cross-brace triplex, SB48 ST48, attract black		1
28	SP1603749-0454	Cross-brace triplex, SB50 ST48, attract black		1
29	SP1603567-0454	Cross-brace triplex, SB44 ST50, attract black		1
30	SP1603624-0454	Cross-brace triplex, SB46 ST50, attract black		1
31	SP1603742-0454	Cross-brace triplex, SB48 ST50, attract black		1

**Cross brace double, SB44-50**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
32	SP1603750-0454	Cross-brace triplex, SB50 ST50, attract black		1
33	SP1603751	Retaining lever SB36/40/44/48, pair		1
34	SP1603752	Retaining lever SB38/42/46/50, pair		1
35	SP1603764	Screw kit to triplex crossbrace		1
36	SP1603774	Seat lock kit, complete		1
37	SP1603393	Seat lock kit		1
38	SP1603765	Cross-brace plug kit, plastic		1


## Rear frames

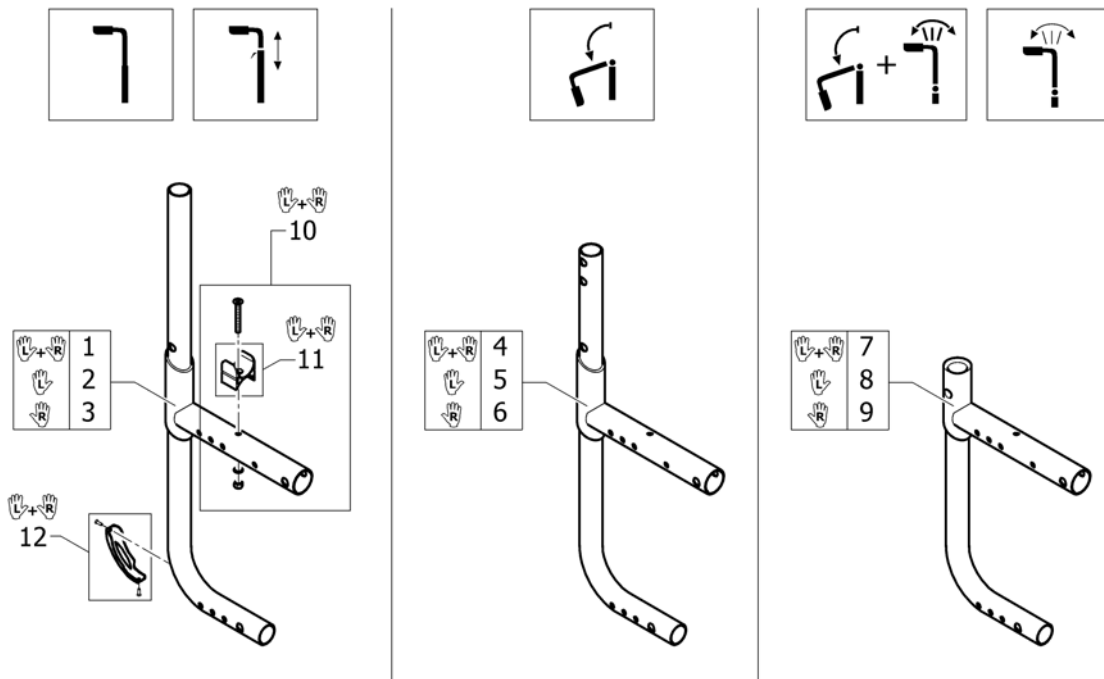

Pos.	Part number	Description	Valid from → until	Qty
1	SP1603766-0454	Rear frame, attract black, Pair		1
2	SP1603767-0454	Rear frame left, attract black		1
3	SP1603768-0454	Rear frame right, attract black		1
4	SP1606530-0454	Rear frame medium to foldable back, attract black, Pair	2016-07-01 →	1
5	SP1606531-0454	Rear frame left medium to foldable attract black	2016-07-01 →	1
6	SP1606532-0454	Rear frame right medium to foldable attract black	2016-07-01 →	1
7	SP1603769-0454	Rear frame short to angle adjustable back, attract black, Pair		1
8	SP1603770-0454	Rear frame left short to angle adjustable attract black, matt		1
9	SP1603771-0454	Rear frame right short to angle adjustable attract black, matt		1
10	SP1603774	Seat lock kit, complete		1
11	SP1603393	Seat lock kit		1
12	1498774	Rear frame protection, pair		1




## Rear wheel fixation

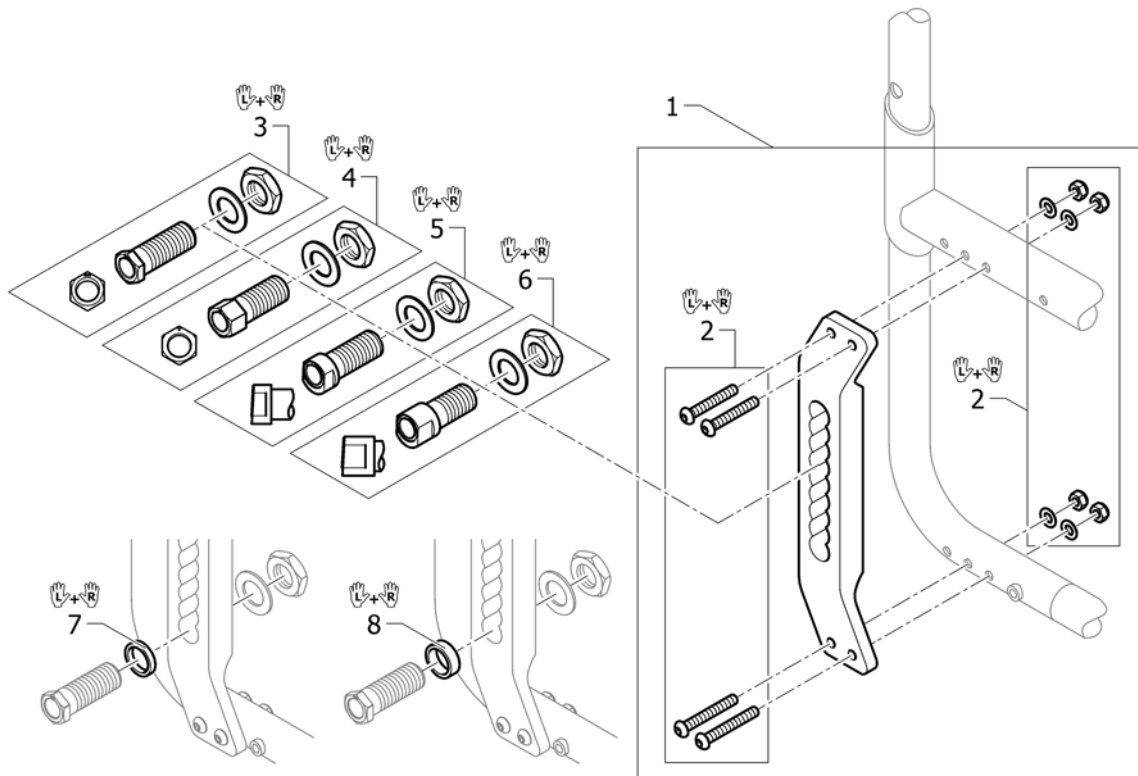


Pos.	Part number	Description	Valid from → until	Qty
1	SP1603772	Vertical brace, complete		1
2	SP1603773	Fixation kit for Vertical brace, pair		1
3	SP1589279	Adapter sleeve kit 2°, head short, pair		1
4	SP1603776	Adapter sleeve kit 1°, head long, pair		1
5	SP1589282	Adapter sleeve kit 4°, head short, pair		1
6	SP1589281	Adapter sleeve kit 4°, head long, pair		1
7	SP1603775	Spacer L=4mm for adapter sleeve, pair		1
8	SP1589284	Spacer L=7mm for adapter sleeve, pair		1

# SEAT

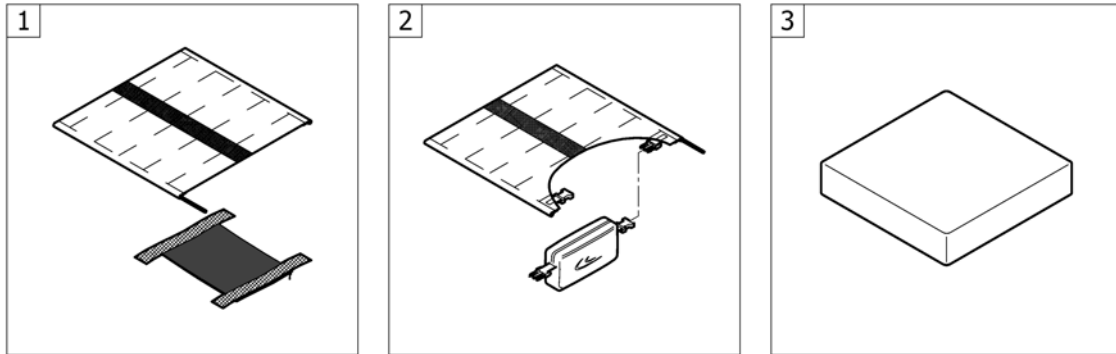

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat cover with 1 Velcro band (soft band)		1
2	-	Seat cover with 1 Velcro band (soft band) for seat bag		1
3	-	Seat cushion standard		1

# Seat cover with 1 Velcro band (soft band)

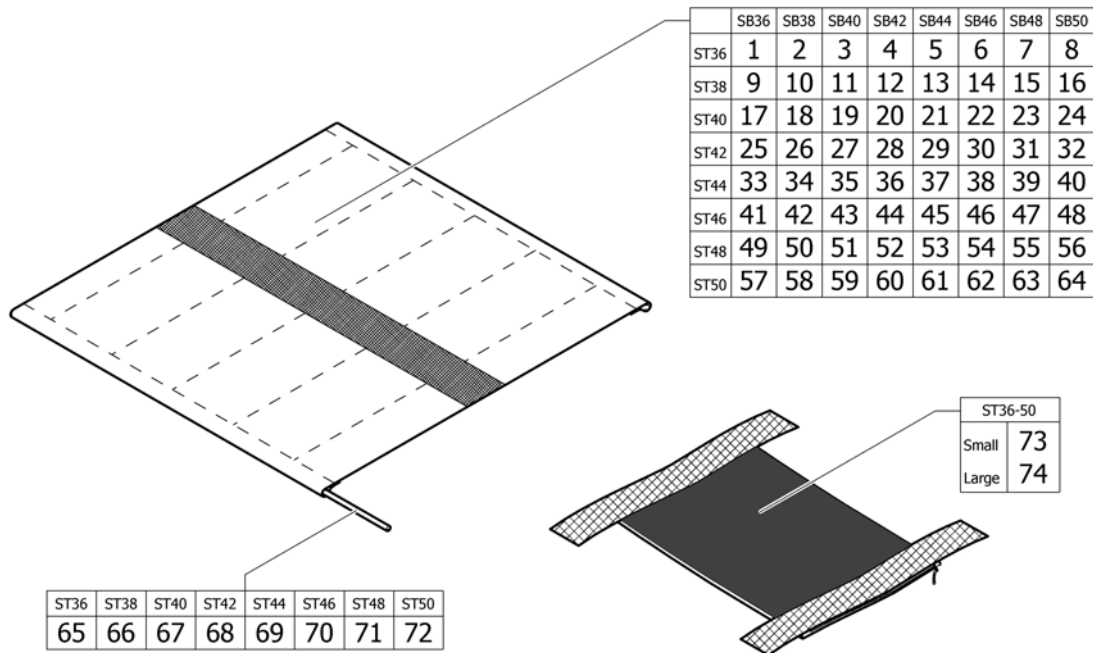


Pos.	Part number	Description	Valid from → until	Qty
1	1540208	Seat cover SB36 ST36 with 1 Velcro band (soft band)		1
2	1540209	Seat cover SB38 ST36 with 1 Velcro band (soft band)		1
3	1540210	Seat cover SB40 ST36 with 1 Velcro band (soft band)		1
4	1540211	Seat cover SB42 ST36 with 1 Velcro band (soft band)		1
5	1540212	Seat cover SB44 ST36 with 1 Velcro band (soft band)		1
6	1540213	Seat cover SB46 ST36 with 1 Velcro band (soft band)		1
7	1540214	Seat cover SB48 ST36 with 1 Velcro band (soft band)		1
8	1540215	Seat cover SB50 ST36 with 1 Velcro band (soft band)		1
9	1540220	Seat cover SB36 ST38 with 1 Velcro band (soft band)		1
10	1540221	Seat cover SB38 ST38 with 1 Velcro band (soft band)		1
11	1540222	Seat cover SB40 ST38 with 1 Velcro band (soft band)		1
12	1540223	Seat cover SB42 ST38 with 1 Velcro band (soft band)		1
13	1540224	Seat cover SB44 ST38 with 1 Velcro band (soft band)		1
14	1540225	Seat cover SB46 ST38 with 1 Velcro band (soft band)		1
15	1540226	Seat cover SB48 ST38 with 1 Velcro band (soft band)		1
16	1540227	Seat cover SB50 ST38 with 1 Velcro band (soft band)		1
17	1540232	Seat cover SB36 ST40 with 1 Velcro band (soft band)		1
18	1540233	Seat cover SB38 ST40 with 1 Velcro band (soft band)		1
19	1540234	Seat cover SB40 ST40 with 1 Velcro band (soft band)		1
20	1540235	Seat cover SB42 ST40 with 1 Velcro band (soft band)		1
21	1540236	Seat cover SB44 ST40 with 1 Velcro band (soft band)		1
22	1540237	Seat cover SB46 ST40 with 1 Velcro band (soft band)		1
23	1540238	Seat cover SB48 ST40 with 1 Velcro band (soft band)		1
24	1540239	Seat cover SB50 ST40 with 1 Velcro band (soft band)		1
25	1540244	Seat cover SB36 ST42 with 1 Velcro band (soft band)		1
26	1540245	Seat cover SB38 ST42 with 1 Velcro band (soft band)		1
27	1540246	Seat cover SB40 ST42 with 1 Velcro band (soft band)		1
28	1540247	Seat cover SB42 ST42 with 1 Velcro band (soft band)		1
29	1540248	Seat cover SB44 ST42 with 1 Velcro band (soft band)		1
30	1540249	Seat cover SB46 ST42 with 1 Velcro band (soft band)		1
31	1540250	Seat cover SB48 ST42 with 1 Velcro band (soft band)		1

**Seat cover with 1 Velcro band (soft band)**

Pos.	Part number	Description	Valid from → until	Qty
32	1540251	Seat cover SB50 ST42 with 1 Velcro band (soft band)		1
33	1540256	Seat cover SB36 ST44 with 1 Velcro band (soft band)		1
34	1540257	Seat cover SB38 ST44 with 1 Velcro band (soft band)		1
35	1540258	Seat cover SB40 ST44 with 1 Velcro band (soft band)		1
36	1540259	Seat cover SB42 ST44 with 1 Velcro band (soft band)		1
37	1540682	Seat cover SB44 ST44 with 1 Velcro band (soft band)		1
38	1540683	Seat cover SB46 ST44 with 1 Velcro band (soft band)		1
39	1540684	Seat cover SB48 ST44 with 1 Velcro band (soft band)		1
40	1540685	Seat cover SB50 ST44 with 1 Velcro band (soft band)		1
41	1540690	Seat cover SB36 ST46 with 1 Velcro band (soft band)		1
42	1540691	Seat cover SB38 ST46 with 1 Velcro band (soft band)		1
43	1540692	Seat cover SB40 ST46 with 1 Velcro band (soft band)		1
44	1540693	Seat cover SB42 ST46 with 1 Velcro band (soft band)		1
45	1540694	Seat cover SB44 ST46 with 1 Velcro band (soft band)		1
46	1540695	Seat cover SB46 ST46 with 1 Velcro band (soft band)		1
47	1540696	Seat cover SB48 ST46 with 1 Velcro band (soft band)		1
48	1540697	Seat cover SB50 ST46 with 1 Velcro band (soft band)		1
49	1540702	Seat cover SB36 ST48 with 1 Velcro band (soft band)		1
50	1540703	Seat cover SB38 ST48 with 1 Velcro band (soft band)		1
51	1540704	Seat cover SB40 ST48 with 1 Velcro band (soft band)		1
52	1540705	Seat cover SB42 ST48 with 1 Velcro band (soft band)		1
53	1540706	Seat cover SB44 ST48 with 1 Velcro band (soft band)		1
54	1540707	Seat cover SB46 ST48 with 1 Velcro band (soft band)		1
55	1540708	Seat cover SB48 ST48 with 1 Velcro band (soft band)		1
56	1540709	Seat cover SB50 ST48 with 1 Velcro band (soft band)		1
57	1540714	Seat cover SB36 ST50 with 1 Velcro band (soft band)		1
58	1540715	Seat cover SB38 ST50 with 1 Velcro band (soft band)		1
59	1540716	Seat cover SB40 ST50 with 1 Velcro band (soft band)		1
60	1540717	Seat cover SB42 ST50 with 1 Velcro band (soft band)		1
61	1540718	Seat cover SB44 ST50 with 1 Velcro band (soft band)		1
62	1540719	Seat cover SB46 ST50 with 1 Velcro band (soft band)		1
63	1540720	Seat cover SB48 ST50 with 1 Velcro band (soft band)		1
64	1540721	Seat cover SB50 ST50 with 1 Velcro band (soft band)		1
65	1564334	Seat cover bar kit ST36		1
66	1564336	Seat cover bar kit ST38		1
67	1564337	Seat cover bar kit ST40		1
68	1564338	Seat cover bar kit ST42		1
69	1564339	Seat cover bar kit ST44		1
70	1564341	Seat cover bar kit ST46		1
71	1564342	Seat cover bar kit ST48		1
72	1564343	Seat cover bar kit ST50		1
73	1547623	Seat upholstery bag, small		1
74	1547624	Seat upholstery bag, big		1


## Seat cover with 1 Velcro band (soft band) for seat bag

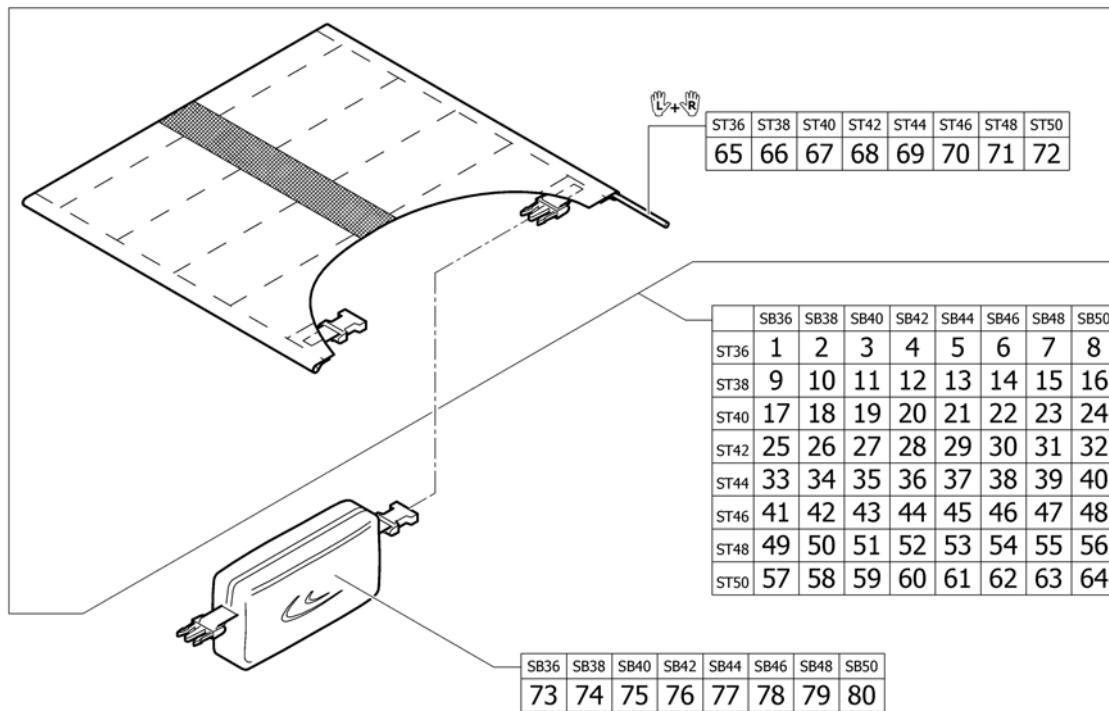

Pos.	Part number	Description	Valid from → until	Qty
1	1576589	Seat cover SB36 ST36 for seat bag with 1 Velcro band (soft band)		1
2	1576695	Seat cover SB38 ST36 for seat bag with 1 Velcro band (soft band)		1
3	1576696	Seat cover SB40 ST36 for seat bag with 1 Velcro band (soft band)		1
4	1576697	Seat cover SB42 ST36 for seat bag with 1 Velcro band (soft band)		1
5	1576698	Seat cover SB44 ST36 for seat bag with 1 Velcro band (soft band)		1
6	1576699	Seat cover SB46 ST36 for seat bag with 1 Velcro band (soft band)		1
7	1576700	Seat cover SB48 ST36 for seat bag with 1 Velcro band (soft band)		1
8	1576701	Seat cover SB50 ST36 for seat bag with 1 Velcro band (soft band)		1
9	1576706	Seat cover SB36 ST38 for seat bag with 1 Velcro band (soft band)		1
10	1576707	Seat cover SB38 ST38 for seat bag with 1 Velcro band (soft band)		1
11	1576708	Seat cover SB40 ST38 for seat bag with 1 Velcro band (soft band)		1
12	1576709	Seat cover SB42 ST38 for seat bag with 1 Velcro band (soft band)		1
13	1576710	Seat cover SB44 ST38 for seat bag with 1 Velcro band (soft band)		1
14	1577138	Seat cover SB46 ST38 for seat bag with 1 Velcro band (soft band)		1
15	1577139	Seat cover SB48 ST38 for seat bag with 1 Velcro band (soft band)		1
16	1577288	Seat cover SB50 ST38 for seat bag with 1 Velcro band (soft band)		1
17	1577992	Seat cover SB36 ST40 for seat bag with 1 Velcro band (soft band)		1
18	1578183	Seat cover SB38 ST40 for seat bag with 1 Velcro band (soft band)		1
19	1578184	Seat cover SB40 ST40 for seat bag with 1 Velcro band (soft band)		1
20	1578185	Seat cover SB42 ST40 for seat bag with 1 Velcro band (soft band)		1
21	1578186	Seat cover SB44 ST40 for seat bag with 1 Velcro band (soft band)		1
22	1578187	Seat cover SB46 ST40 for seat bag with 1 Velcro band (soft band)		1
23	1578188	Seat cover SB48 ST40 for seat bag with 1 Velcro band (soft band)		1
24	1578189	Seat cover SB50 ST40 for seat bag with 1 Velcro band (soft band)		1
25	1578323	Seat cover SB36 ST42 for seat bag with 1 Velcro band (soft band)		1
26	1578324	Seat cover SB38 ST42 for seat bag with 1 Velcro band (soft band)		1
27	1578325	Seat cover SB40 ST42 for seat bag with 1 Velcro band (soft band)		1
28	1578400	Seat cover SB42 ST42 for seat bag with 1 Velcro band (soft band)		1
29	1578401	Seat cover SB44 ST42 for seat bag with 1 Velcro band (soft band)		1
30	1578423	Seat cover SB46 ST42 for seat bag with 1 Velcro band (soft band)		1
31	1578424	Seat cover SB48 ST42 for seat bag with 1 Velcro band (soft band)		1

**Seat cover with 1 Velcro band (soft band) for seat bag**

Pos.	Part number	Description	Valid from → until	Qty
32	1578425	Seat cover SB50 ST42 for seat bag with 1 Velcro band (soft band)		1
33	1578430	Seat cover SB36 ST44 for seat bag with 1 Velcro band (soft band)		1
34	1578618	Seat cover SB38 ST44 for seat bag with 1 Velcro band (soft band)		1
35	1578619	Seat cover SB40 ST44 for seat bag with 1 Velcro band (soft band)		1
36	1578620	Seat cover SB42 ST44 for seat bag with 1 Velcro band (soft band)		1
37	1578621	Seat cover SB44 ST44 for seat bag with 1 Velcro band (soft band)		1
38	1578622	Seat cover SB46 ST44 for seat bag with 1 Velcro band (soft band)		1
39	1578623	Seat cover SB48 ST44 for seat bag with 1 Velcro band (soft band)		1
40	1578624	Seat cover SB50 ST44 for seat bag with 1 Velcro band (soft band)		1
41	1578629	Seat cover SB36 ST46 for seat bag with 1 Velcro band (soft band)		1
42	1578630	Seat cover SB38 ST46 for seat bag with 1 Velcro band (soft band)		1
43	1578631	Seat cover SB40 ST46 for seat bag with 1 Velcro band (soft band)		1
44	1578632	Seat cover SB42 ST46 for seat bag with 1 Velcro band (soft band)		1
45	1578633	Seat cover SB44 ST46 for seat bag with 1 Velcro band (soft band)		1
46	1578634	Seat cover SB46 ST46 for seat bag with 1 Velcro band (soft band)		1
47	1578635	Seat cover SB48 ST46 for seat bag with 1 Velcro band (soft band)		1
48	1578636	Seat cover SB50 ST46 for seat bag with 1 Velcro band (soft band)		1
49	1578641	Seat cover SB36 ST48 for seat bag with 1 Velcro band (soft band)		1
50	1578642	Seat cover SB38 ST48 for seat bag with 1 Velcro band (soft band)		1
51	1578643	Seat cover SB40 ST48 for seat bag with 1 Velcro band (soft band)		1
52	1578644	Seat cover SB42 ST48 for seat bag with 1 Velcro band (soft band)		1
53	1578645	Seat cover SB44 ST48 for seat bag with 1 Velcro band (soft band)		1
54	1578646	Seat cover SB46 ST48 for seat bag with 1 Velcro band (soft band)		1
55	1578647	Seat cover SB48 ST48 for seat bag with 1 Velcro band (soft band)		1
56	1578648	Seat cover SB50 ST48 for seat bag with 1 Velcro band (soft band)		1
57	1578653	Seat cover SB36 ST50 for seat bag with 1 Velcro band (soft band)		1
58	1578654	Seat cover SB38 ST50 for seat bag with 1 Velcro band (soft band)		1
59	1578655	Seat cover SB40 ST50 for seat bag with 1 Velcro band (soft band)		1
60	1578656	Seat cover SB42 ST50 for seat bag with 1 Velcro band (soft band)		1
61	1578657	Seat cover SB44 ST50 for seat bag with 1 Velcro band (soft band)		1
62	1578658	Seat cover SB46 ST50 for seat bag with 1 Velcro band (soft band)		1
63	1578659	Seat cover SB48 ST50 for seat bag with 1 Velcro band (soft band)		1
64	1578660	Seat cover SB50 ST50 for seat bag with 1 Velcro band (soft band)		1
65	1564334	Seat cover bar kit ST36		1
66	1564336	Seat cover bar kit ST38		1
67	1564337	Seat cover bar kit ST40		1
68	1564338	Seat cover bar kit ST42		1
69	1564339	Seat cover bar kit ST44		1
70	1564341	Seat cover bar kit ST46		1
71	1564342	Seat cover bar kit ST48		1
72	1564343	Seat cover bar kit ST50		1
73	1578788	Seat bag SB36		1
74	1578789	Seat bag SB38		1
75	1578790	Seat bag SB40		1
76	1578791	Seat bag SB42		1
77	1578792	Seat bag SB44		1
78	1578793	Seat bag SB46		1
79	1578794	Seat bag SB48		1
80	1578795	Seat bag SB50		1

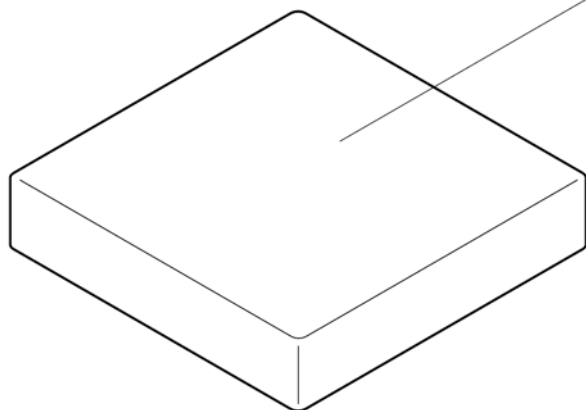

	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
ST36	1	2	3	4	5	6	7	8
ST38	2	9	10	11	12	13	14	15
ST40	3	10	16	17	18	19	20	21
ST42	4	11	17	22	23	24	25	26
ST44	5	12	18	23	27	28	29	30
ST82	6	13	19	24	28	31	32	33
ST48	7	14	20	25	29	32	34	35
ST50	8	15	21	26	30	33	35	36


Pos.	Part number	Description	Valid from → until	Qty
1	1556335	Seat cushion 360mm x 360mm		1
2	1556336	Seat cushion 360mm x 380mm		1
3	1556337	Seat cushion 360mm x 400mm		1
4	1556338	Seat cushion 360mm x 420mm		1
5	1556339	Seat cushion 360mm x 440mm		1
6	1556340	Seat cushion 360mm x 460mm		1
7	1556341	Seat cushion 360mm x 480mm		1
8	1556342	Seat cushion 360mm x 500mm		1
9	1556344	Seat cushion 380mm x 380mm		1
10	1556345	Seat cushion 380mm x 400mm		1
11	1556346	Seat cushion 380mm x 420mm		1
12	1556347	Seat cushion 380mm x 440mm		1
13	1556348	Seat cushion 380mm x 460mm		1
14	1556349	Seat cushion 380mm x 480mm		1
15	1556350	Seat cushion 380mm x 500mm		1
16	1556352	Seat cushion 400mm x 400mm		1
17	1556353	Seat cushion 400mm x 420mm		1
18	1556354	Seat cushion 400mm x 440mm		1
19	1556355	Seat cushion 400mm x 460mm		1
20	1556356	Seat cushion 400mm x 480mm		1
21	1556357	Seat cushion 400mm x 500mm		1
22	1556359	Seat cushion 420mm x 420mm		1
23	1556360	Seat cushion 420mm x 440mm		1
24	1556361	Seat cushion 420mm x 460mm		1
25	1556362	Seat cushion 420mm x 480mm		1
26	1556363	Seat cushion 420mm x 500mm		1
27	1556365	Seat cushion 440mm x 440mm		1
28	1556366	Seat cushion 440mm x 460mm		1
29	1556367	Seat cushion 440mm x 480mm		1
30	1556368	Seat cushion 440mm x 500mm		1
31	1556370	Seat cushion 460mm x 460mm		1

## Seat cushion standard

Pos.	Part number	Description	Valid from → until	Qty
32	1556371	Seat cushion 460mm x 480mm		1
33	1556372	Seat cushion 460mm x 500mm		1
34	1556374	Seat cushion 480mm x 480mm		1
35	1556375	Seat cushion 480mm x 500mm		1
36	1556377	Seat cushion 500mm x 500mm		1




# BACKREST

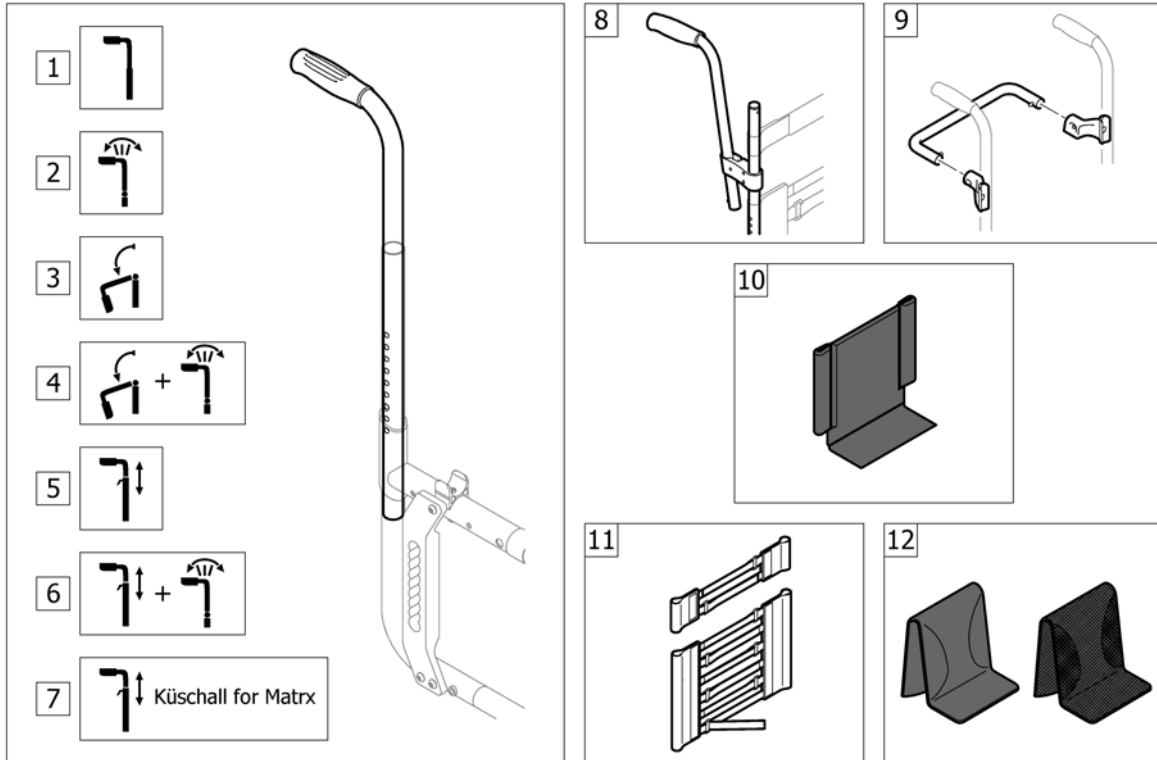


Pos.	Part number	Description	Valid from → until	Qty
1	-	Push handles for backrest standard		1
2	-	Push handles for backrest angle adjustable		1
3	-	Push handles for backrest foldable	2016-07-01 →	1
4	-	Push handles for backrest foldable and angle adjustable	2016-07-01 →	1
5	-	Push handles height adjustable		1
6	-	Push handles height adjustable for backrest angle adjustable		1
7	-	Küschall backrest parts for Matrix backs		1
8	-	Push handles, inclined		1
9	-	Backrest stabilization bar		1
10	-	Back cover, standard		1
11	-	Bands to adjustable backrest		1
12	-	Cover to adjustable backrest		1

## Push handles for backrest standard

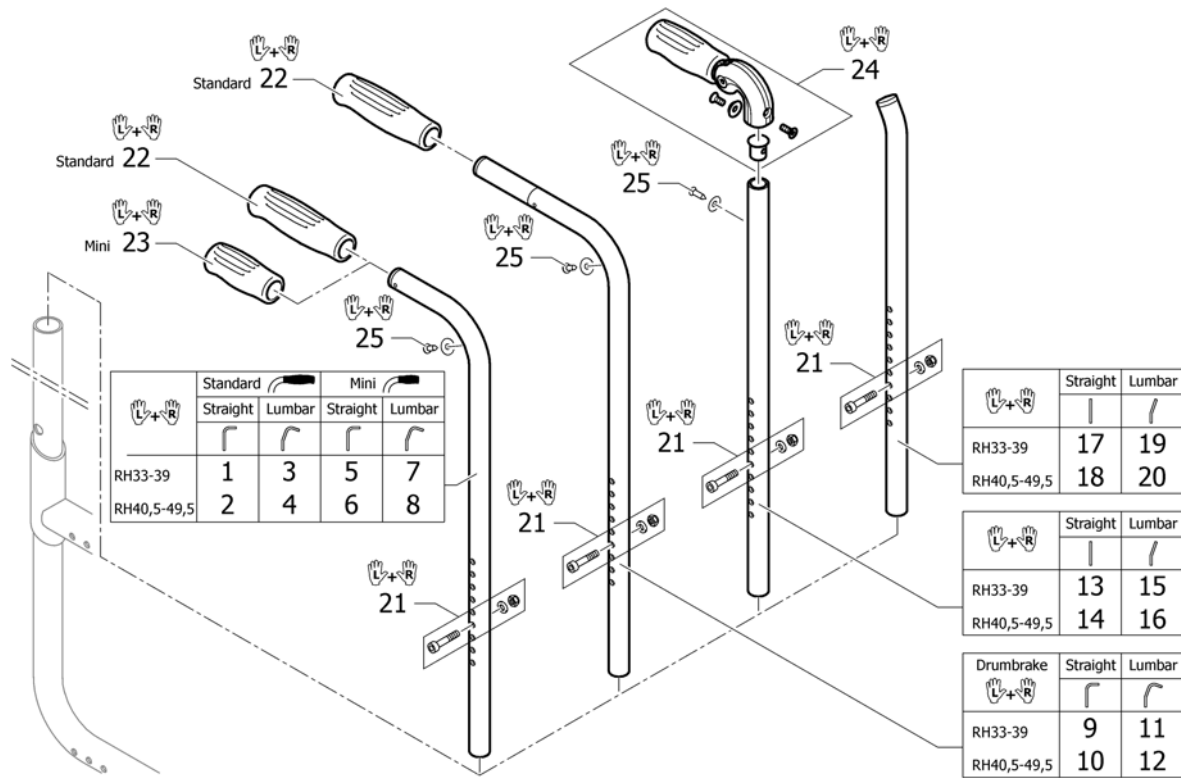


Pos.	Part number	Description	Valid from → until	Qty
1	1540072	Push handles standard, short, pair		1
2	1540073	Push handles standard, medium, pair		1
3	1540074	Push handles standard lumbar short, pair		1
4	1540075	Push handles standard lumbar medium, pair		1
5	1540076	Push handles mini, short, pair		1
6	1540077	Push handles mini, medium, pair		1
7	1540078	Push handles mini lumbar short, pair		1
8	1540079	Push handles mini lumbar medium, pair		1
9	1540084	Push handles short to drum brake, pair		1
10	1540085	Push handles medium to drum brake, pair		1
11	1540086	Push handles lumbar short to drum brake, pair		1
12	1540087	Push handles lumbar medium to drum brake, pair		1
13	1540080	Backrest tube straight short to foldable push handles, pair		1
14	1540081	Backrest tube straight medium to foldable push handles, pair		1
15	1540082	Backrest tube lumbar short to foldable push handles, pair		1
16	1540083	Backrest tube lumbar medium to foldable push handles, pair		1
17	1540089	Backrest tubes L without push handles, pair		1
18	1540090	Backrest tubes XL without push handles, pair		1
19	1540091	Backrest tubes lumbar L without push handles, pair		1
20	1540092	Backrest tubes lumbar XL without push handles, pair		1
21	1540088	Fixation kit to backrest tube		1
22	1507734	Handle black, pair		1
23	1525126	Handle black, mini, pair		1
24	1507733	Push handles foldable, pair		1
25	1432083	Fixation kit for backrest band		1

# Push handles for backrest angle adjustable

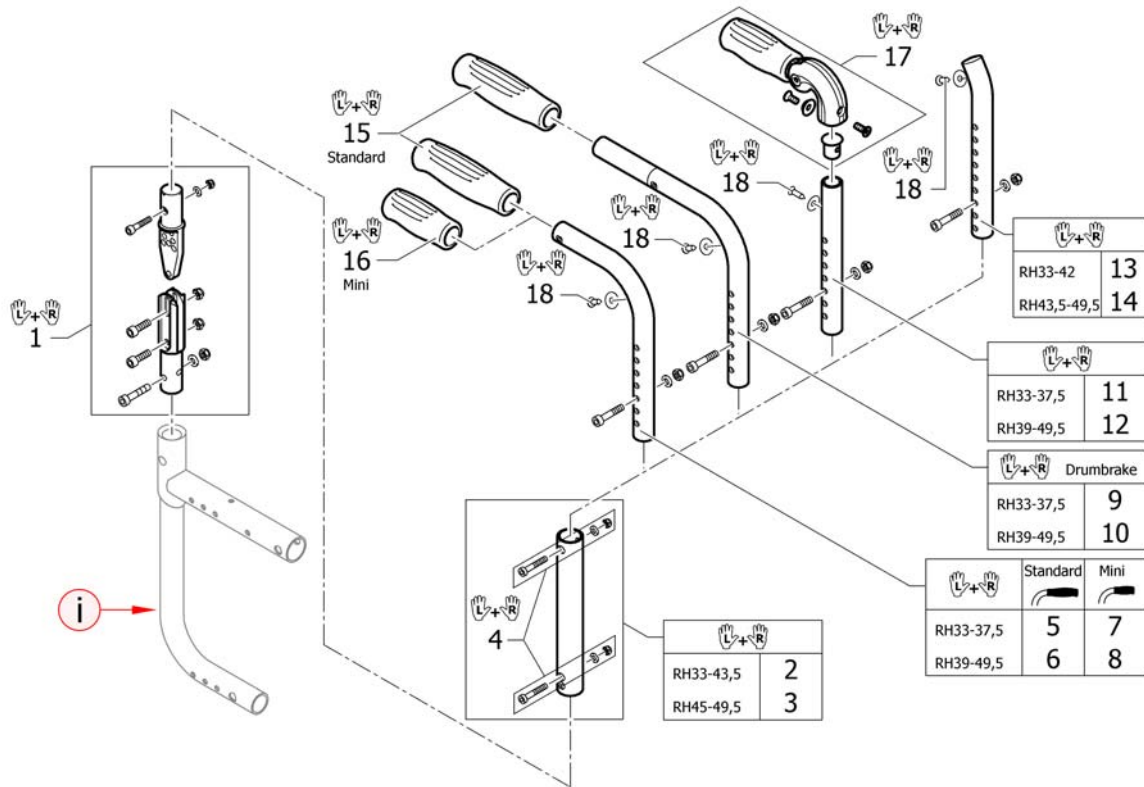


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1540094	Backrest angle kit		1
2	1540063	Tube kit medium to angle adjustable back		1
3	1540095	Tube kit long to angle adjustable back		1
4	1540065	Screw kit for telescopic tube		1
5	1540055	Push handles standard, XS, pair		1
6	1540056	Push handles standard, S, pair		1
7	1540069	Push handles mini, XS, pair		1
8	1540070	Push handles mini, S, pair		1
9	1540057	Push handles XS to drum brake, pair		1
10	1540058	Push handles S to drum brake, pair		1
11	1540059	Backrest tube straight XS to foldable push handles, pair		1
12	1540060	Backrest tube straight S to foldable push handles, pair		1
13	1540061	Backrest tubes S without push handles, pair		1
14	1540062	Backrest tubes M without push handles, pair		1
15	1507734	Handle black, pair		1
16	1525126	Handle black, mini, pair		1
17	1507733	Push handles foldable, pair		1
18	1432083	Fixation kit for backrest band		1

## Push handles for backrest foldable

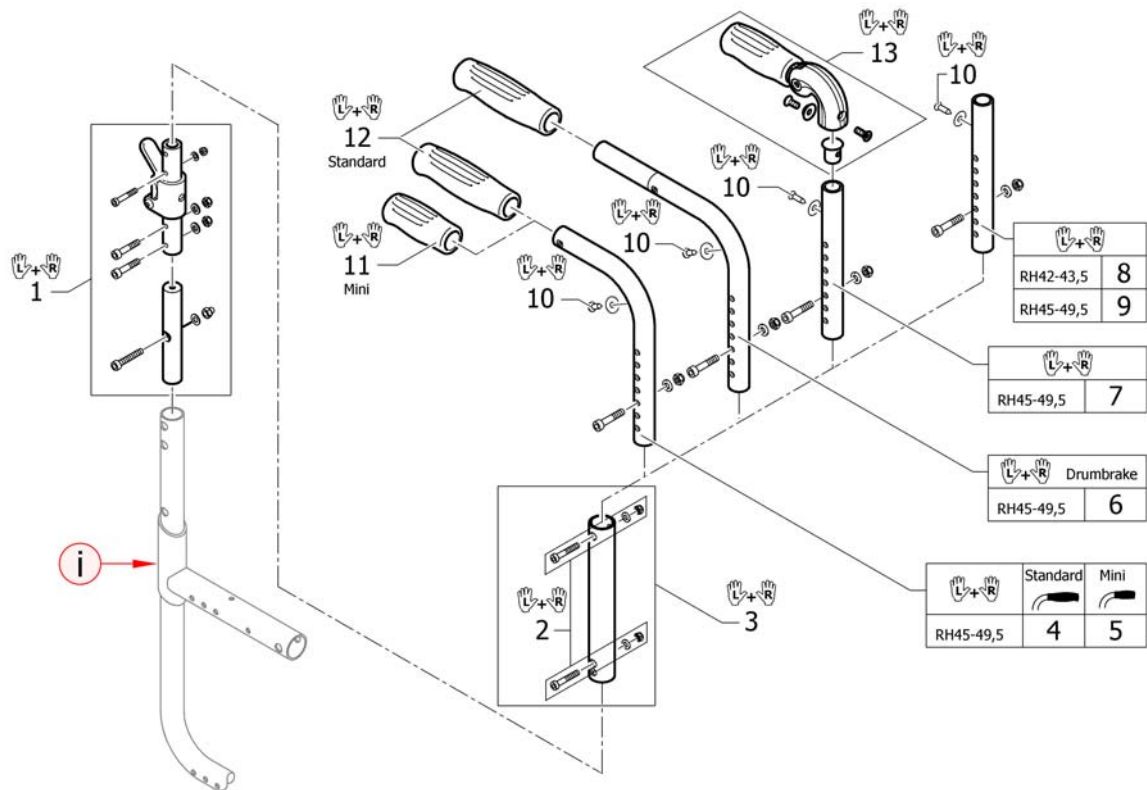


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	SP1606525	Hinge kit to backrest foldable, pair		1
2	1509134	Screw kit for telescopic tube		1
3	1438209	Tube kit medium to angle adjustable back		1
4	1507739	Push handles standard, XS, pair		1
5	1525342	Push handles mini, XS, pair		1
6	1507743	Push handles XS to drum brake, pair		1
7	1507735	Backrest tube straight XS to foldable push handles, pair		1
8	SP1606526	Backrest tube straight XXS for backrest foldable, pair		1
9	SP1606527	Backrest tube straight XS for backrest foldable, pair		1
10	1432083	Fixation kit for backrest band		1
11	1525126	Handle black, mini, pair		1
12	1507734	Handle black, pair		1
13	1507733	Push handles foldable, pair		1

## Push handles for backrest foldable and angle adjustable

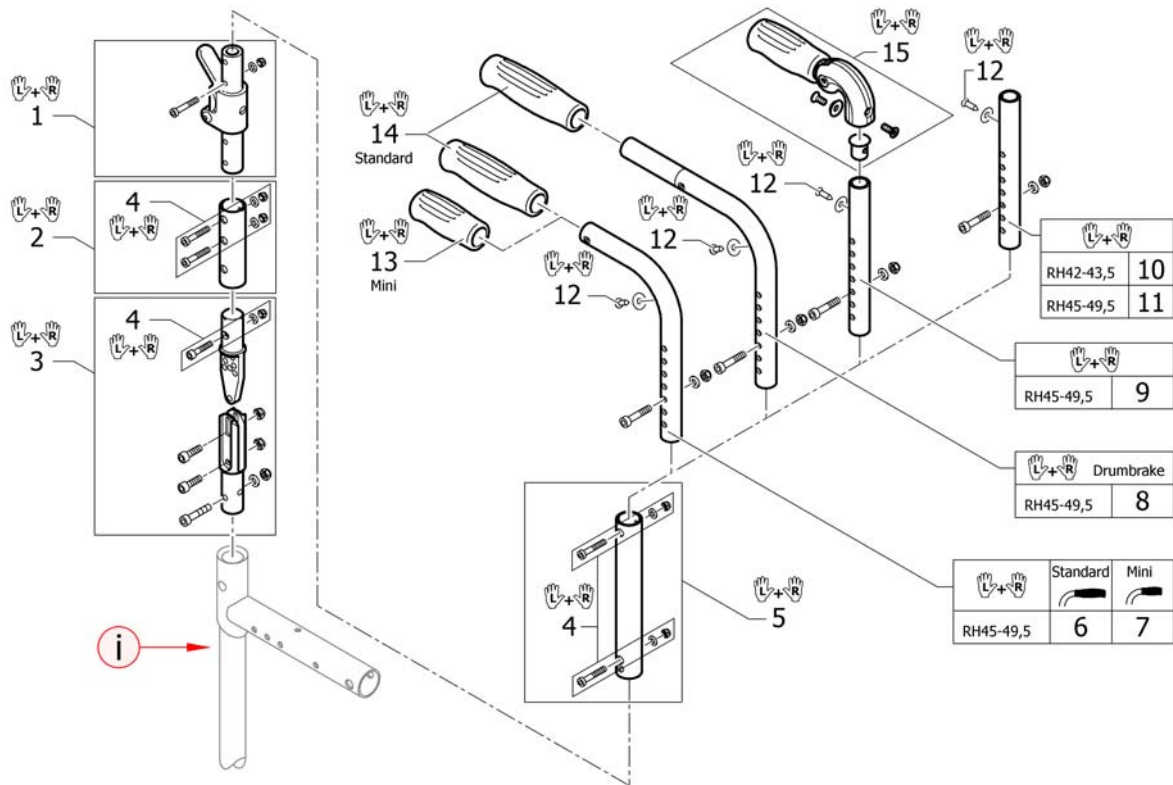


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1507749	Hinge kit to backrest foldable+angle adjustable, pair		1
2	1507748	Connection tube to backrest foldable+angle adjustable, pair		1
3	1540094	Backrest angle kit		1
4	1509135	Screw kit for connection tube		1
5	1438209	Tube kit medium to angle adjustable back		1
6	1507739	Push handles standard, XS, pair		1
7	1525342	Push handles mini, XS, pair		1
8	1507743	Push handles XS to drum brake, pair		1
9	1507735	Backrest tube straight XS to foldable push handles, pair		1
10	SP1606526	Backrest tube straight XXS for backrest foldable, pair		1
11	SP1606527	Backrest tube straight XS for backrest foldable, pair		1
12	1432083	Fixation kit for backrest band		1
13	1525126	Handle black, mini, pair		1
14	1507734	Handle black, pair		1
15	1507733	Push handles foldable, pair		1

## Push handles height adjustable

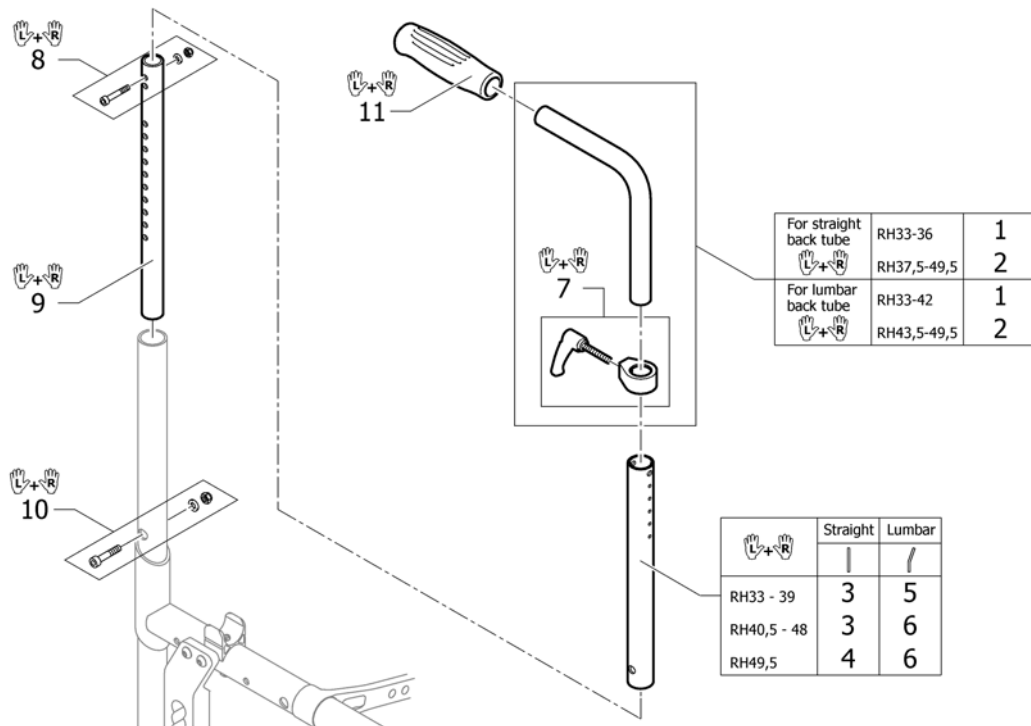


Pos.	Part number	Description	Valid from → until	Qty
1	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
2	1438204	Height adjustable, integrated push handles standard, kit long UL/C04		1
3	1507753	Backrest tube L.280mm straight to integrated push handles, pair		1
4	1509103	Backrest tube L.355mm straight to integrated push handles, pair		1
5	1509117	Backrest tube L.280mm lumbar to integrated push handles, pair		1
6	1509122	Backrest tube L.355mm lumbar to integrated push handles, pair		1
7	1549985	Adjusting lever for height adjustable, integrated push handles, pair		1
8	1540071	Screw kit for backrest tube		1
9	1507747	Telescopic tube frame-backrest, pair		1
10	1540088	Fixation kit to backrest tube		1
11	1507734	Handle black, pair		1

## Push handles height adjustable for backrest angle adjustable

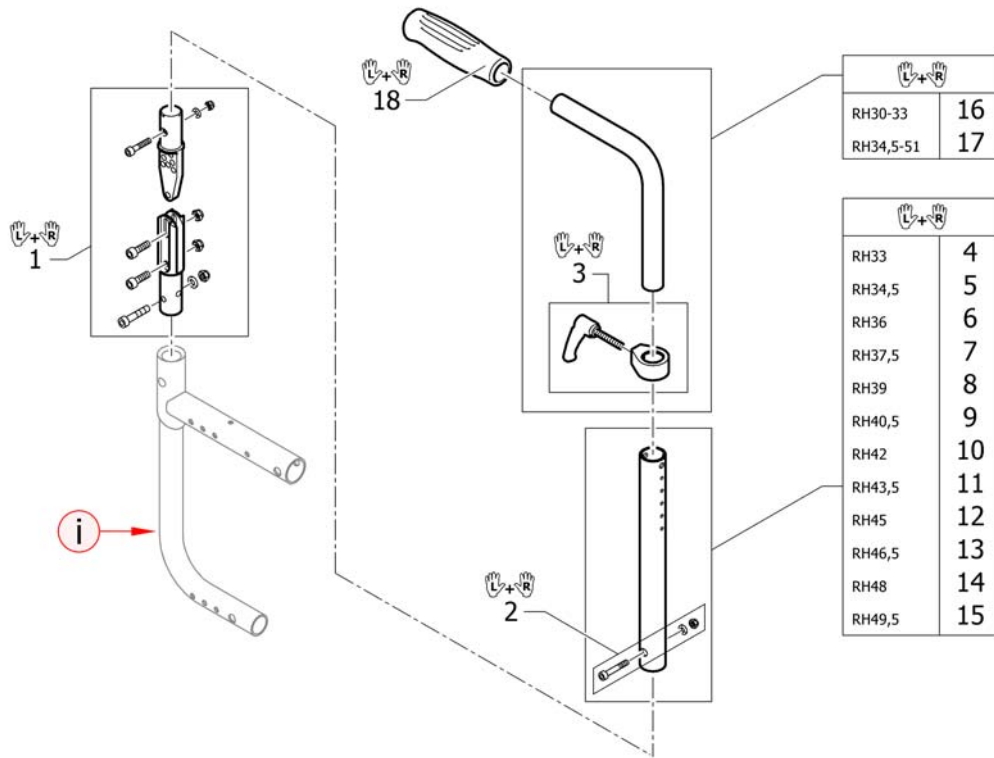

Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1540094	Backrest angle kit		1
2	1540071	Screw kit for backrest tube		1
3	1549985	Adjusting lever for height adjustable, integrated push handles, pair		1
4	1507753	Backrest tube L.280mm straight to integrated push handles, pair		1
5	1507862	Backrest tube L.295mm straight to integrated push handles, pair		1
6	1508438	Backrest tube L.310mm straight to integrated push handles, pair		1
7	1508466	Backrest tube L.325mm straight to integrated push handles, pair		1
8	1508498	Backrest tube L.340mm straight to integrated push handles, pair		1
9	1509103	Backrest tube L.355mm straight to integrated push handles, pair		1
10	1509104	Backrest tube L.370mm straight to integrated push handles, pair		1
11	1509105	Backrest tube L.385mm straight to integrated push handles, pair		1
12	1509106	Backrest tube L.400mm straight to integrated push handles, pair		1
13	1509107	Backrest tube L.415mm straight to integrated push handles, pair		1
14	1509108	Backrest tube L.430mm straight to integrated push handles, pair		1
15	1509109	Backrest tube L.445mm straight to integrated push handles, pair		1
16	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
17	1438204	Height adjustable, integrated push handles standard, kit long UL/C04		1
18	1507734	Handle black, pair		1


## Küschall backrest parts for Matrix backs

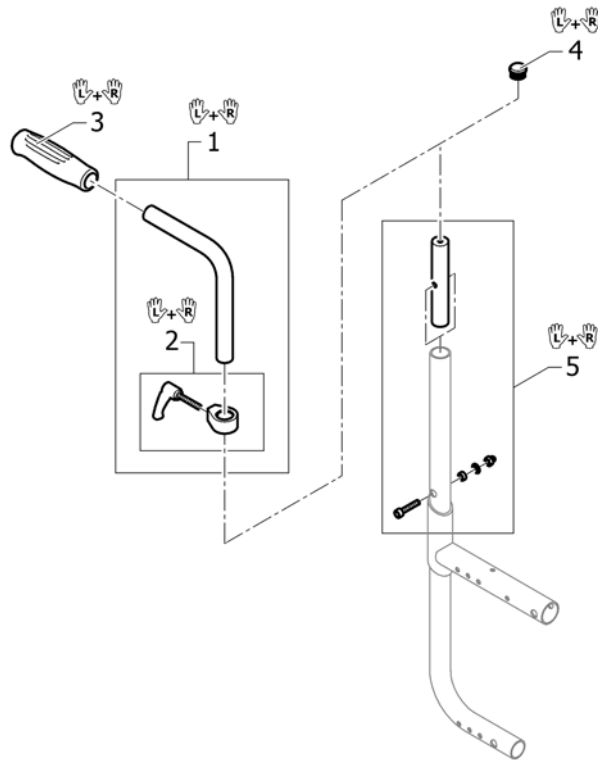

Pos.	Part number	Description	Valid from → until	Qty
1	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
2	1549985	Adjusting lever for height adjustable, integrated push handles, pair		1
3	1507734	Handle black, pair		1
4	1561018	Tube plug Ø24mm, pair		1
5	SP1602179	Backrest bolt with fixation, pair		1




## Push handles, inclined

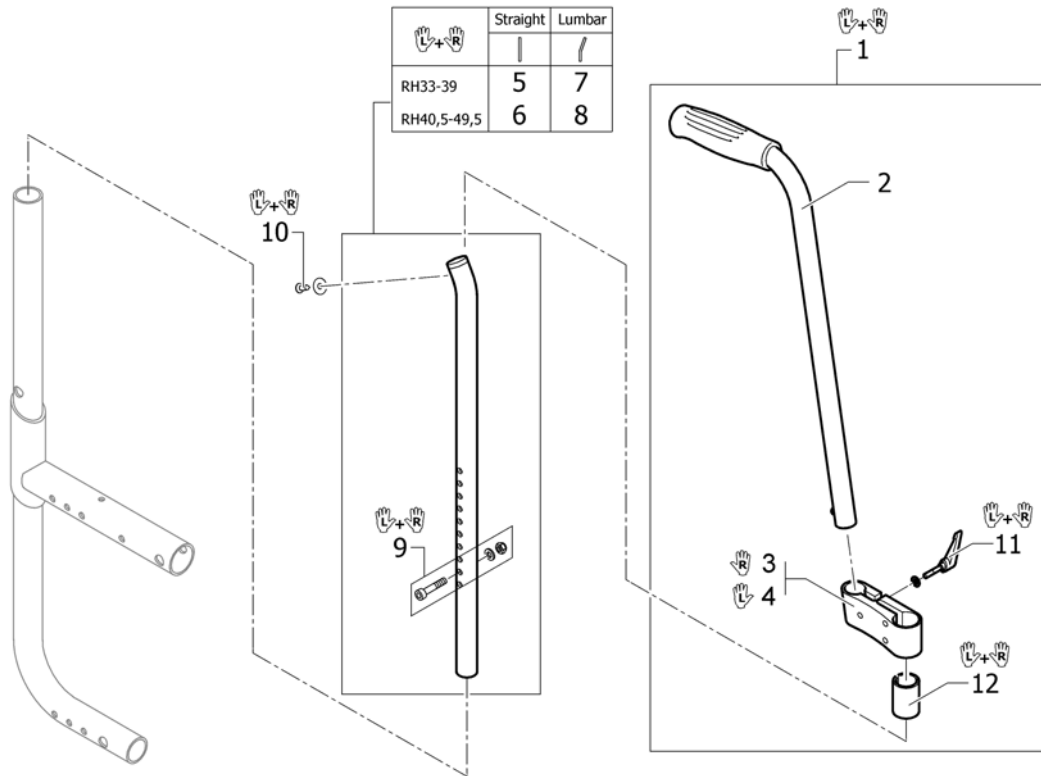


Pos.	Part number	Description	Valid from → until	Qty
1	1517876	Push handles, height adjustable, rear set, pair		1
2	1438938	Push handle tube complete, rearset		2
3	1438939	Mounting fixture right complete, for rearset push handles		1
4	1438940	Mounting fixture left complete, for rearset push handles		1
5	1540089	Backrest tubes L without push handles, pair		1
6	1540090	Backrest tubes XL without push handles, pair		1
7	1540091	Backrest tubes lumbar L without push handles, pair		1
8	1540092	Backrest tubes lumbar XL without push handles, pair		1
9	1540088	Fixation kit to backrest tube		1
10	1432083	Fixation kit for backrest band		1
11	1529799	Adjusting lever, pair		1
12	1581293	Sleeves for mounting fixture, for rearset push handles, pair		1

## Backrest stabilization bar

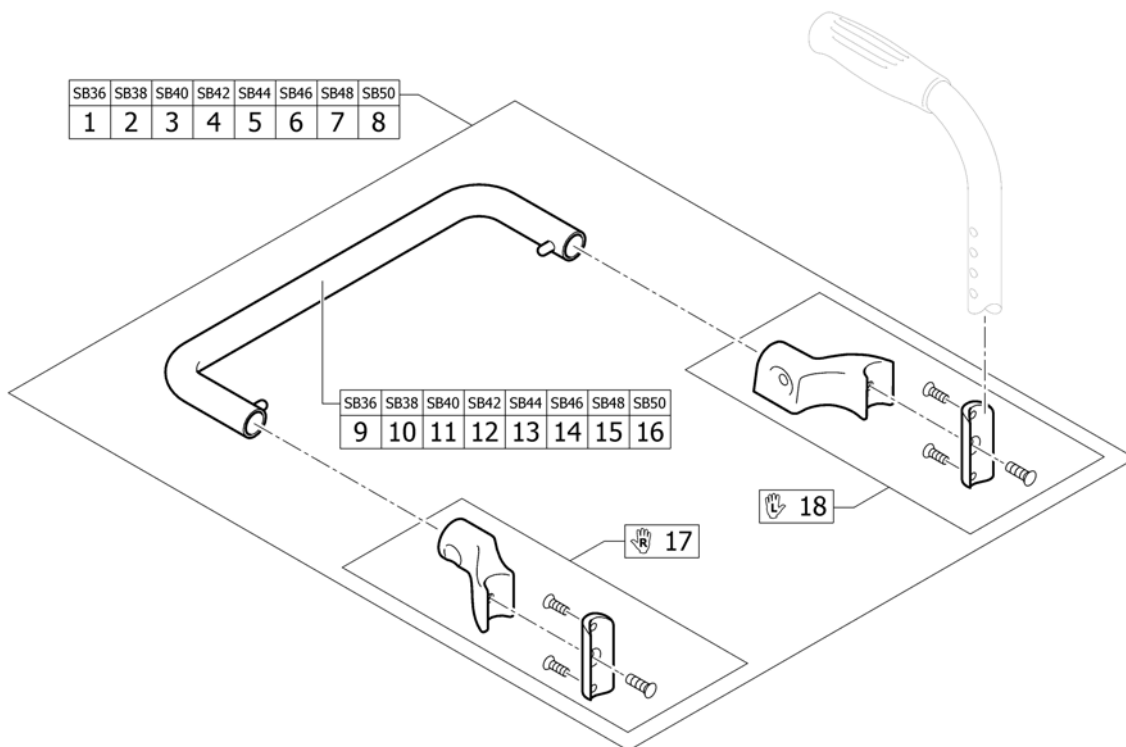


Pos.	Part number	Description	Valid from → until	Qty
1	1517980	Stabilizing bar complete SB36 for backrest tubes Ø20		1
2	1517981	Stabilizing bar complete SB38 for backrest tubes Ø20		1
3	1517982	Stabilizing bar complete SB40 for backrest tubes Ø20		1
4	1517983	Stabilizing bar complete SB42 for backrest tubes Ø20		1
5	1517984	Stabilizing bar complete SB44 for backrest tubes Ø20		1
6	1517985	Stabilizing bar complete SB46 for backrest tubes Ø20		1
7	1517986	Stabilizing bar complete SB48 for backrest tubes Ø20		1
8	1517987	Stabilizing bar complete SB50 for backrest tubes Ø20		1
9	1518000	Stabilizing bar SB36		1
10	1518001	Stabilizing bar SB38		1
11	1518002	Stabilizing bar SB40		1
12	1518003	Stabilizing bar SB42		1
13	1518004	Stabilizing bar SB44		1
14	1518005	Stabilizing bar SB46		1
15	1518006	Stabilizing bar SB48		1
16	1518007	Stabilizing bar SB50		1
17	1518009	Fixation kit to stabilizing bar, right, for backrest tubes Ø20		1
18	1518008	Fixation kit to stabilizing bar, left, for backrest tubes Ø20		1

## Back cover, standard

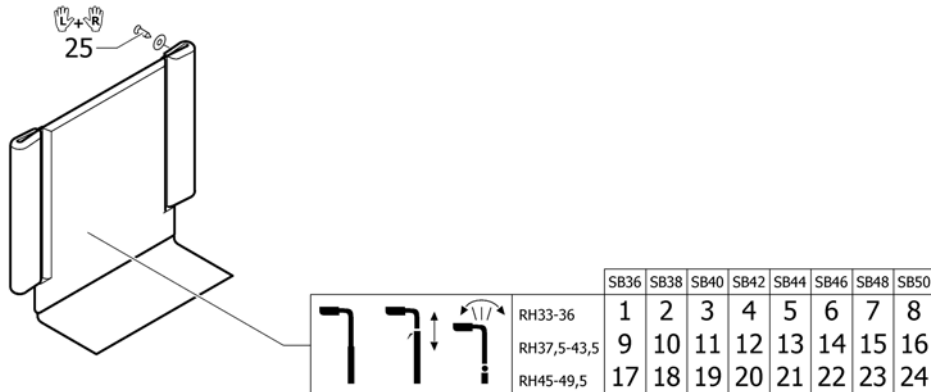


Pos.	Part number	Description	Valid from → until	Qty
1	1552084	Backrest cover SB36 medium		1
2	1552085	Backrest cover SB38 medium		1
3	1552086	Backrest cover SB40 medium		1
4	1552087	Backrest cover SB42 medium		1
5	1552088	Backrest cover SB44 medium		1
6	1552089	Backrest cover SB46 medium		1
7	1552090	Backrest cover SB48 medium		1
8	1552091	Backrest cover SB50 medium		1
9	1552096	Backrest cover SB36 long		1
10	1552097	Backrest cover SB38 long		1
11	1552098	Backrest cover SB40 long		1
12	1552099	Backrest cover SB42 long		1
13	1552100	Backrest cover SB44 long		1
14	1552101	Backrest cover SB46 long		1
15	1552102	Backrest cover SB48 long		1
16	1552103	Backrest cover SB50 long		1
17	1552108	Backrest cover SB36 X-long		1
18	1552109	Backrest cover SB38 X-long		1
19	1552110	Backrest cover SB40 X-long		1
20	1552111	Backrest cover SB42 X-long		1
21	1552112	Backrest cover SB44 X-long		1
22	1552113	Backrest cover SB46 X-long		1
23	1552114	Backrest cover SB48 X-long		1
24	1552115	Backrest cover SB50 X-long		1
25	1432083	Fixation kit for backrest band		1

## Bands to adjustable backrest

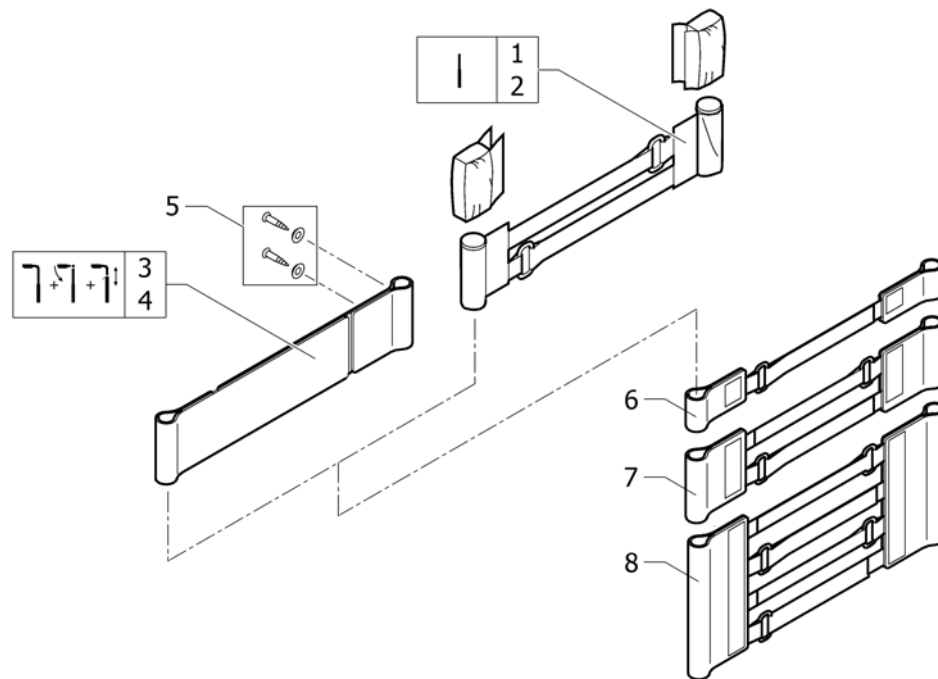


Pos.	Part number	Description	Valid from → until	Qty
1	1513172	Velcro end part, 5 cm		1
2	1513173	Velcro end part, 10cm		1
3	1513170	Backrest band 2 parts, width: 5cm		1
4	1513171	Backrest band 2 parts, width: 10cm		1
5	1432083	Fixation kit for backrest band		1
6	1513167	Velcro adjustment part with 1 strap, pair		1
7	1513168	Velcro adjustment strap with 2 straps, pair		1
8	1513169	Velcro adjustment strap with 4 straps, pair		1

## Cover to adjustable backrest

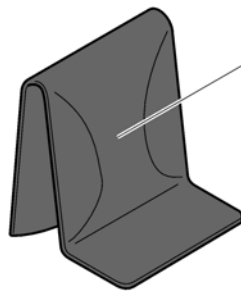


Std. (black)	SB36-38	SB40-42	SB44-46	SB48
RH33-36	1	2	3	4
RH37,5-42	5	6	7	8
RH43,5-49,5	9	10	11	12

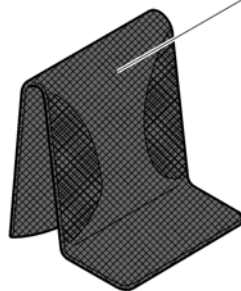


AeroBack	SB36-38	SB40-42	SB44-46	SB48
RH33-36	13	14	15	16
RH37,5-42	17	18	19	20
RH43,5-49,5	21	22	23	24

Pos.	Part number	Description	Valid from → until	Qty
1	1583109	Cover black to adjustable backrest, SB36-38, short		1
2	1583110	Cover black to adjustable backrest, SB40-42, short		1
3	1583111	Cover black to adjustable backrest, SB44-46, short		1
4	1583112	Cover black to adjustable backrest, SB48-50, short		1
5	1583113	Cover black to adjustable backrest, SB36-38, medium		1
6	1583114	Cover black to adjustable backrest, SB40-42, medium		1
7	1583115	Cover black to adjustable backrest, SB44-46, medium		1
8	1583116	Cover black to adjustable backrest, SB48-50, medium		1
9	1583117	Cover black to adjustable backrest, SB36-38, long		1
10	1583118	Cover black to adjustable backrest, SB40-42, long		1
11	1583119	Cover black to adjustable backrest, SB44-46, long		1
12	1583120	Cover black to adjustable backrest, SB48-50, long		1
13	1512157	Cover 'Aero' to adjustable backrest, SB36-38, short		1
14	1512158	Cover 'Aero' to adjustable backrest, SB40-42, short		1
15	1512159	Cover 'Aero' to adjustable backrest, SB44-46, short		1
16	1512160	Cover 'Aero' to adjustable backrest, SB48-50, short		1
17	1512162	Cover 'Aero' to adjustable backrest, SB36-38, medium		1
18	1512163	Cover 'Aero' to adjustable backrest, SB40-42, medium		1
19	1512164	Cover 'Aero' to adjustable backrest, SB44-46, medium		1
20	1512165	Cover 'Aero' to adjustable backrest, SB48-50, medium		1
21	1512167	Cover 'Aero' to adjustable backrest, SB36-38, long		1
22	1512168	Cover 'Aero' to adjustable backrest, SB40-42, long		1
23	1512169	Cover 'Aero' to adjustable backrest, SB44-46, long		1
24	1512170	Cover 'Aero' to adjustable backrest, SB48-50, long		1

# LEGRESTS



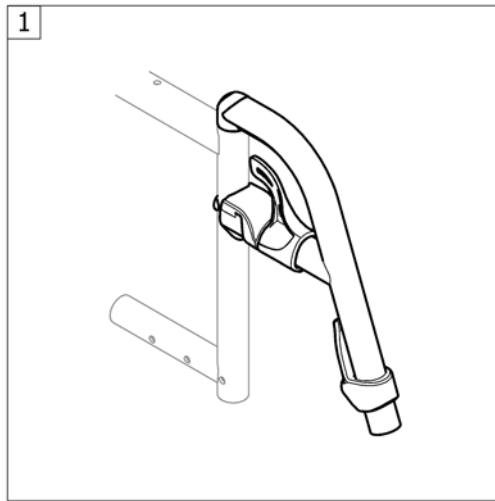


Pos.	Part number	Description	Valid from → until	Qty
1	-	Legrest hanger		1

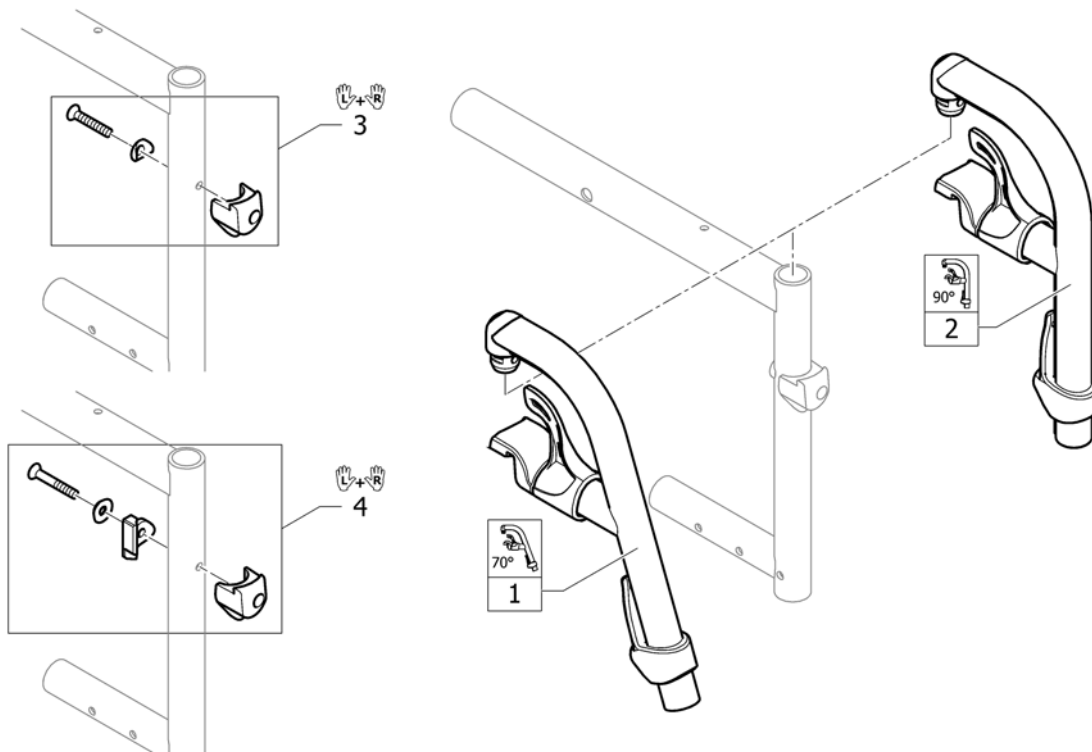**Legrest hanger**

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584369-0453	Swing Away 80°, attract blue, complete		2
1	SP1584369-0454	Swing Away 80°, attract black, complete		2
1	SP1584369-0455	Swing Away 80°, attract grey, complete		2
1	SP1584369-0456	Swing Away 80°, attract red, complete		2
2	SP1584370-0453	Swing Away 90°, attract blue, complete		2
2	SP1584370-0454	Swing Away 90°, attract black, complete		2
2	SP1584370-0455	Swing Away 90°, attract grey, complete		2
2	SP1584370-0456	Swing Away 90°, attract red, complete		2
3	SP1603392	Position mechanism for swing-away, kit, pair		1
4	SP1603777	Rotating limiter for swing-away, kit, pair		1


# FOOTREST

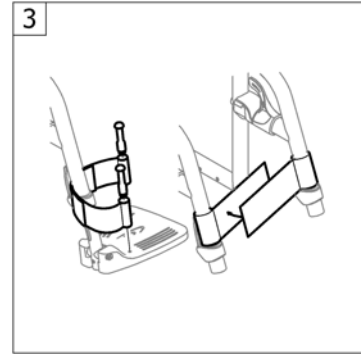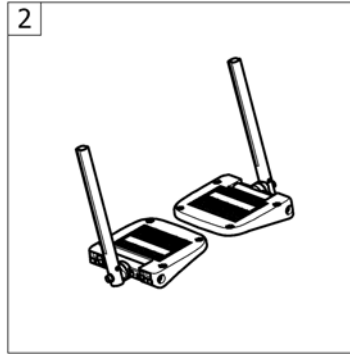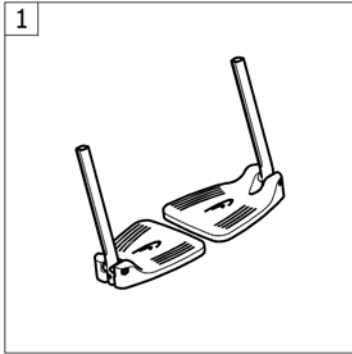

Pos.	Part number	Description	Valid from → until	Qty
1	-	Footrest 2-piece		1
2	-	Footrest 2-piece, angle adjustable		1
3	-	Footbands		1



## Footrest 2-piece

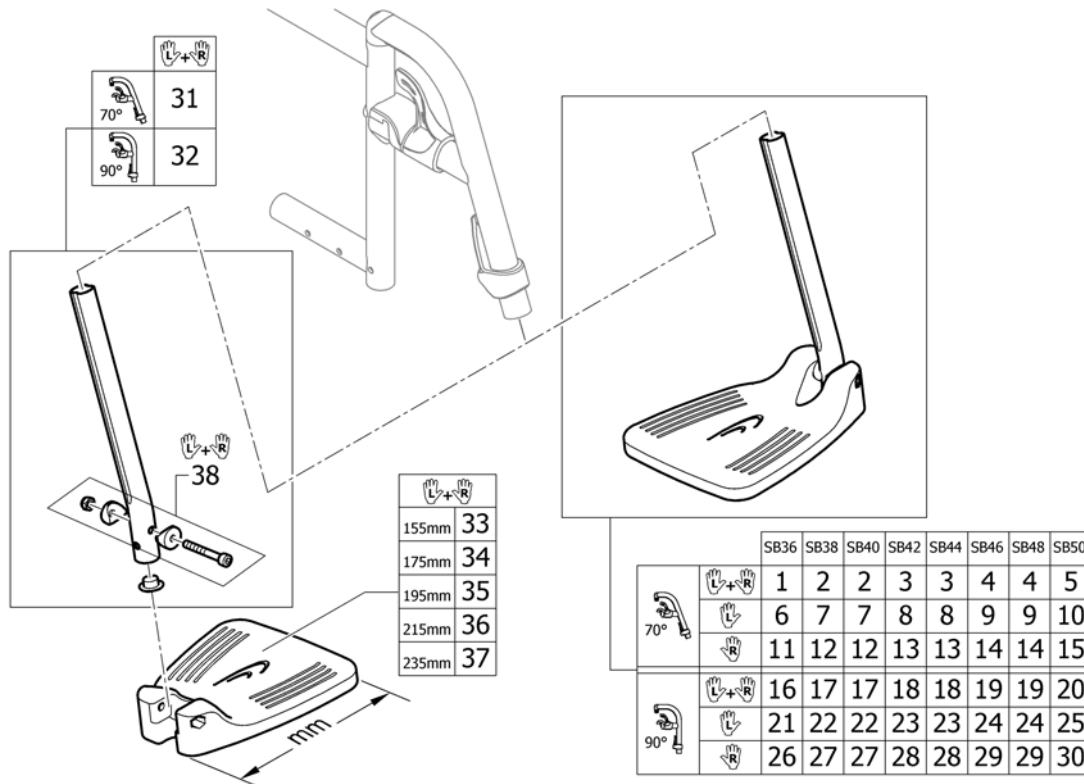


Pos.	Part number	Description	Valid from → until	Qty
1	SP1603867	Footrest 2-pc. 155mm, for frame 70°, pair, complete		1
2	SP1603868	Footrest 2-pc. 175mm, for frame 70°, pair, complete		1
3	SP1603869	Footrest 2-pc. 195mm, for frame 70°, pair, complete		1
4	SP1603884	Footrest 2-pc. 215mm, for frame 70°, pair, complete		1
5	SP1603885	Footrest 2-pc. 235mm, for frame 70°, pair, complete		1
6	SP1603886	Footrest 2-pc. 155mm, for frame 70°, left, complete		1
7	SP1603887	Footrest 2-pc. 175mm, for frame 70°, left, complete		1
8	SP1603888	Footrest 2-pc. 195mm, for frame 70°, left, complete		1
9	SP1603889	Footrest 2-pc. 215mm, for frame 70°, left, complete		1
10	SP1603890	Footrest 2-pc. 235mm, for frame 70°, left, complete		1
11	SP1603891	Footrest 2-pc. 155mm, for frame 70°, right, complete		1
12	SP1603892	Footrest 2-pc. 175mm, for frame 70°, right, complete		1
13	SP1603893	Footrest 2-pc. 195mm, for frame 70°, right, complete		1
14	SP1603894	Footrest 2-pc. 215mm, for frame 70°, right, complete		1
15	SP1603895	Footrest 2-pc. 235mm, for frame 70°, right, complete		1
16	SP1603908	Footrest 2-pc. 155mm, for frame 90°, pair, complete		1
17	SP1603909	Footrest 2-pc. 175mm, for frame 90°, pair, complete		1
18	SP1603910	Footrest 2-pc. 195mm, for frame 90°, pair, complete		1
19	SP1603911	Footrest 2-pc. 215mm, for frame 90°, pair, complete		1
20	SP1603912	Footrest 2-pc. 235mm, for frame 90°, pair, complete		1
21	SP1603913	Footrest 2-pc. 155mm, for frame 90°, left, complete		1
22	SP1603914	Footrest 2-pc. 175mm, for frame 90°, left, complete		1
23	SP1603915	Footrest 2-pc. 195mm, for frame 90°, left, complete		1
24	SP1603940	Footrest 2-pc. 215mm, for frame 90°, left, complete		1
25	SP1603941	Footrest 2-pc. 235mm, for frame 90°, left, complete		1
26	SP1603942	Footrest 2-pc. 155mm, for frame 90°, right, complete		1
27	SP1604002	Footrest 2-pc. 175mm, for frame 90°, right, complete		1
28	SP1604003	Footrest 2-pc. 195mm, for frame 90°, right, complete		1
29	SP1604004	Footrest 2-pc. 215mm, for frame 90°, right, complete		1
30	SP1604005	Footrest 2-pc. 235mm, for frame 90°, right, complete		1
31	SP1603907	Telescopic tube kit for footrest 2-pc. for frame 70°, pair		1

**Footrest 2-piece**


<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
32	SP1604006	Telescopic tube kit for footrest 2-pc. for frame 90°, pair		1
33	1579331	Footrest 2-pc. 155mm, pair		1
34	1579332	Footrest 2-pc. 175mm, pair		1
35	1579333	Footrest 2-pc. 195mm, pair		1
36	1579334	Footrest 2-pc. 215mm, pair		1
37	1579335	Footrest 2-pc. 235mm, pair		1
38	1582352	Fixation kit for telescopic tube on footrest 2-pcs., pair		1

## Footrest 2-piece, angle adjustable

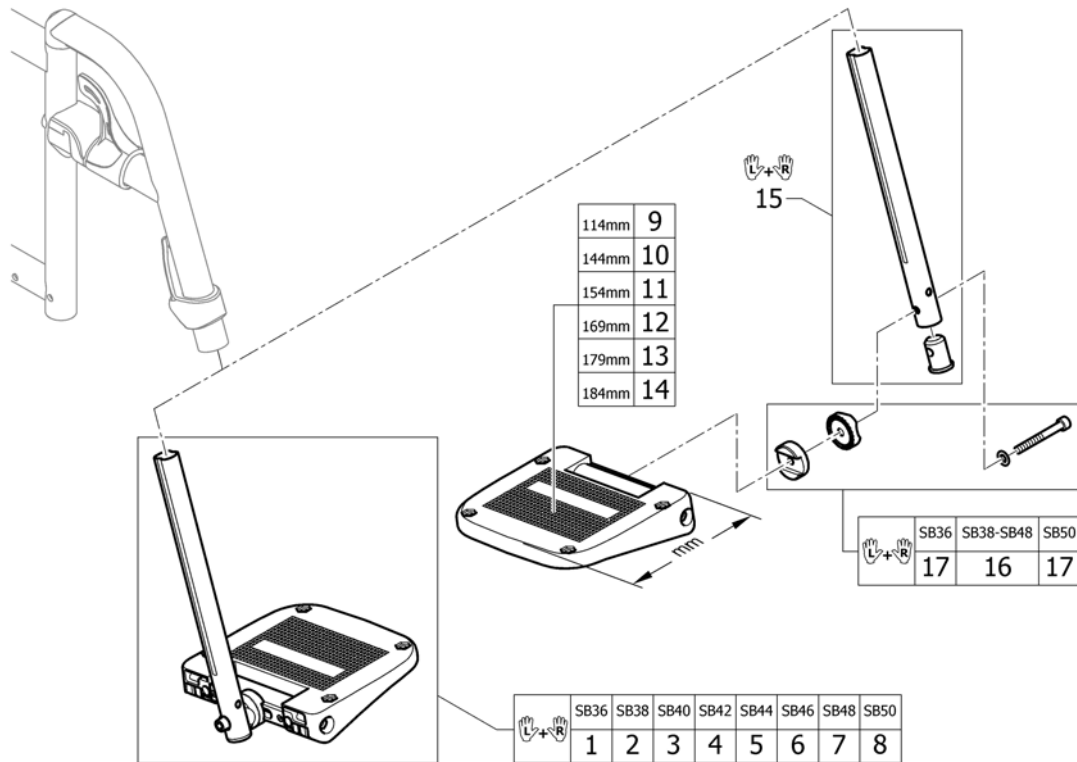


Pos.	Part number	Description	Valid from → until	Qty
1	SP1603789	Footrest 2-pc. angle adjustable SB36, pair, complete		1
2	SP1603790	Footrest 2-pc. angle adjustable SB38, pair, complete		1
3	SP1603791	Footrest 2-pc. angle adjustable SB40, pair, complete		1
4	SP1603792	Footrest 2-pc. angle adjustable SB42, pair, complete		1
5	SP1603793	Footrest 2-pc. angle adjustable SB44, pair, complete		1
6	SP1603835	Footrest 2-pc. angle adjustable SB46, pair, complete		1
7	SP1603836	Footrest 2-pc. angle adjustable SB48, pair, complete		1
8	SP1603837	Footrest 2-pc. angle adjustable SB50, pair, complete		1
9	SP1603838	Footrest 2-pc. angle adjustable 114mm		2
10	SP1603839	Footrest 2-pc. angle adjustable 144mm		2
11	SP1603841	Footrest 2-pc. angle adjustable 154mm		2
12	SP1603842	Footrest 2-pc. angle adjustable 169mm		2
13	SP1603843	Footrest 2-pc. angle adjustable 179mm		2
14	SP1603844	Footrest 2-pc. angle adjustable 184mm		2
15	SP1603846	Telescopic tube kit for footrest 2-pc. angle adjustable, pair		1
16	SP1603865	Lateral fixation, short, pair		1
17	SP1603866	Lateral fixation, long, pair		1

## Footbands

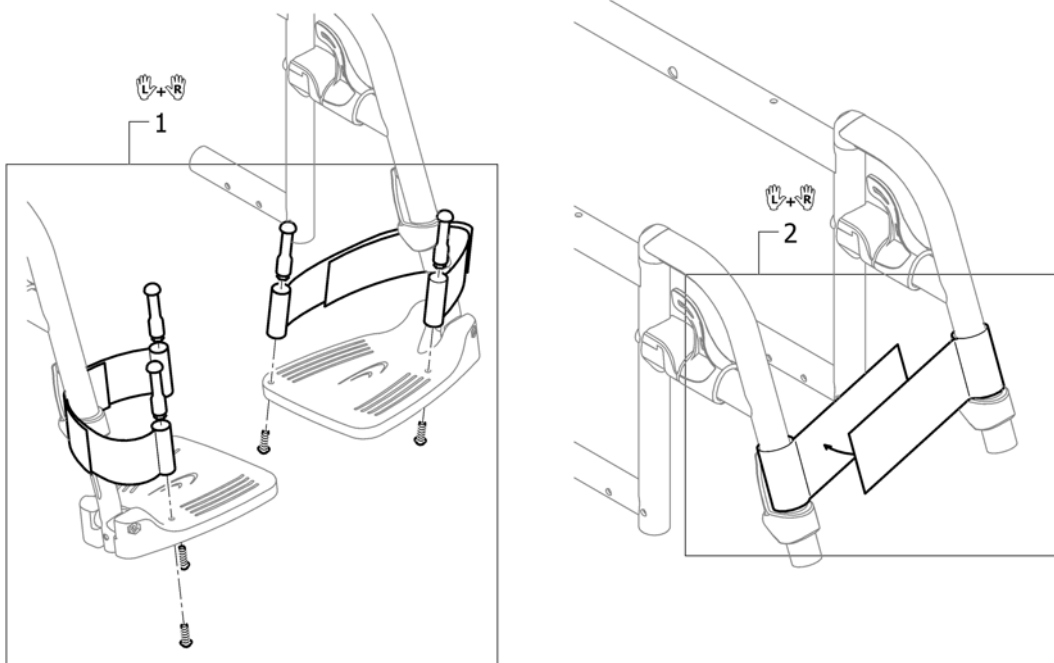


Pos.	Part number	Description	Valid from → until	Qty
1	1437834	Heel loop kit short		1
2	1536082	Calf strap 2-pcs complete		1

# SIDERESTS

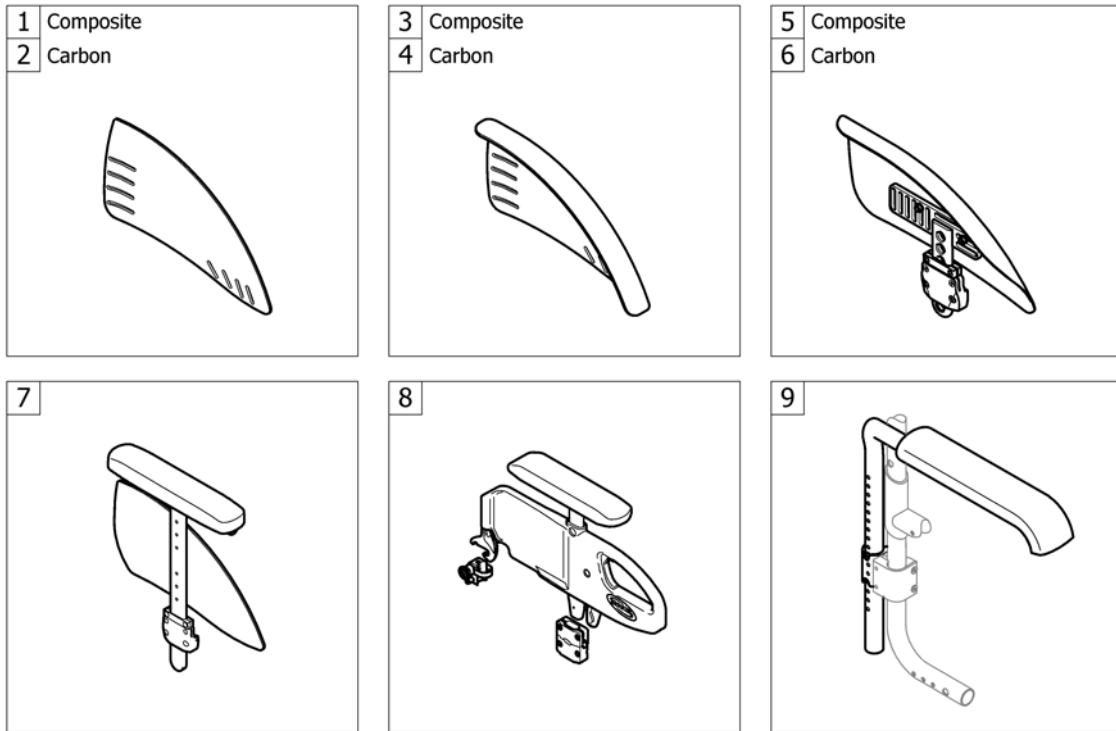


Pos.	Part number	Description	Valid from → until	Qty
1	-	Clothes-guards composite		1
2	-	Clothes-guards carbon		1
3	-	Mudguards composite		1
4	-	Mudguards carbon		1
5	-	Mudguards composite removable		1
6	-	Mudguards carbon removable		1
7	-	Siderest height adjustable		1
8	-	Siderest with desk function		1
9	-	Tubular armrest	2016-07-01 →	1

## Clothes-guards composite

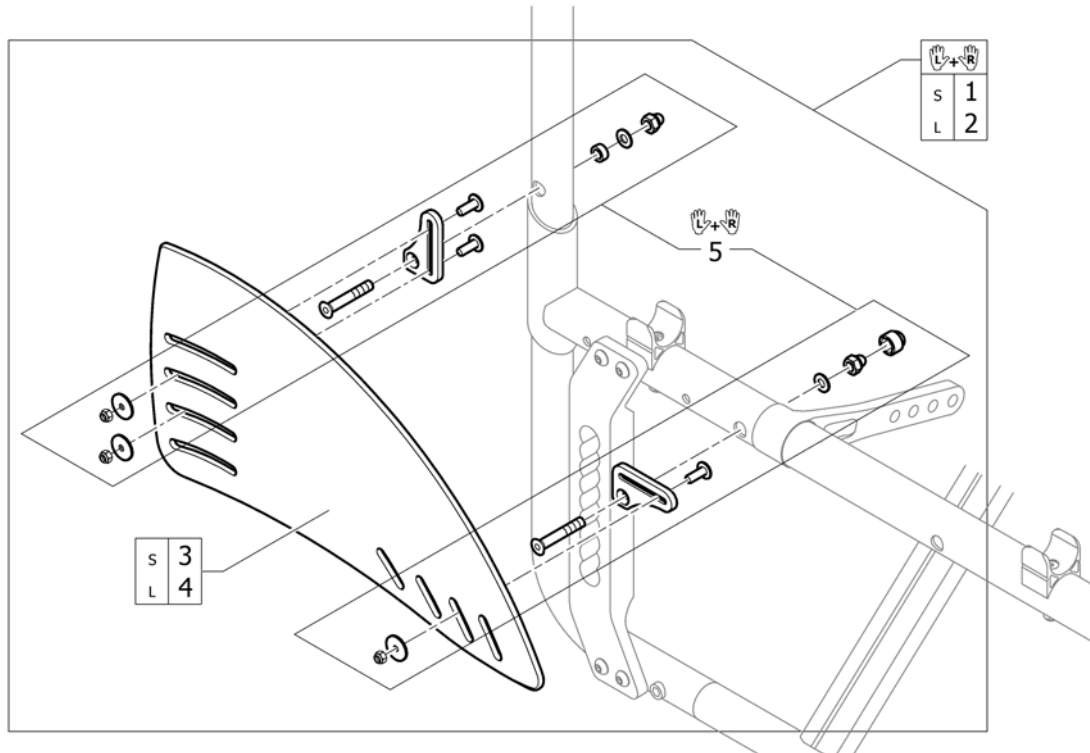


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604011	Clothes-guards composite S with fixation kit, pair		1
2	SP1604012	Clothes-guards composite L with fixation kit, pair		1
3	SP1604013	Clothes-guard composite S		2
4	SP1604029	Clothes-guard composite L		2
5	SP1604771	Fixation kit for clothes-guard and mudguard composite		1

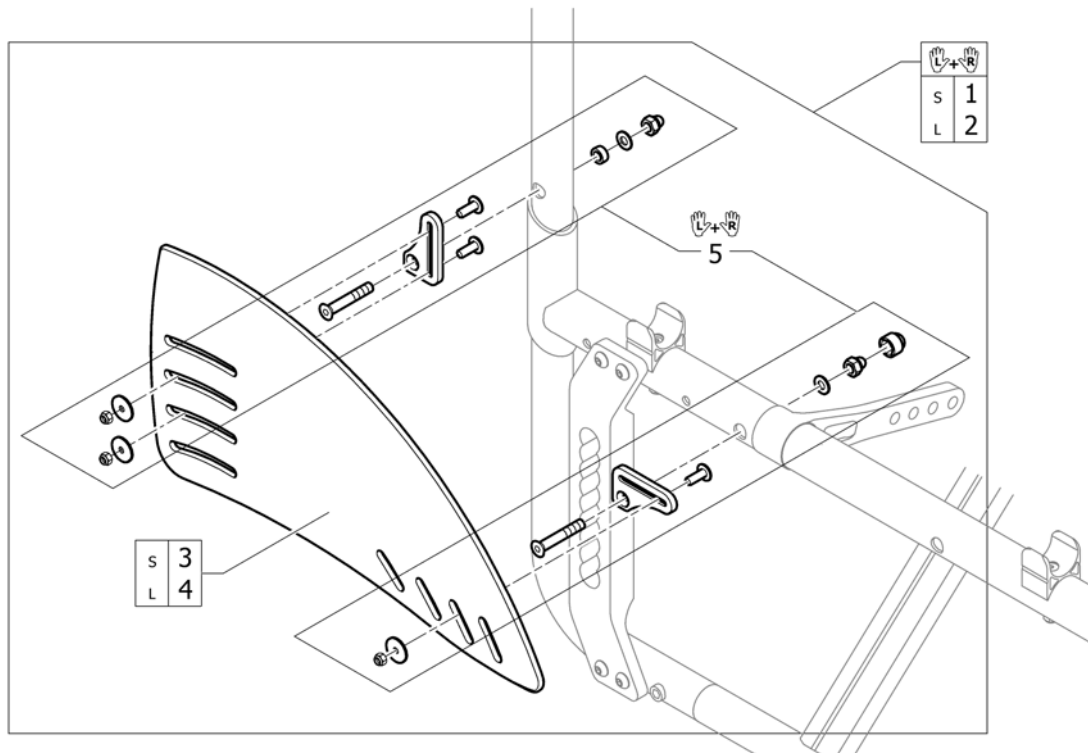**Clothes-guards carbon**


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604880	Clothes-guards carbon S with fixation kit, pair		1
2	SP1604945	Clothes-guards carbon L with fixation kit, pair		1
3	1540003	Clothes-guard S		2
4	1540004	Clothes-guard L		2
5	SP1604795	Fixation kit for clothes-guard and mudguard carbon		1

# Mudguards composite

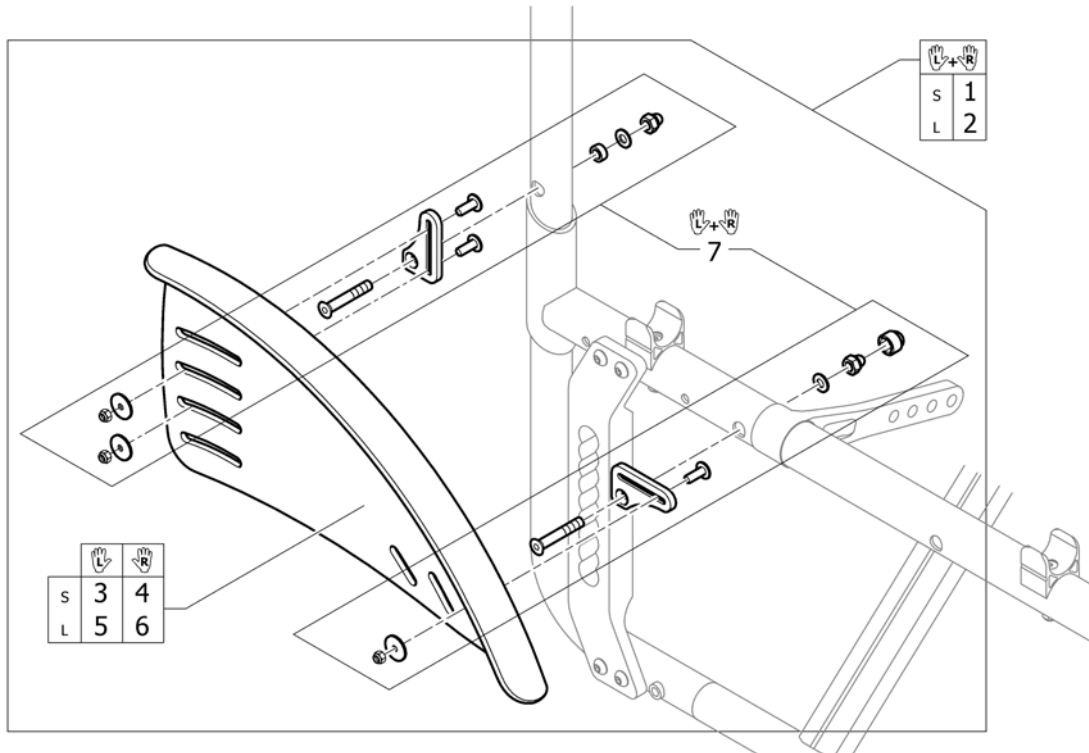


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604772	Mudguard composite S with fixation kit, pair		1
2	SP1604773	Mudguard composite L with fixation kit, pair		1
3	SP1604774	Mudguard composite S left		1
4	SP1604775	Mudguard composite S right		1
5	SP1604776	Mudguard composite L left		1
6	SP1604777	Mudguard composite L right		1
7	SP1604771	Fixation kit for clothes-guard and mudguard composite		1

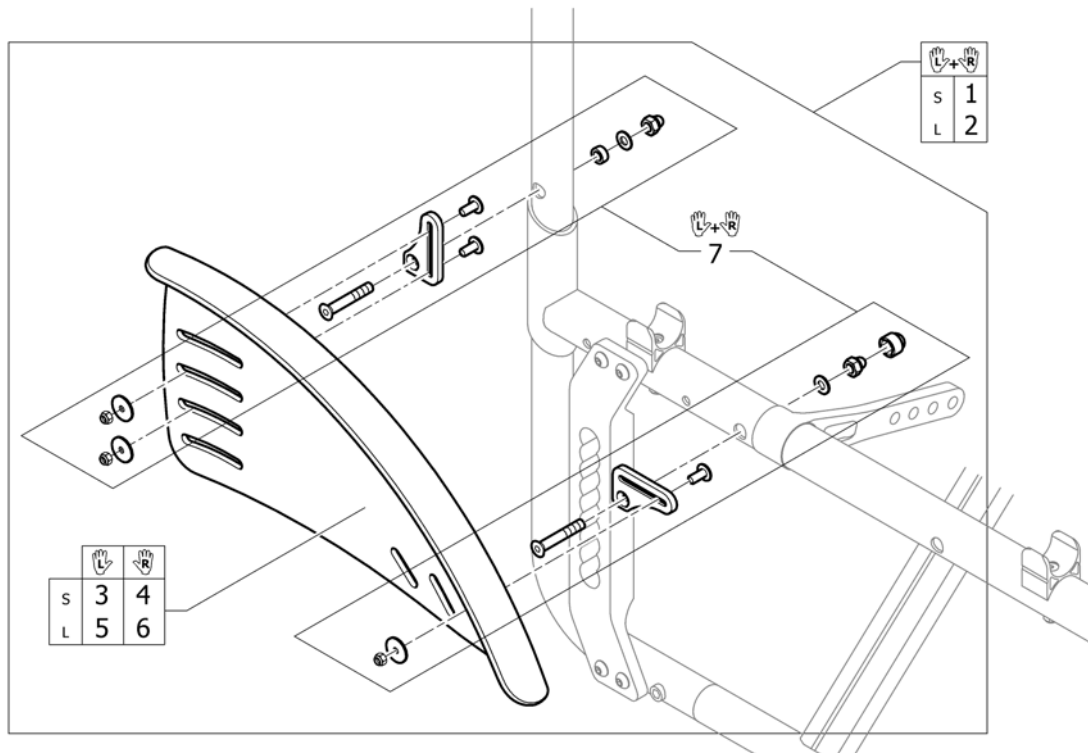

**Mudguards carbon**


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604793	Mudguard carbon S with fixation kit, pair		1
2	SP1604794	Mudguard carbon L with fixation kit, pair		1
3	1529781	Mudguard Carbon S short left		1
4	1529782	Mudguard Carbon S short right		1
5	1529783	Mudguard Carbon L long left		1
6	1529784	Mudguard Carbon L long right		1
7	SP1604795	Fixation kit for clothes-guard and mudguard carbon		1

# Mudguards composite removable

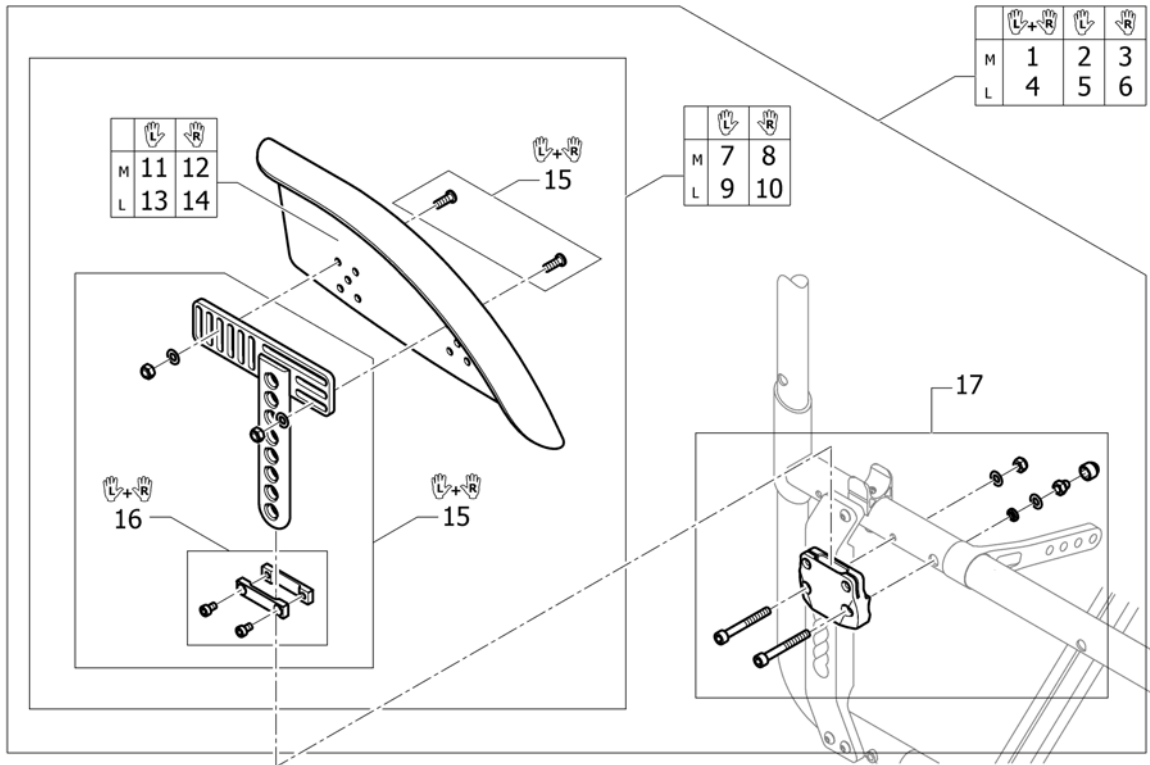


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604952	Mudguard composite M long complete, pair		1
2	SP1604953	Mudguard composite left M long complete		1
3	SP1604954	Mudguard composite right M long complete		1
4	SP1604955	Mudguard composite L long complete, pair		1
5	SP1604956	Mudguard composite left L long complete		1
6	SP1604957	Mudguard composite right L long complete		1
7	SP1604969	Mudguard composite left M, holder long		1
8	SP1604970	Mudguard composite right M, holder long		1
9	SP1604971	Mudguard composite left L, holder long		1
10	SP1604972	Mudguard composite right L, holder long		1
11	SP1604959	Mudguard composite M left		1
12	SP1604960	Mudguard composite M right		1
13	SP1604961	Mudguard composite L left		1
14	SP1604962	Mudguard composite L right		1
15	SP1604963	Mudguard removable holder long, pair		1
16	1547537	Siderests height clamp kit, pair		1
17	SP1604958	Sidepart Adaption complete		2

## Mudguards carbon removable

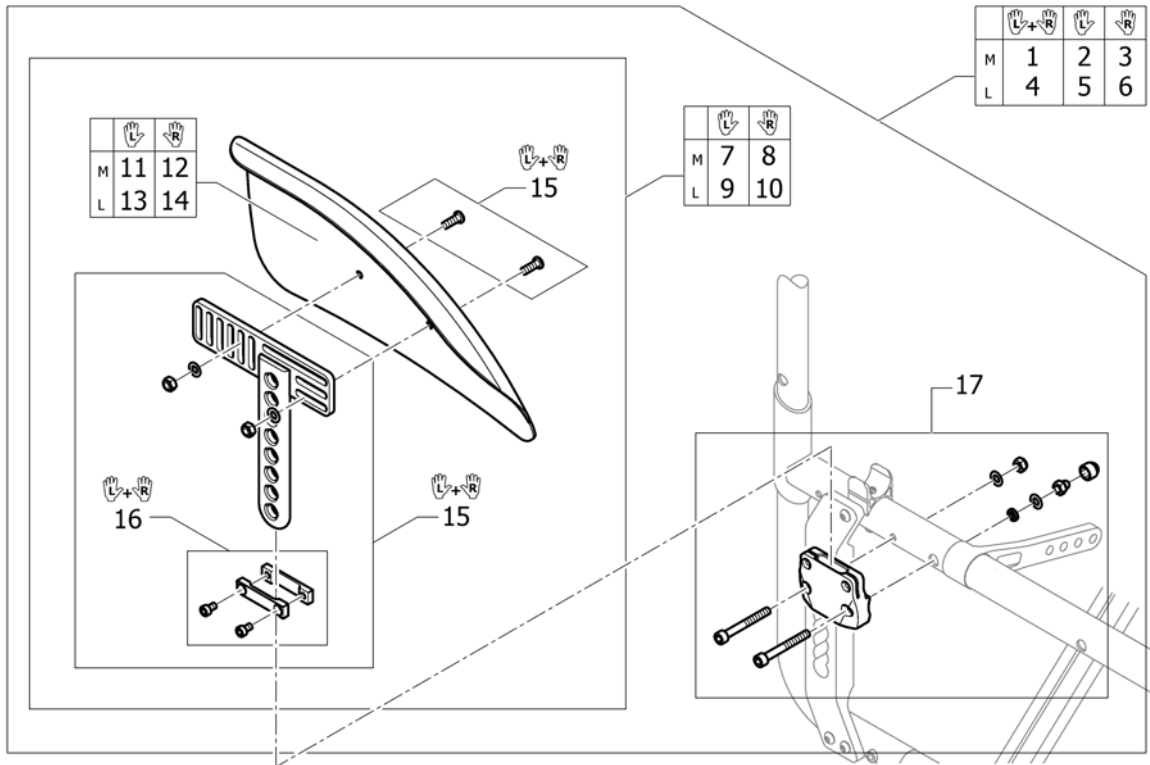


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604946	Mudguard carbon M long complete, pair		1
2	SP1604947	Mudguard carbon left M long complete		1
3	SP1604948	Mudguard carbon right M long complete		1
4	SP1604949	Mudguard carbon L long complete, pair		1
5	SP1604950	Mudguard carbon left L long complete		1
6	SP1604951	Mudguard carbon right L long complete		1
7	SP1604965	Mudguard carbon left M, holder long		1
8	SP1604966	Mudguard carbon right M, holder long		1
9	SP1604967	Mudguard carbon left L, holder long		1
10	SP1604968	Mudguard carbon right L, holder long		1
11	1510275	Mudguard Carbon left M		1
12	1510276	Mudguard Carbon right M		1
13	1510277	Mudguard Carbon left L		1
14	1510278	Mudguard Carbon right L		1
15	SP1604963	Mudguard removable holder long, pair		1
16	1547537	Siderests height clamp kit, pair		1
17	SP1604958	Sidepart Adaption complete		2

## Siderest height adjustable

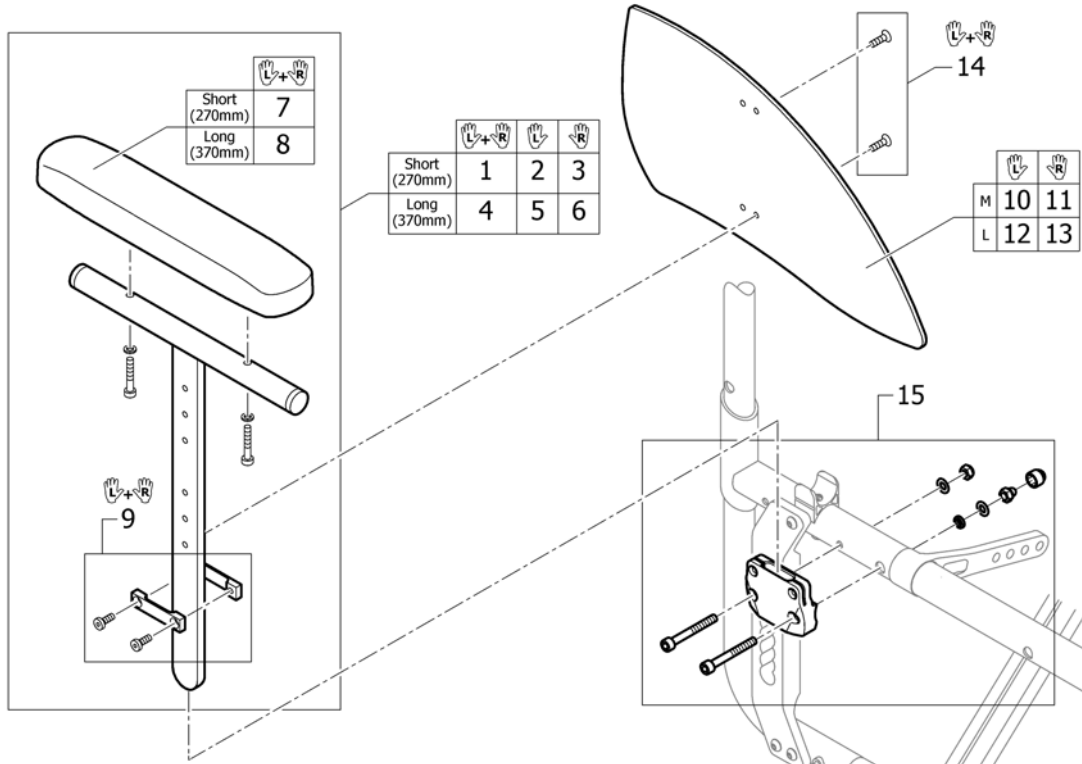


Pos.	Part number	Description	Valid from → until	Qty
1	1517682	Siderest height adjustable short, pair		1
2	1517678	Siderest height adjustable short, left		1
3	1517677	Siderest height adjustable short, right		1
4	1517683	Siderest height adjustable long, pair		1
5	1517680	Siderest height adjustable long, left		1
6	1517679	Siderest height adjustable long, right		1
7	1439497	Ampad S (270mm) incl. screws, pair		1
8	1439498	Ampad L (370mm) incl. screws, pair		1
9	1547537	Siderests height clamp kit, pair		1
10	1517689	Clothes-guard small left to siderest		1
11	1517690	Clothes-guard small right to siderest		1
12	1517691	Clothes-guard large left to siderest		1
13	1517692	Clothes-guard large right to siderest		1
14	1517693	Clothes-guard fixation kit		1
15	SP1604958	Sidepart Adaption complete		2

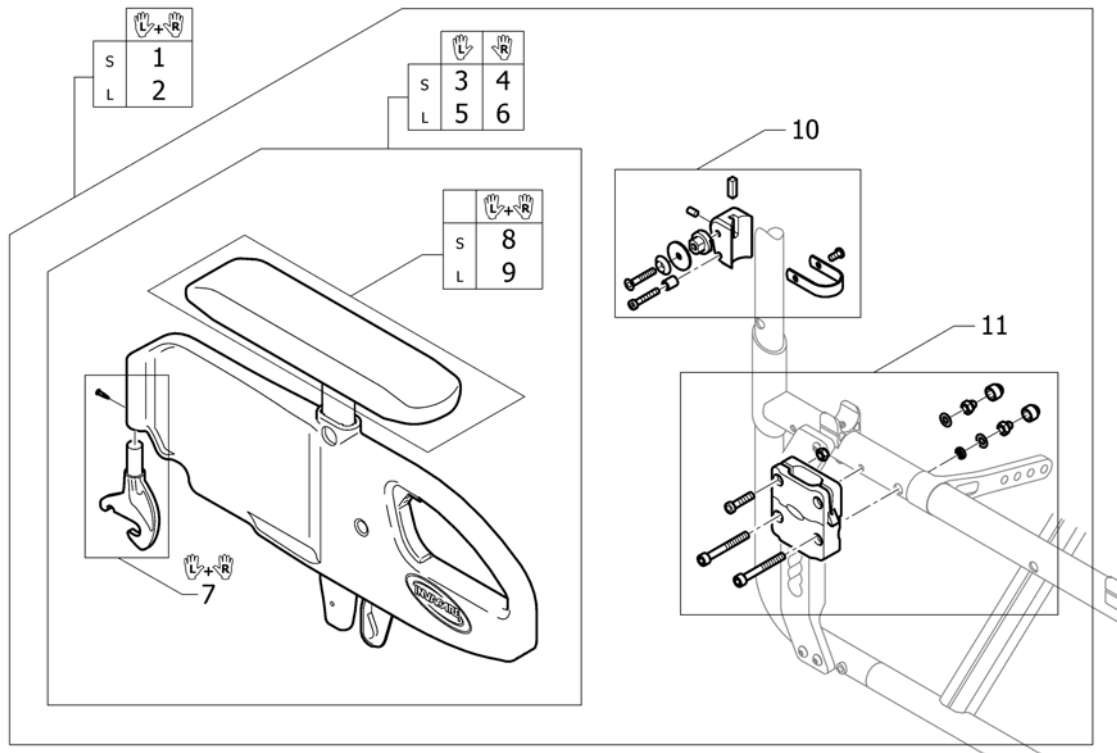



Pos.	Part number	Description	Valid from → until	Qty
1	1540043	Siderest with desk function, flip-back, height adj., removable, pair short, complete		1
2	1540044	Siderest with desk function, flip-back, height adj., removable, pair long, complete		1
3	1540040	Siderest with desk function, flip-back, height adj., removable, short, left		1
4	1540039	Siderest with desk function, flip-back, height adj., removable, short, right		1
5	1540042	Siderest with desk function, flip-back, height adj., removable, long left		1
6	1540041	Siderest with desk function, flip-back, height adj., removable, long, right		1
7	1540031	Hook bracket for siderest, pair		1
8	1521532	Armpad short incl. Screws, pair		1
9	1521533	Armpad long incl. Screws, pair		1
10	1540032	Siderest with desk function fixation kit		2
11	SP1604973	Fixation kit front for siderest		2

## Tubular armrest

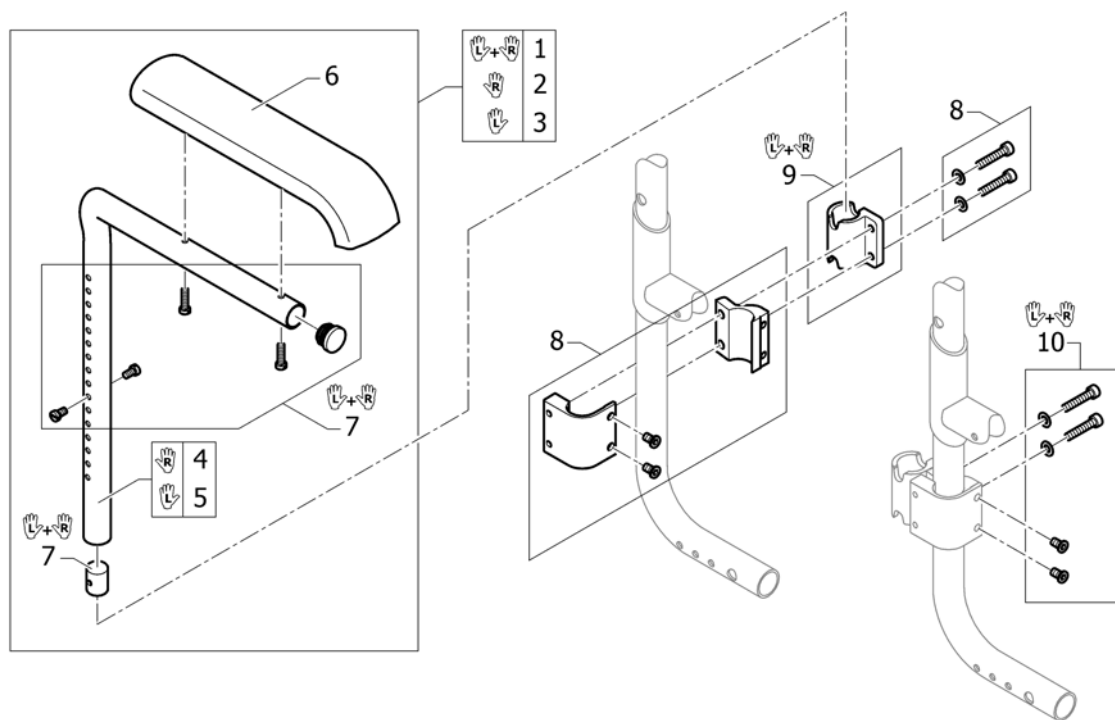


Pos.	Part number	Description	Valid from → until	Qty
1	SP1606428	Pair of armrests with standard pads, bent		1
2	SP1606429	Armrest with standard pad, bent, right		1
3	SP1606430	Armrest with standard pad, bent, left		1
4	SP1606431	Armrest right		1
5	SP1606432	Armrest left		1
6	SP1589276	Standard pad		2
7	SP1606433	Screws kit for tubular armrest, pair		1
8	SP1604985	Clamping piece to adaption		2
9	SP1606434	Armrest holder, pair		1
10	SP1606435	Fixation kit for clamping piece to adaption, pair		1

# FRONT WHEELS

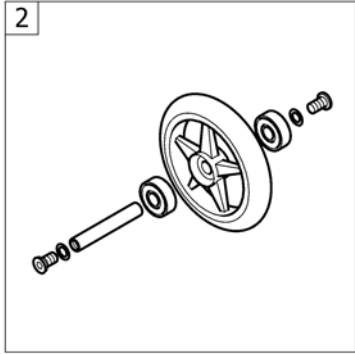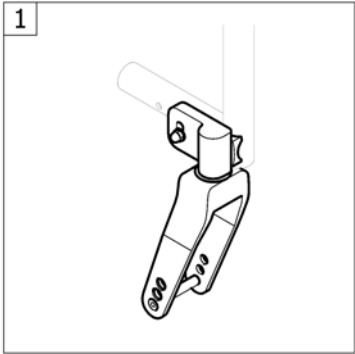

Pos.	Part number	Description	Valid from → until	Qty
1	-	Front wheel supporter		1
2	-	Front wheels		1


# Front wheel supporter

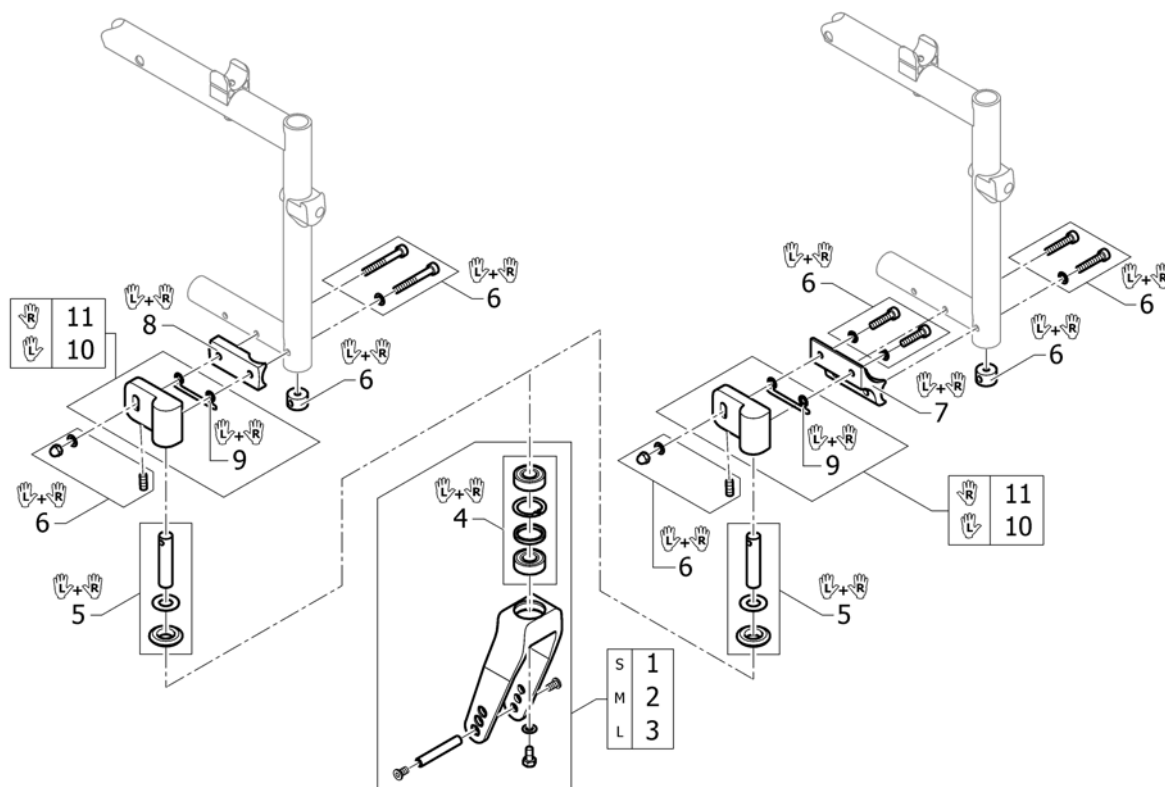

Pos.	Part number	Description	Valid from → until	Qty
1	SP1603779-0004	Front fork S, black anodized		2
2	SP1603780-0004	Front fork M, black anodized		2
3	SP1603781-0004	Front fork L, black anodized		2
4	1510219	Bearing kit		1
5	SP1603786	Front fork pin kit, pair		1
6	1532034	Fixation kit for front fork castor Compact		1
7	1532035-0004	Fixation parts for front fork castor high or low mounted, black anodized, Compact, pair		1
8	1532037	Fixation parts Standard for front fork castor, Compact, pair		1
9	1563141	Front fork castor adjusting parts, pair		1
10	SP1603787-0004	Front fork castor left, black anodized		1
11	SP1603788-0004	Front fork castor right, black anodized		1




## Front wheels

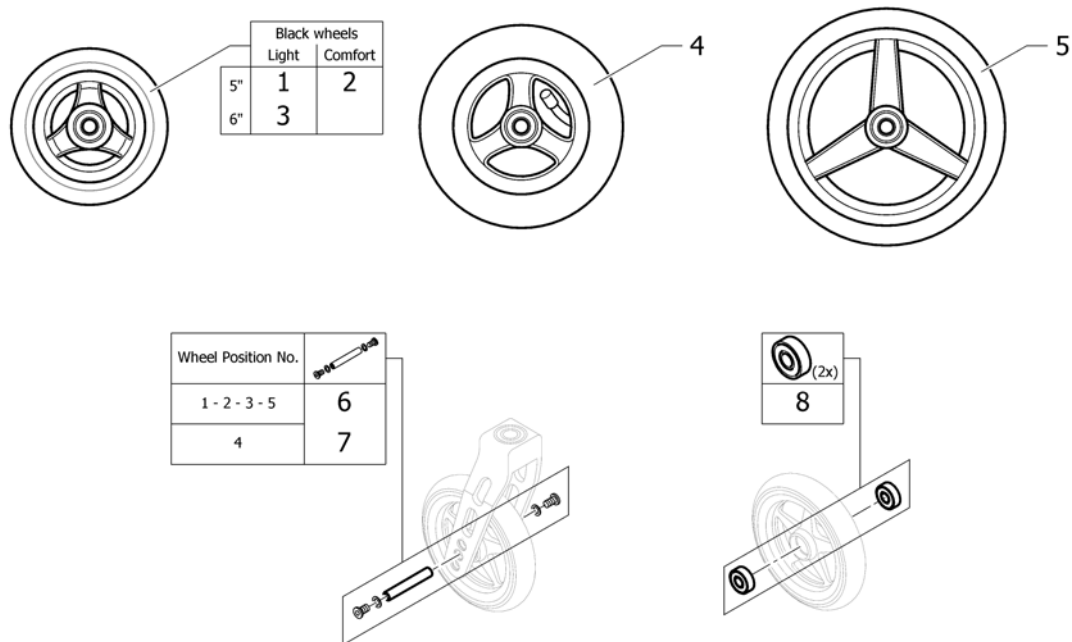


Pos.	Part number	Description	Valid from → until	Qty
1	1562627	5" Everyday wheel, 3 spokes, black		2
2	1542640	5" Comfort wheel, black		2
3	1562628	6" Everyday wheel, 3 spokes, black		2
4	1513189	6" Pneumatic wheel		2
5	1513192	7" Sports wheel		2
6	1570108	Axle set without spacer		2
7	1570109	Axle set complete 6" pneumatic wheel		2
8	1570105	Ball bearing 6000-2RS, pair		2

## REAR WHEELS

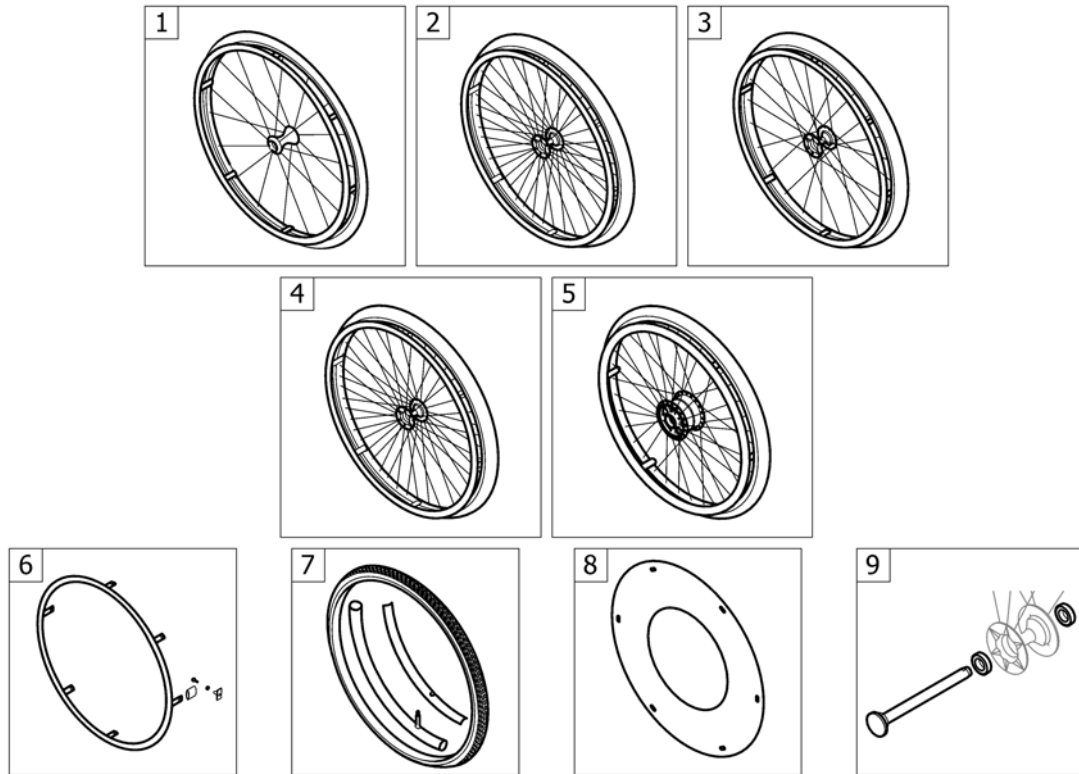


Pos.	Part number	Description	Valid from → until	Qty
1	-	Active wheel		1
2	-	24" Küschall rear wheels complete		1
3	-	High Performance wheel		1
4	-	Spinerger LXX wheel		1
5	-	Drumbrake wheel		1
6	-	Handrim		1
7	-	Tyre		1
8	-	Spoke guard		1
9	-	Quick release axle and bearing		1

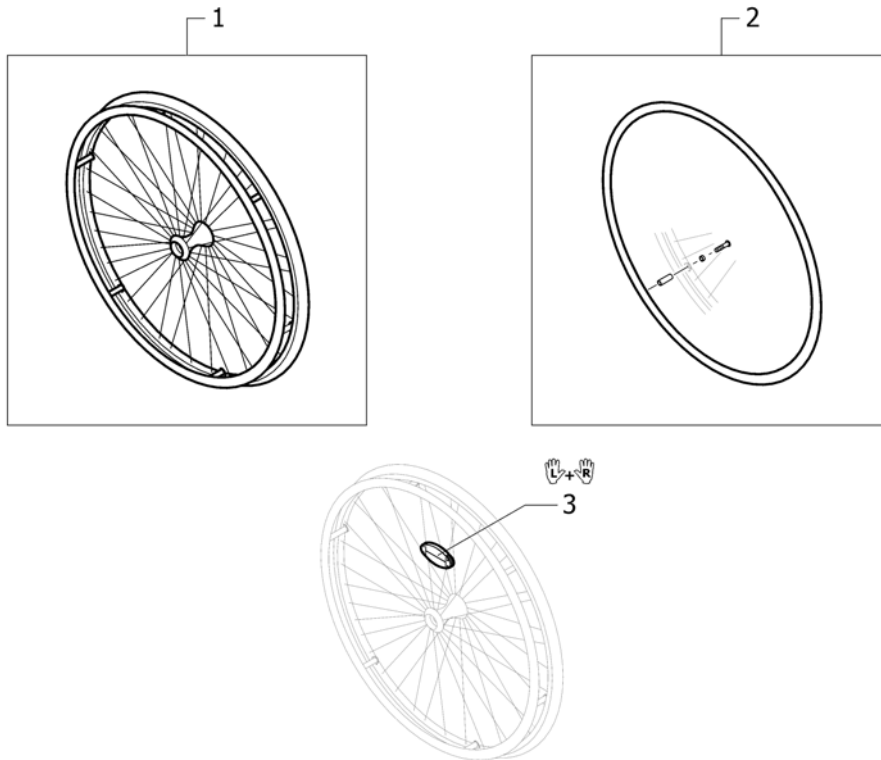
**Active wheel**

Pos.	Part number	Description	Valid from → until	Qty
1	1583058	Rear wheel 24" active		2
2	1583059	Handrim 24" Alu incl. Fixation kit for rear wheel 24" Economy		2
3	1571073	Spokereflection, pair		1


## 24" Küschall rear wheels complete

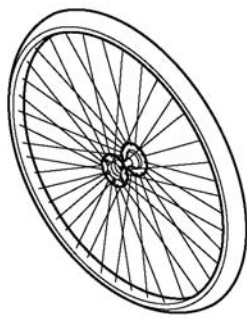

	no tyre		
	1	2	3

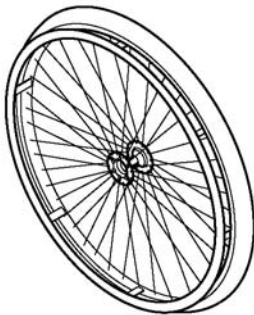


	no tyre		
	-	4	5

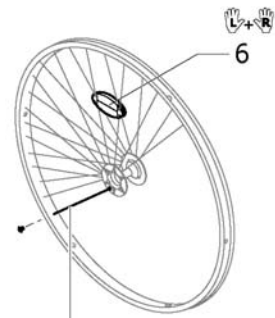

	10 Pcs.
	7


Pos.	Part number	Description	Valid from → until	Qty
1	1567104-0407	Rear wheel 24", Starec X, colourless, without handrim, without tyre		2
2	SP1601745	Rear wheel 24", Starec hub X, without handrim, Rightrun black/grey tyre		2
3	1554399-0407	Rear wheel 24", Starec X, colourless, without handrim, marathon		2
4	SP1601747	Rear wheel 24", Starec hub X, handrim, Rightrun black/grey tyre		2
5	1554400-0407	Rear wheel 24", Starec X, colourless, handrim, marathon evolution		2
6	1571073	Spokereflection, pair		1
7	1579086	Spokes 253mm complete, kit 10 Pcs.		1

## High Performance wheel

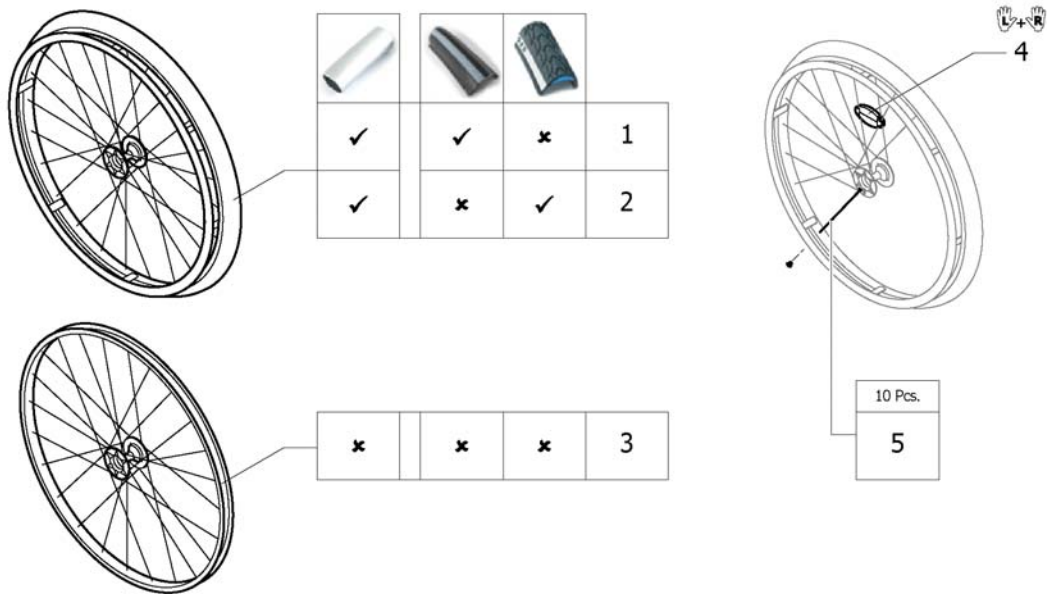


Pos.	Part number	Description	Valid from → until	Qty
1	SP1601754-0004	Rear wheel 24" High Performance black anodized, handrim, Rightrun black/ grey		2
2	1527046-0004	Wheel 24", High Performance, black, handrim alu, tyre Marathon Evolution		2
3	1517723-0004	Wheel 24", High Performance, black, without handrim, without tyre		2
4	1571073	Spokereflection, pair		1
5	1578867	Spokes kit for High Performance wheel 24", 10 Pcs.		1

# Spinergy LXX wheel

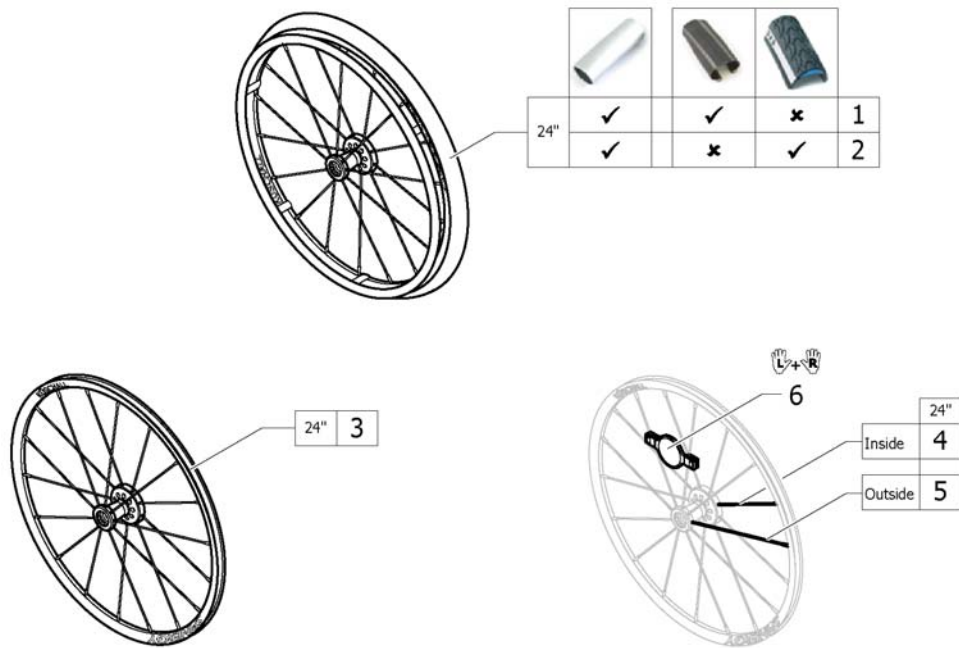

Pos.	Part number	Description	Valid from → until	Qty
1	1585007	Rear wheel 24", Spinergy LXX, handrim alu, tyre Schwalbe One		2
2	1585008	Rear wheel 24", Spinergy LXX, handrim alu, tyre Marathon Evolution		2
3	1579102	Rear wheel 24", Spinergy LXX, without handrim, without tyre		2
4	1579103	Spoke inside for Spinergy LXX wheel 24", 236mm., 1 Pcs.		9
5	1579104	Spoke outside for Spinergy LXX wheel 24", 249mm., 1 Pcs.		9
6	1571088	Spoke reflection for Spinergy wheel, pair		1


## Drumbrake wheel

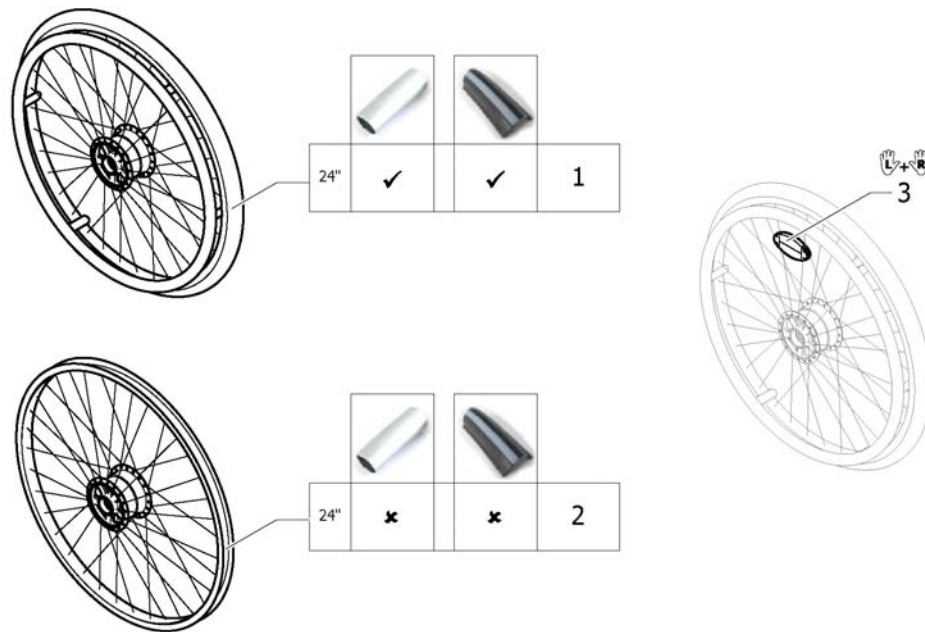

Pos.	Part number	Description	Valid from → until	Qty
1	SP1604974	Rear wheel 24" Drumbrake, handrim, Rightrun black/grey		2
2	SP1604975	Rear wheel 24" Drumbrake, without handrim, without tyre		2
3	1571073	Spokereflection, pair		1

# Handrim



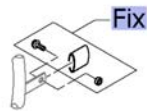




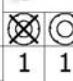
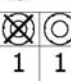







							
		3cm	24"	24"	24"	24"	24"
							
		4cm					
	3cm	1	2	3	4	5	
	4cm	-	-	-	4	-	
	3cm	1	2	3	4	5	
	4cm	-	-	-	4	-	
	3cm	1	2	3	4	5	
	4cm	-	-	-	4	-	
	3cm	1	2	3	4	5	
	4cm	-	-	-	4	-	




Pos.	Part number	Description	Valid from → until	Qty
Fix	-	Fixation handrim		2
1	1568714-0005	Handrim 24" Aluminium, distance 3cm		2
2	1568716	Handrim 24" Titanium, distance 3cm		2
3	1568715	Handrim 24" steel polished, distance 3cm		2
4	1568703	Handrim 24" Super-Grip		2
5	1556916	Handrim 24" LT Surge		2







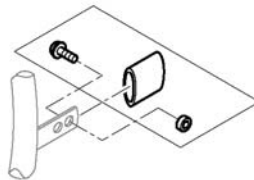




# Fixation handrim

		Aluminium	Titanium	Steel	Supergrip	Surge				
	3cm									
	4cm									
	3cm	1	1	1	1	1	1	3	3	
	4cm	-	-	-	-	-	2	3	-	-
	3cm	4	4	4	4	4	4	4	6	4
	4cm	-	-	-	-	-	5	6	-	-
	3cm	4	-	4	-	4	-	6	-	-
	4cm	-	-	-	-	-	5	-	-	-
	3cm	1	1	1	1	1	1	1	3	3
	4cm	-	-	-	-	-	2	3	-	-



Pos.	Part number	Description	Valid from → until	Qty
1	1495958	Fixation kit handrim, 3 cm		2
2	1495959	Fixation kit handrim, 4 cm		2
3	1495961	Fixation kit handrim, 4 cm with spokes guard		2
4	1496019	Fixation kit handrim for High Performance wheel, 3 cm		2
5	1496020	Fixation kit handrim for High Performance wheel, 4 cm		2
6	1497111	Fixation kit handrim for High Performance wheel, 4 cm with spokes guard		2










# Tyre



				
	24"	24"	24"	24"
 Active wheels	1	2	3	4
 Küschall wheels	1	2	3	4
 High Performance	1	2	3	-
 Spinergy LxK	1	2	3	-
 Drumbrake	1	2	3	4




Pos.	Part number	Description	Valid from → until	Qty
Acc.	-	Inner tube and rim protector		2
1	SP1590118	Tyre Rightrun black/grey 24"		2
2	1570838	Tyre 24" Marathon Evolution		2
3	1578802	Tyre 24" Schwalbe One		2
4	1570831	Solid tyre 24", profile		2

# Inner tube and rim protector







					
		24"	24"	24"	24"
 Active wheels	A	1	1	1	-
	B	2	2	2	2
 Küschall wheels	A	1	1	1	-
	B	2	2	2	2
 High Performance	A	1	1	1	-
	B	3	3	3	-
 Spinergy LxK	A	1	1	1	-
	B	3	3	3	-
 Drumbrake	A	1	1	1	-
	B	2	2	2	2

 Inner Tube (A)       Rim protector (B)

 4

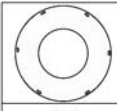






Pos.	Part number	Description	Valid from → until	Qty
1	1570849	Inner Tube 24" x 1"		2
2	1570845	Rim protection 24" Large		2
3	1495962	Rim Protection for High Performance / Spinergy wheel		2
4	1571072	Rear wheel inset anti-flat for 20,22,24,26" tyre		1

# Spoke guard

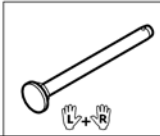		
		24"
Active wheels		-
Küschall wheels		1
High Performance		1
Spinergy LXX		-
Drumbrake		1



Pos.	Part number	Description	Valid from → until	Qty
1	1564457	Spokes guard Transparent 24"		2

Küschall® Compact attract | REAR WHEELS  
**Quick release axle and bearing**

		
	L+R	L+R
 Active wheels	1	-
 Küschall wheels	2	5
 High Performance	1	5
 Spinergy LXX	3	5
 Drumbrake	4	-


Pos.	Part number	Description	Valid from → until	Qty
1	1509137-0004	Quick release axles long with cap, pair, black		1
2	1509137-0407	Quick release axles long with cap, pair, colourless		1
3	1507627	Quickrelease axle long with cap for Spinergy wheel, pair		1
4	SP1606165-0407	Quick release axles for drumbrake with cap, pair, colourless	2016-03-08 →	1
5	1571069	Bearing R8-2RS 1/2"/1.1/8"x5/16", pair		2

# BRAKES

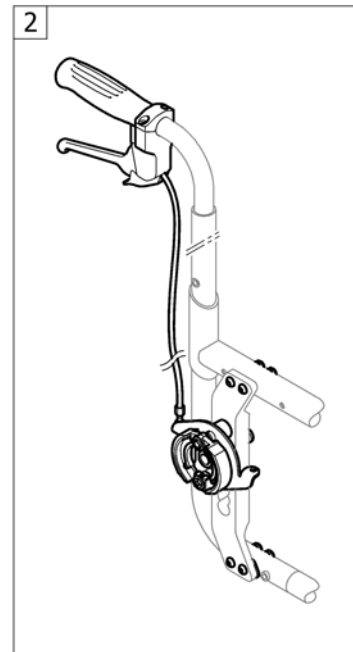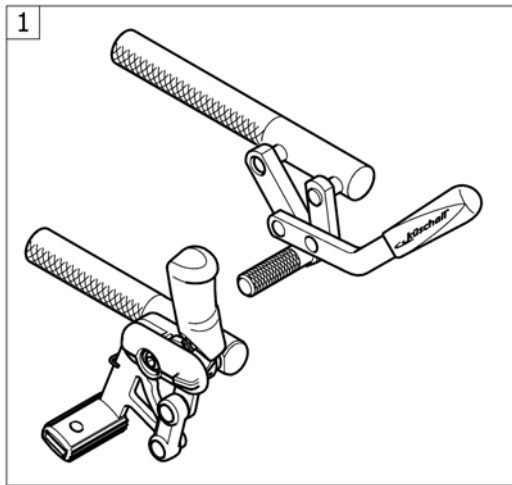


Pos.	Part number	Description	Valid from → until	Qty
1	-	Brakes		1
2	-	Drum brake		1

# Brakes

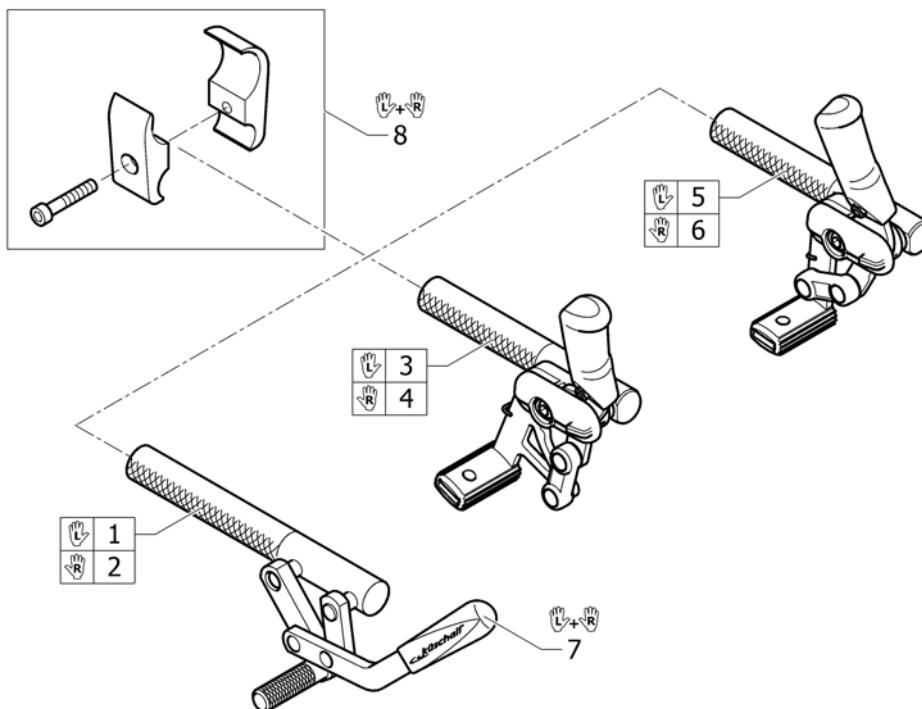

Pos.	Part number	Description	Valid from → until	Qty
1	1512129	Pull brake left		1
2	1512130	Pull brake right		1
3	SP1604007	Attract Pull brake left		1
4	SP1604008	Attract Pull brake right		1
5	SP1604009	Attract Push brake left		1
6	SP1604010	Attract Push brake right		1
7	1513177	Brake handle standard, pair		1
8	1513176	Clamping piece to brake, set		2



## Drum brake

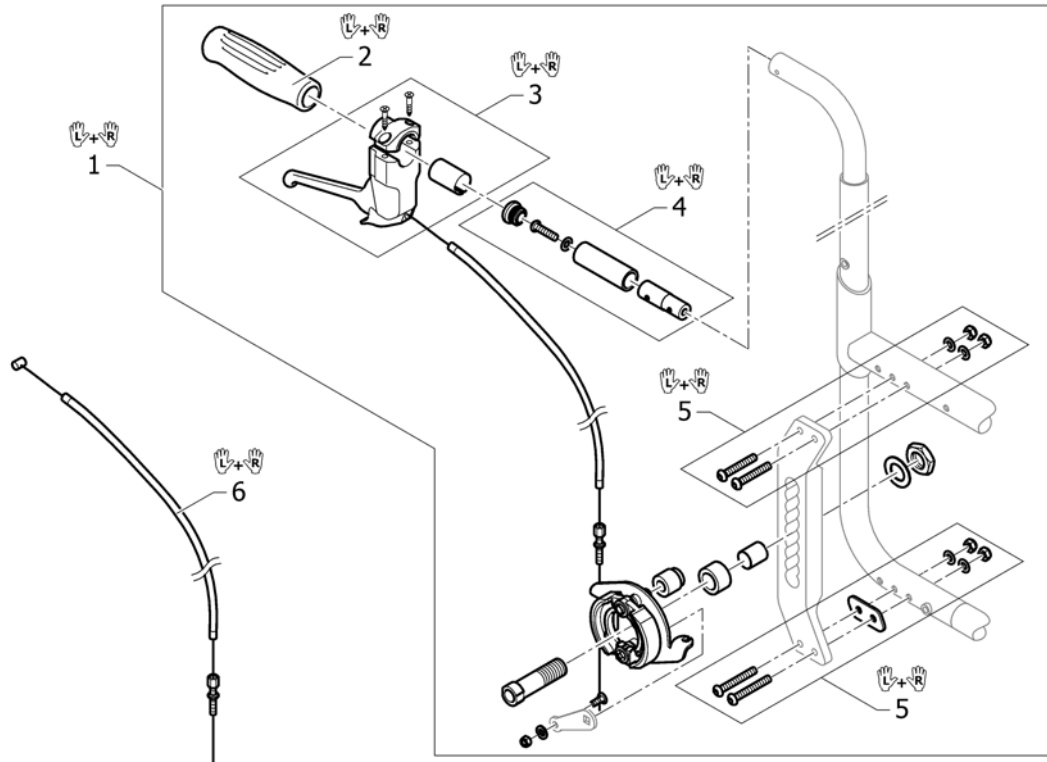

Pos.	Part number	Description	Valid from → until	Qty
1	SP1604976	Drumbrake set, pair		1
2	1507734	Handle black, pair		1
3	SP1604978	Drumbrake handle, pair		1
4	1527108	Push handle extension for drumbrake, pair		1
5	SP1604979	Fixation kit for Vertical brace with drumbrake, pair		1
6	SP1604977	Drumbrake cable with hose, pair		1




## OPTIONS & ACCESSORIES

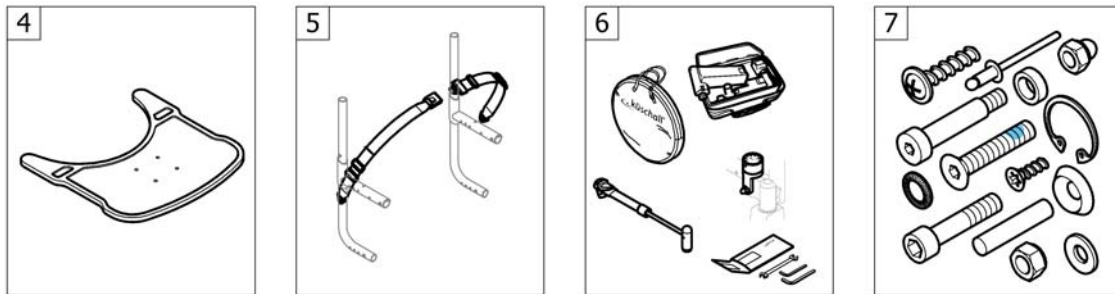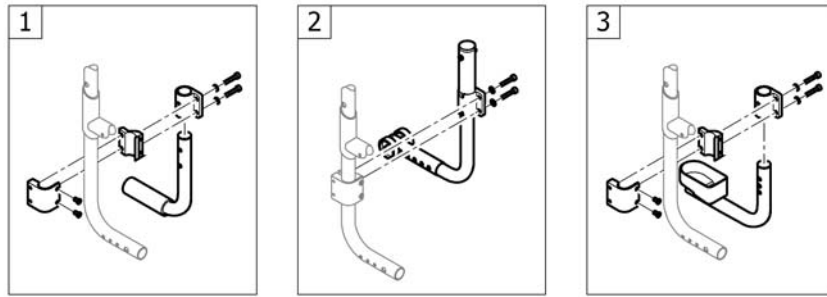


Pos.	Part number	Description	Valid from → until	Qty
1	-	Tipper aid		1
2	-	Antitipper		1
3	-	Cane holder		1
4	-	Therapy table		1
5	-	Seat belt		1
6	-	Chair accessories		1
7	-	Fixation: screws, ...		1

## Tipper aid

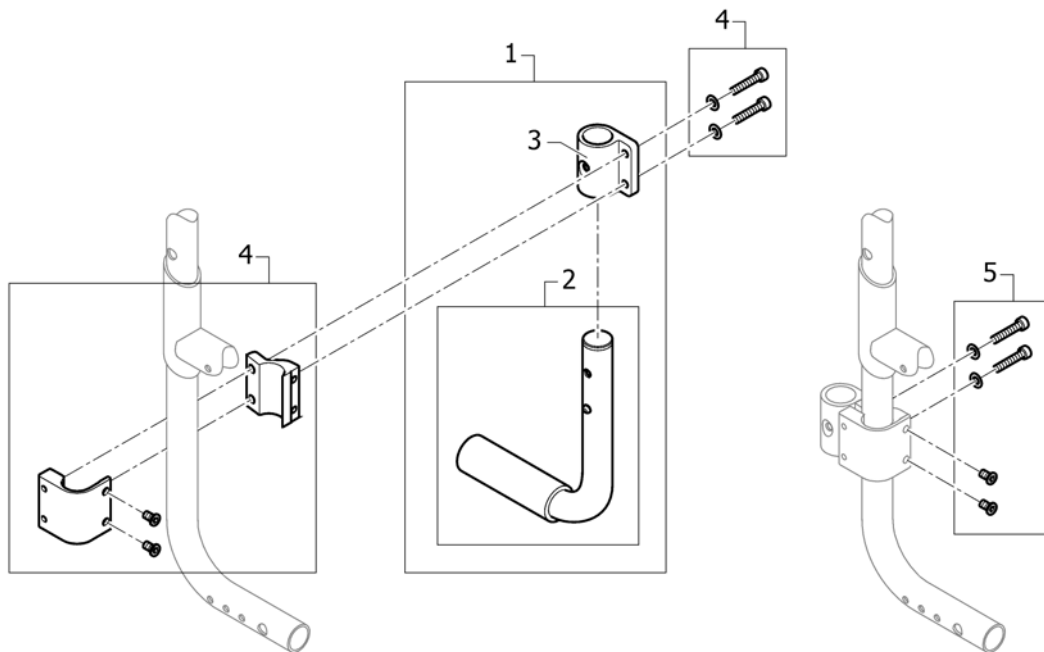


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604987	Tipper-Aid with holder		1
2	SP1604988	Tipper-Aid		1
3	SP1604984	Holder for adaption		1
4	SP1604985	Clamping piece to adaption		1
5	SP1604986	Fixation kit for clamping piece to adaption		1

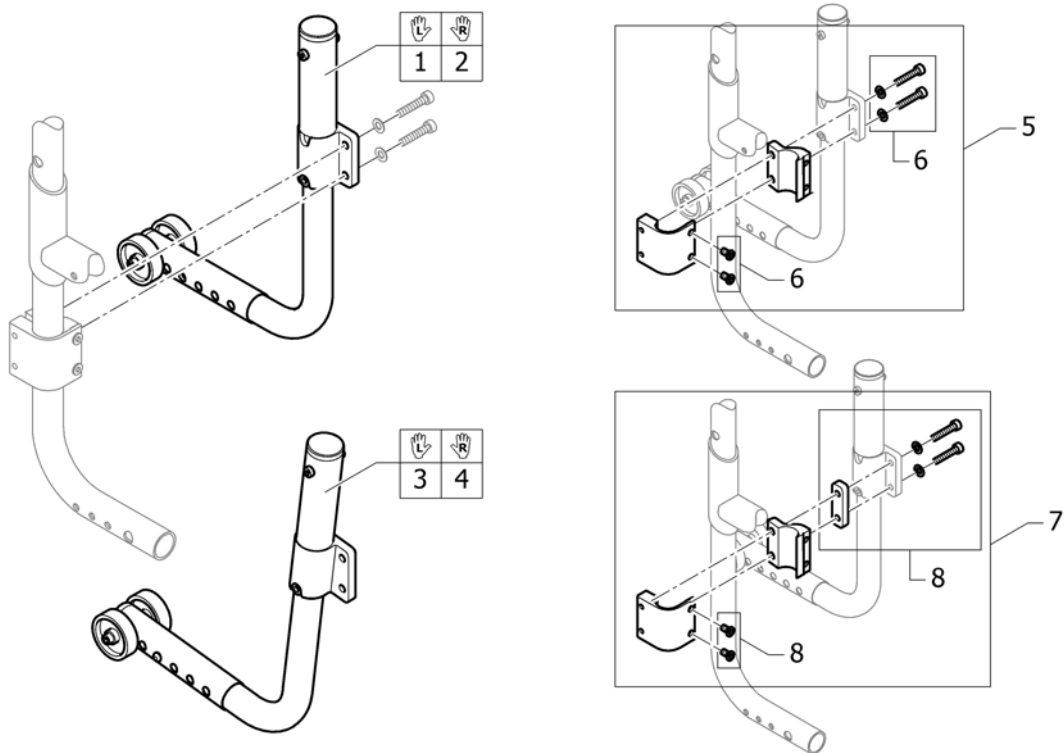**Antitipper**


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604989	Antitipper, left		1
2	SP1604990	Antitipper, right		1
3	SP1604991	Antitipper 7°, left		1
4	SP1604992	Antitipper 7°, right		1
5	SP1604985	Clamping piece to adaption		1
6	SP1604986	Fixation kit for clamping piece to adaption		1
7	SP1604993	Clamping piece to adaption with spacer <i>Needed in combination with siderest with desk function</i>		1
8	SP1604994	Fixation kit for clamping piece to adaption with spacer <i>Needed in combination with siderest with desk function</i>		1

## Cane holder

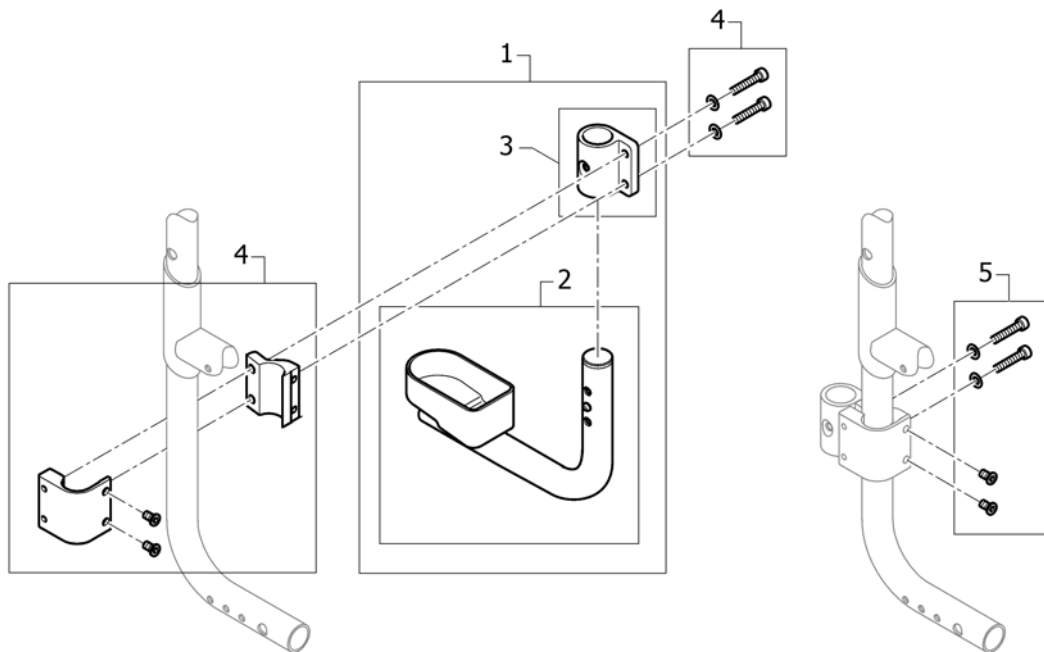


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604982	cane holder with holder		1
2	SP1604983	cane holder		1
3	SP1604984	Holder for adaption		1
4	SP1604985	Clamping piece to adaption		1
5	SP1604986	Fixation kit for clamping piece to adaption		1

## Therapy table

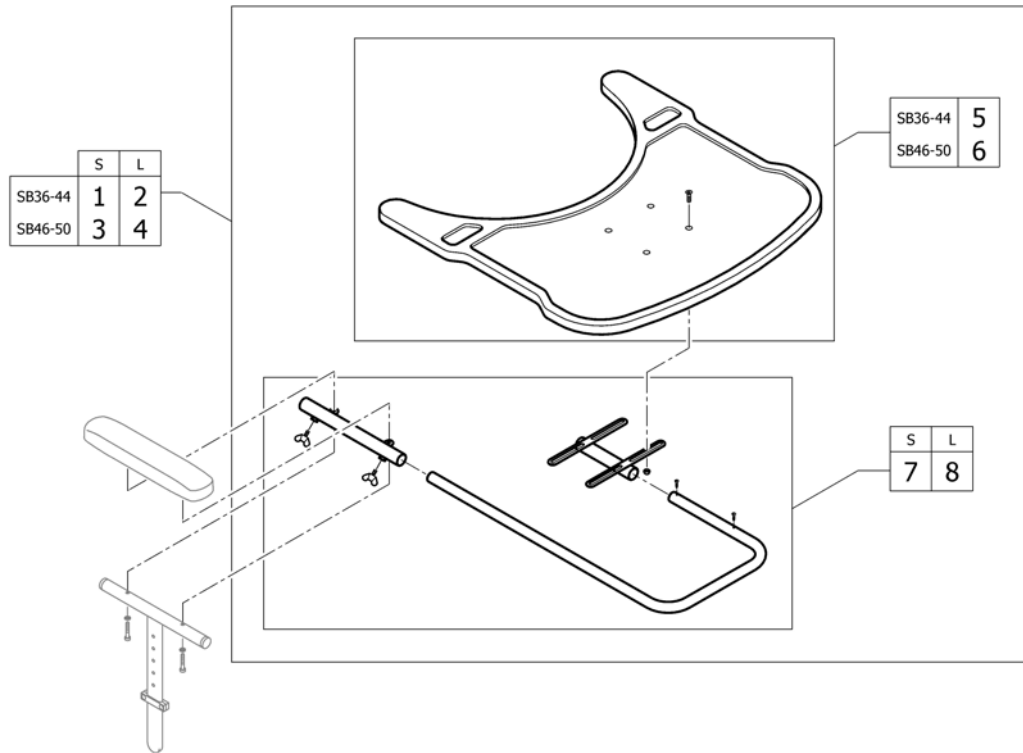


Pos.	Part number	Description	Valid from → until	Qty
1	1439492	Therapy table M complete for Armrest short		1
2	1438374	Therapy table M complete to Armrest long		1
3	1439493	Therapy table L complete for Armrest short		1
4	1438375	Therapy table L complete to Armrest long		1
5	1582670	Therapy table M		1
6	1582671	Therapy table L		1
7	1582672	Fixation kit for therapy table on armrest short		1
8	1582673	Fixation kit for therapy table on armrest long		1

## Seat belt

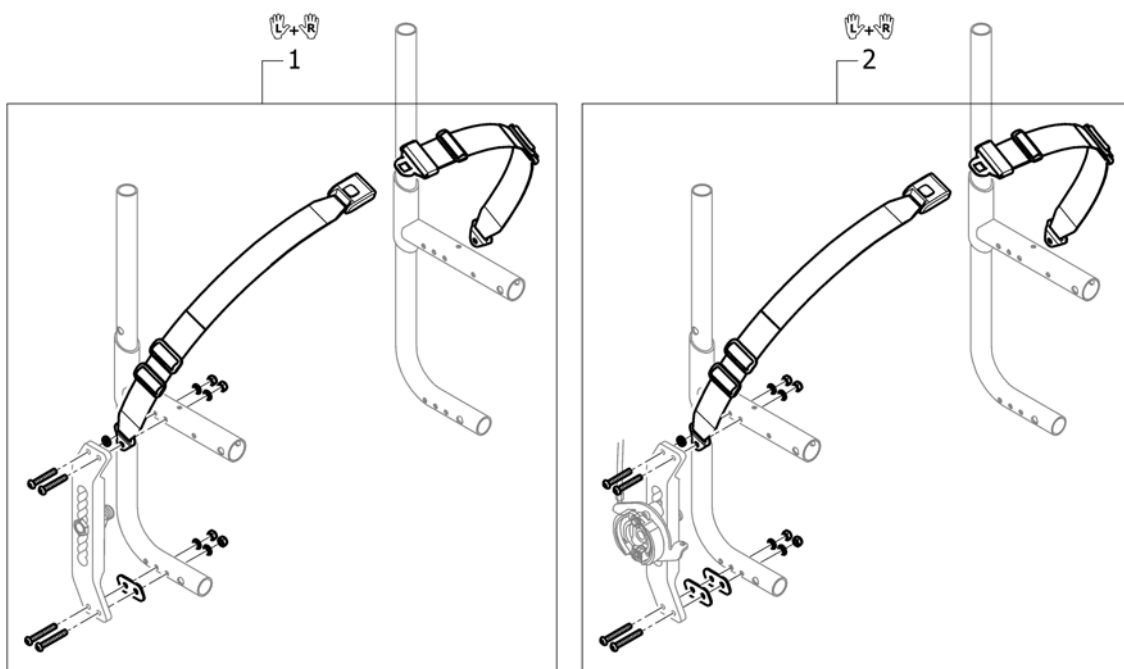


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604980	Seat belt		1
2	SP1604981	Seat belt mounted with drumbrake wheel		1

## Chair accessories

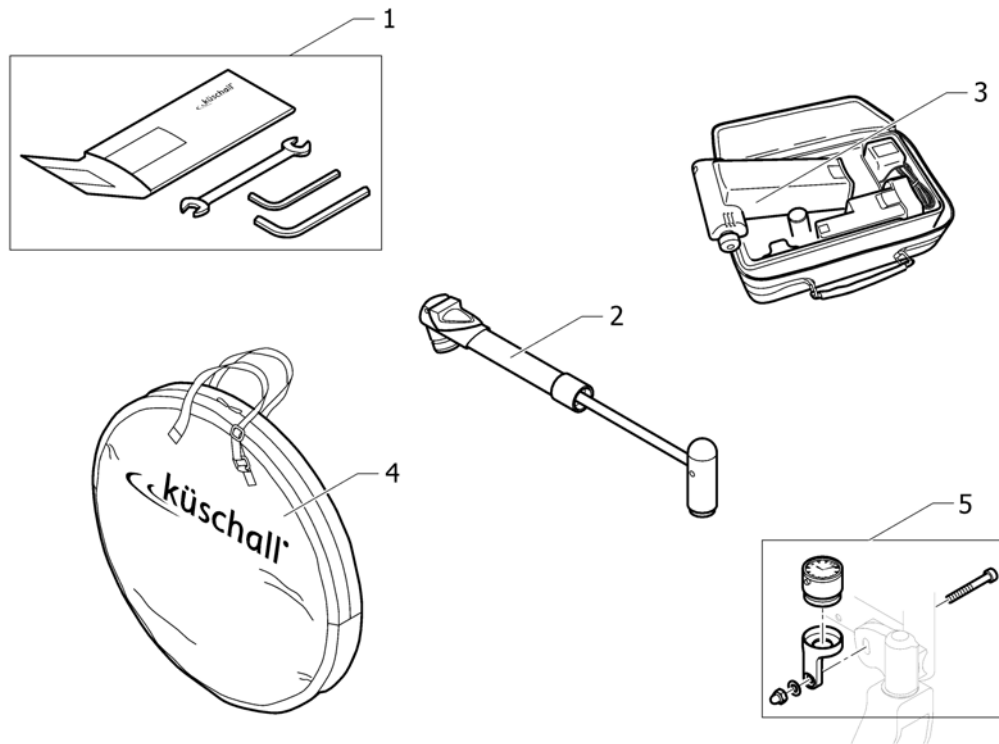

Pos.	Part number	Description	Valid from → until	Qty
1	1518285	Tool Kit		1
2	1517924	Air pump		1
3	1518286	Air man		1
4	1518287	Wheel bag 24" - 26"		1
5	SP1601738	Küschall watch mounted on front fork castor, kit		1


## Fixation: screws, ...

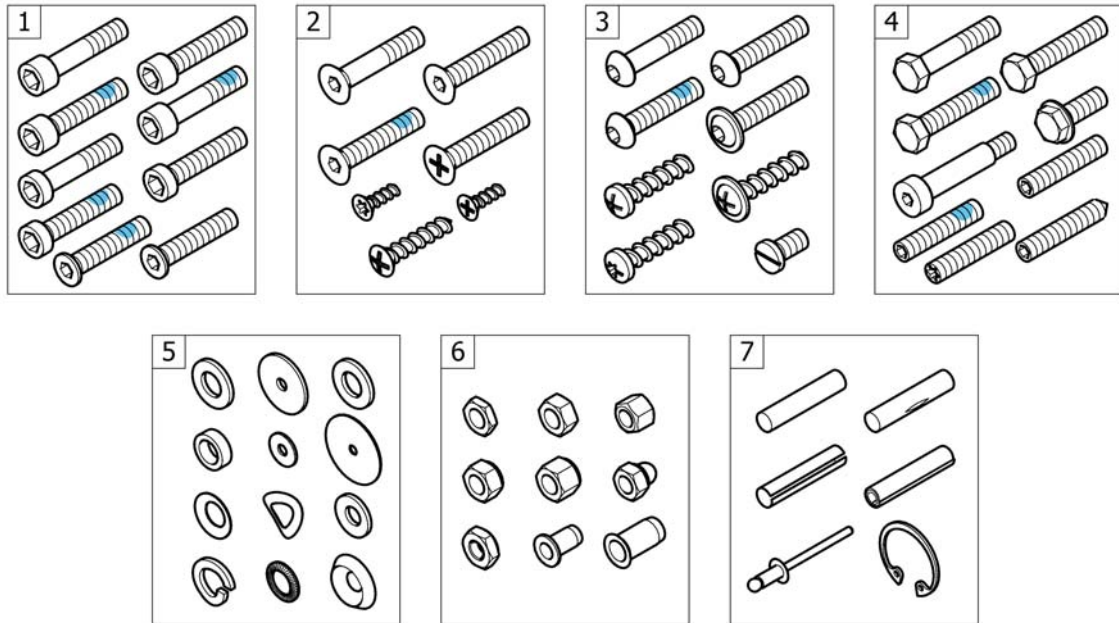

Pos.	Part number	Description	Valid from → until	Qty
1	-	Socket head cap screws		1
2	-	Countersunk screws		1
3	-	Button head screws		1
4	-	Hex-head screws, diverse, ...		1
5	-	Washers		1
6	-	Nuts		1
7	-	Pins, rivets, ...		1



## Socket head cap screws

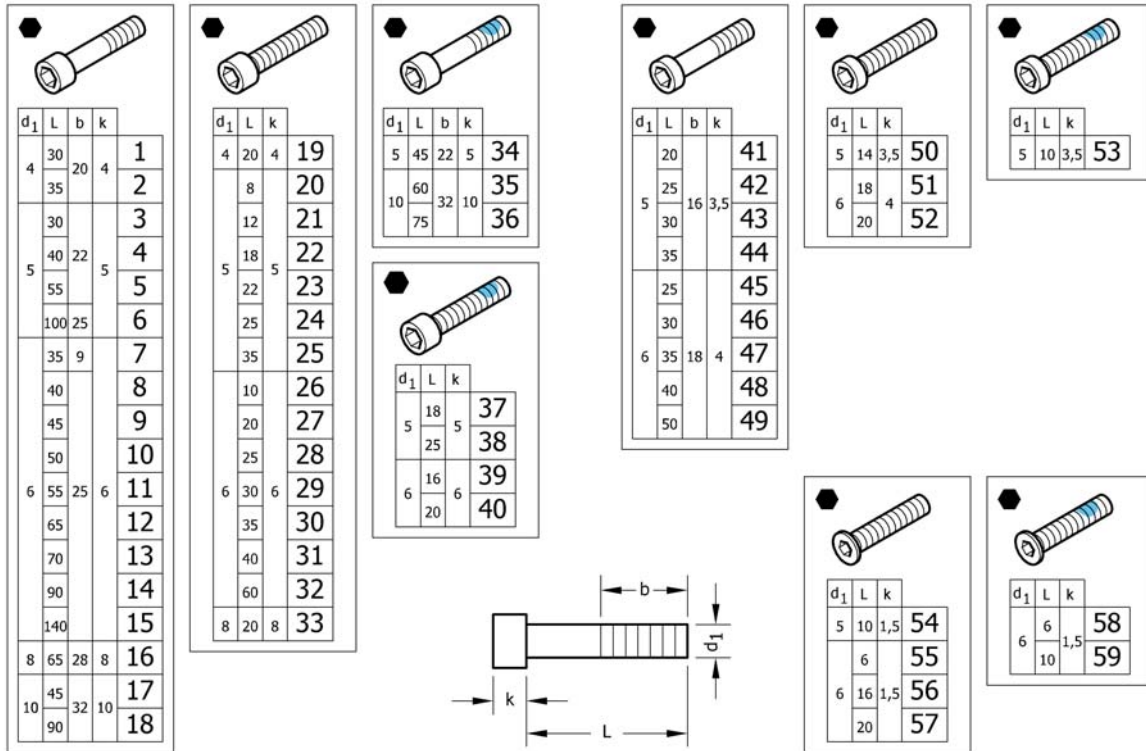

Pos.	Part number	Description	Valid from → until	Qty
1	1579115	Hex socket head cap screws partially threaded, zi.Blue, M4x30, kit 25 <i>Replaces 1560191 (25x)</i>		1
2	1579116	Hex socket head cap screws partially threaded, zi.Blue, M4x35, kit 25 <i>Replaces 1563765 (25x)</i>		1
3	1579117	Hex socket head cap screws partially threaded, black, M5x30, kit 25 Pcs. <i>Replaces 100.1091 (25x)</i>		1
4	1579118	Hex socket head cap screws partially threaded, black, M5x40, kit 25 Pcs. <i>Replaces 100.1093 (25x)</i>		1
5	1579119	Hex socket head cap screws partially threaded, black, M5x55, kit 25 Pcs. <i>Replaces 100.1096 (25x)</i>		1
6	1579120	Hex socket head cap screws partially threaded, black, M5x100, kit 10 Pcs. <i>Replaces 1425240 (10x)</i>		1
7	1579121	Hex socket head cap screws partially threaded, black, M6x35, kit 5 Pcs. <i>Replaces 107.1001 (5x)</i>		1
8	1579122	Hex socket head cap screws partially threaded, black, M6x40, kit 25 Pcs. <i>Replaces 100.1133 (25x)</i>		1
9	1579123	Hex socket head cap screws partially threaded, black, M6x45, kit 25 Pcs. <i>Replaces 100.1134 (25x)</i>		1
10	1579124	Hex socket head cap screws partially threaded, black, M6x50, kit 25 Pcs. <i>Replaces 100.1135 (25x)</i>		1
11	1579125	Hex socket head cap screws partially threaded, black, M6x55, kit 25 Pcs. <i>Replaces 100.1136 (25x)</i>		1
11	1579126	Hex socket head cap screws partially threaded, zi.Blue, M6x55, kit 25 <i>Replaces ZM006551A (25x)</i>		1
12	1579127	Hex socket head cap screws partially threaded, black, M6x65, kit 10 Pcs. <i>Replaces 100.1138 (10x)</i>		1
13	1579128	Hex socket head cap screws partially threaded, black, M6x70, kit 10 Pcs. <i>Replaces 100.1139 (10x)</i>		1
14	1579129	Hex socket head cap screws partially threaded, black, M6x90, kit 10 Pcs. <i>Replaces 1529118 (10x)</i>		1
15	1579130	Hex socket head cap screws partially threaded, black, M6x140, kit 10 Pcs. <i>Replaces 1529117 (10x)</i>		1
16	1579131	Hex socket head cap screws partially threaded, black, M8x65, kit 10 Pcs. <i>Replaces 1510316 (10x)</i>		1
17	1579132	Hex socket head cap screws partially threaded, black, M10x45, kit 2 Pcs. <i>Replaces 1496010 (2x)</i>		1


**Socket head cap screws**

Pos.	Part number	Description	Valid from → until	Qty
18	1579133	Hex socket head cap screws partially threaded, black, M10x90, kit 2 Pcs. <i>Replaces 1496013 (2x)</i>		1
19	1579134	Hex socket head cap screws fully threaded, black, M4x20, kit 25 Pcs. <i>Replaces 100.1050 (25x)</i>		1
20	1579135	Hex socket head cap screws fully threaded, black, M5x8, kit 5 Pcs. <i>Replaces 1493753 (5x)</i>		1
21	1579136	Hex socket head cap screws fully threaded, black, M5x12, kit 25 Pcs. <i>Replaces 100.1084 (25x)</i>		1
22	1579137	Hex socket head cap screws fully threaded, black, M5x18, kit 25 Pcs. <i>Replaces 100.1087 (25x)</i>		1
23	1579138	Hex socket head cap screws fully threaded, black, M5x22, kit 25 Pcs. <i>Replaces 100.1089 (25x)</i>		1
24	1579139	Hex socket head cap screws fully threaded, black, M5x25, kit 25 Pcs. <i>Replaces 100.1090 (25x)</i>		1
25	1579140	Hex socket head cap screws fully threaded, black, M5x35, kit 25 Pcs. <i>Replaces 100.1092 (25x)</i>		1
26	1579141	Hex socket head cap screws fully threaded, black, M6x10, kit 25 Pcs. <i>Replaces 100.1123 (25x)</i>		1
27	1579142	Hex socket head cap screws fully threaded, black, M6x20, kit 25 Pcs. <i>Replaces 100.1128 (25x)</i>		1
28	1579143	Hex socket head cap screws fully threaded, black, M6x25, kit 25 Pcs. <i>Replaces 100.1130 (25x)</i>		1
29	1579144	Hex socket head cap screws fully threaded, black, M6x30, kit 25 Pcs. <i>Replaces 100.1131 (25x)</i>		1
30	1579145	Hex socket head cap screws fully threaded, black, M6x35, kit 25 Pcs. <i>Replaces 100.1132 (25x)</i>		1
31	1579146	Hex socket head cap screws fully threaded, black, M6x40, kit 10 Pcs. <i>Replaces 1563349 (10x)</i>		1
32	1579147	Hex socket head cap screws fully threaded, black, M6x60, kit 10 Pcs. <i>Replaces 100.1137 (10x)</i>		1
33	1579148	Hex socket head cap screws fully threaded, black, M8x20, kit 10 Pcs. <i>Replaces 1510307 (10x)</i>		1
34	1579149	Hex sock.Head cap screws partial.Thread., black, TufLok, M5x45, kit 5 Pcs. <i>Replaces 1496014 (5x)</i>		1
35	1579150	Hex sock.Head cap screws partial.Thread., black, TufLok, M10x60, kit 2 Pcs. <i>Replaces 1496011 (2x)</i>		1
36	1579151	Hex sock.Head cap screws partial.Thread., black, TufLok, M10x75, kit 2 Pcs. <i>Replaces 1496012 (2x)</i>		1
37	1579152	Hex socket head cap screws fully threaded, black, TufLok, M5x18, kit 10 Pcs. <i>Replaces 1425241 (10x)</i>		1
38	1579153	Hex socket head cap screws fully threaded, black, TufLok, M5x25, kit 10 Pcs. <i>Replaces 1570153 (10x)</i>		1
39	1579154	Hex socket head cap screws full.Thread., zi.Blue, TufLok, M6x16, kit 10 Pcs. <i>Replaces 1560219 (10x)</i>		1
40	1579155	Hex socket head cap screws fully threaded, black, TufLok, M6x20, kit 10 Pcs. <i>Replaces 1498775 (10x)</i>		1
41	1579156	Hex sock.Head cap screws, low head, part.Thread., black, M5x20, kit 25 Pcs. <i>Replaces 1438501 (25x)</i>		1
42	1579157	Hex sock.Head cap screws, low head, part.Thread., black, M5x25, kit 25 Pcs. <i>Replaces 1438518 (25x)</i>		1
43	1579158	Hex sock.Head cap screws, low head, part.Thread., black, M5x30, kit 25 Pcs. <i>Replaces 100.5090 (25x)</i>		1
44	1579159	Hex sock.Head cap screws, low head, part.Thread., black, M5x35, kit 25 Pcs. <i>Replaces 100.5091 (25x)</i>		1

**Socket head cap screws**

Pos.	Part number	Description	Valid from → until	Qty
45	1579160	Hex sock.Head cap screws, low head, part.Thread., black, M6x25, kit 25 Pcs. <i>Replaces 1432120 (25x)</i>		1
46	1579161	Hex sock.Head cap screws, low head, part.Thread., black, M6x30, kit 25 Pcs. <i>Replaces 100.5130 (25x)</i>		1
47	1579162	Hex sock.Head cap screws, low head, part.Thread., black, M6x35, kit 25 Pcs. <i>Replaces 1438504 (25x)</i>		1
48	1579163	Hex sock.Head cap screws, low head, part.Thread., black, M6x40, kit 25 Pcs. <i>Replaces 100.5132 (25x)</i>		1
49	1579164	Hex sock.Head cap screws, low head, part.Thread., black, M6x50, kit 25 Pcs. <i>Replaces 100.5134 (25x)</i>		1
50	1579165	Hex sock.Head cap screws, low head, part.Thread., black, M5x14, kit 25 Pcs. <i>Replaces 100.5085 (25x)</i>		1
51	1579166	Hex sock.Head cap screws, low head, part.Thread., black, M6x18, kit 25 Pcs. <i>Replaces 100.5126 (25x)</i>		1
52	1579167	Hex sock.Head cap screws, low head, part.Thread., black, M6x20, kit 25 Pcs. <i>Replaces 100.5127 (25x)</i>		1
53	1579168	Hex soc.Head cap scr., low head, part.Threa., bl., TufL., M5x10, kit 10 Pcs. <i>Replaces 1496026 (10x)</i>		1
54	1579169	Hex socket head cap screws, special low head, black, M5x10, kit 25 Pcs. <i>Replaces 1437290 (25x)</i>		1
55	1579170	Hex socket head cap screws, special low head, black, M6x6, kit 25 Pcs. <i>Replaces 1425182 (25x)</i>		1
56	1579171	Hex socket head cap screws, special low head, black, M6x16, kit 25 Pcs. <i>Replaces 1541181 (25x)</i>		1
57	1579172	Hex socket head cap screws, special low head, black, M6x20, kit 25 Pcs. <i>Replaces 107.1022 (25x)</i>		1
58	1579173	Hex socket head cap screws, spec.Low head, black, TufLok, M5x6, kit 10 Pcs. <i>Replaces 1532039 (10x)</i>		1
59	1579174	Hex socket head cap screws, spec.Low head, black, TufLok, M6x10, kit 10 Pcs. <i>Replaces 1420946 (10x)</i>		1

# Countersunk screws

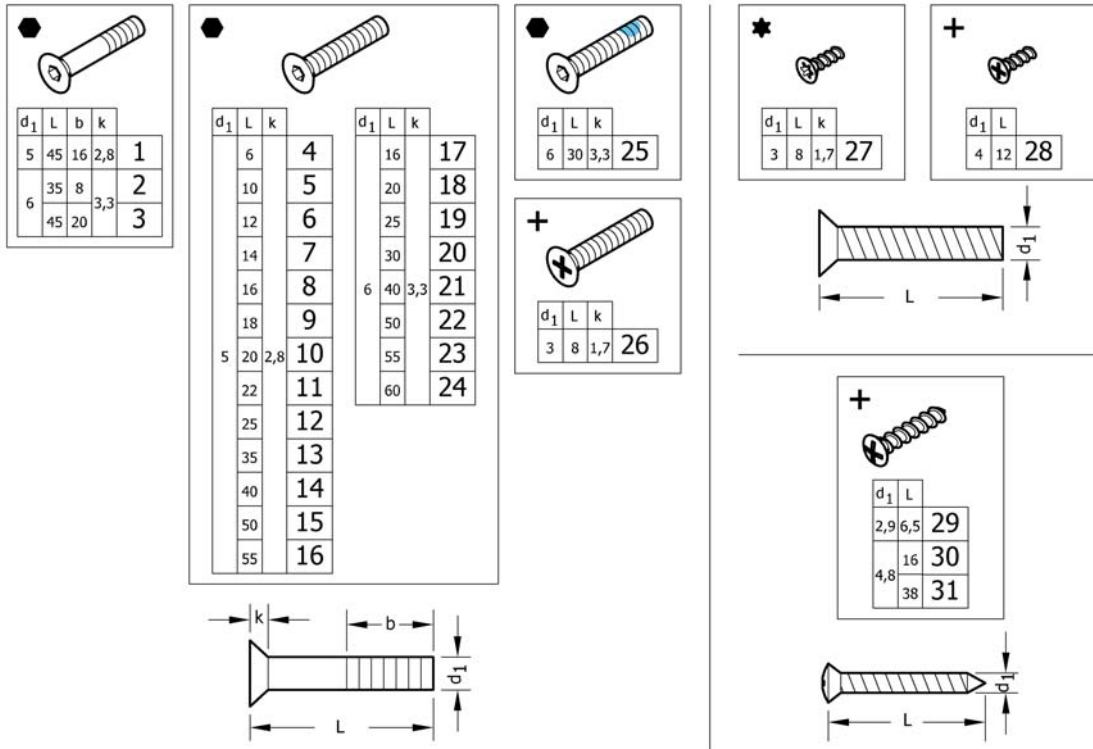


Pos.	Part number	Description	Valid from → until	Qty
1	1579175	Countersunk head screws partially threaded, Black, M5x45, kit 10 Pcs. <i>Replaces 1438773 (10x)</i>		1
2	1579176	Countersunk head screws partially threaded, Black, M6x35, kit 10 Pcs. <i>Replaces 107.1004 (10x)</i>		1
3	1579177	Countersunk head screws partially threaded, Black, M6x45, kit 10 Pcs. <i>Replaces 100.3133 (10x)</i>		1
4	1579178	Countersunk head screws fully threaded, Black, M5x6, kit 25 Pcs. <i>Replaces 1445273 (25x)</i>		1
5	1579179	Countersunk head screws fully threaded, Black, M5x10, kit 25 Pcs. <i>Replaces 100.3082 (25x)</i>		1
6	1579180	Countersunk head screws fully threaded, Black, M5x12, kit 25 Pcs. <i>Replaces 1438999 (25x)</i>		1
7	1579181	Countersunk head screws fully threaded, Black, M5x14, kit 25 Pcs. <i>Replaces 107.1015 (25x)</i>		1
8	1579182	Countersunk head screws fully threaded, Black, M5x16, kit 25 Pcs. <i>Replaces 100.3085 (25x)</i>		1
9	1579183	Countersunk head screws fully threaded, Black, M5x18, kit 25 Pcs. <i>Replaces 100.3086 (25x)</i>		1
10	1579184	Countersunk head screws fully threaded, Black, M5x20, kit 25 Pcs. <i>Replaces 100.3087 (25x)</i>		1
11	1579185	Countersunk head screws fully threaded, Black, M5x22, kit 25 Pcs. <i>Replaces 1416930 (25x)</i>		1
12	1579186	Countersunk head screws fully threaded, Black, M5x25, kit 25 Pcs. <i>Replaces 100.3089 (25x)</i>		1
13	1579187	Countersunk head screws fully threaded, Black, M5x35, kit 25 Pcs. <i>Replaces 100.3091 (25x)</i>		1
14	1579188	Countersunk head screws fully threaded, Black, M5x40, kit 25 Pcs. <i>Replaces 1432148 (25x)</i>		1
15	1579189	Countersunk head screws fully threaded, Black, M5x50, kit 25 Pcs. <i>Replaces 100.3094 (25x)</i>		1
16	1579190	Countersunk head screws fully threaded, Black, M5x55, kit 25 Pcs. <i>Replaces 100.3095 (25x)</i>		1
17	1579191	Countersunk head screws fully threaded, Black, M6x16, kit 25 Pcs. <i>Replaces 100.3125 (25x)</i>		1
18	1579192	Countersunk head screws full.Threaded, zinc flake coat., M6x20, kit 25 Pcs. <i>Replaces 100.3127 (25x)</i>		1

**Countersunk screws**

Pos.	Part number	Description	Valid from → until	Qty
18	1579193	Countersunk head screws fully threaded, Black, M6x20, kit 25 Pcs. <i>Replaces 1437809 (25x)</i>		1
19	1579194	Countersunk head screws fully threaded, Black, M6x25, kit 25 Pcs. <i>Replaces 100.3129 (25x)</i>		1
20	1579195	Countersunk head screws fully threaded, Black, M6x30, kit 25 Pcs. <i>Replaces 100.3130 (25x)</i>		1
21	1579196	Countersunk head screws fully threaded, Black, M6x40, kit 25 Pcs. <i>Replaces 100.3132 (25x)</i>		1
22	1579197	Countersunk head screws fully threaded, Black, M6x50, kit 25 Pcs. <i>Replaces 100.3134 (25x)</i>		1
23	1579198	Countersunk head screws fully threaded, Black, M6x55, kit 25 Pcs. <i>Replaces 100.3135 (25x)</i>		1
24	1579199	Countersunk head screws fully threaded, Black, M6x60, kit 25 Pcs. <i>Replaces 100.3136 (25x)</i>		1
25	1579200	Countersunk head screws fully threaded, Black, TufLok, M6x30, kit 25 Pcs. <i>Replaces 1550479 (25x)</i>		1
26	1579201	Countersunk head screws fully threaded, zi.Blue, M3x8, kit 25 Pcs. <i>Replaces 1529279 (25x)</i>		1
27	1579202	Countersunk head PT screws, Torx, stainless steel, 3x8, kit 25 Pcs. <i>Replaces 1529278 (25x)</i>		1
28	1579203	Countersunk head PT screws, Black, KA40x12, kit 25 Pcs. <i>Replaces 1438513 (25x)</i>		1
29	1579204	Countersunk head tapping screws with cone end, Black, ST2,9x6,5, kit 25 Pcs. <i>Replaces 126.5041 (25x)</i>		1
30	1579205	Countersunk head tapping screws with cone end, Black, ST4,8x16, kit 25 Pcs. <i>Replaces 126.5202 (25x)</i>		1
31	1579206	Countersunk head tapping screws with cone end, Black, ST4,8x38, kit 25 Pcs. <i>Replaces 126.5207 (25x)</i>		1

## Button head screws

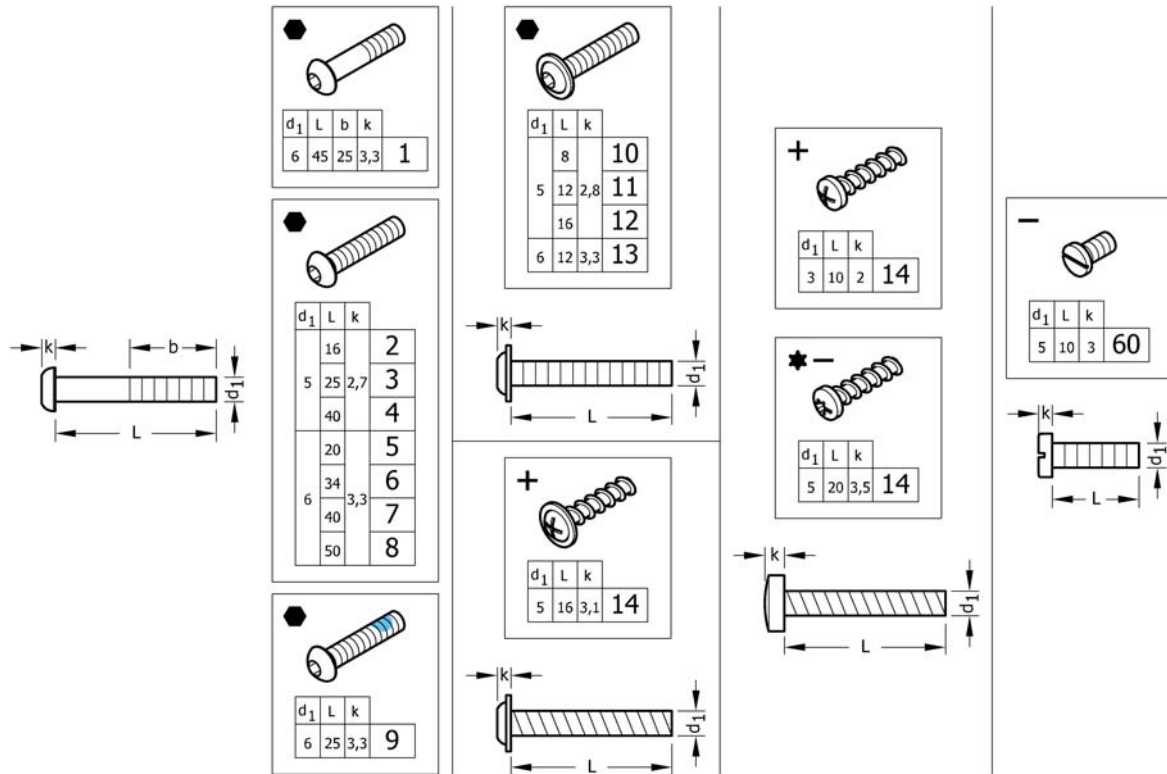


Pos.	Part number	Description	Valid from → until	Qty
1	1579259	Hex sock.Button head cap screws partially thread., Black, M6x45, kit 25 Pcs. <i>Replaces 1438507 (25x)</i>		1
2	1579260	Hex socket button head cap screws fully threaded, Black, M5x16, kit 25 Pcs. <i>Replaces 1445341-0003 (25x)</i>		1
3	1579261	Hex socket button head cap screws fully threaded, Black, M5x25, kit 25 Pcs. <i>Replaces 107.1021 (25x)</i>		1
4	1579262	Hex socket button head cap screws fully threaded, Black, M5x40, kit 25 Pcs. <i>Replaces 1445101-0003 (25x)</i>		1
5	1579263	Hex socket button head cap screws fully threaded, Black, M6x20, kit 25 Pcs. <i>Replaces 1439153 (25x)</i>		1
6	1579264	Hex socket button head cap screws fully threaded, Black, M6x34, kit 25 Pcs. <i>Replaces 1438506-0003 (25x)</i>		1
7	1579265	Hex socket button head cap screws fully threaded, Black, M6x40, kit 25 Pcs. <i>Replaces 1439530-0003 (25x)</i>		1
8	1579266	Hex socket button head cap screws fully threaded, Black, M6x50, kit 25 Pcs. <i>Replaces 1439531-0003 (25x)</i>		1
9	1579267	Hex sock.But.Head cap screws full.Thread., Black, TufLok, M6x25, kit 25 Pcs. <i>Replaces 1438505 (25x)</i>		1
10	1579268	Button head socket cap screws with flange, Black, M5x8, kit 25 Pcs. <i>Replaces 1536268 (25x)</i>		1
11	1579269	Button head socket cap screws with flange, Black, M5x12, kit 25 Pcs. <i>Replaces 1529290 (25x)</i>		1
12	1579270	Button head socket cap screws with flange, Black, M5x16, kit 25 Pcs. <i>Replaces 1445102-0003 (25x)</i>		1
13	1579271	Button head socket cap screws with flange, Black, M6x12, kit 25 Pcs. <i>Replaces 1432204 (25x)</i>		1
14	1579272	Round washer head PT® screws, form H, zinc plated blue, M5x16, kit 25 Pcs. <i>Replaces 1536057 (25x)</i>		1

**Button head screws**

Pos.	Part number	Description	Valid from → until	Qty
15	1579273	Pan head PT screws, form H, zinc plated blue, M3x10, kit 25 Pcs. <i>Replaces ZM003510A (25x)</i>		1
16	1579274	Pan head PT screws, Torx, Black, M5x20, kit 25 Pcs. <i>Replaces 1424569 (25x)</i>		1
17	1579275	Slotted machine screws, Black, M5x10, kit 25 Pcs. <i>Replaces 102.2083 (25x)</i>		1



# Hex-head screws, diverse, ...

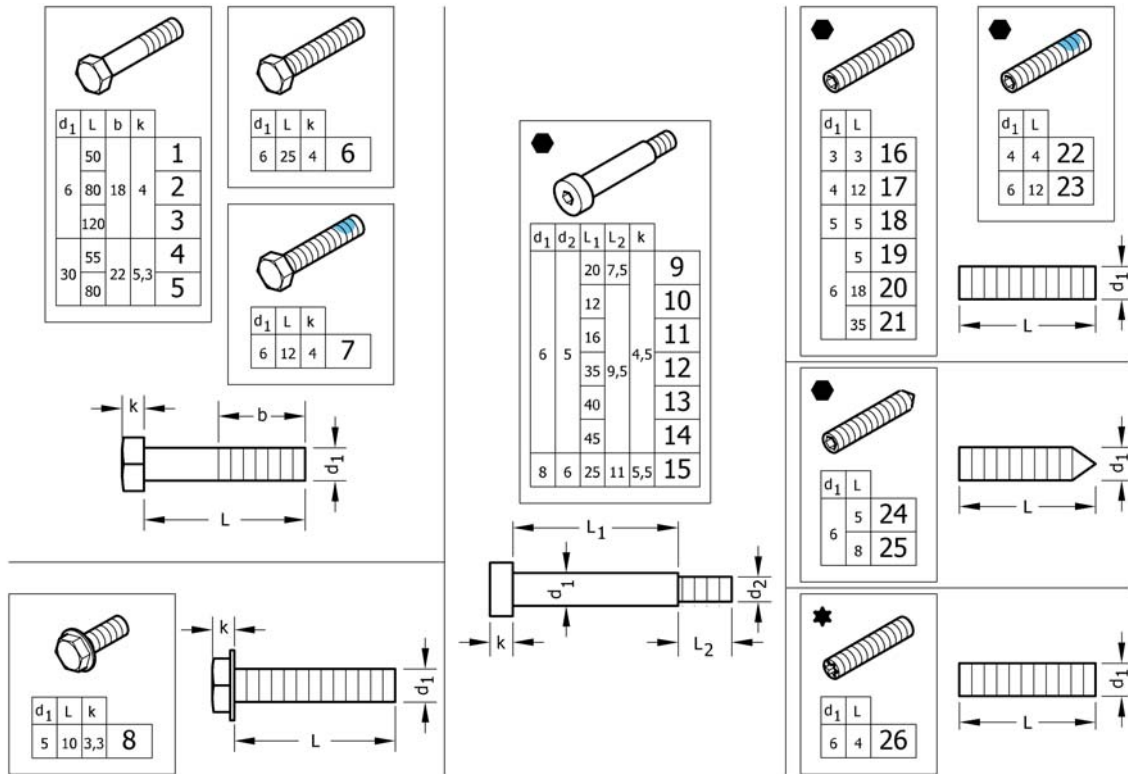


Pos.	Part number	Description	Valid from → until	Qty
1	1579276	Hex head bolts partially threaded, Black, M6x50, kit 10 Pcs. <i>Replaces 101.1134 (10x)</i>		1
2	1579277	Hex head bolts partially threaded, Black, M6x80, kit 10 Pcs. <i>Replaces 1438508 (10x)</i>		1
3	1579278	Hex head bolts partially threaded, Black, M6x120, kit 10 Pcs. <i>Replaces 1438509 (10x)</i>		1
4	1579279	Hex head bolts partially threaded, Black, M8x55, kit 10 Pcs. <i>Replaces 101.1174 (10x)</i>		1
5	1579280	Hex head bolts partially threaded, Black, M8x80, kit 10 Pcs. <i>Replaces 101.1179 (10x)</i>		1
6	1579281	Hex head screws fully threaded, Black, M6x25, kit 10 Pcs. <i>Replaces 1437299 (10x)</i>		1
7	1579282	Hex head screws fully threaded, Black, TufLok, M6x12, kit 10 Pcs. <i>Replaces 1564245 (10x)</i>		1
8	1579283	Hex flange head machine screws with flangeBlack, M5x10, kit 10 Pcs. <i>Replaces 107.1014 (10x)</i>		1
9	1579295	Hex socket head shoulder bolts, Black, Ø6x20 / M5, kit 4 Pcs. <i>Replaces 1488934-0003 (4x)</i>		1
10	1579296	Hex socket head shoulder bolts, Black, Ø6x12 / M5, kit 4 Pcs. <i>Replaces 1564246 (4x)</i>		1
11	1579297	Hex socket head shoulder bolts, Black, Ø6x16 / M5, kit 4 Pcs. <i>Replaces 1529291 (4x)</i>		1
12	1579298	Hex socket head shoulder bolts, Black, Ø6x35 / M5, kit 4 Pcs. <i>Replaces 1496034 (4x)</i>		1
13	1579299	Hex socket head shoulder bolts, Black, Ø6x40 / M5, kit 4 Pcs. <i>Replaces 107.1023 (4x)</i>		1
14	1579300	Hex socket head shoulder bolts, Black, Ø6x45 / M5, kit 4 Pcs. <i>Replaces 1496030 (4x)</i>		1
15	1579301	Hex socket head shoulder bolts, Black, Ø8x25 / M6, kit 4 Pcs. <i>Replaces ZM008252A (4x)</i>		1
16	1579284	Hex socket set screws with flat point, Black, M3x3, kit 25 Pcs. <i>Replaces 116.1001 (25x)</i>		1
17	1579285	Hex socket set screws with flat point, Black, M4x12, kit 25 Pcs. <i>Replaces 116.1046 (25x)</i>		1
18	1579286	Hex socket set screws with flat point, zinc plated blue, M5x5, kit 25 Pcs. <i>Replaces 116.1081 (25x)</i>		1



**Hex-head screws, diverse, ...**

Pos.	Part number	Description	Valid from → until	Qty
19	1579287	Hex socket set screws with flat point, Black, M6x5, kit 25 Pcs. <i>Replaces 1437272 (25x)</i>		1
20	1579288	Hex socket set screws with flat point, zinc plated blue, M6x18, kit 25 Pcs. <i>Replaces 1559106 (25x)</i>		1
21	1579289	Hex socket set screws with flat point, Black, M6x35, kit 25 Pcs. <i>Replaces 116.1133 (25x)</i>		1
22	1579290	Hex socket set screws with flat point, Black, TufLok, M4x4, kit 10 Pcs. <i>Replaces 1493792 (10x)</i>		1
23	1579291	Hex socket set screws with flat point, Z.Plat.Bl., TufLok, M6x12, kit 10Pcs. <i>Replaces 1439057 (10x)</i>		1
24	1579292	Hex socket set screws with cone point, Black, M6x5, kit 25 Pcs. <i>Replaces 116.2121 (25x)</i>		1
25	1579293	Hex socket set screws with cone point, Black, M6x8, kit 25 Pcs. <i>Replaces 116.2123 (25x)</i>		1
26	1579294	Screws with cup point, Torx, zinc plated blue, M6x4, kit 25 Pcs. <i>Replaces 1567763 (25x)</i>		1

# Washers

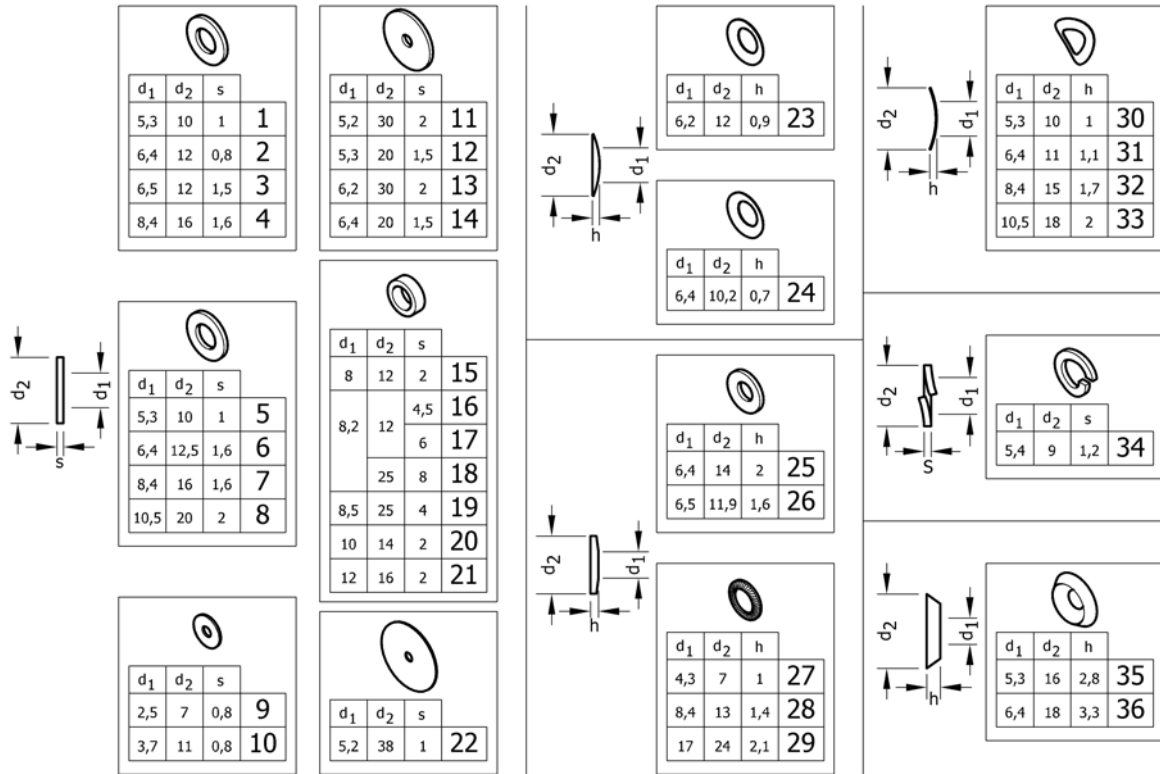


Pos.	Part number	Description	Valid from → until	Qty
1	1579225	Flat washers without chamfer, black, 5,3x10x1, kit 50 Pcs. <i>Replaces 150.1004 (50x)</i>		1
2	1579226	Flat washers without chamfer, black, 6,4x12x0,8, kit 50 Pcs. <i>Replaces 150.2001 (50x)</i>		1
3	1579227	Flat washers without chamfer, zinc plated blue, 6,5x12x1,5, kit 50 Pcs. <i>Replaces 5729091 (50x)</i>		1
4	1579228	Flat washers without chamfer, black, 8,4x16x1,6, kit 50 Pcs. <i>Replaces 150.1012 (50x)</i>		1
5	1579229	Flat washers without chamfer, polyamide, 5,3x10x1, kit 50 Pcs. <i>Replaces 150.3003 (50x)</i>		1
6	1579230	Flat washers without chamfer, polyamide, 6,4x12,5x1,6, kit 50 Pcs. <i>Replaces 150.3004 (50x)</i>		1
7	1579231	Flat washers without chamfer, polyamide, 8,4x16x1,6, kit 50 Pcs. <i>Replaces 150.3005 (50x)</i>		1
8	1579232	Flat washers without chamfer, polyamide, 10,5x20x2, kit 50 Pcs. <i>Replaces 150.3006 (50x)</i>		1
9	1579233	Flat washers without chamfer, zinc plated blue, 2,5x7x0,8, kit 50 Pcs. <i>Replaces 150.2012 (50x)</i>		1
10	1579234	Flat washers without chamfer, zinc plated blue, 3,7x11x0,8, kit 50 Pcs. <i>Replaces ZM203501A (50x)</i>		1
11	1579235	Flat washers without chamfer, black, 5,2x30x2, kit 20 Pcs. <i>Replaces 150.2006 (20x)</i>		1
12	1579236	Flat washers without chamfer, black, 5,3x20x1,5, kit 50 Pcs. <i>Replaces 150.2010 (50x)</i>		1
13	1579237	Flat washers without chamfer, black, 6,2x30x2, kit 20 Pcs. <i>Replaces 150.2009 (20x)</i>		1
14	1579238	Flat washers without chamfer, black, 6,4x25x1,5, kit 50 Pcs. <i>Replaces 1451753 (50x)</i>		1
15	1579239	Hinge rings, black, 8x12x2, kit 50 Pcs. <i>Replaces 150.5002 (50x)</i>		1
16	1579240	Distance sleeve, alu, 8,2x12x4,5, kit 20 Pcs. <i>Replaces 1420970 (20x)</i>		1
17	1579241	Distance sleeve, alu, 8,2x12x6, kit 20 Pcs. <i>Replaces 1420969 (20x)</i>		1
18	1581292	Distance sleeve, alu, 8,2x25x8, kit 2 Pcs. <i>Replaces 151.1002 (2x)</i>		1

**Washers**

Pos.	Part number	Description	Valid from → until	Qty
19	1583052	Distance sleeve, alu, black, 8,5x25x4, kit 10 Pcs. <i>Replaces 151.1014 (10x)</i>		1
20	1579242	Hinge rings, zinc plated blue, 10x14x2, kit 50 Pcs. <i>Replaces 1506666 (50x)</i>		1
21	1579243	Hinge rings, black, 12x16x2, kit 50 Pcs. <i>Replaces 150.5006 (50x)</i>		1
22	1579244	Rubber washers, 5,2x38x1, kit 20 Pcs. <i>Replaces 1424891 (20x)</i>		1
23	1579245	Disc springs, black, 6,2x12x0,9, kit 50 Pcs. <i>Replaces 160.1015 (50x)</i>		1
24	1579246	Conical spring washers, zinc plated blue, 6,4x10,2x0,7, kit 50 Pcs. <i>Replaces 161.1004 (50x)</i>		1
25	1579247	Conical spring washers, black, 6,4x14x2, kit 50 Pcs. <i>Replaces 161.2004 (50x)</i>		1
26	1579248	Conical spring washers, zinc plated blue, 6,5x11,9x1,6, kit 50 Pcs. <i>Replaces ZM206001A (50x)</i>		1
27	1579249	Ribbed lock washers, black, 4,3x7x1, kit 50 Pcs. <i>Replaces 155.2002 (50x)</i>		1
28	1579250	Ribbed lock washers, zinc plated blue, 8,4x13x1,4, kit 50 Pcs. <i>Replaces 155.2005 (50x)</i>		1
29	1579251	Ribbed lock washers, zinc plated blue, 17x24x2,1, kit 50 Pcs. <i>Replaces 155.2009 (50x)</i>		1
30	1579252	Curved spring washers, black, 5,3x10x1, kit 50 Pcs. <i>Replaces 1438515 (50x)</i>		1
31	1579253	Curved spring washers, black, 6,4x11x1,1, kit 50 Pcs. <i>Replaces 1438516 (50x)</i>		1
32	1579254	Curved spring washers, zinc plated blue, 8,4x15x1,7, kit 50 Pcs. <i>Replaces ZM208003A (50x)</i>		1
33	1579255	Curved spring washers, zinc plated blue, 10,5x18x2, kit 50 Pcs. <i>Replaces ZM201581A (50x)</i>		1
34	1579256	Split spring lock washers, black, 5,4x9x1,2, kit 50 Pcs. <i>Replaces 155.1003 (50x)</i>		1
35	1579257	Finishing washers 90° countersunk head screw, black, 5,3x16x2,8, kit 50 Pcs. <i>Replaces 150.6003 (50x)</i>		1
36	1579258	Finishing washers 90° countersunk head screw, black, 6,4x18x3,3, kit 50 Pcs. <i>Replaces 150.6004 (50x)</i>		1

# Nuts

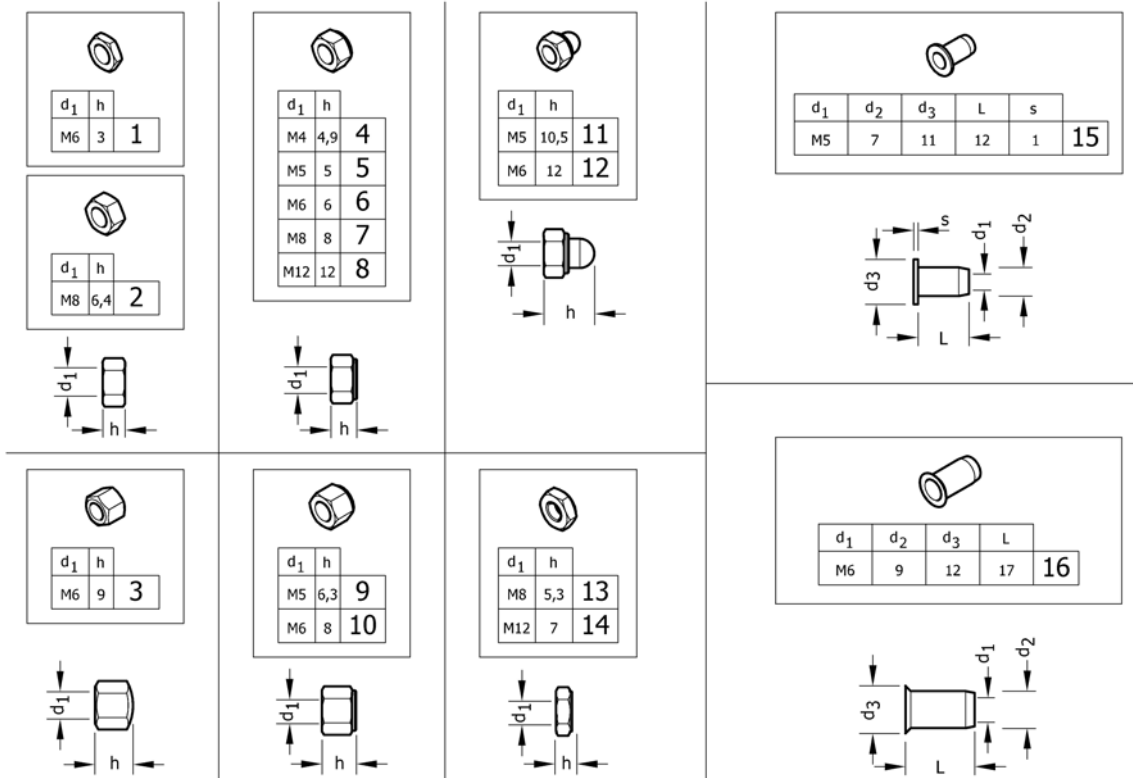


Pos.	Part number	Description	Valid from → until	Qty
1	1579207	Hex jam nuts, black, M6, kit 25 Pcs. <i>Replaces 108.3004 (25x)</i>		1
2	1579208	Hexagon nuts , black, M8, kit 25 Pcs. <i>Replaces 1420997 (25x)</i>		1
3	1579209	Hex cap nuts, low type, black, M6, kit 25 Pcs. <i>Replaces 109.5009 (25x)</i>		1
4	1579210	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M4, kit 25 Pcs. <i>Replaces 109.1002 (25x)</i>		1
5	1579211	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M5, kit 25 Pcs. <i>Replaces 109.1003 (25x)</i>		1
5	1579212	Prev.Torque lock nuts, thin type, with polya.Insert, zi.Bl., M5, kit 25 Pcs. <i>Replaces 109.5008 (25x)</i>		1
6	1579213	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M6, kit 25 Pcs. <i>Replaces 109.1004 (25x)</i>		1
6	1579214	Prev.Torque lock nuts, thin type, with polya.Insert, zi.Bl., M6, kit 25 Pcs. <i>Replaces ZM106001A (25x)</i>		1
7	1579215	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M8, kit 25 Pcs. <i>Replaces 109.1005 (25x)</i>		1
8	1579216	Prev.Torque lock nuts, thin type, with polya.Insert, black, M12, kit 4 Pcs. <i>Replaces 109.1007 (4x)</i>		1
9	1579217	Preva.Torq.Hex lock nuts, high type with poly.Insert, black, M5, kit 25 Pcs. <i>Replaces 1493771 (25x)</i>		1
10	1579218	Preva.Torq.Hex lock nuts, high type with poly.Insert, black, M6, kit 25 Pcs. <i>Replaces 109.5001 (25x)</i>		1
11	1579219	Prevail.Torque hex domed cap nuts with polyam.Insert, black, M5, kit 25 Pcs. <i>Replaces 109.5006 (25x)</i>		1
12	1579220	Prevail.Torque hex domed cap nuts with polyam.Insert, black, M6, kit 25 Pcs. <i>Replaces 109.5003 (25x)</i>		1
13	1579221	Prevailing torque hex lock nuts, all-metal, thin typeblack, M8, kit 10 Pcs. <i>Replaces 109.5005 (10x)</i>		1
14	1579222	Prevailing torque hex lock nuts, all-metal, thin typeblack, M12, kit 4 Pcs. <i>Replaces 109.5004 (4x)</i>		1
15	1579223	Blind rivet nuts, flat head, open type, black, M5, kit 10 Pcs. <i>Replaces 180.4007 (10x)</i>		1

**Nuts**

Pos.	Part number	Description	Valid from → until	Qty
16	1579224	Blind rivet nut with low countersunk head, open type, black, M6, kit 10 Pcs. <i>Replaces 180.4105 (10x)</i>		1

## Pins, rivets, ...

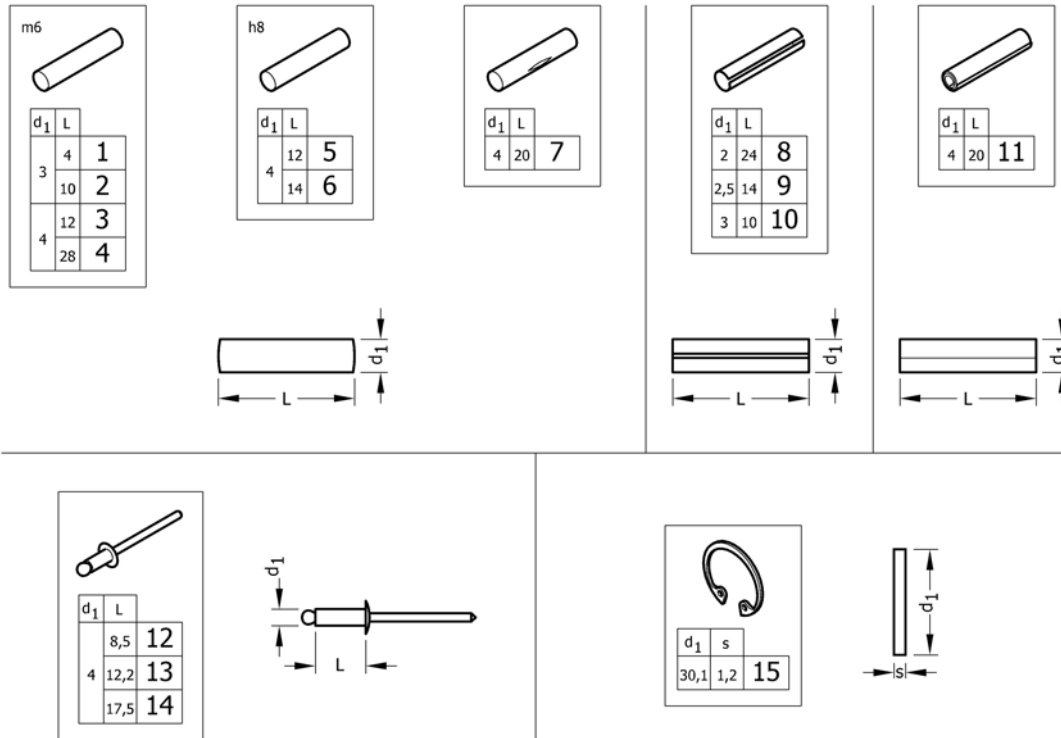

Pos.	Part number	Description	Valid from → until	Qty
1	1579302	Parallel pins, Ø3x4 m6, kit 10 Pcs. <i>Replaces 1439336 (10x)</i>		1
2	1579303	Parallel pins, Ø3x10 m6, kit 10 Pcs. <i>Replaces 1439337 (10x)</i>		1
3	1579304	Parallel pins, Ø4x12 m6, kit 20 Pcs. <i>Replaces 200.1124 (20x)</i>		1
4	1579305	Parallel pins, Ø4x28 m6, kit 20 Pcs. <i>Replaces 1437344 (20x)</i>		1
5	1579306	Parallel pins, Ø4x12 h8, kit 20 Pcs. <i>Replaces 1487663 (20x)</i>		1
6	1579307	Parallel pins, Ø4x14 h8, kit 20 Pcs. <i>Replaces 1487662 (20x)</i>		1
7	1579308	Grooved pins, center grooved, Ø4x20, kit 20 Pcs. <i>Replaces 1517718 (20x)</i>		1
8	1579309	Slotted spring pins, heavy duty, Black, Ø2x24, kit 20 Pcs. <i>Replaces 200.4011 (20x)</i>		1
9	1579310	Slotted spring pins, heavy duty, Black, Ø2,5x14, kit 20 Pcs. <i>Replaces 200.4007 (20x)</i>		1
10	1579311	Slotted spring pins, heavy duty, stainless steel, Ø3x10, kit 20 Pcs. <i>Replaces 1567997 (20x)</i>		1
11	1579312	Spiral spring pins, Black, Ø4x20, kit 20 Pcs. <i>Replaces 200.3167 (20x)</i>		1
12	1579313	Dome head blind rivets, Ø4x8,5, kit 20 Pcs. <i>Replaces 1439292 (20x)</i>		1
13	1579314	Dome head blind rivets, Ø4x12,2, kit 20 Pcs. <i>Replaces 180.5116 (20x)</i>		1
14	1579315	Dome head blind rivets, Ø4x17,5, kit 20 Pcs. <i>Replaces 180.5019 (20x)</i>		1
15	1579316	Retaining rings for bores, Std.Design, Z.plat.Blue, Ø30,1X1,2, kit 20 Pcs. <i>Replaces ZM50J281A (20x)</i>		1





Yes, you can.®



---

**Belgium & Luxemburg:**

Invacare nv, Autobaan 22, B-8210 Loppem  
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11  
belgium@invacare.com / www.invacare.be

**Denmark:**

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01  
denmark@invacare.com / www.invacare.dk

**Deutschland:**

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny  
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66  
info@invacare-aquatec.com / www.invacare-aquatec.de

**European Distributor Organisation:**

Invacare, Kleiststraße 49, D-32457 Porta Westfalica  
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541  
edo@invacare.com / www.invacare.eu.com

**España:**

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20  
contactsp@invacare.com / www.invacare.es

**France:**

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com / www.invacare.fr

**Ireland:**

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland  
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085  
ireland@invacare.com / www.invacare.ie

**Italia:**

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34  
italia@invacare.com / www.invacare.it

**Nederland:**

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede  
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758  
nederland@invacare.com, csede@invacare.com / www.invacare.nl

**Norge:**

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo  
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01  
norway@invacare.com, island@invacare.com / www.invacare.no

**Österreich:**

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee  
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4  
info-austria@invacare.com / www.invacare.at

**Portugal:**

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39  
portugal@invacare.com / www.invacare.pt

**Sverige & Suomi:**

Invacare AB, Fagerstagatan 9, S-163 91 Spånga  
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08  
sweden@invacare.com, finland@invacare.com / www.invacare.se

**Switzerland:**

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81  
switzerland@invacare.com / www.invacare.ch

**United Kingdom:**

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ  
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201  
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220  
uk@invacare.com / www.invacare.co.uk