PYÖRÄTUOLI

V300

V300XL

KÄYTTÖOHJE

CE

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*The manufacturer or his authorized representative :*

N.V. VERMEIREN N.V

**Address :**

Vermeirenplein 1/15

2920 Kalmthout

Belgium

*declares under his sole responsibility that the CE marked medical devices :*

Productgroup: Manual wheelchairs

Brand: Vermeiren

Type: V300

*have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,*

*and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :*

Medical devices directive MDD 93/42/EEC

*and is in conformity with the relevant European harmonized standards:*

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A series of horizontal dotted lines for taking notes.

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**Camp Mobility**

**Patamäenkatu 5**

**33900 Tampere**

**09-350 76 310**

**info@campmobility.fi**

**campmobility.fi**

| Active substance   | Product name                                | Laundry disinfection |                     | Surface disinfection (scrubbing-/wiping disinfection) |                     | Disinfection of excretions<br>1 part sputum or stools + 2 parts diluted solution or 1 part urine + 1 part diluted solution |     |        |     |       |     | Area of effectiveness | Manufacturer or Supplier |
|--|---|----------------------|---------------------|---|---------------------|--|-----|--------|-----|-------|-----|-----------------------|--------------------------|
|  |   | Diluted solution     | Time to take effect | Diluted solution                                      | Time to take effect | Sputum   |     | Stools |     | Urine |     |                       |                          |
|  |   |                      |                     |   |                     | %  | Hr. | %      | Hr. | %     | Hr. |                       |                          |
|  |   | %                    | Hr.                 | %   | Hr.                 | %  | Hr. | %      | Hr. | %     | Hr. |                       |                          |
| Phenol or phenol derivative                                    | Amocid                                      | 1                    | 12                  | 5   | 6                   | 5  | 4   | 5      | 6   | 5     | 2   | A                     | Lysoform                 |
|  | Gevisol                                     | 0,5                  | 12                  | 5   | 4                   | 5  | 4   | 5      | 6   | 5     | 2   | A                     | Schülke & Mayr           |
|  | Helipur                                     |                      |                     | 6   | 4                   | 6  | 4   | 6      | 6   | 6     | 2   | A                     | B. Braun                 |
|  | m-cresylic soap solution (DAB 6)            | 1                    | 12                  | 5   | 4                   |  |     |        |     |       |     | A                     |                          |
|  | Phenol                                      | 1                    | 12                  | 3   | 2                   |  |     |        |     |       |     | A                     |                          |
| Chlorine, organic or inorganic substances with active chlorine | Chloramin-T DAB 9                           | 1,5                  | 12                  | 2,5   | 2                   | 5  | 4   |        |     |       |     | A <sup>1</sup> B      |                          |
|  | Clorina                                     | 1,5                  | 12                  | 2,5   | 2                   | 5  | 4   |        |     |       |     | A <sup>1</sup> B      | Lysoform                 |
|  | Trichlorol                                  | 2                    | 12                  | 3   | 2                   | 6  | 4   |        |     |       |     | A <sup>1</sup> B      | Lysoform                 |
| Per combinations   | Apesin AP100 <sup>2</sup>                   |                      |                     | 4   | 4                   |  |     |        |     |       |     | AB                    | Tana PROFESSIONAL        |
|  | Dismozon pur <sup>2</sup>                   |                      |                     | 4   | 1                   |  |     |        |     |       |     | AB                    | Bode Chemie              |
|  | Perform <sup>2</sup>                        |                      |                     | 3   | 4                   |  |     |        |     |       |     | AB                    | Schülke & Mayr           |
|  | Wofesterii <sup>2</sup>                     |                      |                     | 2   | 4                   |  |     |        |     |       |     | AB                    | Kesla Pharma             |
| Formaldehyde and/or other aldehydes or derivatives             | Aldasan 2000                                |                      |                     | 4   | 4                   |  |     |        |     |       |     | AB                    | Lysoform                 |
|  | Antifect FD 10                              |                      |                     | 3   | 4                   |  |     |        |     |       |     | AB                    | Schülke & Mayr           |
|  | Antiseptica surface disinfection 7          |                      |                     | 3   | 6                   |  |     |        |     |       |     | AB                    | Antiseptica              |
|  | Apesin AP30                                 |                      |                     | 5   | 4                   |  |     |        |     |       |     | A                     | Tana PROFESSIONAL        |
|  | Bacillocid special                          |                      |                     | 6   | 4                   |  |     |        |     |       |     | AB                    | Bode Chemie              |
|  | Buraton 10F                                 |                      |                     | 3   | 4                   |  |     |        |     |       |     | AB                    | Schülke & Mayr           |
|  | Desomed A 2000                              |                      |                     | 3   | 6                   |  |     |        |     |       |     | AB                    | Desomed                  |
|  | Hospital disinfectant cleaner               |                      |                     | 8   | 6                   |  |     |        |     |       |     | AB                    | Dreiturm                 |
|  | Desomed Perfekt                             |                      |                     | 7   | 4                   |  |     |        |     |       |     | AB*                   | Desomed                  |
|  | Formaldehyde-solution (DAB 10), (formaline) | 1,5                  | 12                  | 3   | 4                   |  |     |        |     |       |     | AB                    |                          |
|  | Incidin Perfekt                             | 1                    | 12                  | 3   | 4                   |  |     |        |     |       |     | AB                    | Ecolab                   |
|  | Incidin Plus                                |                      |                     | 8   | 6                   |  |     |        |     |       |     | A                     | Ecolab                   |
|  | Kohrsolin                                   | 2                    | 12                  | 3   | 4                   |  |     |        |     |       |     | AB                    | Bode Chemie              |
|  | Lysoform                                    | 4                    | 12                  | 5   | 6                   |  |     |        |     |       |     | AB                    | Lysoform                 |
|  | Lysoformin                                  | 3                    | 12                  | 5   | 6                   |  |     |        |     |       |     | AB                    | Lysoform                 |
|  | Lysoformin 2000                             |                      |                     | 4   | 6                   |  |     |        |     |       |     | AB                    | Lysoform                 |
|  | Melsept                                     | 2                    | 12                  | 4   | 6                   |  |     |        |     |       |     | AB                    | B. Braun                 |
|  | Melsitt                                     | 4                    | 12                  | 10  | 4                   |  |     |        |     |       |     | AB                    | B. Braun                 |
|  | Minutil                                     | 2                    | 12                  | 6   | 4                   |  |     |        |     |       |     | AB                    | Ecolab                   |
|  | Multidor                                    |                      |                     | 3   | 6                   |  |     |        |     |       |     | AB                    | Ecolab                   |
| Nüscosept  |   |                      | 5                   | 4   |                     |  |     |        |     |       | AB  | Dr. Nüsken Chemie     |                          |
| Optisept   |   |                      | 7                   | 4   |                     |  |     |        |     |       | AB* | Dr. Schumacher        |                          |
| Pursept-FD   |   |                      | 7                   | 4   |                     |  |     |        |     |       | AB* | Merz                  |                          |
| Ultrasol F   | 3   | 12                   | 5                   | 4   |                     |  |     |        |     |       | AB  | Fresenius Kabi        |                          |
| Amphoteric surfactants (amfotensiden)                          | Tensodur 103                                | 2                    | 12                  |   |                     |  |     |        |     |       |     | A                     | MFH Marienfelde          |
| Lye  | Lime-milk <sup>3</sup>                      |                      |                     |   |                     |  |     | 20     | 6   |       |     | A <sup>3</sup> B      |                          |

1 Not effective against myco-bacteria when service disinfecting, especially in the presence of blood.  
2 Not suitable for disinfecting blood-contaminated or porous surfaces (e.g. raw wood).  
3 Useless for tuberculosis; preparation of Lime-milk: 1 part dissolved lime (calcium hydroxide) + 3 parts water.  
\* Checked for effectiveness on viruses in accordance with checking methods of the RKI (Federal Health Reporting 38 (1995) 242).  
A: Suitable for killing vegetative bacterial germs including myco-bacteria as well as fungi, including fungal spores.  
B: Suitable for deactivating viruses.

Table 7: Disinfectants

Kindly consult your specialist dealer if you have queries on matters related to disinfection; he will gladly assist you.

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## ENGLISH

### CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

### APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

### EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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© N.V. Vermeiren N.V. 2011
### Vermeiren Nederland B.V.