

Spare Parts Catalogue

Version: 2016/05/20a EN





Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Information

Abbreviations	7
Frame colours & CPS colours	8

Chassis

Chassis	9
Swing away frame	10
Fixed front end 80 degree angle frame	12
Folding system	14

Seat

Seat	15
Seat upholstery, depth adjustable	16
Seat upholstery, basic	18
Seat upholstery, tension & depth adjustable	20

Backrest

Backrest type

Backrest type	22
Backrest attachment type	23
Backrest fixed height, fixed pushhandle	24
Backrest height adjustable 7°, without & rearsset pushhandle	25
Backrest height adjustable, Integrated pushhandle	26

Backrest Upholsteries

Backrest Upholsteries	27
Backrest upholstery fixed, for fixed & integrated pushhandle	28
Backrest upholstery adjustable, for fixed & integrated pushhandle	30
Backrest upholstery adjustable, for without & rearsset pushhandle	33
Backrest Upholsteries adjustable télescopique	35
Backrest upholstery MatrX	37
Mounting Hardware MatrX	38

Legrests

Legrests	39
Hanger 70°	40
Hanger 80°	41
Hanger 90°	42
Elevating legrest	43
Calf Pad	44
Amputee Legrest	45

Footrests

Footrests	46
Footrest fixed, swing away frame	47
Footrest fixed, fixed frame	49



Yes, you can.®

TABLE OF CONTENTS

Footrest fixed, high mounted tube, swing away frame	51
Footrest angle & depth adjustable, swing away frame	53
Footrest angle & depth adjustable, fixed frame	56
Footrest angle & depth adjustable, high mounted tube, swing away frame	58
One piece footplate for fixed frame	59

Siderests

Siderests	61
Side guard	62
Mudguard	63
Armrest n°1, Removable, height adjustable T-arm, desk armpad (280mm)	65
Hemi Dartex armpad multiposition standard	66
Armrest n°3, Swing away detachable & height adjustable, desk armpad (280mm)	67

Front wheels

Front wheels	69
Forks for swing away frame	70
Forks for fixed frame	72
Forks support for swing away frame	74
Forks support for fixed frame	75
Front wheels	76

Rear Wheels

Rear Wheels	77
Rear wheel 22"/24 " , cross spoked	78
Rear wheel Lightweight 22"/24"/25"	79
Rear wheel Composite 22"-24"	80
Rear wheel Spinergy 24"-25"	81
Rear wheel hub brake 22"-24"	82
Standard & hub brake wheels accessories	83
Lightweight wheels accessories	85
Composite wheels accessories	87
Spinergy wheels accessories	88
Spoke protector	89
Rear wheel support	90
Pump	91

Brakes

Brakes	92
Push Brakes	93
Pull Brakes	94
Attendant Hub Brake	95

Safety Options

Safety Options	96
Buckled belt	97
Buckled belt (UK)	98
Anti-tipper	99
Reflectors	100
Kit anti barrier	101



Yes, you can.[®]

TABLE OF CONTENTS


Seating Options

Seating Options	102
Cushion foam Nylon & anti-slip cover	103
Matrx Flo-Tech CONTOUR VISCO cushions	104
Matrx Flo-Tech IMAGE cushions	105
Matrx Flo-Tech LITE cushions	106
Matrx Flo-Tech XTRA cushions	107
Matrx VI Cushions	108
Matrx LIBRA Cushions	109
Clip to keep the chair folded	110
Heels strap	111
Calf strap	112
Folding backbrace	113

Other Options & Accessories

Other Options & Accessories	114
Cane holder	115
Tipping aid	116
Tool Kit	117
Case holder	118


Abbreviations

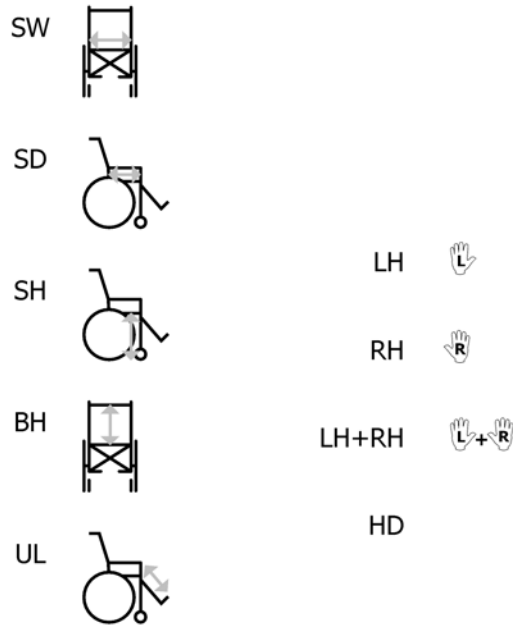

15P5133721

Pos.	Part number	Description	Valid from → until	Qty
-	SW	Seat width (2 digits in cm) (3 digits in mm)		1
-	SD	Seat depth (2 digits in cm) (3 digits in mm)		1
-	SH	Seat height (2 digits in cm) (3 digits in mm)		1
-	BH	Backrest height (2 digits in cm) (3 digits in mm)		1
-	UL	Knee-to-heel height (2 digits in cm) (3 digits in mm)		1
-	LH	Left side (sitting in the wheelchair)		1
-	RH	Right side (sitting in the wheelchair)		1
-	LH+RH	An assembly of a left and a right part, referred to as "pair"		1
-	HD	Heavy duty version		1




Frame colours & CPS colours

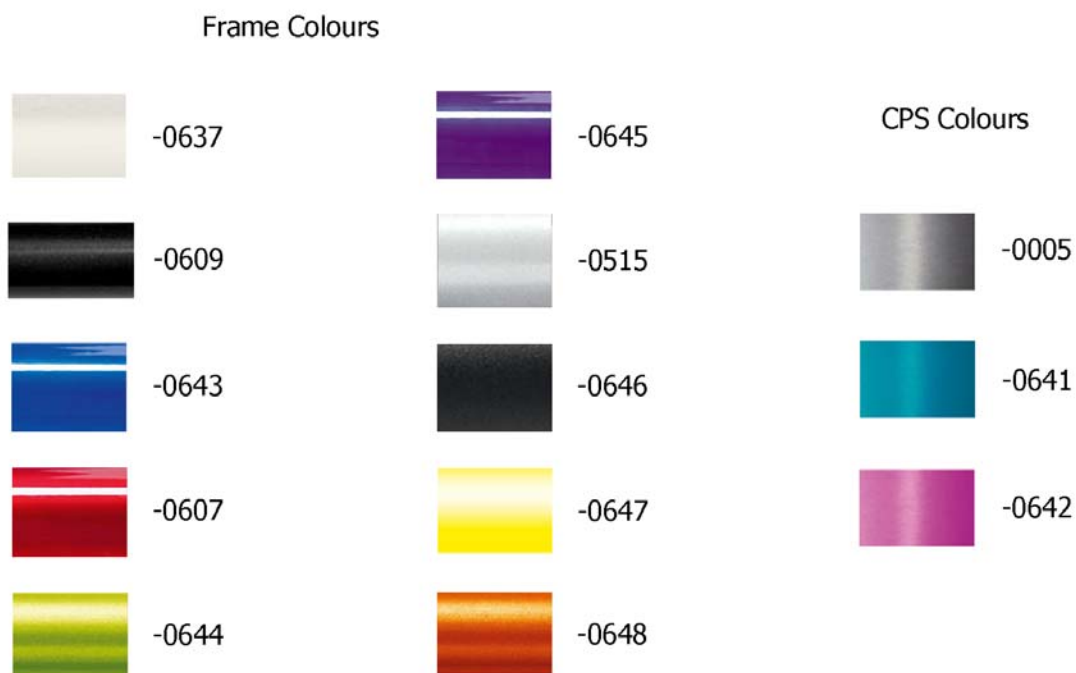

ISPS:1602217

Pos.	Part number	Description	Valid from → until	Qty
-	-0637	Colour code for frame, Pearl White		1
-	-0609	Colour code for frame, Anthracite Black		1
-	-0643	Colour code for frame, Pop Blue		1
-	-0607	Colour code for frame, Dark Red		1
-	-0644	Colour code for frame, Electric Green		1
-	-0645	Colour code for frame, Hypnotic Purple		1
-	-0515	Colour code for frame, Pearl Grey		1
-	-0646	Colour code for frame, Matt Black		1
-	-0647	Colour code for frame, Intense Yellow		1
-	-0648	Colour code for frame, Vibrating Orange		1
-	-0005	Colour code for CPS, without accent		1
-	-0641	Colour code for CPS, Blue		1
-	-0642	Colour code for CPS, Purple		1


Chassis

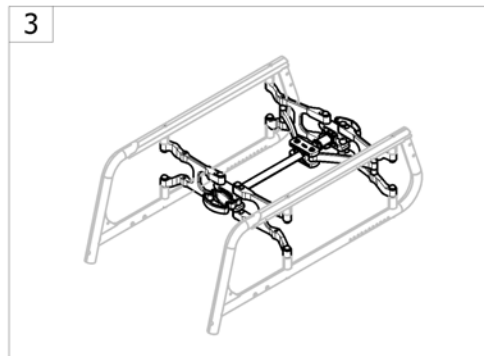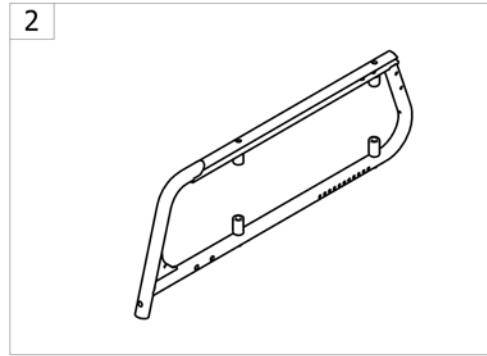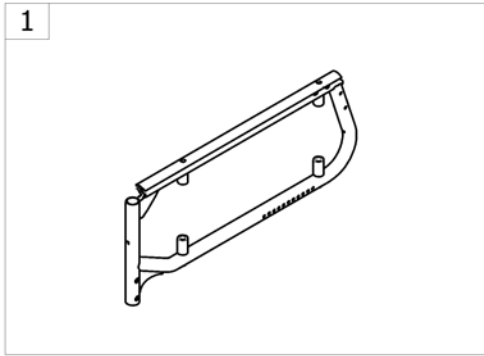

->SPCACTIONS10

Pos.	Part number	Description	Valid from → until	Qty
1	-	Swing away frame <i>[SJ_0200]</i>		1
2	-	Fixed front end 80 degree angle frame <i>[SJ_0210]</i>		1
3	-	Folding system		1

Swing away frame

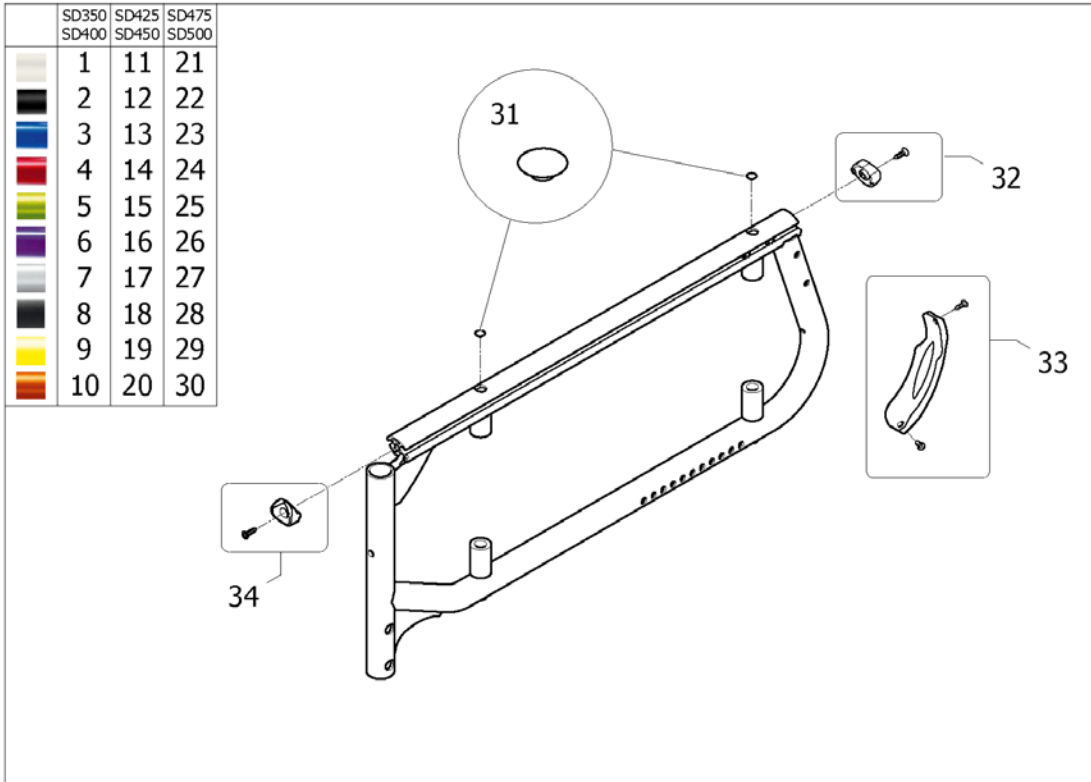


ISPS:1601836

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582901-0637	Kit swing away frames, SD350/375/400mm, Pearl White (pair) [SJ_0200+SJ_6000]		1
2	SP1582901-0609	Kit swing away frames, SD350/375/400mm, Anthracite (pair) [SJ_0200+SJ_6010]		1
3	SP1582901-0643	Kit swing away frames, SD350/375/400mm, Pop Blue (pair) [SJ_0200+SJ_6020]		1
4	SP1582901-0607	Kit swing away frames, SD350/375/400mm, Dark Red (pair) [SJ_0200+SJ_6030]		1
5	SP1582901-0644	Kit swing away frames, SD350/375/400mm, Electric Green (pair) [SJ_0200+SJ_6040]		1
6	SP1582901-0645	Kit swing away frames, SD350/375/400mm, Hypnotic Purple (pair) [SJ_0200+SJ_6050]		1
7	SP1582901-0515	Kit swing away frames, SD350/375/400mm, Pearl Grey (pair) [SJ_0200+SJ_6060]		1
8	SP1582901-0646	Kit swing away frames, SD350/375/400mm, Matt Black (pair) [SJ_0200+SJ_6070]		1
9	SP1582901-0647	Kit swing away frames, SD350/375/400mm, Intense Yellow (pair) [SJ_0200+SJ_6080]		1
10	SP1582901-0648	Kit swing away frames, SD350/375/400mm, Vibrating Orange (pair) [SJ_0200+SJ_6090]		1
11	SP1582902-0637	Kit swing away frames, SD425/450mm, Pearl White (pair) [SJ_0200+SJ_6000]		1
12	SP1582902-0609	Kit swing away frames, SD425/450mm, Anthracite (pair) [SJ_0200+SJ_6010]		1
13	SP1582902-0643	Kit swing away frames, SD425/450mm, Pop Blue (pair) [SJ_0200+SJ_6020]		1
14	SP1582902-0607	Kit swing away frames, SD425/450mm, Dark Red (pair) [SJ_0200+SJ_6030]		1
15	SP1582902-0644	Kit swing away frames, SD425/450mm, Electric Green (pair) [SJ_0200+SJ_6040]		1
16	SP1582902-0645	Kit swing away frames, SD425/450mm, Hypnotic Purple (pair) [SJ_0200+SJ_6050]		1
17	SP1582902-0515	Kit swing away frames, SD425/450mm, Pearl Grey (pair) [SJ_0200+SJ_6060]		1
18	SP1582902-0646	Kit swing away frames, SD425/450mm, Matt Black (pair) [SJ_0200+SJ_6070]		1

Swing away frame

Pos.	Part number	Description	Valid from → until	Qty
19	SP1582902-0647	Kit swing away frames, SD425/450mm, Intense Yellow (pair) <i>[SJ_0200+SJ_6080]</i>		1
20	SP1582902-0648	Kit swing away frames, SD425/450mm, Vibrating Orange (pair) <i>[SJ_0200+SJ_6090]</i>		1
21	SP1582903-0637	Kit swing away frames, SD475/500mm, Pearl White (pair) <i>[SJ_0200+SJ_6000]</i>		1
22	SP1582903-0609	Kit swing away frames, SD475/500mm, Anthracite (pair) <i>[SJ_0200+SJ_6010]</i>		1
23	SP1582903-0643	Kit swing away frames, SD475/500mm, Pop Blue (pair) <i>[SJ_0200+SJ_6020]</i>		1
24	SP1582903-0607	Kit swing away frames, SD475/500mm, Dark Red (pair) <i>[SJ_0200+SJ_6030]</i>		1
25	SP1582903-0644	Kit swing away frames, SD475/500mm, Electric Green (pair) <i>[SJ_0200+SJ_6040]</i>		1
26	SP1582903-0645	Kit swing away frames, SD475/500mm, Hypnotic Purple (pair) <i>[SJ_0200+SJ_6050]</i>		1
27	SP1582903-0515	Kit swing away frames, SD475/500mm, Pearl Grey (pair) <i>[SJ_0200+SJ_6060]</i>		1
28	SP1582903-0646	Kit swing away frames, SD475/500mm, Matt Black (pair) <i>[SJ_0200+SJ_6070]</i>		1
29	SP1582903-0647	Kit swing away frames, SD475/500mm, Intense Yellow (pair) <i>[SJ_0200+SJ_6080]</i>		1
30	SP1582903-0648	Kit swing away frames, SD475/500mm, Vibrating Orange (pair) <i>[SJ_0200+SJ_6090]</i>		1
31	SP1590794	Kit end cap (pair)		2
32	SP1589683	Kit rear cap reflector for frame		2
33	SP1589684	Kit frame plastic protector		2
34	SP1589692	Kit front cap for frame		2
35	SP1584209	Sticker MyOn HC		2

Fixed front end 80 degree angle frame

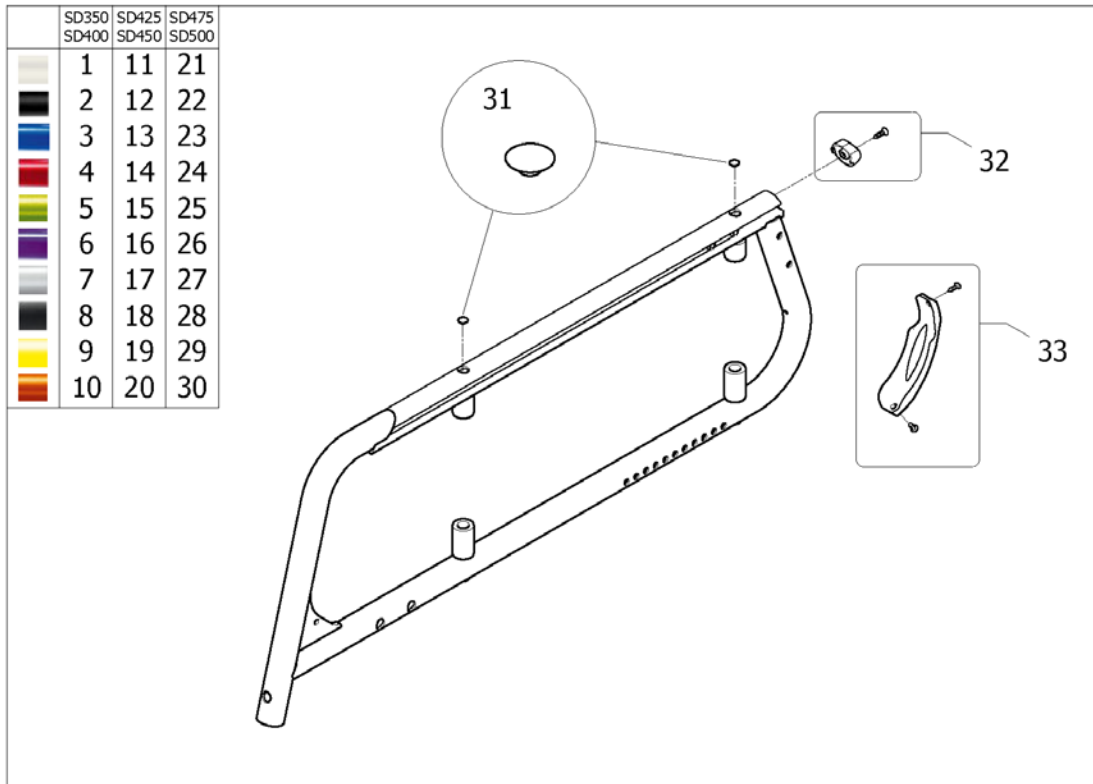


ISPS I601837

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582904-0637	Kit fixed frames, SD350/375/400, Pearl White (pair) <i>[SJ_0210+SJ_6000]</i>		1
2	SP1582904-0609	Kit fixed frames, SD350/375/400, Anthracite (pair) <i>[SJ_0210+SJ_6010]</i>		1
3	SP1582904-0643	Kit fixed frames, SD350/375/400, Pop Blue (pair) <i>[SJ_0210+SJ_6020]</i>		1
4	SP1582904-0607	Kit fixed frames, SD350/375/400, Dark Red (pair) <i>[SJ_0210+SJ_6030]</i>		1
5	SP1582904-0644	Kit fixed frames, SD350/375/400, Electric Green (pair) <i>[SJ_0210+SJ_6040]</i>		1
6	SP1582904-0645	Kit fixed frames, SD350/375/400, Hypnotic Purple (pair) <i>[SJ_0210+SJ_6050]</i>		1
7	SP1582904-0515	Kit fixed frames, SD350/375/400, Pearl Grey (pair) <i>[SJ_0210+SJ_6060]</i>		1
8	SP1582904-0646	Kit fixed frames, SD350/375/400, Matt Black (pair) <i>[SJ_0210+SJ_6070]</i>		1
9	SP1582904-0647	Kit fixed frames, SD350/375/400, Intense Yellow (pair) <i>[SJ_0210+SJ_6080]</i>		1
10	SP1582904-0648	Kit fixed frames, SD350/375/400, Vibrating Orange (pair) <i>[SJ_0210+SJ_6090]</i>		1
11	SP1582905-0637	Kit fixed frames, SD425/450, Pearl White (pair) <i>[SJ_0210+SJ_6000]</i>		1
12	SP1582905-0609	Kit fixed frames, SD425/450, Anthracite (pair) <i>[SJ_0210+SJ_6010]</i>		1
13	SP1582905-0643	Kit fixed frames, SD425/450, Pop Blue (pair) <i>[SJ_0210+SJ_6020]</i>		1
14	SP1582905-0607	Kit fixed frames, SD425/450, Dark Red (pair) <i>[SJ_0210+SJ_6030]</i>		1
15	SP1582905-0644	Kit fixed frames, SD425/450, Electric Green (pair) <i>[SJ_0210+SJ_6040]</i>		1
16	SP1582905-0645	Kit fixed frames, SD425/450, Hypnotic Purple (pair) <i>[SJ_0210+SJ_6050]</i>		1
17	SP1582905-0515	Kit fixed frames, SD425/450, Pearl Grey (pair) <i>[SJ_0210+SJ_6060]</i>		1
18	SP1582905-0646	Kit fixed frames, SD425/450, Matt Black (pair) <i>[SJ_0210+SJ_6070]</i>		1

Fixed front end 80 degree angle frame




Pos.	Part number	Description	Valid from → until	Qty
19	SP1582905-0647	Kit fixed frames, SD425/450, Intense Yellow (pair) <i>[SJ_0210+SJ_6080]</i>		1
20	SP1582905-0648	Kit fixed frames, SD425/450, Vibrating Orange (pair) <i>[SJ_0210+SJ_6090]</i>		1
21	SP1582906-0637	Kit fixed frames, SD475/500, Pearl White (pair) <i>[SJ_0210+SJ_6000]</i>		1
22	SP1582906-0609	Kit fixed frames, SD475/500, Anthracite (pair) <i>[SJ_0210+SJ_6010]</i>		1
23	SP1582906-0643	Kit fixed frames, SD475/500, Pop Blue (pair) <i>[SJ_0210+SJ_6020]</i>		1
24	SP1582906-0607	Kit fixed frames, SD475/500, Dark Red (pair) <i>[SJ_0210+SJ_6030]</i>		1
25	SP1582906-0644	Kit fixed frames, SD475/500, Electric Green (pair) <i>[SJ_0210+SJ_6040]</i>		1
26	SP1582906-0645	Kit fixed frames, SD475/500, Hypnotic Purple (pair) <i>[SJ_0210+SJ_6050]</i>		1
27	SP1582906-0515	Kit fixed frames, SD475/500, Pearl Grey (pair) <i>[SJ_0210+SJ_6060]</i>		1
28	SP1582906-0646	Kit fixed frames, SD475/500, Matt Black (pair) <i>[SJ_0210+SJ_6070]</i>		1
29	SP1582906-0647	Kit fixed frames, SD475/500, Intense Yellow (pair) <i>[SJ_0210+SJ_6080]</i>		1
30	SP1582906-0648	Kit fixed frames, SD475/500, Vibrating Orange (pair) <i>[SJ_0210+SJ_6090]</i>		1
31	SP1590794	Kit end cap (pair)		2
32	SP1589683	Kit rear cap reflector for frame		2
33	SP1589684	Kit frame plastic protector		2
34	SP1584209	Sticker MyOn HC		2

Folding system

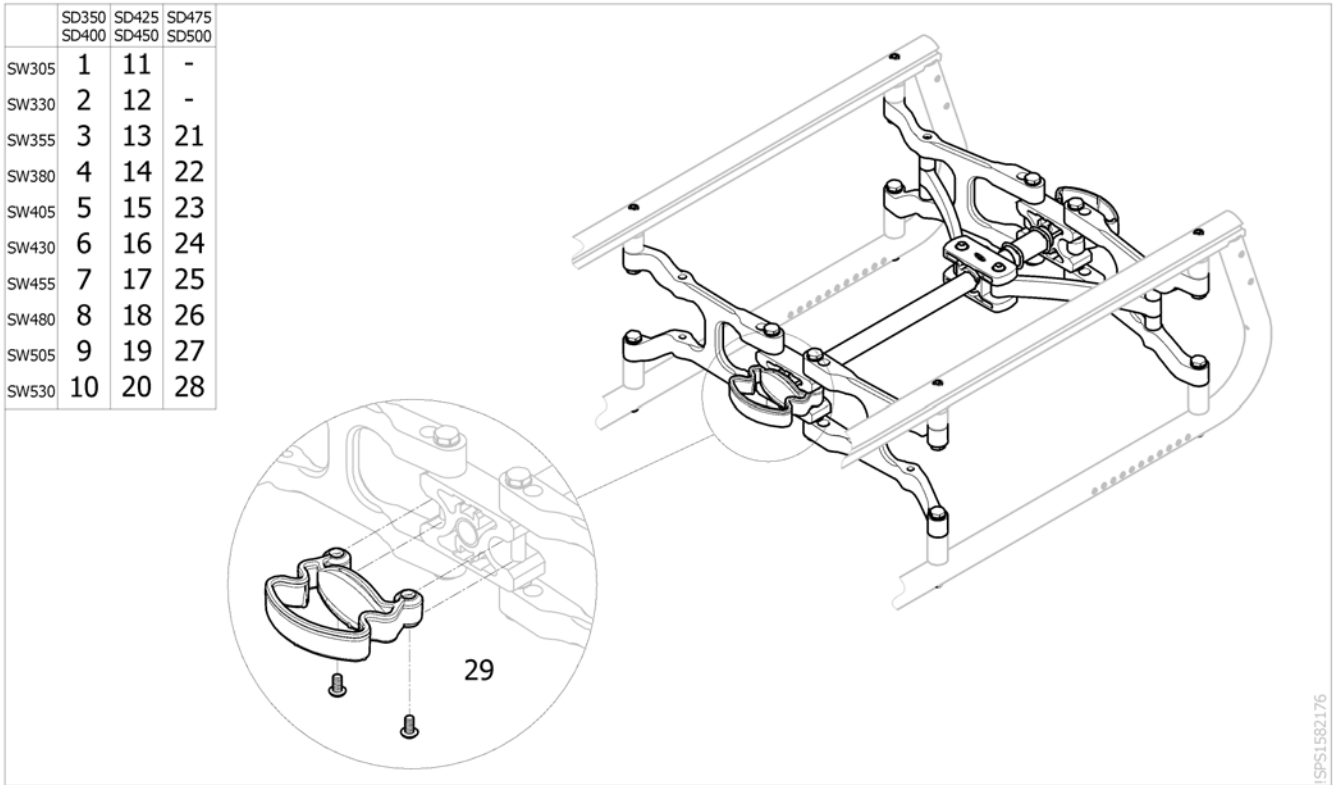

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583191	Folding system, SW305 mm x SD350/375/400 mm		1
2	SP1583193	Folding system, SW330 mm x SD350/375/400 mm		1
3	SP1583195	Folding system, SW355 mm x SD350/375/400 mm		1
4	SP1583198	Folding system, SW380 mm x SD350/375/400 mm		1
5	SP1583253	Folding system, SW405 mm x SD350/375/400 mm		1
6	SP1583262	Folding system, SW430 mm x SD350/375/400 mm		1
7	SP1583265	Folding system, SW455 mm x SD350/375/400 mm		1
8	SP1583268	Folding system, SW480 mm x SD350/375/400 mm		1
9	SP1589975	Folding system, SW505 mm x SD400 mm		1
10	SP1589976	Folding system, SW530 mm x SD400 mm		1
11	SP1583192	Folding system, SW305 mm x SD425/450 mm		1
12	SP1583194	Folding system, SW330 mm x SD425/450 mm		1
13	SP1583196	Folding system, SW355 mm x SD425/450 mm		1
14	SP1583199	Folding system, SW380 mm x SD425/450 mm		1
15	SP1583260	Folding System, SW405 mm x SD425/450 mm		1
16	SP1583263	Folding system, SW430 mm x SD425/450 mm		1
17	SP1583266	Folding system, SW455 mm x SD425/450 mm		1
18	SP1583269	Folding system, SW425 mm x SD425/450 mm		1
19	SP1583271	Folding system, SW505 mm x SD425/450 mm		1
20	SP1583313	Folding system, SW530 mm x SD425/450 mm		1
21	SP1583197	Folding system, SW355 mm x SD475/500 mm		1
22	SP1583252	Folding system, SW380 mm x SD475/500 mm		1
23	SP1583261	Folding system, SW475 mm x SD475/500 mm		1
24	SP1583264	Folding system, SW430 mm x SD475/500 mm		1
25	SP1583267	Folding system, SW455 mm x SD475/500 mm		1
26	SP1583270	Folding system, SW480 mm x SD475/500 mm		1
27	SP1583272	Folding system, SW505 mm x SD475/500 mm		1
28	SP1583314	Folding system, SW530 mm x SD475/500 mm		1
29	SP1589803	Kit handle + screw for folding system		1

Seat

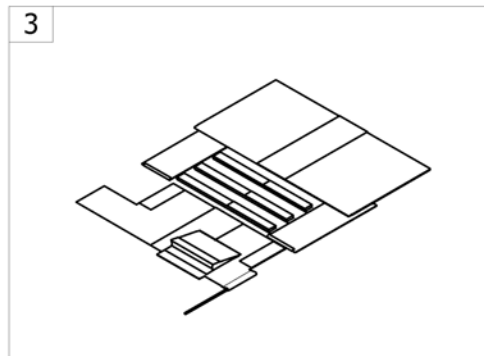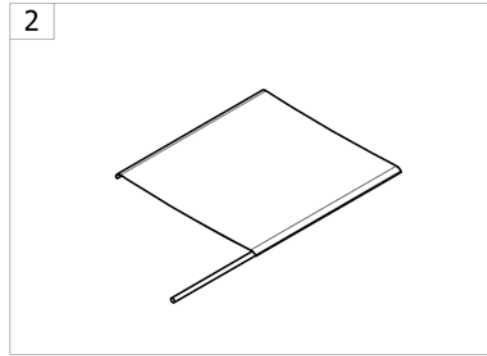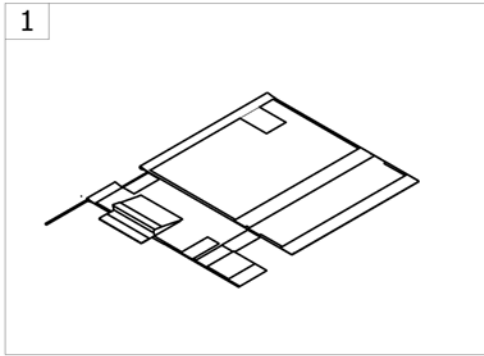


-<SPACTION520

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat upholstery, depth adjustable <i>[SJ_1500]</i>		1
2	-	Seat upholstery, basic <i>[SJ_1510]</i>		1
3	-	Seat upholstery, tension & depth adjustable <i>[SJ_1520]</i>		1

Seat upholstery, depth adjustable

	SD350 SD400	SD400 SD450	SD450 SD500
SW305	1	2	-
SW330	1	2	-
SW355	3	4	5
SW380	3	4	5
SW405	6	7	8
SW430	6	7	8
SW455	9	10	11
SW480	9	10	11
SW505	-	12	13
SW530	-	12	13

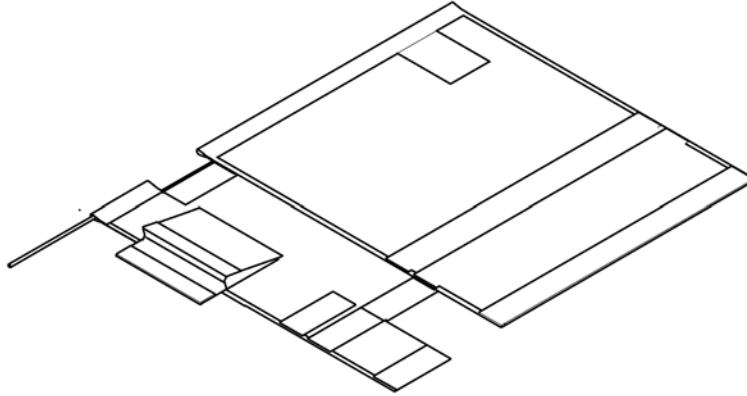

SPS1582178 ~ sps1577822


Pos.	Part number	Description	Valid from → until	Qty
1	SP1582406	Seat upholstery depth adjustable, complete, SW305/330mm SD350/400mm <i>[SJ_1500]</i>		1
2	SP1582407	Seat upholstery depth adjustable, complete, SW305/330mm SD400/450mm <i>[SJ_1500]</i>		1
3	SP1582408	Seat upholstery depth adjustable, complete, SW355/380mm SD350/400mm <i>[SJ_1500]</i>		1
4	SP1582433	Seat upholstery depth adjustable, complete, SW355/380mm SD400/450mm <i>[SJ_1500]</i>		1
5	SP1582434	Seat upholstery depth adjustable, complete, SW355/380mm SD450/500mm <i>[SJ_1500]</i>		1
6	SP1582435	Seat upholstery depth adjustable, complete, SW405/430mm SD350/400mm <i>[SJ_1500]</i>		1
7	SP1582589	Seat upholstery depth adjustable, complete, SW405/430mm SD400/450mm <i>[SJ_1500]</i>		1
8	SP1582590	Seat upholstery depth adjustable, complete, SW405/430mm SD450/500mm <i>[SJ_1500]</i>		1
9	SP1582591	Seat upholstery depth adjustable, complete, SW455/480mm SD350/400mm <i>[SJ_1500]</i>		1
10	SP1582592	Seat upholstery depth adjustable, complete, SW455/480mm SD400/450mm <i>[SJ_1500]</i>		1
11	SP1582593	Seat upholstery depth adjustable, complete, SW455/480mm SD450/500mm <i>[SJ_1500]</i>		1
12	SP1582594	Seat upholstery depth adjustable, complete, SW505/530mm SD400/450mm <i>[SJ_1500]</i>		1

Seat upholstery, depth adjustable

Pos.	Part number	Description	Valid from → until	Qty
13	SP1582595	Seat upholstery depth adjustable, complete, SW505/530mm SD450/500mm <i>[SJ_1500]</i>		1

Seat upholstery, basic

	SD350	SD375	SD400	SD425	SD450	SD475	SD500
SW305	1	2	3	4	5	-	-
SW330	1	2	3	4	5	-	-
SW355	6	7	8	9	10	11	12
SW380	6	7	8	9	10	11	12
SW405	13	14	15	16	17	18	19
SW430	13	14	15	16	17	18	19
SW455	20	21	22	23	24	25	26
SW480	20	21	22	23	24	25	26
SW505	-	-	27	28	29	30	31
SW530	-	-	27	28	29	30	31

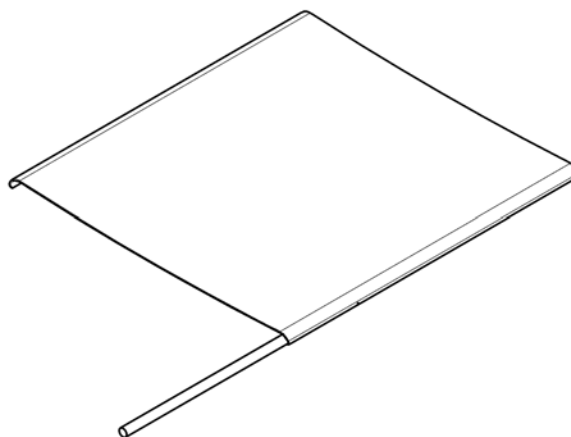


ISPS1582177

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582200	Basic complete seat upholstery, SW305/330mm, SD350mm <i>[SJ_1510]</i>		1
2	SP1582201	Basic complete seat upholstery, SW305/330mm, SD375mm <i>[SJ_1510]</i>		1
3	SP1582202	Basic complete seat upholstery, SW305/330mm, SD400mm <i>[SJ_1510]</i>		1
4	SP1582203	Basic complete seat upholstery, SW305/330mm, SD425mm <i>[SJ_1510]</i>		1
5	SP1582204	Basic complete seat upholstery, SW305/330mm, SD450mm <i>[SJ_1510]</i>		1
6	SP1582205	Basic complete seat upholstery, SW355/380mm, SD350mm <i>[SJ_1510]</i>		1
7	SP1582220	Basic complete seat upholstery, SW355/380mm, SD375mm <i>[SJ_1510]</i>		1
8	SP1582221	Basic complete seat upholstery, SW355/380mm, SD400mm <i>[SJ_1510]</i>		1
9	SP1582222	Basic complete seat upholstery, SW355/380mm, SD425mm <i>[SJ_1510]</i>		1
10	SP1582223	Basic complete seat upholstery, SW355/380mm, SD450mm <i>[SJ_1510]</i>		1
11	SP1582224	Basic complete seat upholstery, SW355/380mm, SD475mm <i>[SJ_1510]</i>		1
12	SP1582225	Basic complete seat upholstery, SW355/380mm, SD500mm <i>[SJ_1510]</i>		1
13	SP1582226	Basic complete seat upholstery, SW405/430mm, SD355mm <i>[SJ_1510]</i>		1
14	SP1582227	Basic complete seat upholstery, SW405/430mm, SD375mm <i>[SJ_1510]</i>		1
15	SP1582294	Basic complete seat upholstery, SW405/430mm, SD400mm <i>[SJ_1510]</i>		1
16	SP1582295	Basic complete seat upholstery, SW405/430mm, SD425mm <i>[SJ_1510]</i>		1
17	SP1582296	Basic complete seat upholstery, SW405/430mm, SD450mm <i>[SJ_1510]</i>		1
18	SP1582297	Basic complete seat upholstery, SW405/430mm, SD475mm <i>[SJ_1510]</i>		1

Pos.	Part number	Description	Valid from → until	Qty
19	SP1582298	Basic complete seat upholstery, SW405/430mm, SD500mm <i>[SJ_1510]</i>		1
20	SP1582299	Basic complete seat upholstery, SW455/480mm, SD355mm <i>[SJ_1510]</i>		1
21	SP1582300	Basic complete seat upholstery, SW455/480mm, SD375mm <i>[SJ_1510]</i>		1
22	SP1582301	Basic complete seat upholstery, SW455/480mm, SD400mm <i>[SJ_1510]</i>		1
23	SP1582376	Basic complete seat upholstery, SW455/480mm, SD425mm <i>[SJ_1510]</i>		1
24	SP1582377	Basic complete seat upholstery, SW455/480mm, SD455mm <i>[SJ_1510]</i>		1
25	SP1582378	Basic complete seat upholstery, SW455/480mm, SD475mm <i>[SJ_1510]</i>		1
26	SP1582379	Basic complete seat upholstery, SW455/480mm, SD500mm <i>[SJ_1510]</i>		1
27	SP1589983	Basic complete seat Upholstery, SW505/530mm, SD400mm <i>[SJ_1510]</i>		1
28	SP1582380	Basic complete seat upholstery, SW505/530mm, SD425mm <i>[SJ_1510]</i>		1
29	SP1582381	Basic complete seat upholstery, SW505/530mm, SD450mm <i>[SJ_1510]</i>		1
30	SP1582382	Basic complete seat upholstery, SW505/530mm, SD475mm <i>[SJ_1510]</i>		1
31	SP1582405	Basic complete seat upholstery, SW505/530mm, SD500mm <i>[SJ_1510]</i>		1

Seat upholstery, tension & depth adjustable

	SD350 SD400	SD400 SD450	SD450 SD500
SW305	1	2	-
SW330	1	2	-
SW355	3	4	5
SW380	3	4	5
SW405	6	7	8
SW430	6	7	8
SW455	9	10	11
SW480	9	10	11
SW505	-	12	13
SW530	-	12	13

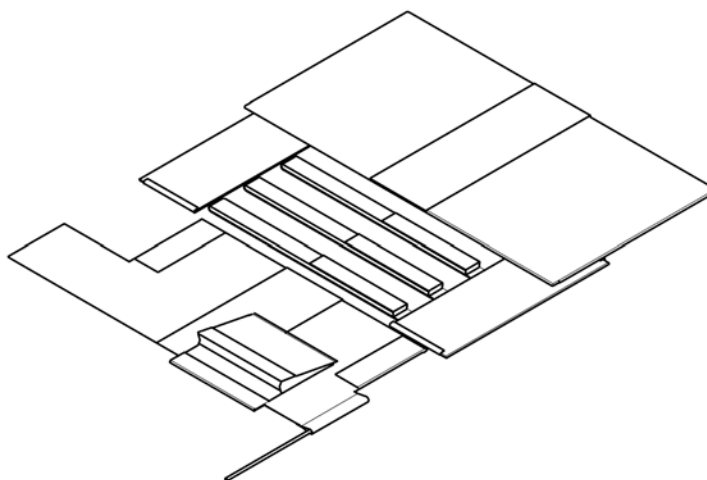





ISPS1582179 - sps1577822

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582596	Seat upholstery tension adjustable, complete, SW305/330mm SD350/400mm <i>[SJ_1520]</i>		1
2	SP1582597	Seat upholstery tension adjustable, complete, SW305/330mm SD400/450mm <i>[SJ_1520]</i>		1
3	SP1582598	Seat upholstery tension adjustable, complete, SW355/380mm SD350/400mm <i>[SJ_1520]</i>		1
4	SP1582599	Seat upholstery tension adjustable, complete, SW355/380mm SD400/450mm <i>[SJ_1520]</i>		1
5	SP1582600	Seat upholstery tension adjustable, complete, SW355/380mm SD450/500mm <i>[SJ_1520]</i>		1
6	SP1582601	Seat upholstery tension adjustable, complete, SW405/430mm SD350/400mm <i>[SJ_1520]</i>		1
7	SP1582773	Seat upholstery tension adjustable, complete, SW405/430mm SD400/450mm <i>[SJ_1520]</i>		1
8	SP1582774	Seat upholstery tension adjustable, complete, SW405/430mm SD450/500mm <i>[SJ_1520]</i>		1
9	SP1582775	Seat upholstery tension adjustable, complete, SW455/480mm SD350/400mm <i>[SJ_1520]</i>		1
10	SP1582776	Seat upholstery tension adjustable, complete, SW455/480mm SD400/450mm <i>[SJ_1520]</i>		1
11	SP1582777	Seat upholstery tension adjustable, complete, SW455/480mm SD450/500mm <i>[SJ_1520]</i>		1
12	SP1582778	Seat upholstery tension adjustable, complete, SW505/530mm SD400/450mm <i>[SJ_1520]</i>		1

Seat upholstery, tension & depth adjustable


Pos.	Part number	Description	Valid from → until	Qty
13	SP1582779	Seat upholstery tension adjustable, complete, SW505/530mm SD450/500mm <i>[SJ_1520]</i>		1

Backrest type

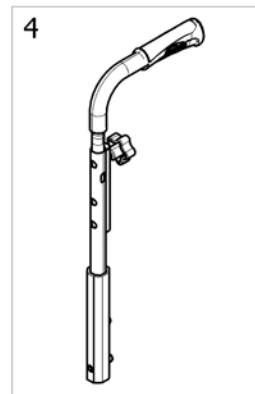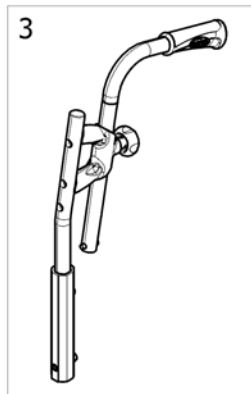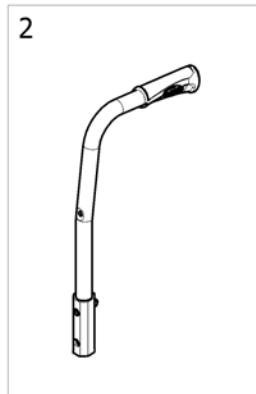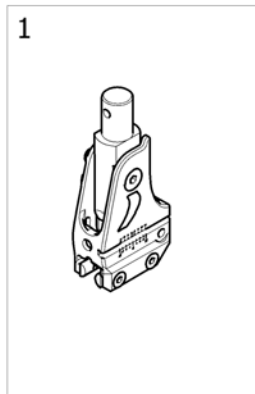

ISPS1601948 ~ <SP>ACTIONS


Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest attachment type <i>[SJ_2000/SJ_2010/SJ_2020]</i>		1
2	-	Backrest fixed height, fixed pushhandle <i>[SJ_2100/SJ_2103/SJ_2106/SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126]</i>		1
3	-	Backrest height adjustable 7°, without & rearsset pushhandle <i>[SJ_2300/SJ_2310/SJ_2400/SJ_2410]</i>		1
4	-	Backrest height adjustable, Integrated pushhandle <i>[SJ_2300/SJ_2310/SJ_2420]</i>		1

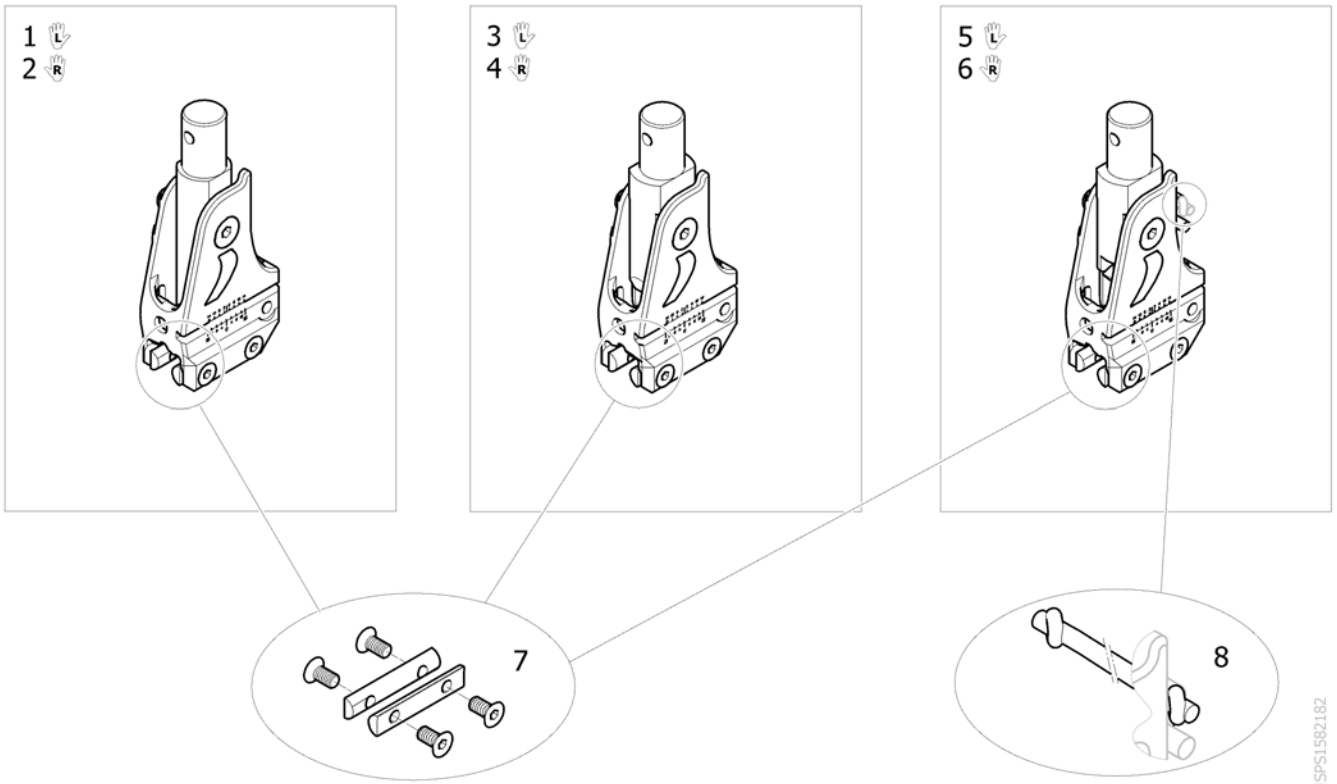

ISPS1582182

Pos.	Part number	Description	Valid from → until	Qty
1	SP1580830	Fixed angle backrest attachment Lh <i>[SJ_2000] ONLY FOR EUROPE</i>		1
2	SP1580829	Fixed angle backrest attachment Rh <i>[SJ_2000] ONLY FOR EUROPE</i>		1
3	SP1580832	Adjustable angle backrest attachment Lh <i>[SJ_2010] ONLY FOR EUROPE</i>		1
4	SP1580831	Adjustable angle backrest attachment Rh <i>[SJ_2010] ONLY FOR EUROPE</i>		1
5	SP1583838	Adjustable angle & fold down backrest attachment Lh <i>[SJ_2020]</i>		1
6	SP1583837	Adjustable angle & fold down backrest attachment Rh <i>[SJ_2020]</i>		1
7	SP1589804	Kit fixation for backrest attachment		2
8	SP1601900	Cord for backbrace & angle/fold down backrest attachment <i>Replace SP1554427</i>		1


Backrest fixed height, fixed pushhandle

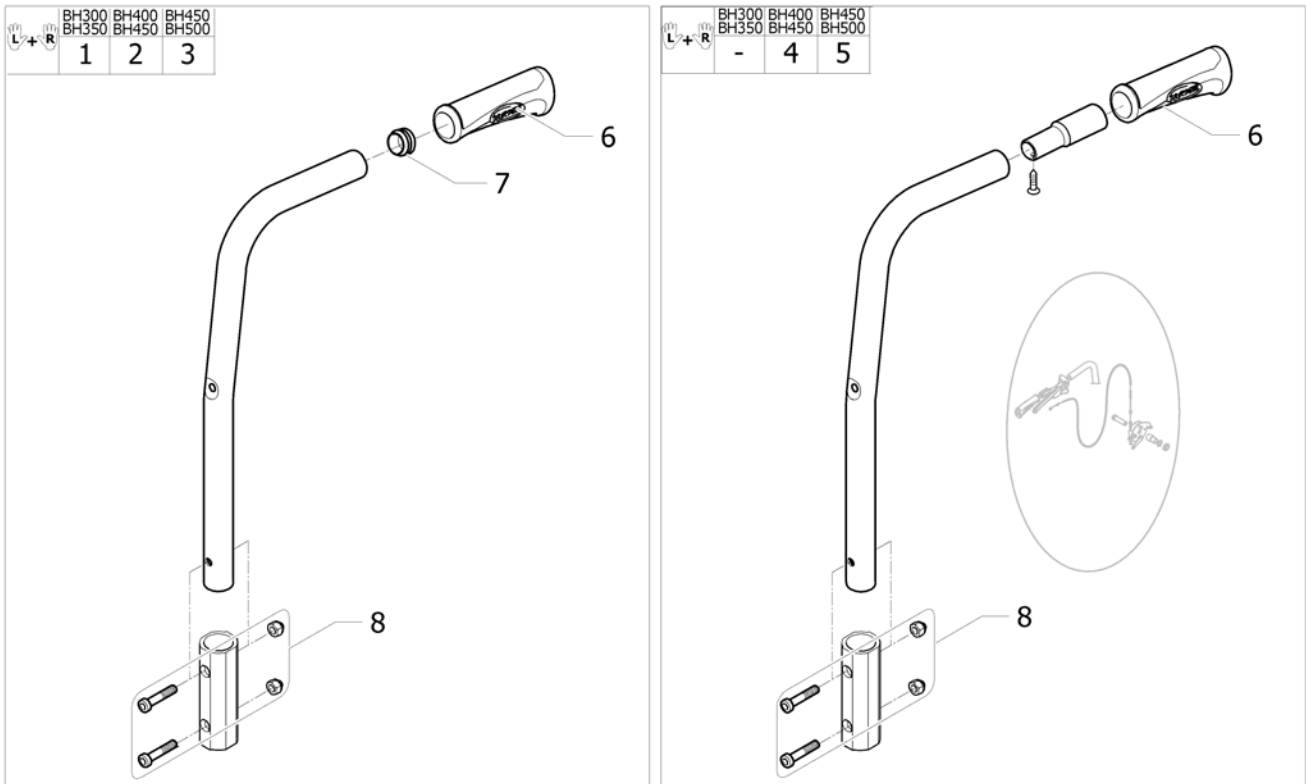

ISPS1582183 ~ sps1582183

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584782	Backrest fixed, low, height 300/350mm + pushhandle [SJ_2100/SJ_2103/SJ_2106]		1
2	SP1584783	Backrest fixed, medium, height 400/450mm + pushhandle [SJ_2110/SJ_2113/SJ_2116]		1
3	SP1584784	Backrest fixed, high, height 450/500mm + pushhandle [SJ_2120/SJ_2123/SJ_2126]		1
4	SP1601901	Backrest fixed with insert, medium, height 400/450mm + pushhandle [SJ_2110/SJ_2113/SJ_2116+SJ_8160]		1
5	SP1601902	Backrest fixed with insert, medium, height 450/500mm + pushhandle [SJ_2120/SJ_2123/SJ_2126+SJ_8160]		1
6	1435420	Handle Invacare for backrest Ø20mm		2
7	1418112	Crossbrace back cap Ø22x15mm		2
8	SP1589826	Kit fixation for fixed backrest		2


Backrest height adjustable 7°, without & rearsset pushhandle

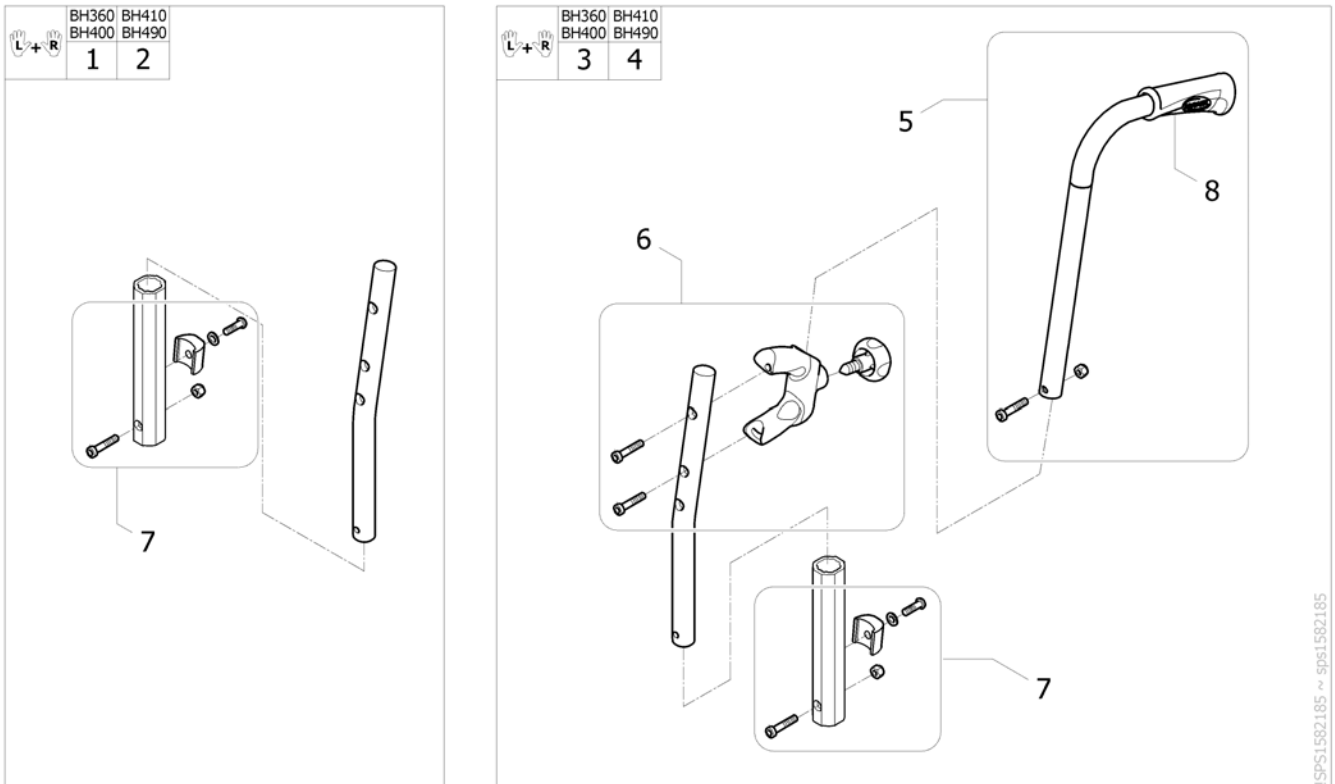

SPS1582185 - SPS1582185






Pos.	Part number	Description	Valid from → until	Qty
1	SP1584776	Backrest 360-400mm, height adjustable, without pushhandle <i>[SJ_2300+SJ_2400]</i>		1
2	SP1584777	Backrest 410-490mm, height adjustable, without pushhandle <i>[SJ_2310+SJ_2400]</i>		1
3	SP1584747	Backrest 360-400mm, height adjustable, rearsset pushhandle <i>[SJ_2300+SJ_2410]</i>		1
4	SP1584764	Backrest 410-490mm, height adjustable, rearsset pushhandle <i>[SJ_2310+SJ_2410]</i>		1
5	1450574	Kit rearsset pushhandle tube		2
6	1451818	Kit rearsset pushhandle support		2
7	SP1589825	Kit adjustable backrest tube fixation		2
8	1435420	Handle Invacare for backrest Ø20mm		2

Backrest height adjustable, Integrated pushhandle

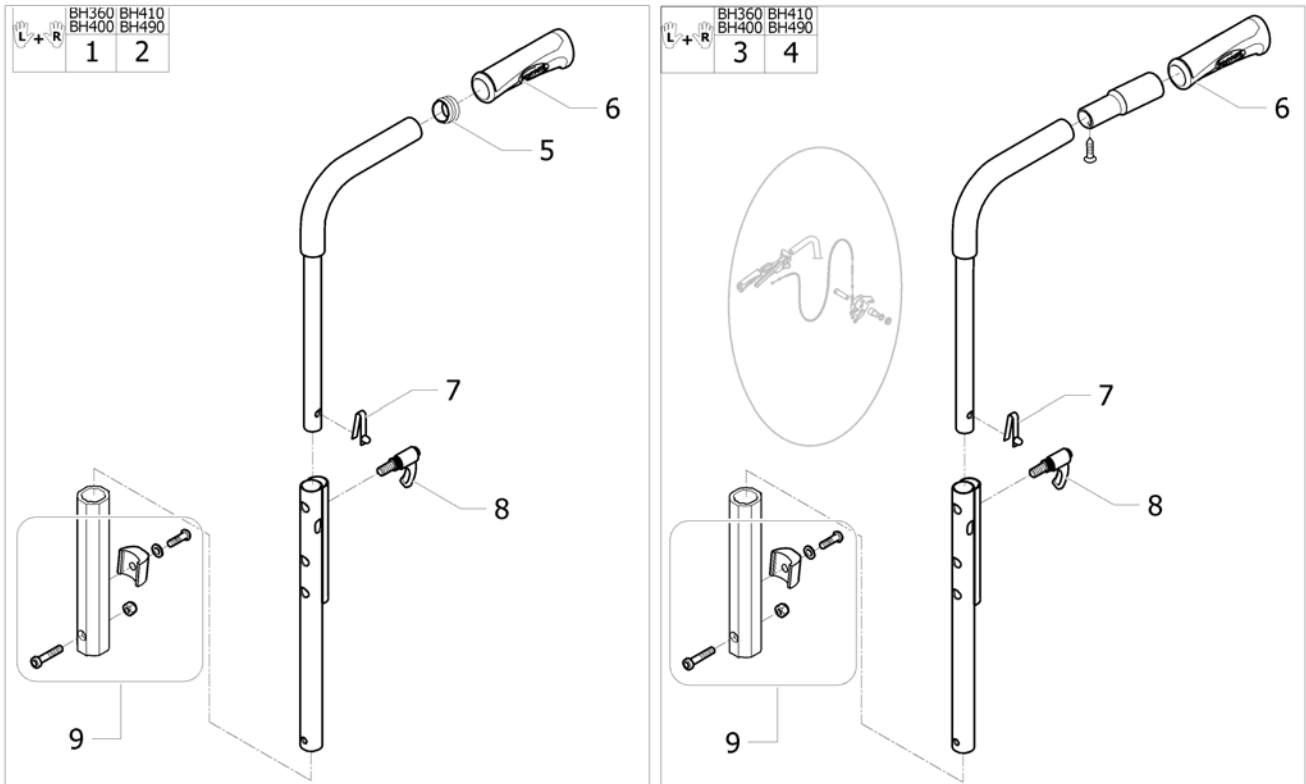

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584778	Backrest height adjustable, 360-400mm, Integrated pushhandle <i>[SJ_2300+SJ_2420]</i>		1
2	SP1584779	Backrest height adjustable, 410-490mm, Integrated pushhandle <i>[SJ_2310+SJ_2420]</i>		1
3	SP1601903	Backrest height adjustable with insert,360-400mm, Integrated pushhandle <i>[SJ_2300+SJ_2420+SJ_8160]</i>		1
4	SP1601922	Backrest height adjustable with insert,400-490mm, Integrated pushhandle <i>[SJ_2310+SJ_2420+SJ_8160]</i>		1
5	1418112	Crossbrace back cap Ø22x15mm		2
6	1425473	Handle Invacare for backrest Ø22mm		2
7	0027-10003	Spring pin locked armrest		2
8	1443023	Lever indexable M8x25		2
9	SP1589825	Kit adjustable backrest tube fixation		2


Backrest Upholsteries

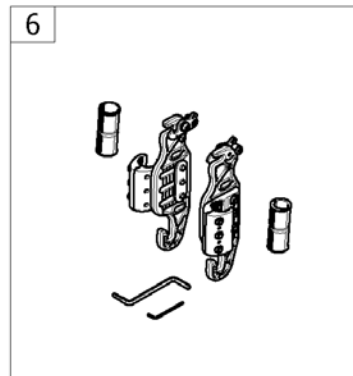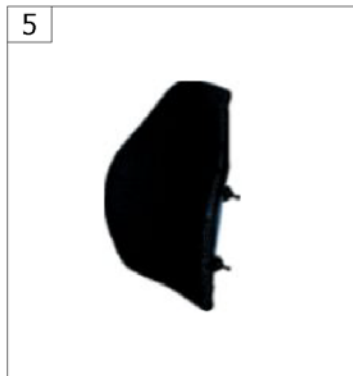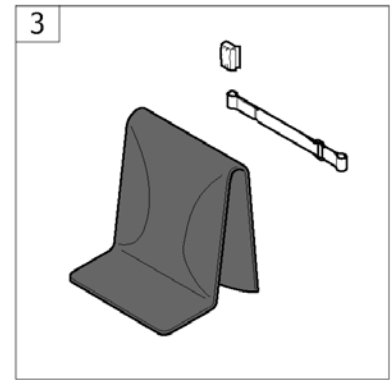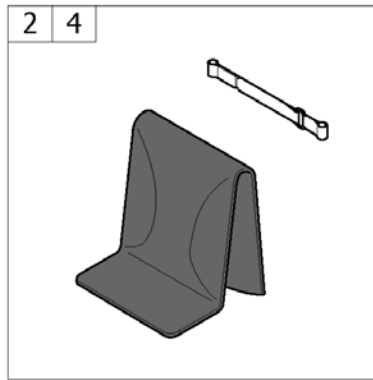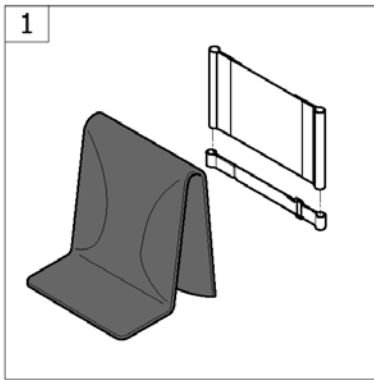

ISPS:1601949

Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest upholstery fixed, for fixed & integrated pushhandle <i>[SJ_2505]</i>		1
2	-	Backrest upholstery adjustable, for fixed & integrated pushhandle <i>[SJ_2510]</i>		1
3	-	Backrest upholstery adjustable, for without & rearsset pushhandle <i>[SJ_2510]</i>		1
4	-	Backrest Upholsteries adjustable télescopique <i>[SJ_2510]</i>		1
5	-	Backrest upholstery MatrX <i>[SJ_2460/SJ_2465/SJ_2470/SJ_2475] ONLY FOR EUROPE</i>		1
6	-	Mounting Hardware MatrX <i>[SJ_2480] ONLY FOR EUROPE</i>		1

Backrest upholstery fixed, for fixed & integrated pushhandle

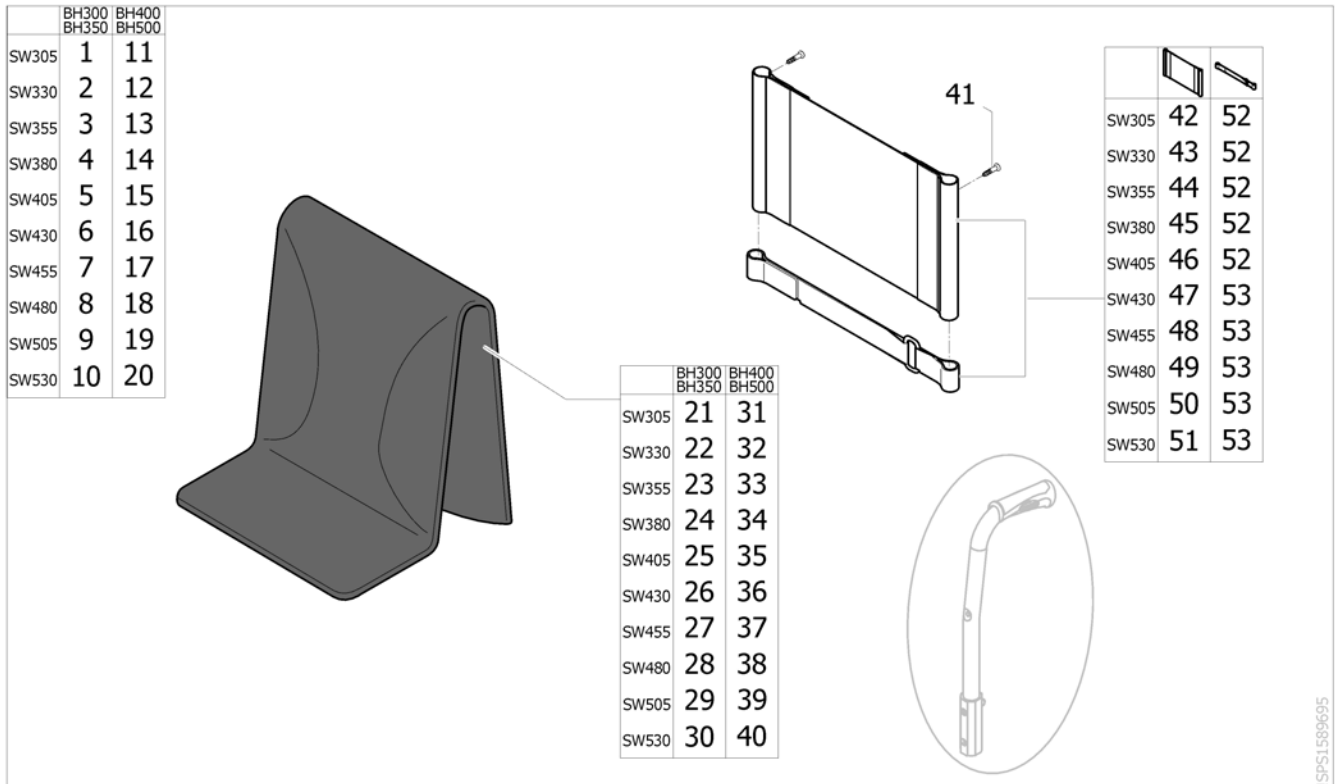


ISPS1589695

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584647	Backrest upholstery fixed, low, SW305mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
2	SP1584648	Backrest upholstery fixed, low, SW330mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
3	SP1584649	Backrest upholstery fixed, low, SW355mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
4	SP1584650	Backrest upholstery fixed, low, SW380mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
5	SP1584651	Backrest upholstery fixed, low, SW405mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
6	SP1584652	Backrest upholstery fixed, low, SW430mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
7	SP1584653	Backrest upholstery fixed, low, SW455mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
8	SP1584654	Backrest upholstery fixed, low, SW480mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
9	SP1584655	Backrest upholstery fixed, low, SW505mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
10	SP1584656	Backrest upholstery fixed, low, SW530mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
11	SP1588315	Backrest upholstery fixed, high, SW305mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1
12	SP1588316	Backrest upholstery fixed, high, SW330mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1
13	SP1588317	Backrest upholstery fixed, high, SW355mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1
14	SP1588318	Backrest upholstery fixed, high, SW380mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1
15	SP1588339	Backrest upholstery fixed, high, SW405mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1
16	SP1588340	Backrest upholstery fixed, high, SW430mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1
17	SP1588341	Backrest upholstery fixed, high, SW455mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1
18	SP1588412	Backrest upholstery fixed, high, SW480mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1

Backrest upholstery fixed, for fixed & integrated pushhandle

Pos.	Part number	Description	Valid from → until	Qty
19	SP1588428	Backrest upholstery fixed, high, SW505mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1
20	SP1588429	Backrest upholstery fixed, high, SW530mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i>		1
21	SP1586150	Shapped cover, short, SW305 mm		1
22	SP1586151	Shapped cover, short, SW330 mm		1
23	SP1586169	Shapped cover, short, SW355 mm		1
24	SP1586170	Shapped cover, short, SW380 mm		1
25	SP1586171	Shapped cover, short, SW405 mm		1
26	SP1586185	Shapped cover, short, SW430 mm		1
27	SP1586186	Shapped cover, short, SW455 mm		1
28	SP1586187	Shapped cover, short, SW480 mm		1
29	SP1586188	Shapped cover, short, SW505 mm		1
30	SP1586189	Shapped cover, short, SW530 mm		1
31	SP1586070	Shapped cover, standard, SW305 mm		1
32	SP1586071	Shapped cover, standard, SW330 mm		1
33	SP1586072	Shapped cover, standard, SW355 mm		1
34	SP1586073	Shapped cover, standard, SW380 mm		1
35	SP1586074	Shapped cover, standard, SW405 mm		1
36	SP1586075	Shapped cover, standard, SW430 mm		1
37	SP1586076	Shapped cover, standard, SW455 mm		1
38	SP1586077	Shapped cover, standard, SW480 mm		1
39	SP1586148	Shapped cover, standard, SW505 mm		1
40	SP1586149	Shapped cover, standard, SW530 mm		1
41	1428603	Screw Tfb 4,8x19 A2r Tx25 DIN7983		2
42	SP1584892	Fix backrest muff SW305 BH 430 mm Action		1
43	SP1584893	Fix backrest muff SW330 BH B430 mm Action		1
44	SP1584894	Fix backrest muff SW355 BH 430 mm Action		1
45	SP1532247	Fix backrest muff SW380 BH 430 mm Action		1
46	SP1532248	Fix backrest muff SW405 BH 430 mm Action		1
47	SP1532249	Fix backrest muff SW430 BH 430 mm Action		1
48	SP1532250	Fix backrest muff SW455 BH 430 mm Action		1
49	SP1532251	Fix backrest muff SW480 BH 430 mm Action		1
50	SP1532252	Fix backrest muff SW505 BH 430 mm Action		1
51	SP1533168	Fix backrest muff SW530 BH 430 mm Action		1
52	SP1601671	Strap 475mm for tension adj. backrest (unit) <i>[SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040]</i>		1
53	SP1601712	Strap 600mm for tension adj. backrest (unit) <i>[SJ_1050/SJ_1060/SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090]</i>		1

Backrest upholstery adjustable, for fixed & integrated pushhandle

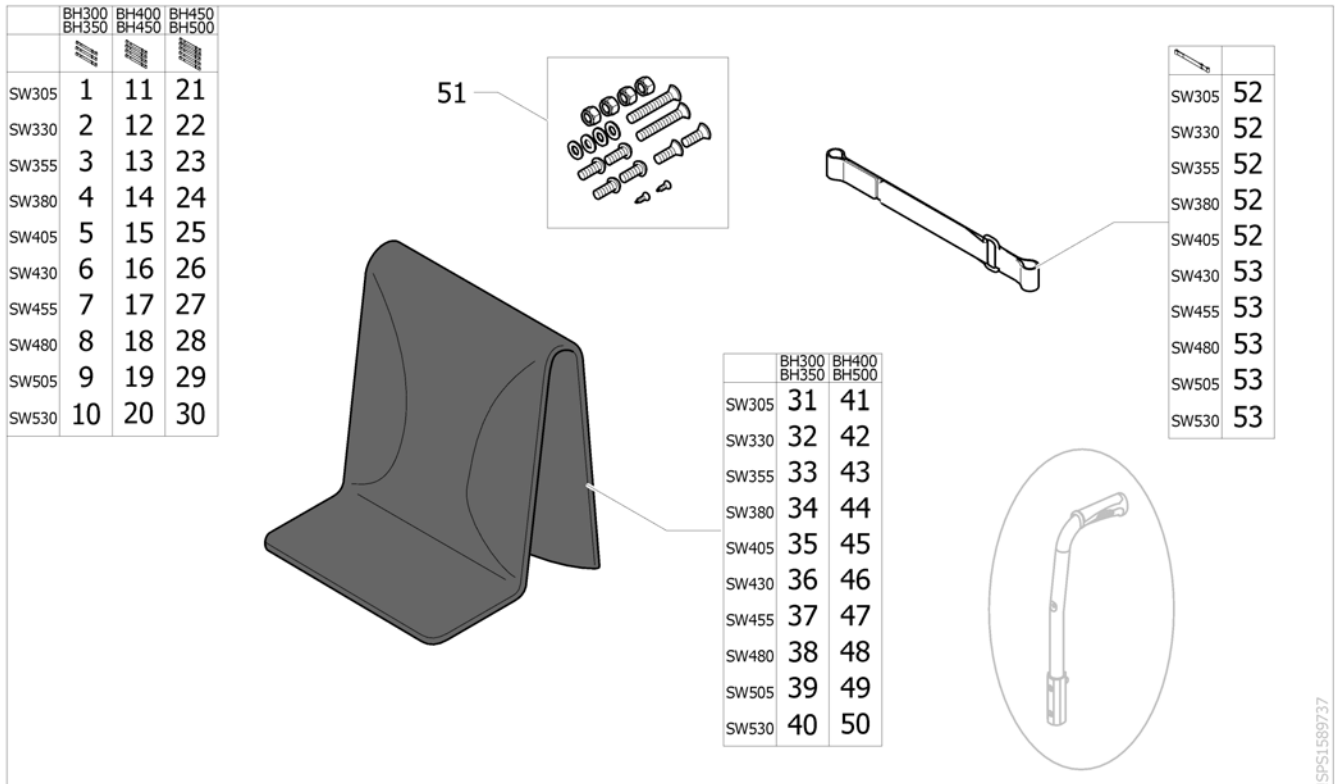

SPS1589737

Pos.	Part number	Description	Valid from → until	Qty
1	SP1589213	Backrest upholstery adjustable, low, SW305mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
2	SP1589214	Backrest upholstery adjustable, low, SW330mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
3	SP1589215	Backrest upholstery adjustable, low, SW355mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
4	SP1589216	Backrest upholstery adjustable, low, SW380mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
5	SP1589217	Backrest upholstery adjustable, low, SW405mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
6	SP1589218	Backrest upholstery adjustable, low, SW430mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
7	SP1589219	Backrest upholstery adjustable, low, SW455mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
8	SP1589220	Backrest upholstery adjustable, low, SW480mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
9	SP1589221	Backrest upholstery adjustable, low, SW505mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
10	SP1589222	Backrest upholstery adjustable, low, SW530mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i>		1
11	SP1602207	Backrest upholstery adjustable, medium, SW305mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]</i>		1
12	SP1602208	Backrest upholstery adjustable, medium, SW330mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]</i>		1


Backrest upholstery adjustable, for fixed & integrated pushhandle

Pos.	Part number	Description	Valid from → until	Qty
13	SP1602209	Backrest upholstery adjustable,medium,SW355mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]		1
14	SP1602210	Backrest upholstery adjustable,medium,SW380mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]		1
15	SP1602211	Backrest upholstery adjustable,medium,SW405mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]		1
16	SP1602212	Backrest upholstery adjustable,medium,SW430mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]		1
17	SP1602213	Backrest upholstery adjustable,medium,SW455mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]		1
18	SP1602214	Backrest upholstery adjustable,medium,SW480mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]		1
19	SP1602215	Backrest upholstery adjustable,medium,SW505mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]		1
20	SP1602216	Backrest upholstery adjustable,medium,SW530mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]		1
21	SP1589413	Backrest upholstery adjustable, high, SW305mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
22	SP1589414	Backrest upholstery adjustable, high, SW330mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
23	SP1589415	Backrest upholstery adjustable, high, SW355mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
24	SP1589416	Backrest upholstery adjustable, high, SW380mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
25	SP1589417	Backrest upholstery adjustable, high, SW405mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
26	SP1589418	Backrest upholstery adjustable, high, SW430mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
27	SP1589419	Backrest upholstery adjustable, high, SW455mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
28	SP1589420	Backrest upholstery adjustable, high, SW480mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
29	SP1589421	Backrest upholstery adjustable, high, SW505mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
30	SP1589422	Backrest upholstery adjustable, high, SW530mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)]		1
31	SP1586150	Shapped cover, short, SW305 mm		1
32	SP1586151	Shapped cover, short, SW330 mm		1
33	SP1586169	Shapped cover, short, SW355 mm		1
34	SP1586170	Shapped cover, short, SW380 mm		1
35	SP1586171	Shapped cover, short, SW405 mm		1
36	SP1586185	Shapped cover, short, SW430 mm		1
37	SP1586186	Shapped cover, short, SW455 mm		1
38	SP1586187	Shapped cover, short, SW480 mm		1
39	SP1586188	Shapped cover, short, SW505 mm		1
40	SP1586189	Shapped cover, short, SW530 mm		1
41	SP1586070	Shapped cover, standard, SW305 mm		1

Backrest upholstery adjustable, for fixed & integrated pushhandle

Pos.	Part number	Description	Valid from → until	Qty
42	SP1586071	Shapped cover, standard, SW330 mm		1
43	SP1586072	Shapped cover, standard, SW355 mm		1
44	SP1586073	Shapped cover, standard, SW380 mm		1
45	SP1586074	Shapped cover, standard, SW405 mm		1
46	SP1586075	Shapped cover, standard, SW430 mm		1
47	SP1586076	Shapped cover, standard, SW455 mm		1
48	SP1586077	Shapped cover, standard, SW480 mm		1
49	SP1586148	Shapped cover, standard, SW505 mm		1
50	SP1586149	Shapped cover, standard, SW530 mm		1
51	SP1601667	Screws kit for adjustable backrest upholstery		1
52	SP1601671	Strap 475mm for tension adj. backrest (unit) <i>[SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040]</i>		5
53	SP1601712	Strap 600mm for tension adj. backrest (unit) <i>[SJ_1050/SJ_1060/SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090]</i>		5

Backrest upholstery adjustable, for without & reaset pushhandle

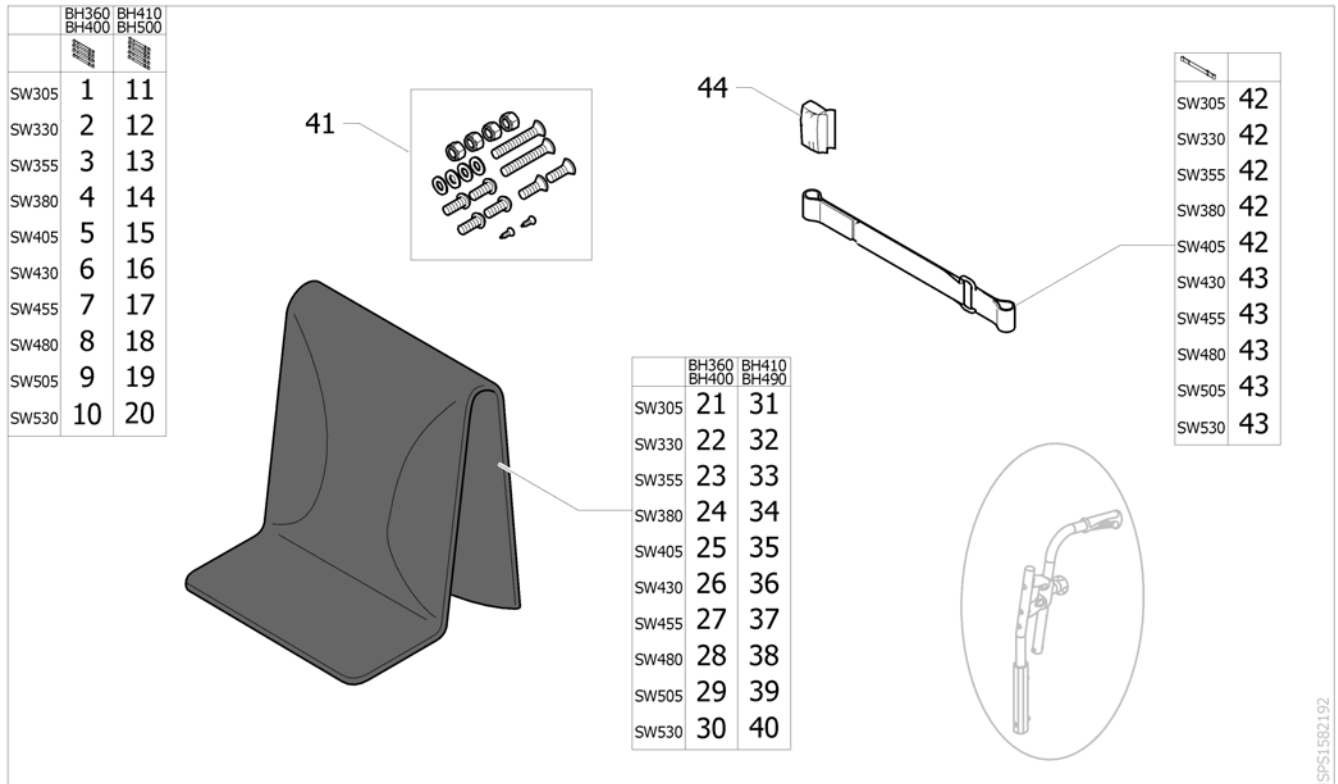


SPS1582192

Pos.	Part number	Description	Valid from → until	Qty
1	SP1589223	Backrest upholstery adjustable, SW305mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
2	SP1589224	Backrest upholstery adjustable, SW330mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
3	SP1589327	Backrest upholstery adjustable, SW355mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
4	SP1589396	Backrest upholstery adjustable, SW380mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
5	SP1589397	Backrest upholstery adjustable, SW405mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
6	SP1589398	Backrest upholstery adjustable, SW430mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
7	SP1589399	Backrest upholstery adjustable, SW455mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
8	SP1589400	Backrest upholstery adjustable, SW480mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
9	SP1589401	Backrest upholstery adjustable, SW505mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
10	SP1589402	Backrest upholstery adjustable, SW530mm, 360-400mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2300]</i>		1
11	SP1589455	Backrest upholstery adjustable, SW305mm, 410-490mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2310]</i>		1
12	SP1589456	Backrest upholstery adjustable, SW330mm, 410-490mm, w/o & reaset pushhandle <i>[SJ_2510+SJ_2310]</i>		1

Backrest upholstery adjustable, for without & rearsset pushhandle

Pos.	Part number	Description	Valid from → until	Qty
13	SP1589457	Backrest upholstery adjustable, SW355mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i>		1
14	SP1589458	Backrest upholstery adjustable, SW380mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i>		1
15	SP1589459	Backrest upholstery adjustable, SW405mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i>		1
16	SP1589460	Backrest upholstery adjustable, SW430mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i>		1
17	SP1589461	Backrest upholstery adjustable, SW455mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i>		1
18	SP1589466	Backrest upholstery adjustable, SW480mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i>		1
19	SP1589467	Backrest upholstery adjustable, SW505mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i>		1
20	SP1589468	Backrest upholstery adjustable, SW530mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i>		1
21	SP1586150	Shapped cover, short, SW305 mm		1
22	SP1586151	Shapped cover, short, SW330 mm		1
23	SP1586169	Shapped cover, short, SW355 mm		1
24	SP1586170	Shapped cover, short, SW380 mm		1
25	SP1586171	Shapped cover, short, SW405 mm		1
26	SP1586185	Shapped cover, short, SW430 mm		1
27	SP1586186	Shapped cover, short, SW455 mm		1
28	SP1586187	Shapped cover, short, SW480 mm		1
29	SP1586188	Shapped cover, short, SW505 mm		1
30	SP1586189	Shapped cover, short, SW530 mm		1
31	SP1586070	Shapped cover, standard, SW305 mm		1
32	SP1586071	Shapped cover, standard, SW330 mm		1
33	SP1586072	Shapped cover, standard, SW355 mm		1
34	SP1586073	Shapped cover, standard, SW380 mm		1
35	SP1586074	Shapped cover, standard, SW405 mm		1
36	SP1586075	Shapped cover, standard, SW430 mm		1
37	SP1586076	Shapped cover, standard, SW455 mm		1
38	SP1586077	Shapped cover, standard, SW480 mm		1
39	SP1586148	Shapped cover, standard, SW505 mm		1
40	SP1586149	Shapped cover, standard, SW530 mm		1
41	SP1601667	Screws kit for adjustable backrest upholstery		1
42	SP1601671	Strap 475mm for tension adj. backrest (unit) <i>[SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040]</i>		5
43	SP1601712	Strap 600mm for tension adj. backrest (unit) <i>[SJ_1050/SJ_1060/SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090]</i>		5
44	SP1601713	Kit protection for backrest without pushhandle		2

Backrest Upholsteries adjustable télescopique

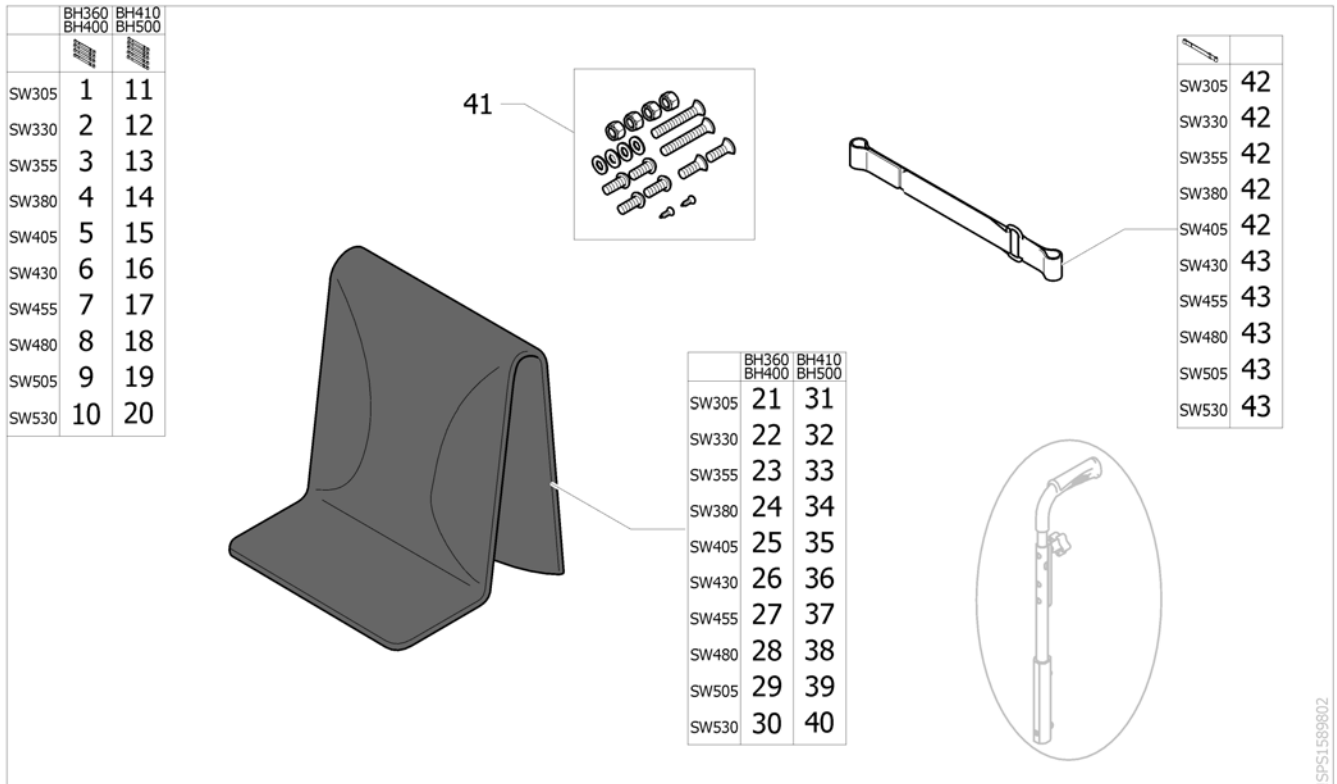

ISPS1589802

Pos.	Part number	Description	Valid from → until	Qty
1	SP1589403	Back uphol. adj low integrated handle SW305 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
2	SP1589404	Back uphol. adj low integrated handle SW330 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
3	SP1589405	Back uphol. adj low integrated handle SW355 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
4	SP1589406	Back uphol. adj low integrated handle SW380 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
5	SP1589407	Back uphol. adj low integrated handle SW405 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
6	SP1589408	Back uphol. adj low integrated handle SW430 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
7	SP1589409	Back uphol. adj low integrated handle SW455 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
8	SP1589410	Back uphol. adj low integrated handle SW480 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
9	SP1589411	Back uphol. adj low integrated handle SW505 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
10	SP1589412	Back uphol. adj low integrated handle SW530 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i>		1
11	SP1589423	Back uphol. adj high integrated handle SW305 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1
12	SP1589446	Back uphol. adj high integrated handle SW330 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1
13	SP1589447	Back uphol. adj high integrated handle SW355 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1
14	SP1589448	Back uphol. adj high integrated handle SW380 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1
15	SP1589449	Back uphol. adj high integrated handle SW405 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1
16	SP1589450	Back uphol. adj high integrated handle SW430 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1
17	SP1589451	Back uphol. adj high integrated handle SW455 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1
18	SP1589452	Back uphol. adj high integrated handle SW480 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1

Backrest Upholsteries adjustable télécospique

Pos.	Part number	Description	Valid from → until	Qty
19	SP1589453	Back uphol. adj high integrated handle SW505 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1
20	SP1589454	Back uphol. adj high integrated handle SW530 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i>		1
21	SP1586150	Shapped cover, short, SW305 mm		1
22	SP1586151	Shapped cover, short, SW330 mm		1
23	SP1586169	Shapped cover, short, SW355 mm		1
24	SP1586170	Shapped cover, short, SW380 mm		1
25	SP1586171	Shapped cover, short, SW405 mm		1
26	SP1586185	Shapped cover, short, SW430 mm		1
27	SP1586186	Shapped cover, short, SW455 mm		1
29	SP1586188	Shapped cover, short, SW505 mm		1
28	SP1586187	Shapped cover, short, SW480 mm		1
30	SP1586189	Shapped cover, short, SW530 mm		1
31	SP1586070	Shapped cover, standard, SW305 mm		1
32	SP1586071	Shapped cover, standard, SW330 mm		1
33	SP1586072	Shapped cover, standard, SW355 mm		1
34	SP1586073	Shapped cover, standard, SW380 mm		1
35	SP1586074	Shapped cover, standard, SW405 mm		1
36	SP1586075	Shapped cover, standard, SW430 mm		1
37	SP1586076	Shapped cover, standard, SW455 mm		1
38	SP1586077	Shapped cover, standard, SW480 mm		1
39	SP1586148	Shapped cover, standard, SW505 mm		1
40	SP1586149	Shapped cover, standard, SW530 mm		1
41	SP1601667	Screws kit for adjustable backrest upholstery		1
42	SP1601671	Strap 475mm for tension adj. backrest (unit) <i>[SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040]</i>		5
43	SP1601712	Strap 600mm for tension adj. backrest (unit) <i>[SJ_1050/SJ_1060/SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090]</i>		5

Backrest upholstery MatrX



ISPS1604045







Pos.	Part number	Description	Valid from → until	Qty
1	1532093	Backrest MatrX Elite W380 x H410 mm - without hardware		1
2	1515720	Backrest MatrX Elite W410 x H410 mm - without hardware		1
3	1515722	Backrest MatrX Elite W430 x H410 mm - without hardware		1
4	1515724	Backrest MatrX Elite W360 x H410 mm - without hardware		1
5	1515726	Backrest MatrX Elite W480 x H410 mm - without hardware		1
6	1532096	Backrest MatrX Elite W510 x H410 mm - without hardware		1
7	1545100	Backrest MatrX Elite W380 x H460 mm - without hardware		1
8	1545102	Backrest MatrX Elite W410 x H460 mm - without hardware		1
9	1545105	Backrest MatrX Elite W430 x H460 mm - without hardware		1
10	1545108	Backrest MatrX Elite W460 x H460 mm - without hardware		1
11	1545111	Backrest MatrX Elite W480 x H460 mm - without hardware		1
12	1545114	Backrest MatrX Elite W510 x H460 mm - without hardware		1
13	1543509	Backrest MatrX Elite Deep W380 x H410 mm - without hardware		1
14	1543513	Backrest MatrX Elite Deep W410 x H410 mm - without hardware		1
15	1543517	Backrest MatrX Elite Deep W430 x H410 mm - without hardware		1
16	1543521	Backrest MatrX Elite Deep W460 x H410 mm - without hardware		1
17	1543525	Backrest MatrX Elite Deep W480 x H410 mm - without hardware		1
18	1543989	Backrest MatrX Elite Deep W510 x H410 mm - without hardware		1
19	1543510	Backrest MatrX Elite Deep W380 x H460 mm - without hardware		1
20	1543514	Backrest MatrX Elite Deep W410 x H460 mm - without hardware		1
21	1543518	Backrest MatrX Elite Deep W430 x H460 mm - without hardware		1
22	1543522	Backrest MatrX Elite Deep W460 x H460 mm - without hardware		1
23	1543526	Backrest MatrX Elite Deep W480 x H460 mm - without hardware		1
24	1543990	Backrest MatrX Elite Deep W510 x H460 mm - without hardware		1

Mounting Hardware MatrX



Pos.	Part number	Description	Valid from → until	Qty
1	1545275	PB Mounting Hardware for all MC Backs (pair)		1


Legrests

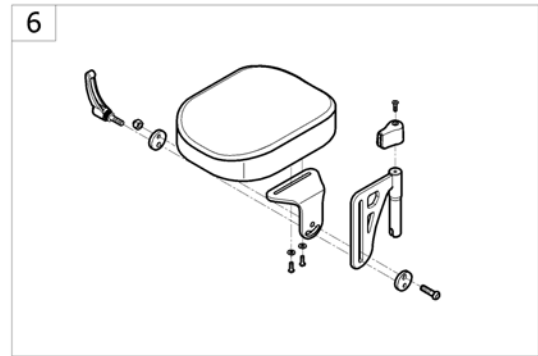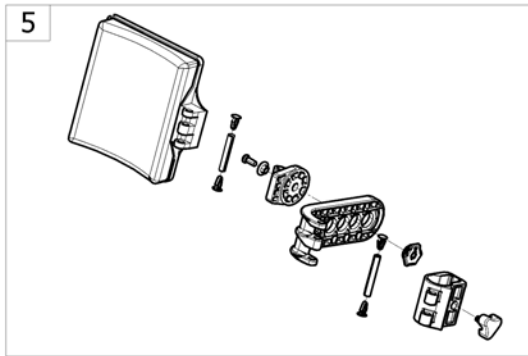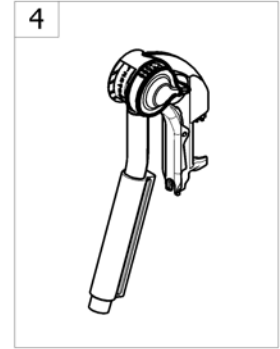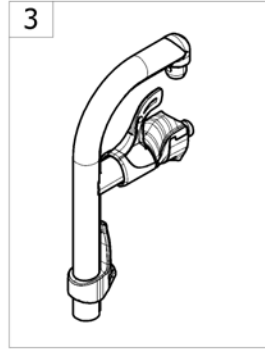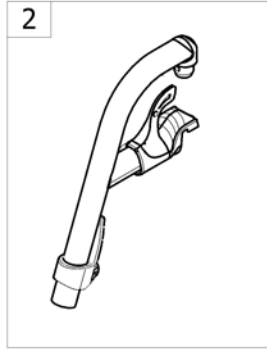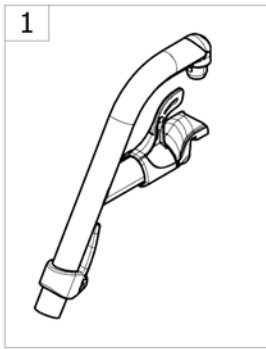

I->SPCACTION530

Pos.	Part number	Description	Valid from → until	Qty
1	-	Hanger 70° [SJ_2810/SJ_3010/SJ_3110]		1
2	-	Hanger 80° [SJ_2820/SJ_3020/SJ_3120]		1
3	-	Hanger 90° [SJ_2830/SJ_3030/SJ_3130]		1
4	-	Elevating legrest [SJ_2840/SJ_3040/SJ_3140]		1
5	-	Calf Pad [SJ_2840/SJ_3040/SJ_3140]		1
6	-	Amputee Legrest [SJ_2850/SJ_3050/SJ_3150]		1


Hanger 70°

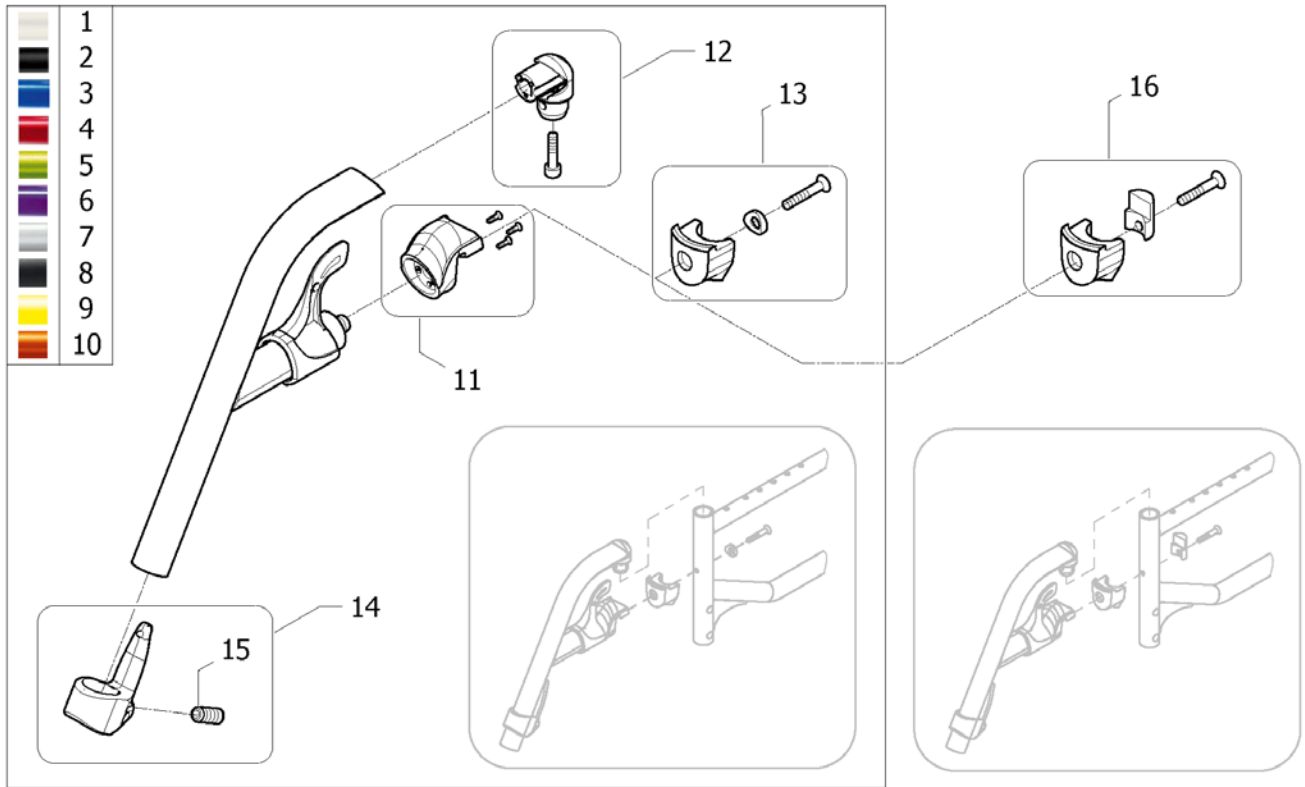

ISPS:1582188 - sps1582188

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584371-0637	Hanger 70°, complete, Pearl White (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6000]		2
2	SP1584371-0609	Hanger 70°, complete, Anthracite (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6010]		2
3	SP1584371-0643	Hanger 70°, complete, Pop Blue (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6020]		2
4	SP1584371-0607	Hanger 70°, complete, Dark Red (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6030]		2
5	SP1584371-0644	Hanger 70°, complete, Electric Green (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6040]		2
6	SP1584371-0645	Hanger 70°, complete, Hypnotic Purple (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6050]		2
7	SP1584371-0515	Hanger 70°, complete, Pearl Grey (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6060]		2
8	SP1584371-0646	Hanger 70°, complete, Matt Black (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6070]		2
9	SP1584371-0647	Hanger 70°, complete, Intense Yellow (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6080]		2
10	SP1584371-0648	Hanger 70°, complete, Vibrating Orange (2015) [[SJ_2810/SJ_3010/SJ_3110)+SJ_6090]		2
11	SP1584383	Kit hanger locking nose (2015)		2
12	SP1584381	Kit hanger pivot (2015)		2
13	SP1584380	Kit hanger locked (2015)		2
14	SP1584382	Calf strap holder + ball screw (2015)		2
15	1424571	Screw stop M10x23		2
16	SP1587483	Kit hanger stopper (2015) for Action NG		2


Hanger 80°

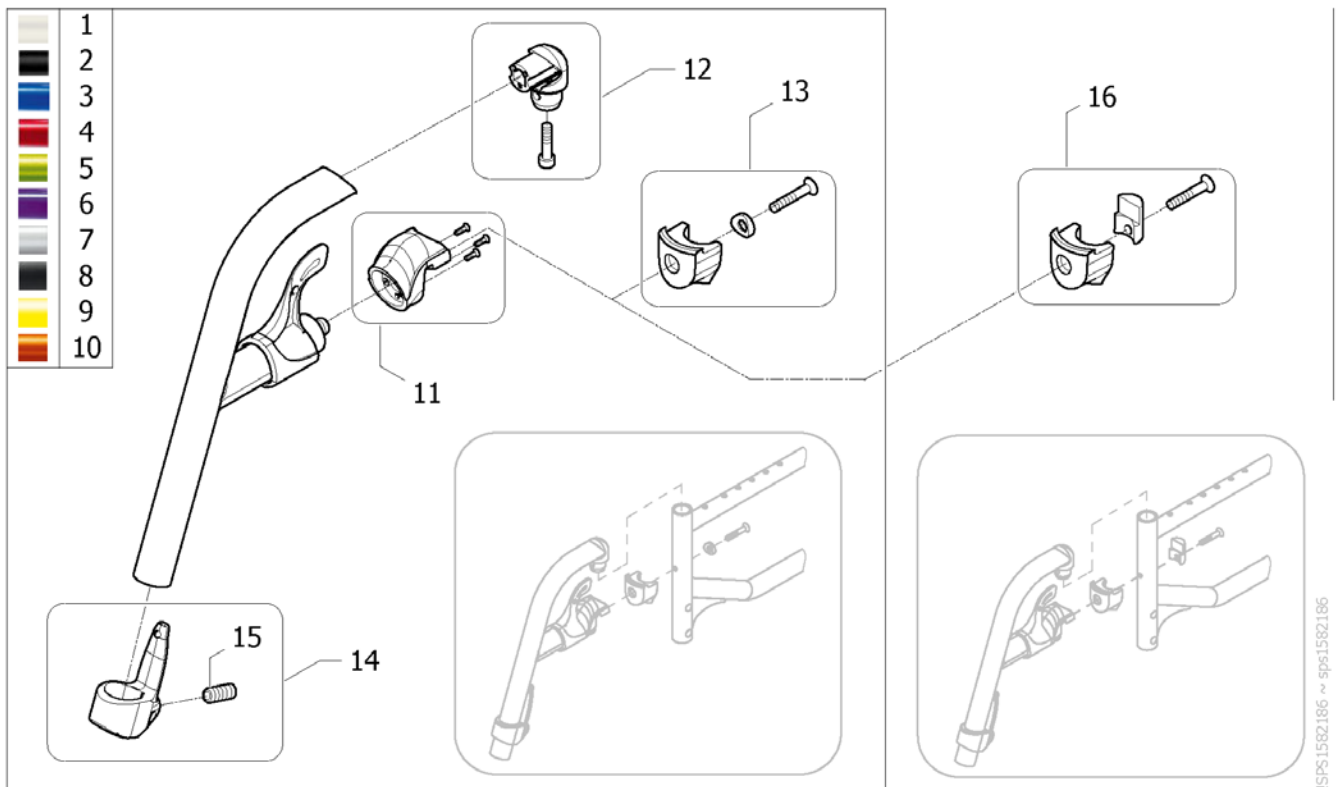

ISPS:1582186 - sps1582186


Pos.	Part number	Description	Valid from → until	Qty
1	SP1584369-0637	Hanger 80°, complete, Pearl White (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6000]		2
2	SP1584369-0609	Hanger 80°, complete, Anthracite (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6010]		2
3	SP1584369-0643	Hanger 80°, complete, Pop Blue (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6020]		2
4	SP1584369-0607	Hanger 80°, complete, Dark Red (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6030]		2
5	SP1584369-0644	Hanger 80°, complete, Electric Green (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6040]		2
6	SP1584369-0645	Hanger 80°, complete, Hypnotic Purple (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6050]		2
7	SP1584369-0515	Hanger 80°, complete, Pearl Grey (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6060]		2
8	SP1584369-0646	Hanger 80°, complete, Matt Black (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6070]		2
9	SP1584369-0647	Hanger 80°, complete, Intense Yellow (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6080]		2
10	SP1584369-0648	Hanger 80°, complete, Vibrating Orange (2015) [(SJ_2820/SJ_3020/SJ_3120)+SJ_6090]		2
11	SP1584383	Kit hanger locking nose (2015)		2
12	SP1584381	Kit hanger pivot (2015)		2
13	SP1584380	Kit hanger locked (2015)		2
14	SP1584382	Calf strap holder + ball screw (2015)		2
15	1424571	Screw stop M10x23		2
16	SP1587483	Kit hanger stopper (2015) for Action NG		2

Hanger 90°

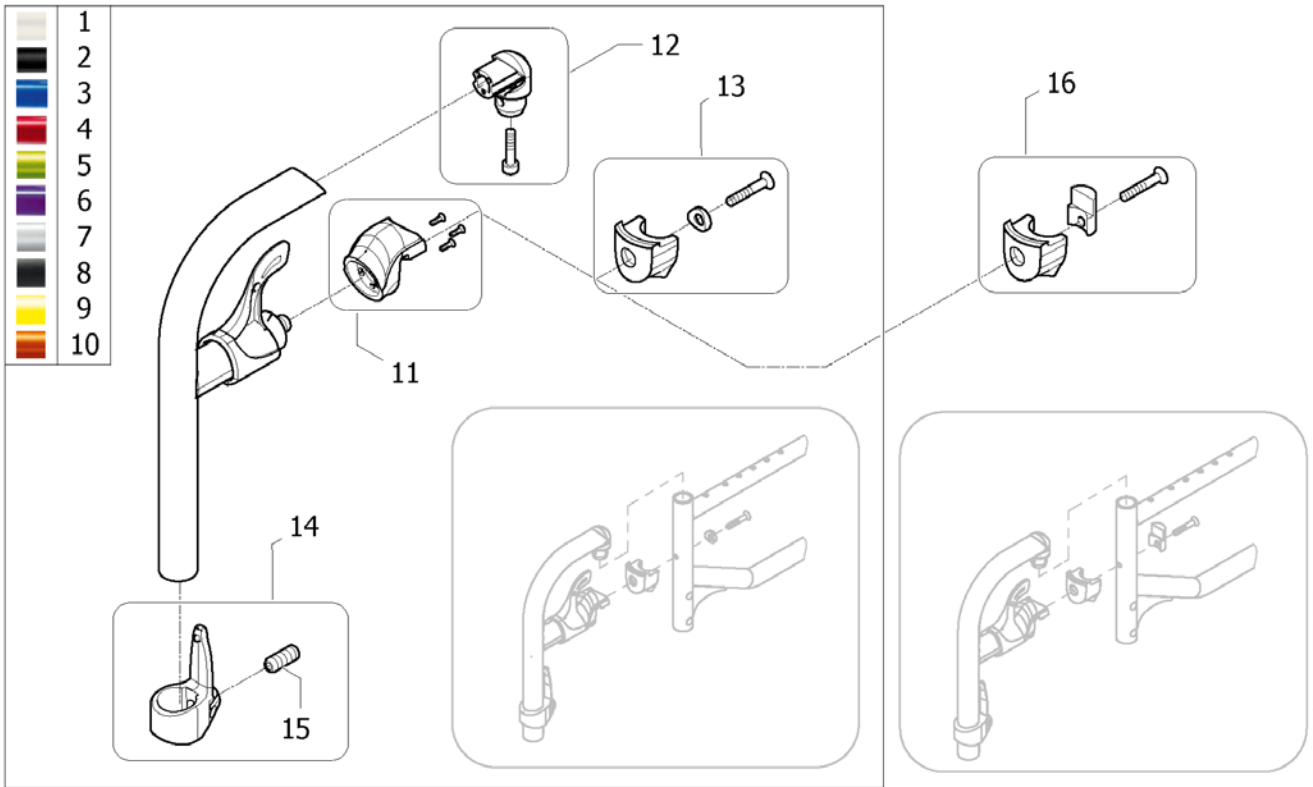

ISPS:1582187 ~ sps1582187


Pos.	Part number	Description	Valid from → until	Qty
1	SP1584370-0637	Hanger 90°, complete, Pearl White (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6000]		2
2	SP1584370-0609	Hanger 90°, complete, Anthracite (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6010]		2
3	SP1584370-0643	Hanger 90°, complete, Pop Blue (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6020]		2
4	SP1584370-0607	Hanger 90°, complete, Dark Red (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6030]		2
5	SP1584370-0644	Hanger 90°, complete, Electric Green (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6040]		2
6	SP1584370-0645	Hanger 90°, complete, Hypnotic Purple (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6050]		2
7	SP1584370-0515	Hanger 90°, complete, Pearl Grey (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6060]		2
8	SP1584370-0646	Hanger 90°, complete, Matt Black (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6070]		2
9	SP1584370-0647	Hanger 90°, complete, Intense Yellow (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6080]		2
10	SP1584370-0648	Hanger 90°, complete, Vibrating Orange (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6090]		2
11	SP1584383	Kit hanger locking nose (2015)		2
12	SP1584381	Kit hanger pivot (2015)		2
13	SP1584380	Kit hanger locked (2015)		2
14	SP1584382	Calf strap holder + ball screw (2015)		2
15	1424571	Screw stop M10x23		2
16	SP1587483	Kit hanger stopper (2015) for Action NG		2

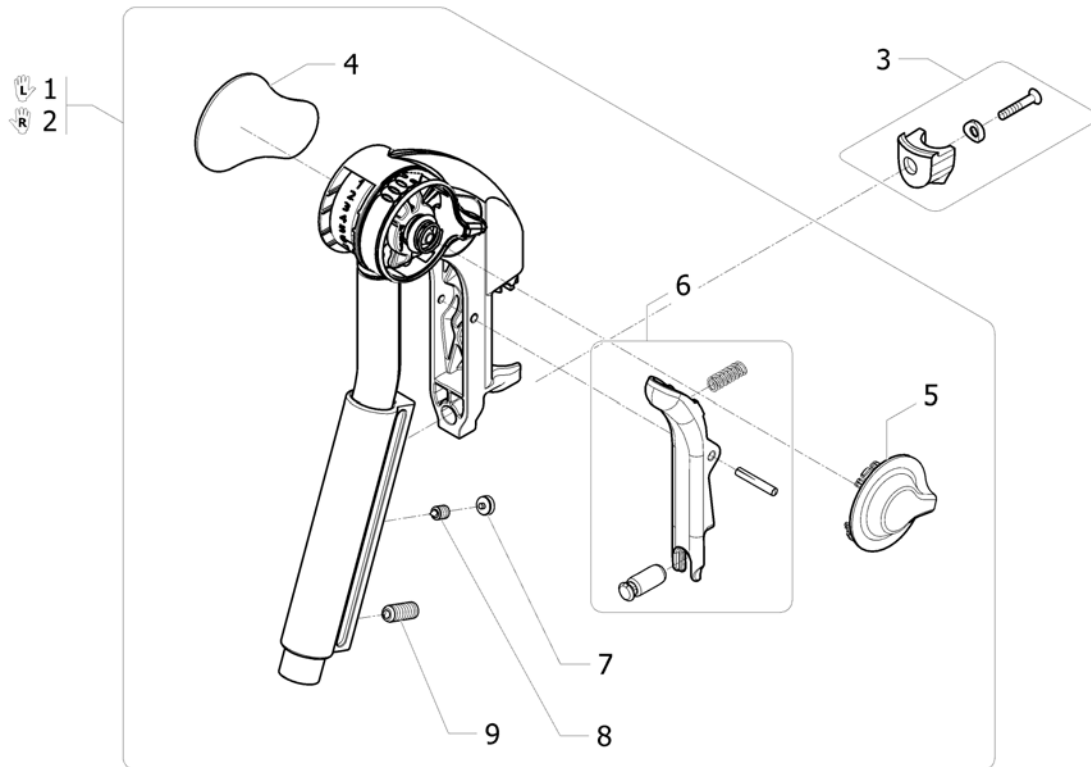

ISPS1535382

Pos.	Part number	Description	Valid from → until	Qty
1	1552881	Elevating legrest Lh + fixation <i>Replace 1508945</i>		1
2	1552882	Elevating legrest Rh + fixation <i>Replace 1508948</i>		1
3	SP1584380	Kit hanger locked (2015) <i>Replace 1430427</i>		2
4	1428336	Kneepad		2
5	1508943	Incline adjustment lever		2
6	1430429	Legrest Trigger Kit		2
7	1435741	Footrest Hanger Plug		2
8	1424572	Screw stop M10x12		2
9	1424571	Screw stop M10x23		2


Calf Pad

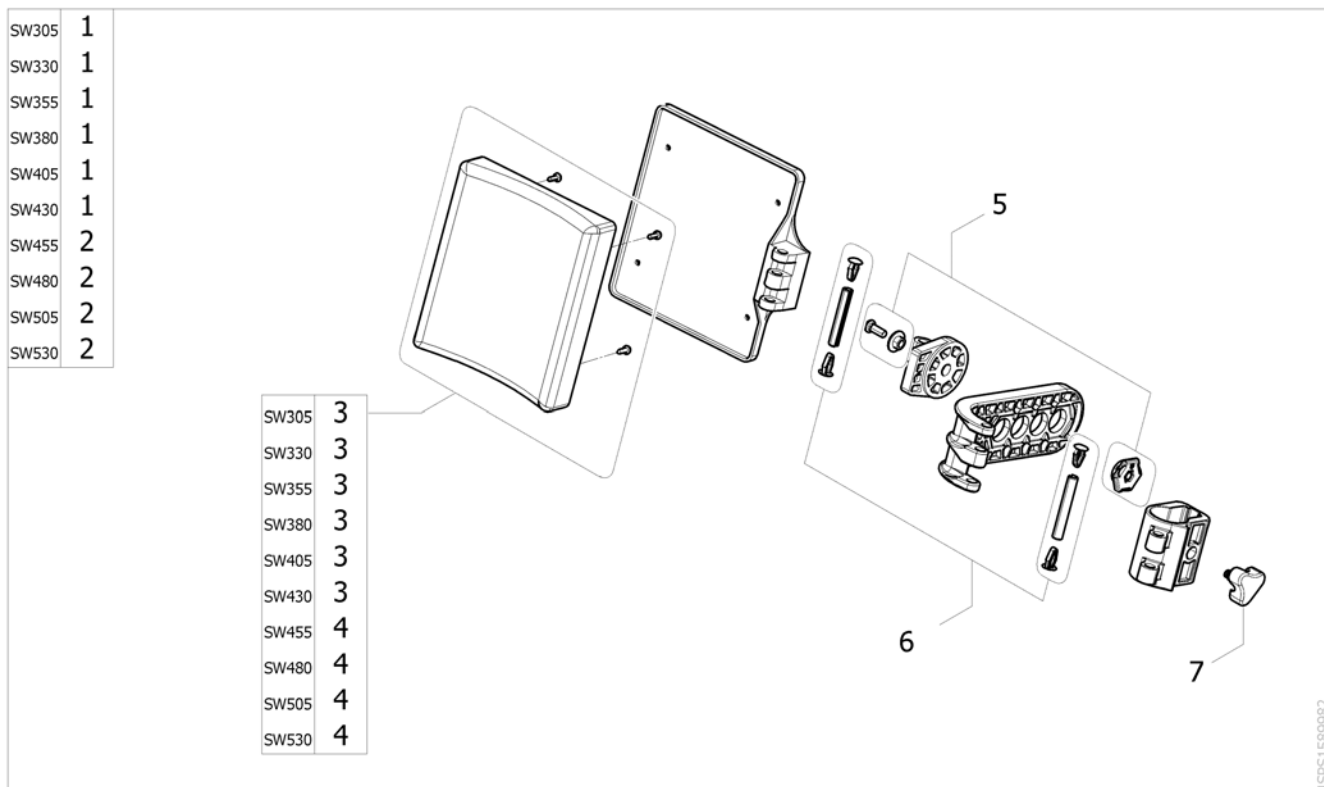

20666015SPS1589982

Pos.	Part number	Description	Valid from → until	Qty
1	1450932	Complete Polyurethane calf pad (cushion with support), SW380/405/430 mm <i>[SJ_2840/SJ_3040/SJ_3140]</i>		2
2	1450933	Complete Polyurethane calf pad (cushion with support), SW455/480/505 mm <i>[SJ_2840/SJ_3040/SJ_3140]</i>		2
3	1430430	Polyurethane calf pad cushion with fixation SW380/405/430 mm		2
4	1430431	Polyurethane calf pad cushion with fixation SW455/480/505 mm		2
5	1440132	Kit fixation angle adjustable calf-pad		2
6	1430432	Kit articulation pin calf pad		2
7	1440943	Locking star knob		2


Amputee Legrest

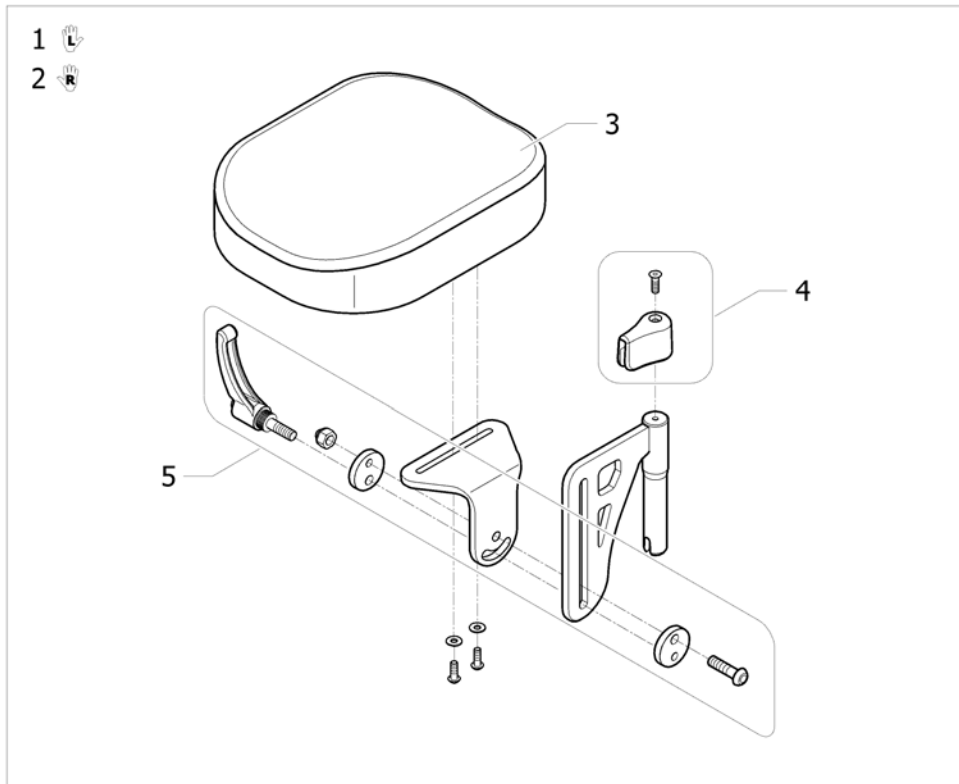

ISP51495897

Pos.	Part number	Description	Valid from → until	Qty
1	1442991	Support Lh, legrest amputee		1
2	1442990	Support Rh, legrest amputee		1
3	1497450	Cushion amputee legrest		2
4	1446556	Amputee legrest stop + fixation		2
5	1446557	Amputee legrest slide fixation		2
6	SP1584380	Kit hanger locked (2015) <i>Replace 1430427</i>		2


Footrests

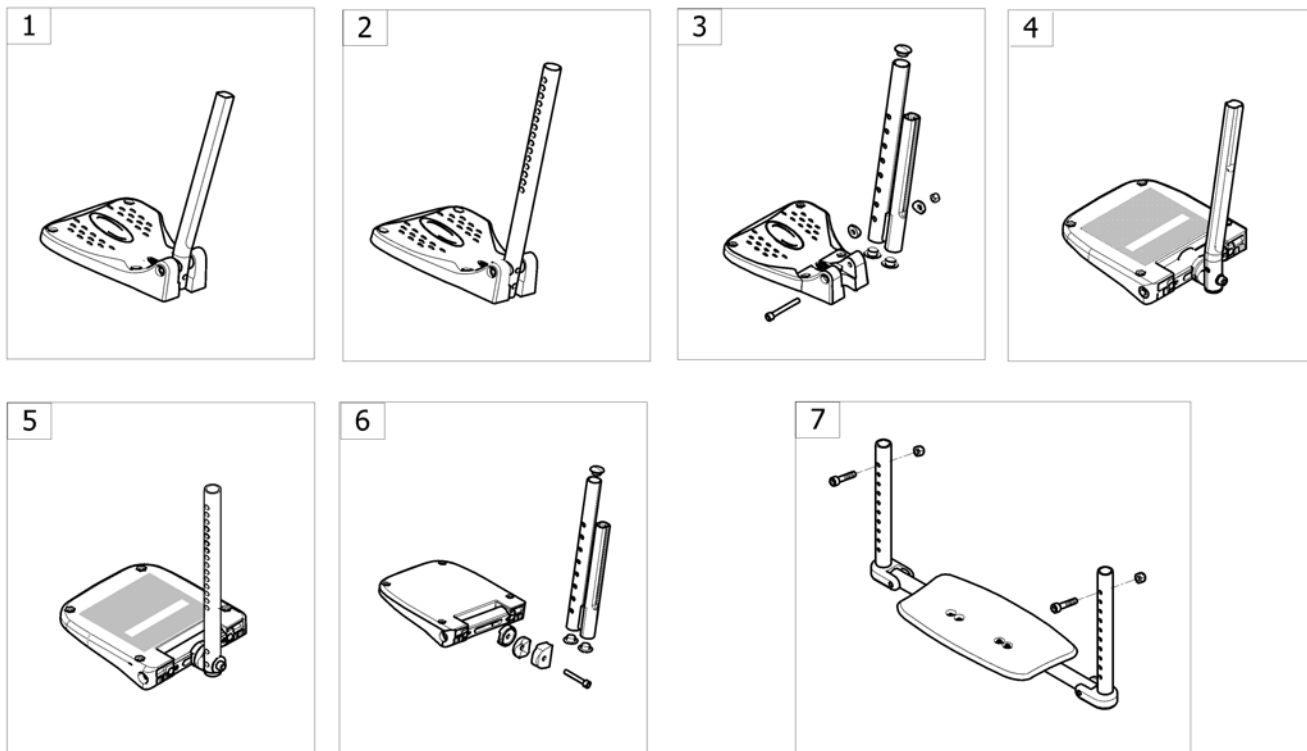

-<SPACED>SPOACTION535

Pos.	Part number	Description	Valid from → until	Qty
1	-	Footrest fixed, swing away frame [SJ_0200+(SJ_3170/SJ_3200/SJ_3300)+(SJ_3420/SJ_3430)]		1
2	-	Footrest fixed, fixed frame [SJ_0210+(SJ_3170/SJ_3200/SJ_3300)]		1
3	-	Footrest fixed, high mounted tube, swing away frame [(SJ_3170/SJ_3200/SJ_3300)+SJ_3400]		1
4	-	Footrest angle & depth adjustable, swing away frame [SJ_0200+(SJ_3175/SJ_3210/SJ_3310)+(SJ_3410/SJ_3420/SJ_3430)]		1
5	-	Footrest angle & depth adjustable, fixed frame [SJ_0210+(SJ_3175/SJ_3210/SJ_3310)+(SJ_3440/SJ_3450)]		1
6	-	Footrest angle & depth adjustable, high mounted tube, swing away frame [(SJ_3175/SJ_3210/SJ_3310)+SJ_3400]		1
7	-	One piece footplate for fixed frame [SJ_0210+(SJ_3185/SJ_3230)+(SJ_3440/SJ_3450)]		1

Footrest fixed, swing away frame

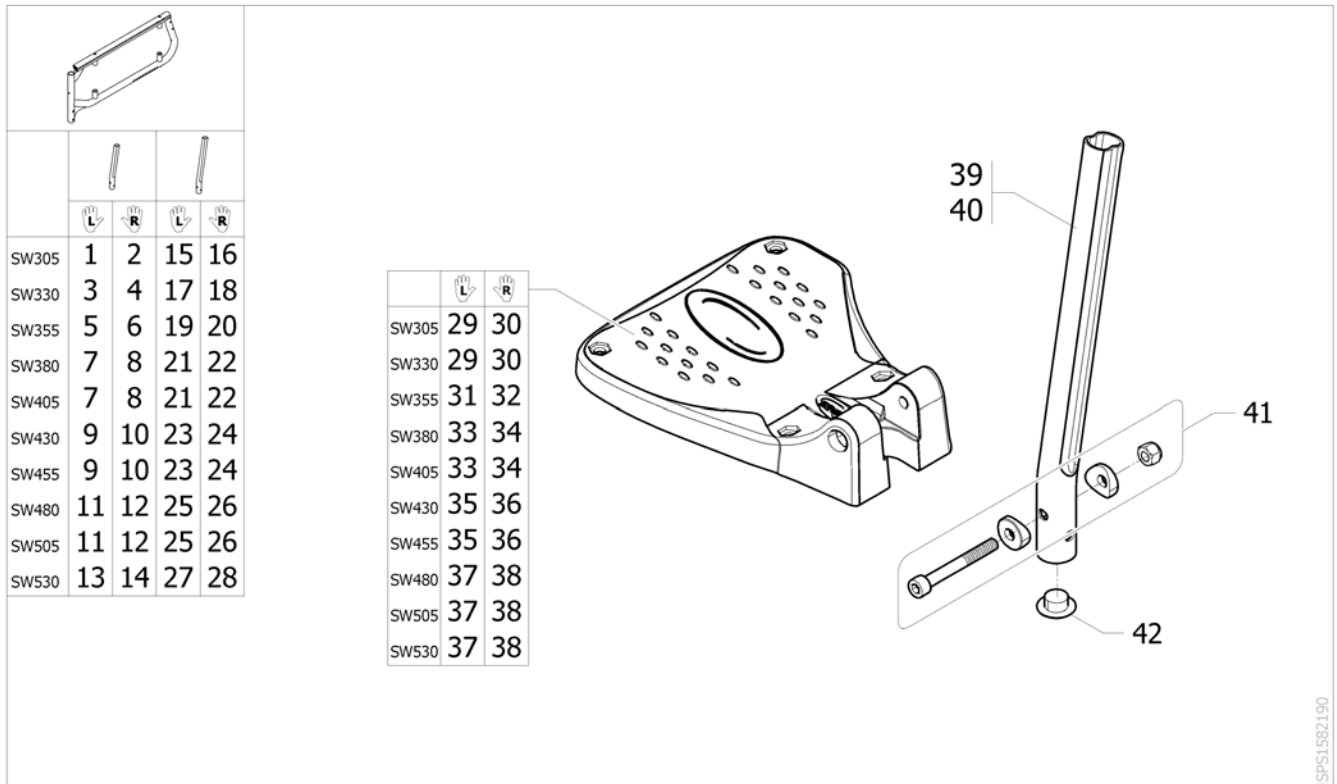


SPS1582190

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586899	Footrest fixed, Lh, SW305mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i>		1
2	SP1586900	Footrest fixed, Rh, SW305mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i>		1
3	SP1586901	Footrest fixed, Lh, SW330mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i>		1
4	SP1587031	Footrest fixed, Rh, SW330mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i>		1
5	SP1587179	Footrest fixed, Lh, SW355mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i>		1
6	SP1587215	Footrest fixed, Rh, SW355mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i>		1
7	1530620	Footrest fixed, Lh, SW380-405mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i>		1
8	1530614	Footrest fixed, Rh, SW380-405mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i>		1
9	1530622	Footrest fixed, Lh, SW430-455mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i>		1
10	1530616	Footrest fixed, Rh, SW430-455mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i>		1
11	1530623	Footrest fixed, Lh, SW480-505mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i>		1
12	1530617	Footrest fixed, Rh, SW480-505mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i>		1
13	SP1601715	Footrest fixed, Lh, SW530mm, standard tube, swing away frame + Heel strap <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i>		1
14	SP1601716	Footrest fixed, Rh, SW530mm, standard tube, swing away frame + Heel strap <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i>		1
15	SP1587273	Footrest fixed, Lh, SW305mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i>		1
16	SP1587274	Footrest fixed, Rh, SW305mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i>		1
17	SP1587275	Footrest fixed, Lh, SW330mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i>		1

Footrest fixed, swing away frame

Pos.	Part number	Description	Valid from → until	Qty
18	SP1587312	Footrest fixed, Rh, SW330mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i>		1
19	SP1587313	Footrest fixed, Lh, SW355mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i>		1
20	SP1587405	Footrest fixed, Rh, SW355mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i>		1
21	1530633	Footrest fixed, Lh, SW380-405mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i>		1
22	1530630	Footrest fixed, Rh, SW380-405mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i>		1
23	1530634	Footrest fixed, Lh, SW330-455mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i>		1
24	1530631	Footrest fixed, Rh, SW330-455mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i>		1
25	1530635	Footrest fixed, Lh, SW480-505mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i>		1
26	1530632	Footrest fixed, Rh, SW480-505mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i>		1
27	SP1601762	Footrest fixed, Lh, SW530mm, long tube, swing away frame + Heel strap <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i>		1
28	SP1601763	Footrest fixed, Rh, SW530mm, long tube, swing away frame + Heel strap <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i>		1
29	0572-04433	Footplate, fixed, Lh, 133mm		1
30	0572-04447	Footplate, fixed, Rh, 133mm		1
31	0572-04434	Footplate, fixed, Lh, 163mm		1
32	0572-04448	Footplate, fixed, Rh, 163mm		1
33	1529643	Footplate, fixed, Lh, 174mm (V2009)		1
34	1529642	Footplate, fixed, Rh, 174mm (V2009)		1
35	1529645	Footplate, fixed, Lh, 199mm (V2009)		1
36	1529644	Footplate, fixed, Rh, 199mm (V2009)		1
37	1529647	Footplate, fixed, Lh, 224mm (V2009)		1
38	1529646	Footplate, fixed, Rh, 224mm (V2009)		1
39	1434561-0010	Standard tube for fixed footrest <i>[SJ_0200+SJ_3420]</i>		2
40	1434563-0010	Long tube for fixed and a&d adjustable footrest <i>[SJ_0200+SJ_3430]</i>		2
41	1430433	Kit screws for fixed footrest		2
42	1424576	Plastic cap black Ø20mm		2

Footrest fixed, fixed frame

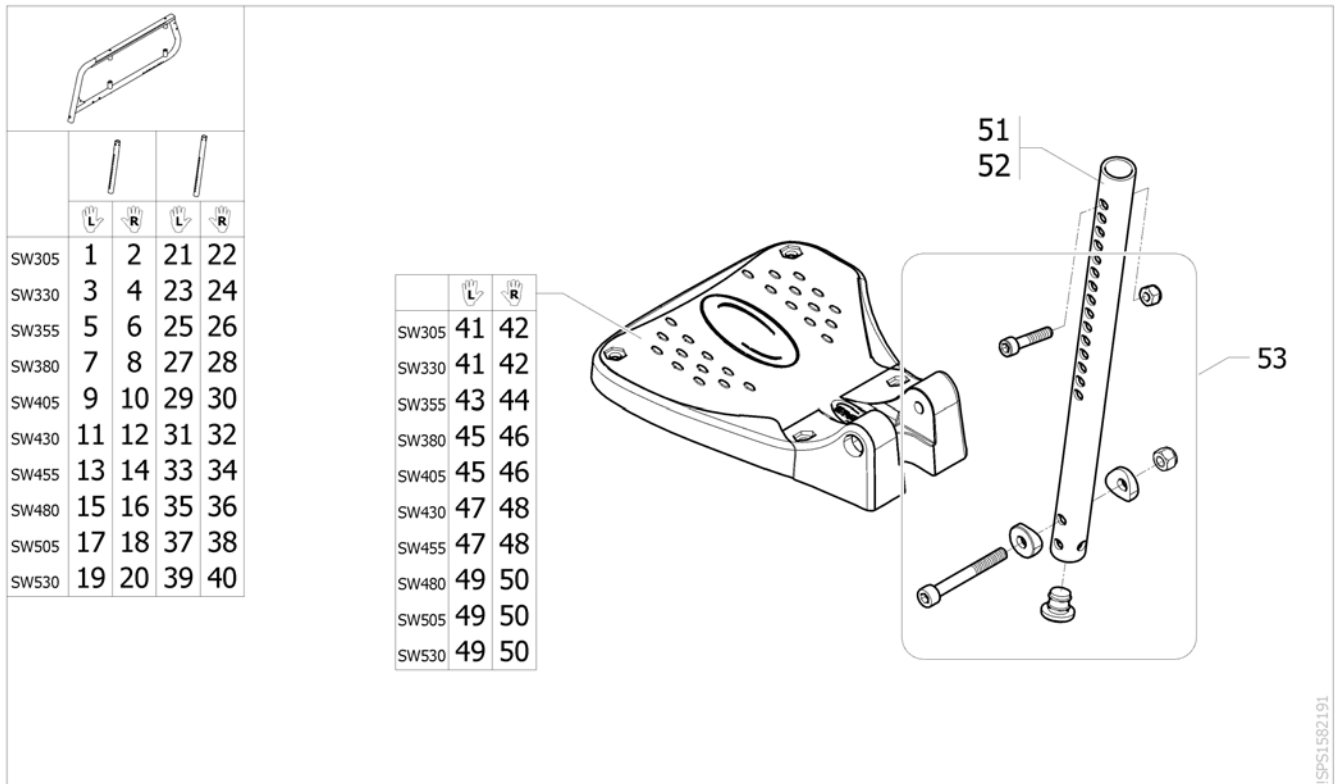


SPS1582191

Pos.	Part number	Description	Valid from → until	Qty
1	SP1590821	Footrest fixed, Lh, SW305mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
2	SP1590822	Footrest fixed, Rh, SW305mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1
3	SP1590823	Footrest fixed, Lh, SW330mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
4	SP1590829	Footrest fixed, Rh, SW330mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1
5	SP1590830	Footrest fixed, Lh, SW355mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
6	SP1590831	Footrest fixed, Rh, SW355mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1
7	SP1590846	Footrest fixed, Lh, SW380mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
8	SP1590848	Footrest fixed, Rh, SW380mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1
9	SP1586809	Footrest fixed, Lh, SW405mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
10	SP1586810	Footrest fixed, Rh, SW405mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1
11	SP1586811	Footrest fixed, Lh, SW430mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
12	SP1586812	Footrest fixed, Rh, SW430mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1
13	SP1586813	Footrest fixed, Lh, SW455mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
14	SP1586814	Footrest fixed, Rh, SW455mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1
15	SP1586815	Footrest fixed, Lh, SW480mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
16	SP1586816	Footrest fixed, Rh, SW480mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1
17	SP1586817	Footrest fixed, Lh, SW505mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
18	SP1586818	Footrest fixed, Rh, SW505mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1

Footrest fixed, fixed frame

Pos.	Part number	Description	Valid from → until	Qty
19	SP1601764	Footrest fixed, Lh, SW530mm, standard tube, fixed frame + Heel strap <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i>		1
20	SP1601783	Footrest fixed, Rh, SW530mm, standard tube, fixed frame + Heel strap <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i>		1
21	SP1590970	Footrest fixed, Lh, SW305mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
22	SP1590971	Footrest fixed, Rh, SW305mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
23	SP1590980	Footrest fixed, Lh, SW330mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
24	SP1590981	Footrest fixed, Rh, SW330mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
25	SP1590982	Footrest fixed, Lh, SW355mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
26	SP1590989	Footrest fixed, Rh, SW355mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
27	SP1590990	Footrest fixed, Lh, SW380mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
28	SP1590991	Footrest fixed, Rh, SW380mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
29	SP1590992	Footrest fixed, Lh, SW405mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
30	SP1590993	Footrest fixed, Rh, SW405mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
31	SP1590994	Footrest fixed, Lh, SW430mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
32	SP1590995	Footrest fixed, Rh, SW430mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
33	SP1590996	Footrest fixed, Lh, SW455mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
34	SP1590997	Footrest fixed, Rh, SW455mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
35	SP1590998	Footrest fixed, Lh, SW480mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
36	SP1591018	Footrest fixed, Rh, SW480mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
37	SP1591019	Footrest fixed, Lh, SW505mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
38	SP1591031	Footrest fixed, Rh, SW505mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
39	SP1601784	Footrest fixed, Lh, SW530mm, long tube, fixed frame + Heel strap <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i>		1
40	SP1601785	Footrest fixed, Rh, SW530mm, long tube, fixed frame + Heel strap <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i>		1
41	0572-04433	Footplate, fixed, Lh, 133mm		1
42	0572-04447	Footplate, fixed, Rh, 133mm		1
43	0572-04434	Footplate, fixed, Lh, 163mm		1
44	0572-04448	Footplate, fixed, Rh, 163mm		1
45	1529643	Footplate, fixed, Lh, 174mm (V2009)		1
46	1529642	Footplate, fixed, Rh, 174mm (V2009)		1
47	1529645	Footplate, fixed, Lh, 199mm (V2009)		1
48	1529644	Footplate, fixed, Rh, 199mm (V2009)		1
49	1529647	Footplate, fixed, Lh, 224mm (V2009)		1
50	1529646	Footplate, fixed, Rh, 224mm (V2009)		1
51	SP1511849-0010	Footrest tube standard straight for fixed frame <i>[SJ_0210+SJ_3440]</i>		2
52	SP1506244-0699	Footrest tube long straight for fixed frame <i>[SJ_0210+SJ_3450]</i>		2
53	1434612	Kit Fixation Footrest Tube		2

Footrest fixed, high mounted tube, swing away frame

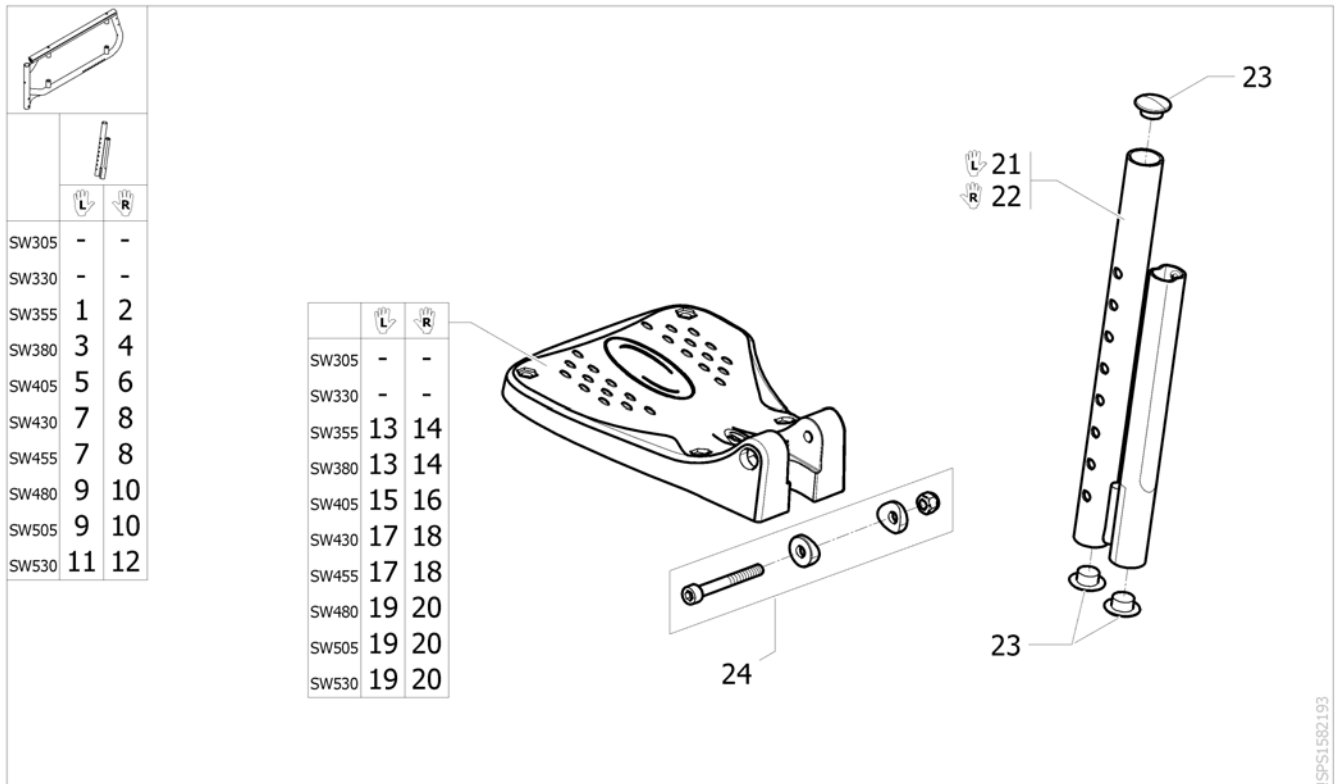


Pos.	Part number	Description	Valid from → until	Qty
1	SP1587842	Footrest fixed, Lh, SW355mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400]		1
2	SP1587843	Footrest fixed, Rh, SW355mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400]		1
3	SP1587844	Footrest fixed, Lh, SW380mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400]		1
4	SP1587845	Footrest fixed, Rh, SW380mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400]		1
5	SP1588156	Footrest fixed, Lh, SW405mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400]		1
6	SP1588157	Footrest fixed, Rh, SW405mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400]		1
7	1532070	Footrest fixed, Lh, SW430-455mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400]		1
8	1532066	Footrest fixed, Rh, SW430-455mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400]		1
9	1532072	Footrest fixed, Lh, SW480-505mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400]		1
10	1532068	Footrest fixed, Rh, SW480-505mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400]		1
11	SP1601786	Footrest fixed, Lh, SW530mm, high mounted tube + Heel strap [(SJ_3170/SJ_3300)+SJ_3400]		1
12	SP1601787	Footrest fixed, Rh, SW530mm, high mounted tube + Heel strap [(SJ_3170/SJ_3200)+SJ_3400]		1
13	0572-04433	Footplate, fixed, Lh, 133mm		1
14	0572-04447	Footplate, fixed, Rh, 133mm		1
15	0572-04434	Footplate, fixed, Lh, 163mm		1
16	0572-04448	Footplate, fixed, Rh, 163mm		1
17	1529643	Footplate, fixed, Lh, 174mm (V2009)		1
18	1529642	Footplate, fixed, Rh, 174mm (V2009)		1
19	1529645	Footplate, fixed, Lh, 199mm (V2009)		1
20	1529644	Footplate, fixed, Rh, 199mm (V2009)		1
21	1531202-0699	Footrest tube high mounted Lh [SJ_3400]		1

Footrest fixed, high mounted tube, swing away frame

Pos.	Part number	Description	Valid from → until	Qty
22	1531203-0699	Footrest tube high mounted Rh <i>[SJ_3400]</i>		1
23	1424576	Plastic cap black Ø20mm		6
24	1430433	Kit screws for fixed footrest		2

Footrest angle & depth adjustable, swing away frame

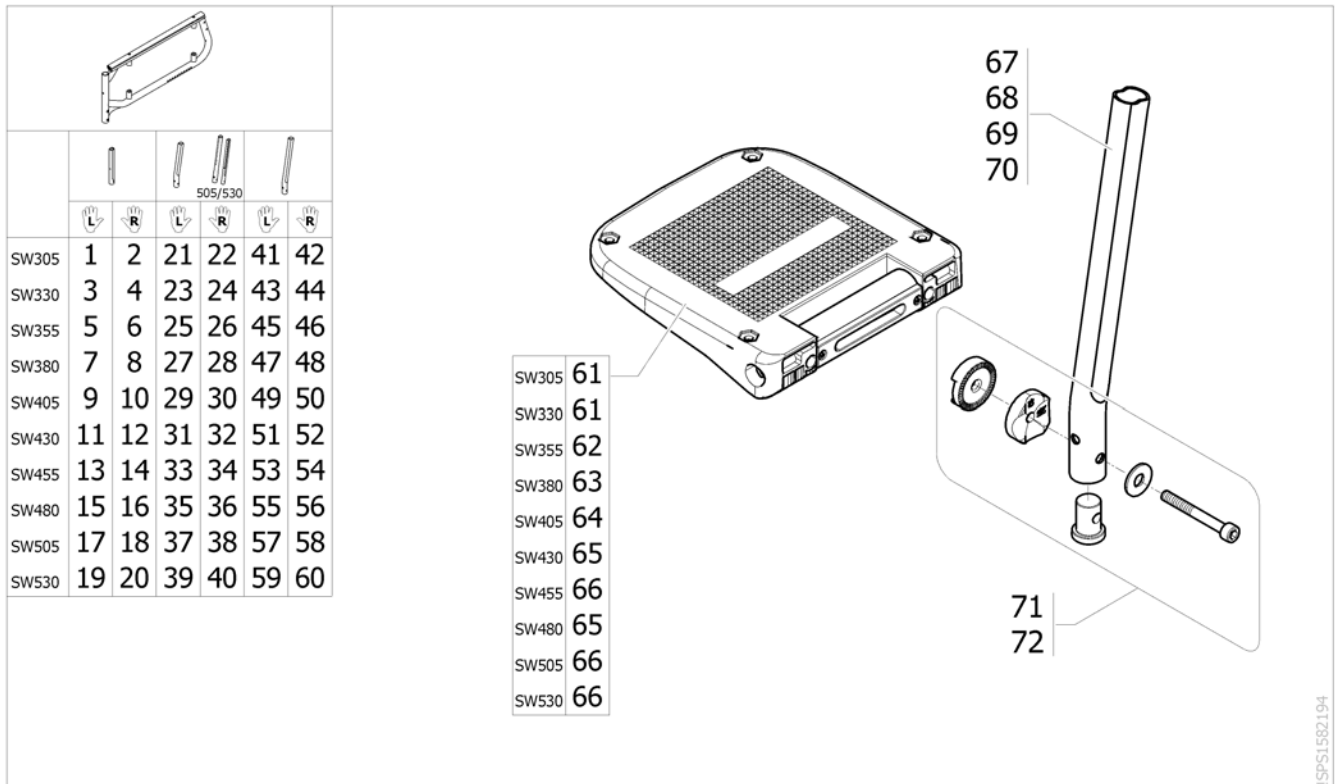

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587834	Footrest a&d adjustable, Lh, SW305mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
2	SP1587835	Footrest a&d adjustable, Rh, SW305mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1
3	SP1587836	Footrest a&d adjustable, Lh, SW330mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
4	SP1587837	Footrest a&d adjustable, Rh, SW330mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1
5	SP1587838	Footrest a&d adjustable, Lh, SW355mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
6	SP1587839	Footrest a&d adjustable, Rh, SW355mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1
7	1548477	Footrest a&d adjustable, Lh, SW380mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
8	1548478	Footrest a&d adjustable, Rh, SW380mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1
9	1548487	Footrest a&d adjustable, Lh, SW405mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
10	1548488	Footrest a&d adjustable, Rh, SW405mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1
11	1548489	Footrest a&d adjustable, Lh, SW430mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
12	1548490	Footrest a&d adjustable, Rh, SW430mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1
13	1548491	Footrest a&d adjustable, Lh, SW455mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
14	1548492	Footrest a&d adjustable, Rh, SW455mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1
15	1548518	Footrest a&d adjustable, Lh, SW480mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
16	1548519	Footrest a&d adjustable, Rh, SW480mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1
17	1548520	Footrest a&d adjustable, Lh, SW505mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
18	1548521	Footrest a&d adjustable, Rh, SW505mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1


Footrest angle & depth adjustable, swing away frame

Pos.	Part number	Description	Valid from → until	Qty
19	SP1601788	Footrest a&d adjustable, Lh, SW530mm, short tube, s/a frame + heel <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i>		1
20	SP1601789	Footrest a&d adjustable, Rh, SW530mm, short tube, s/a frame + heel strap <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i>		1
21	SP1587406	Footrest a&d adjustable, Lh, SW305mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
22	SP1587438	Footrest a&d adjustable, Rh, SW305mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
23	SP1587439	Footrest a&d adjustable, Lh, SW330mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
24	SP1587440	Footrest a&d adjustable, Rh, SW330mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
25	SP1587441	Footrest a&d adjustable, Lh, SW355mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
26	SP1587442	Footrest a&d adjustable, Rh, SW355mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
27	1428657	Footrest a&d adjustable, Lh, SW380mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
28	1428651	Footrest a&d adjustable, Rh, SW380mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
29	1428658	Footrest a&d adjustable, Lh, SW405mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
30	1428652	Footrest a&d adjustable, Rh, SW405mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
31	1428659	Footrest a&d adjustable, Lh, SW430mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
32	1428653	Footrest a&d adjustable, Rh, SW430mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
33	1428660	Footrest a&d adjustable, Lh, SW455mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
34	1428654	Footrest a&d adjustable, Rh, SW455mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
35	1428661	Footrest a&d adjustable, Lh, SW480mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
36	1428655	Footrest a&d adjustable, Rh, SW480mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
37	1532964	Footrest a&d adjustable, Lh, SW505mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
38	1532963	Footrest a&d adjustable, Rh, SW505mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
39	SP1601790	Footrest a&d adjustable, Lh, SW530mm, standard tube, s/a frame + heel strap <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3420]</i>		1
40	SP1601791	Footrest a&d adjustable, Rh, SW530mm, standard tube, s/a frame + heel strap <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3420]</i>		1
41	SP1587443	Footrest a&d adjustable, Lh, SW305mm, long tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]</i>		1
42	SP1587444	Footrest a&d adjustable, Rh, SW305mm, long tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]</i>		1
43	SP1587445	Footrest a&d adjustable, Lh, SW330mm, long tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]</i>		1
44	SP1587446	Footrest a&d adjustable, Rh, SW330mm, long tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]</i>		1
45	SP1587447	Footrest a&d adjustable, Lh, SW355mm, long tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]</i>		1

Footrest angle & depth adjustable, swing away frame

Pos.	Part number	Description	Valid from → until	Qty
46	SP1587448	Footrest a&d adjustable, Rh, SW355mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]		1
47	1440816	Footrest a&d adjustable, Lh, SW380mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]		1
48	1440810	Footrest a&d adjustable, Rh, SW380mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]		1
49	1440817	Footrest a&d adjustable, Lh, SW405mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]		1
50	1440811	Footrest a&d adjustable, Rh, SW405mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]		1
51	1440818	Footrest a&d adjustable, Lh, SW430mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]		1
52	1440812	Footrest a&d adjustable, Rh, SW430mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]		1
53	1440819	Footrest a&d adjustable, Lh, SW455mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]		1
54	1440813	Footrest a&d adjustable, Rh, SW455mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]		1
55	1440820	Footrest a&d adjustable, Lh, SW480mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]		1
56	1440814	Footrest a&d adjustable, Rh, SW480mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]		1
57	1440821	Footrest a&d adjustable, Lh, SW505mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]		1
58	1440815	Footrest a&d adjustable, Rh, SW505mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]		1
59	SP1601792	Footrest a&d adjustable, Lh, SW530mm, long tube, s/a frame + heel strap [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430]		1
60	SP1601793	Footrest a&d adjustable, Rh, SW530mm, long tube, s/a frame + heel strap [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430]		1
61	5106371	Footplate, angle and depht adjustable, 106 mm		2
62	5106373	Footplate, angle and depth adjustable, 131 mm		2
63	5106374	Footplate, angle and depth adjustable, 141 mm		2
64	5106375	Footplate, angle and depth adjustable, 156 mm		2
65	5106594	Footplate, angle and depth adjustable, 166 mm		1
66	5106376	Footplate, angle and depth adjustable, 176 mm		1
67	1541994	Short footrest tube for a&d adjustable footrest [SJ_0200+SJ_3410]		2
68	1558855-0010	Standard footrest tube for a&d adjustable footrest [SJ_0200+(SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040/SJ_1050/SJ_1060/SJ_1070)+SJ_3420]		2
69	1533719	Standard footrest tube with heavy bar [SJ_0200+(SJ_1075/SJ_1078/SJ_1080/SJ_1090)+SJ_3420]		1
70	1434563-0010	Long tube for fixed and a&d adjustable footrest [SJ_0200+SJ_3430]		2
71	1430839	Kit fixation, adjustable footplate, short spacer		2
72	1430840	Kit fixation, adjustable footplate, long spacer		2

Footrest angle & depth adjustable, fixed frame

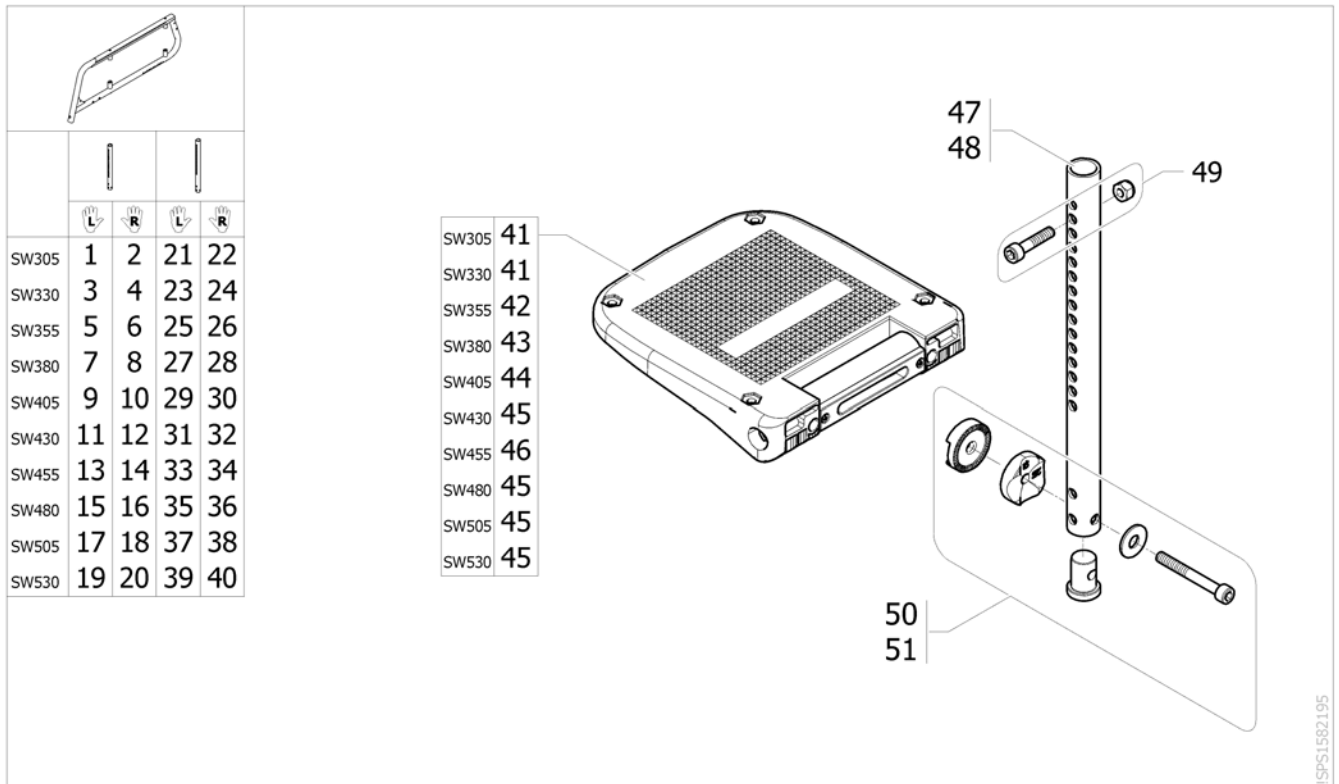


Pos.	Part number	Description	Valid from → until	Qty
1	SP1591045	Footrest a&d adjustable, Lh, SW305mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
2	SP1591046	Footrest a&d adjustable, Rh, SW305mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1
3	SP1591047	Footrest a&d adjustable, Lh, SW330mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
4	SP1591052	Footrest a&d adjustable, Rh, SW330mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1
5	SP1591053	Footrest a&d adjustable, Lh, SW355mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
6	SP1591054	Footrest a&d adjustable, Rh, SW355mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1
7	SP1591055	Footrest a&d adjustable, Lh, SW380mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
8	SP1591056	Footrest a&d adjustable, Rh, SW380mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1
9	SP1587513	Footrest a&d adjustable, Lh, SW405mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
10	SP1587604	Footrest a&d adjustable, Rh, SW405mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1
11	SP1587615	Footrest a&d adjustable, Lh, SW430mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
12	SP1587616	Footrest a&d adjustable, Rh, SW430mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1
13	SP1587617	Footrest a&d adjustable, Lh, SW455mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
14	SP1587698	Footrest a&d adjustable, Rh, SW455mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1
15	SP1587699	Footrest a&d adjustable, Lh, SW480mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
16	SP1587747	Footrest a&d adjustable, Rh, SW480mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1
17	SP1587748	Footrest a&d adjustable, Lh, SW505mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
18	SP1587749	Footrest a&d adjustable, Rh, SW505mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1

Footrest angle & depth adjustable, fixed frame


Pos.	Part number	Description	Valid from → until	Qty
19	SP1590816	Footrest a&d adjustable, Lh, SW530mm, standard tube, fixed frame +heel strap <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i>		1
20	SP1590817	Footrest a&d adjustable, Rh, SW530mm, standard tube, fixed frame +heel strap <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i>		1
21	SP1587451	Footrest a&d adjustable, Lh, SW305mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
22	SP1587452	Footrest a&d adjustable, Rh, SW305mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
23	SP1587453	Footrest a&d adjustable, Lh, SW330mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
24	SP1587454	Footrest a&d adjustable, Rh, SW330mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
25	SP1587455	Footrest a&d adjustable, Lh, SW355mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
26	SP1587456	Footrest a&d adjustable, Rh, SW355mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
27	SP1587457	Footrest a&d adjustable, Lh, SW380mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
28	SP1587458	Footrest a&d adjustable, Rh, SW380mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
29	SP1587459	Footrest a&d adjustable, Lh, SW405mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
30	SP1587460	Footrest a&d adjustable, Rh, SW405mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
31	SP1587461	Footrest a&d adjustable, Lh, SW430mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
32	SP1587462	Footrest a&d adjustable, Rh, SW430mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
33	SP1587463	Footrest a&d adjustable, Lh, SW455mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
34	SP1587464	Footrest a&d adjustable, Rh, SW455mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
35	SP1587465	Footrest a&d adjustable, Lh, SW480mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
36	SP1587466	Footrest a&d adjustable, Rh, SW480mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
37	SP1587467	Footrest a&d adjustable, Lh, SW505mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
38	SP1587468	Footrest a&d adjustable, Rh, SW505mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
39	SP1601794	Footrest a&d adjustable, Lh, SW530mm, long tube, fixed frame + heel strap <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i>		1
40	SP1601795	Footrest a&d adjustable, Rh, SW530mm, long tube, fixed frame + heel strap <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i>		1
41	5106371	Footplate, angle and depht adjustable, 106 mm		2
42	5106373	Footplate, angle and depth adjustable, 131 mm		2
43	5106374	Footplate, angle and depth adjustable, 141 mm		2
44	5106375	Footplate, angle and depth adjustable, 156 mm		2
45	5106594	Footplate, angle and depth adjustable, 166 mm		2
46	5106376	Footplate, angle and depth adjustable, 176 mm		2
47	SP1511849-0010	Footrest tube standard straight for fixed frame <i>[SJ_0210+SJ_3440]</i>		2
48	SP1506244-0699	Footrest tube long straight for fixed frame <i>[SJ_0210+SJ_3450]</i>		2
49	SP1602122	Kit fixation, footrest adj tube with fixed frame		2
50	1430839	Kit fixation, adjustable footplate, short spacer <i>[SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040/SJ_1050/SJ_1060]</i>		2
51	1430840	Kit fixation, adjustable footplate, long spacer <i>[SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090]</i>		2

Footrest angle & depth adjustable, high mounted tube, swing away frame

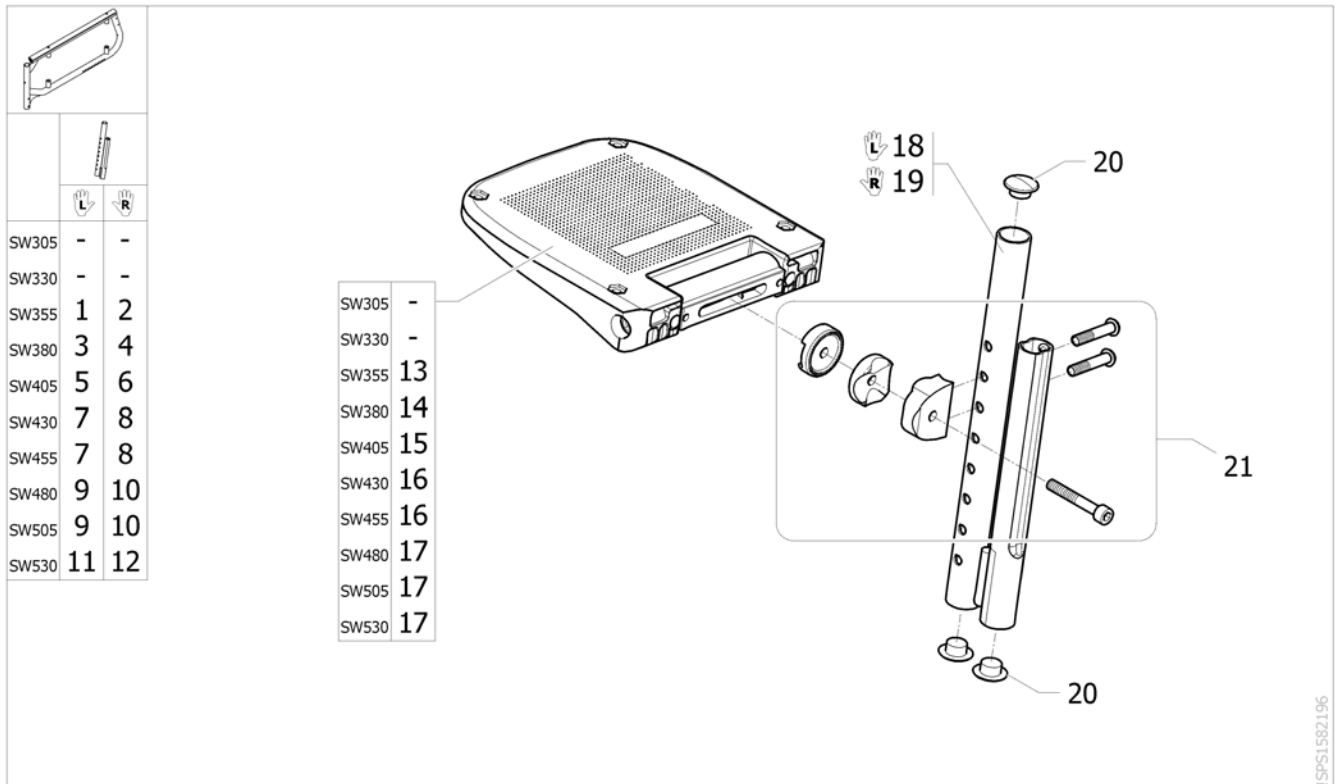

Pos.	Part number	Description	Valid from → until	Qty
1	SP1588277	Footrest a&d adjustable, Lh, SW355mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400]		1
2	SP1588278	Footrest a&d adjustable, Rh, SW355mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400]		1
3	SP1586054	Footrest a&d adjustable, Lh, SW380mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400]		1
4	SP1586052	Footrest a&d adjustable, Rh, SW380mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400]		1
5	SP1586068	Footrest a&d adjustable, Lh, SW405mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400]		1
6	SP1586053	Footrest a&d adjustable, Rh, SW405mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400]		1
7	1532078	Footrest a&d adjustable, Lh, SW430-455mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400]		1
8	1532074	Footrest a&d adjustable, Rh, SW430-455mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400]		1
9	1532080	Footrest a&d adjustable, Lh, SW480-505mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400]		1
10	1532076	Footrest a&d adjustable, Rh, SW480-505mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400]		1
11	SP1601796	Footrest a&d adjustable, Lh, SW530mm, high mounted tube + heel strap [(SJ_3175/SJ_3310)+SJ_3400]		1
12	SP1601797	Footrest a&d adjustable, Rh, SW530mm, high mounted tube + heel strap [(SJ_3175/SJ_3210)+SJ_3400]		1
13	1518638	Footplate, angle and depth adjustable, 82 mm		1
14	5106371	Footplate, angle and depth adjustable, 106 mm		1
15	5106373	Footplate, angle and depth adjustable, 131 mm		1
16	5106374	Footplate, angle and depth adjustable, 141 mm		1
17	5106594	Footplate, angle and depth adjustable, 166 mm		1
18	1531202-0699	Footrest tube high mounted Lh [SJ_3400]		1
19	1531203-0699	Footrest tube high mounted Rh [SJ_3400]		1
20	1424576	Plastic cap black Ø20mm		6
21	1533714	Kit fixation for angle & depth adjustable footrest, high mounted tube		2

One piece footplate for fixed frame

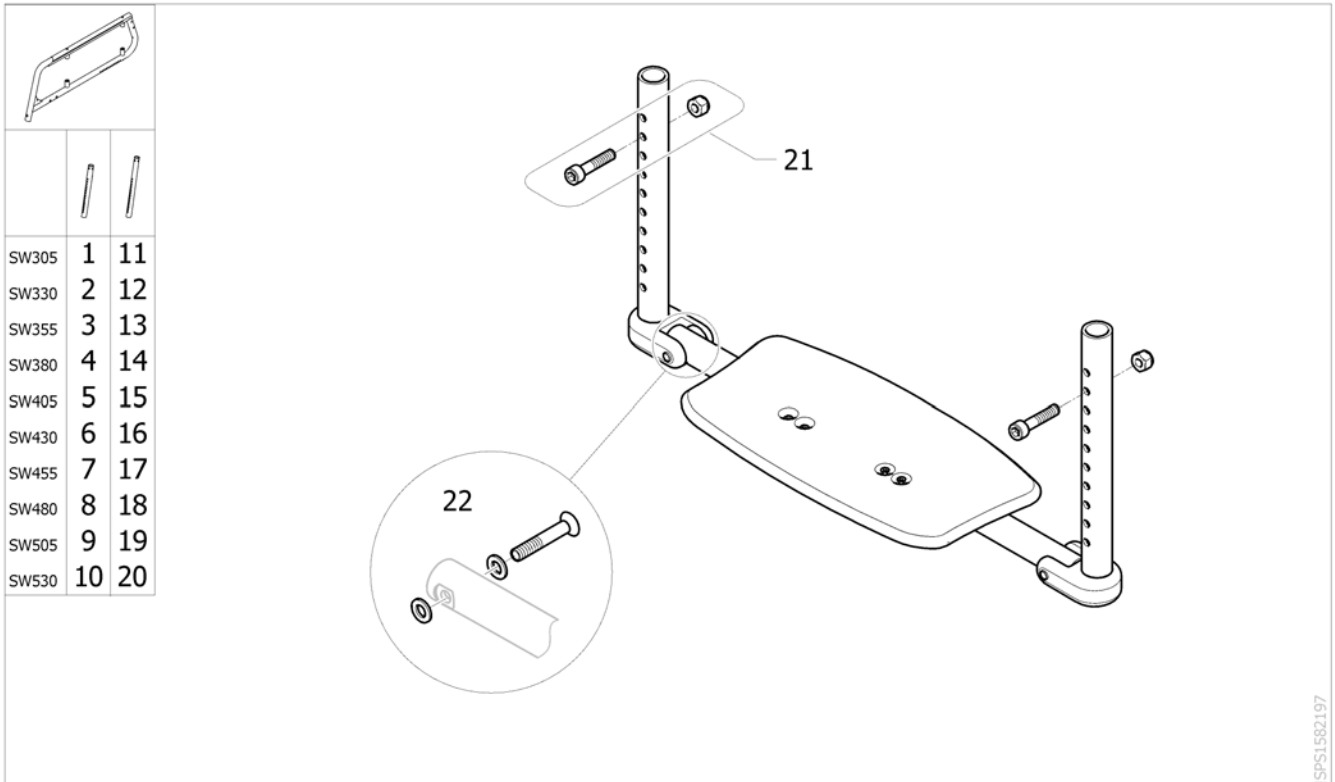


Pos.	Part number	Description	Valid from → until	Qty
1	SP1587750	One piece footplate, standard tube, SW305 mm Action 5 <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
2	SP1587751	One piece footplate, standard tube, SW330 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
3	SP1587767	One piece footplate, standard tube, SW355 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
4	SP1587768	One piece footplate, standard tube, SW380 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
5	SP1587769	One piece footplate, standard tube, SW405 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
6	SP1587770	One piece footplate, standard tube, SW430 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
7	SP1587771	One piece footplate, standard tube, SW455 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
8	SP1587813	One piece footplate, standard tube, SW480 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
9	SP1587814	One piece footplate, standard tube, SW505 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
10	SP1587818	One piece footplate, standard tube, SW530 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i>		1
11	SP1587824	One piece footplate, long tube, SW305 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1
12	SP1587825	One piece footplate, long tube, SW330 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1
13	SP1587826	One piece footplate, long tube, SW355 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1
14	SP1587827	One piece footplate, long tube, SW380 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1
15	SP1587828	One piece footplate, long tube, SW405 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1
16	SP1587829	One piece footplate, long tube, SW430 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1
17	SP1587830	One piece footplate, long tube, SW455 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1
18	SP1587831	One piece footplate, long tube, SW480 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1

One piece footplate for fixed frame

Pos.	Part number	Description	Valid from → until	Qty
19	SP1587832	One piece footplate, long tube, SW505 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1
20	SP1587833	One piece footplate, long tube, SW530 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i>		1
21	SP1602218	Kit fixation, footrest Monobloc with fixed frame		2
22	SP1606180	Fixing kit for support footrest monobloc		2



Siderests

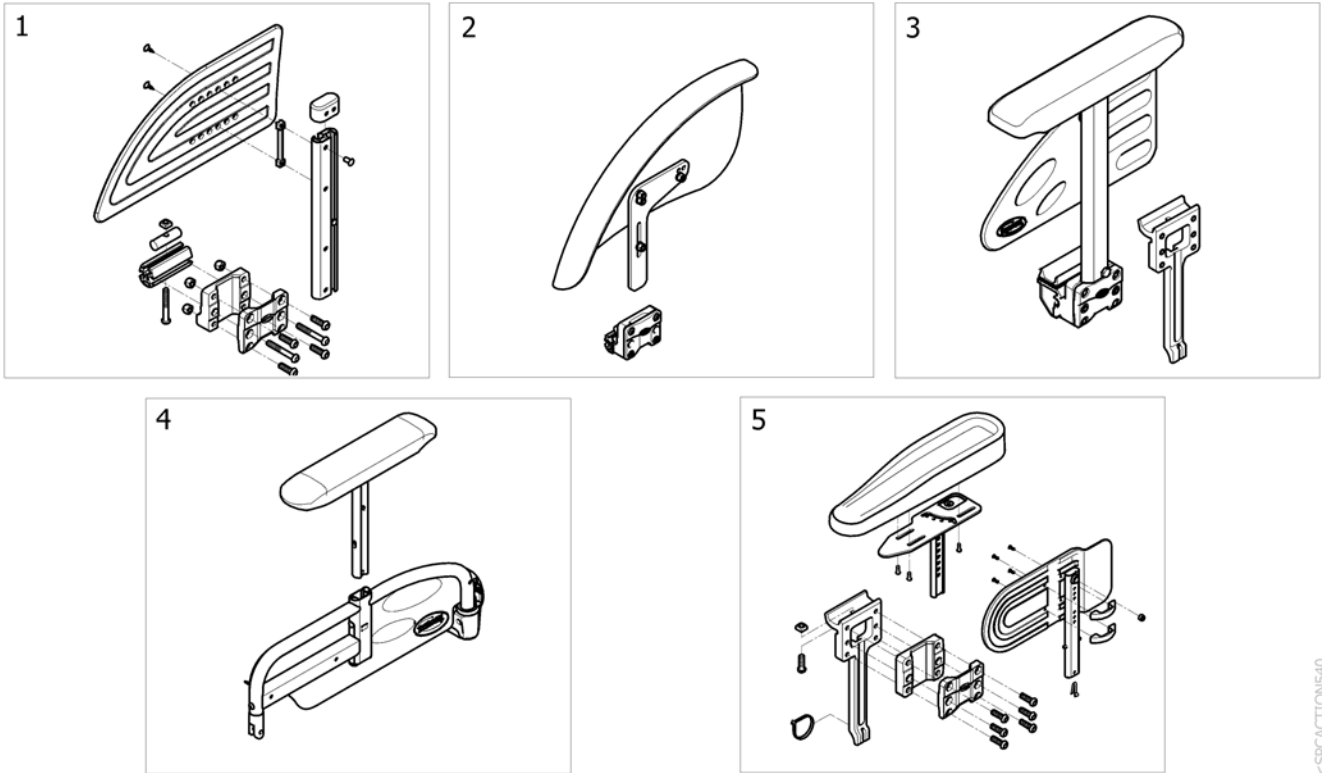

->SPCACTIONS40

Pos.	Part number	Description	Valid from → until	Qty
1	-	Side guard [SJ_3750]		1
2	-	Mudguard [SJ_3800]		1
3	-	Armrest n°1, Removable, height adjustable T-arm, desk armpad (280mm) [SJ_3910/SJ_4010/SJ_4110]		1
5	-	Hemi Dartex armpad multiposition standard [SJ_3960/SJ_4060/SJ_4160]		1
5	-	Armrest n°3, Swing away detachable & height adjustable, desk armpad (280mm) [SJ_3930/SJ_4030/SJ_4130]		1


Side guard

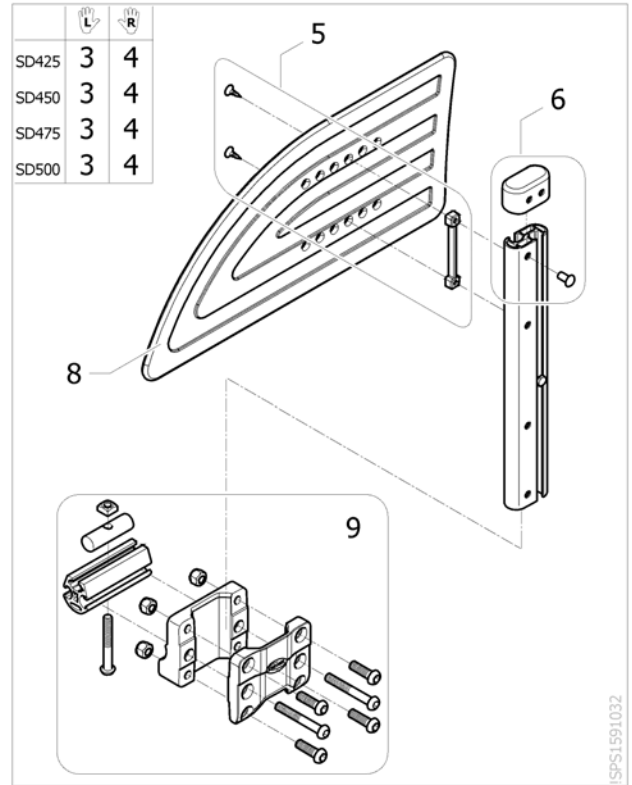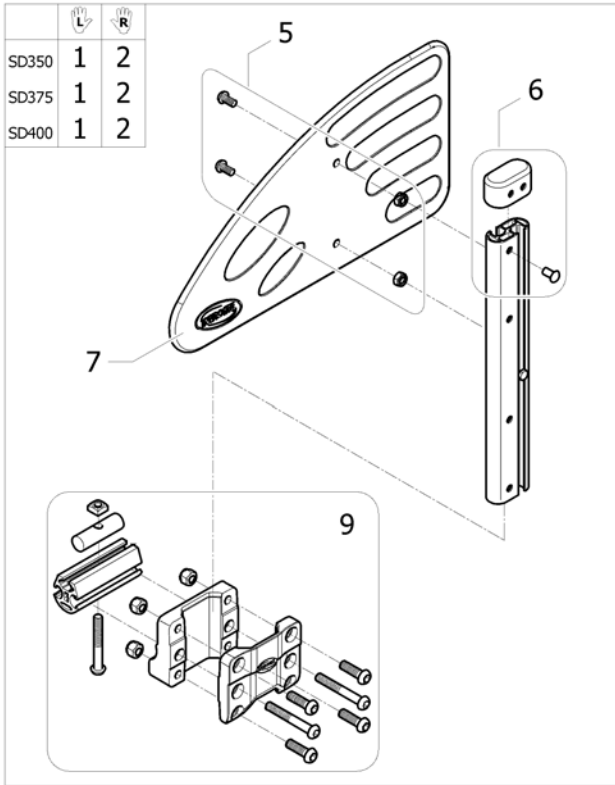

ISPS1591032

Pos.	Part number	Description	Valid from → until	Qty
1	SP1601490	Side Guard, Lh, large, SD350-400mm <i>[SJ_3750]</i>		1
2	SP1601491	Side Guard, Rh, large, SD350-400mm <i>[SJ_3750]</i>		1
3	SP1601492	Side Guard, Lh, extra large, SD425-500mm <i>[SJ_3750]</i>		1
4	SP1601493	Side Guard, Rh, extra large, SD425-500mm <i>[SJ_3750]</i>		1
5	SP1601494	Kit fixation skirtguard & armpad		2
6	1513515	Kit skirtguard Junior cap		2
7	1425601	Skirtguard black for armrest 1		2
8	1434725	Skirtguard Rio height adjustable		2
9	SP1601838	Sideguard support		2

Mudguard

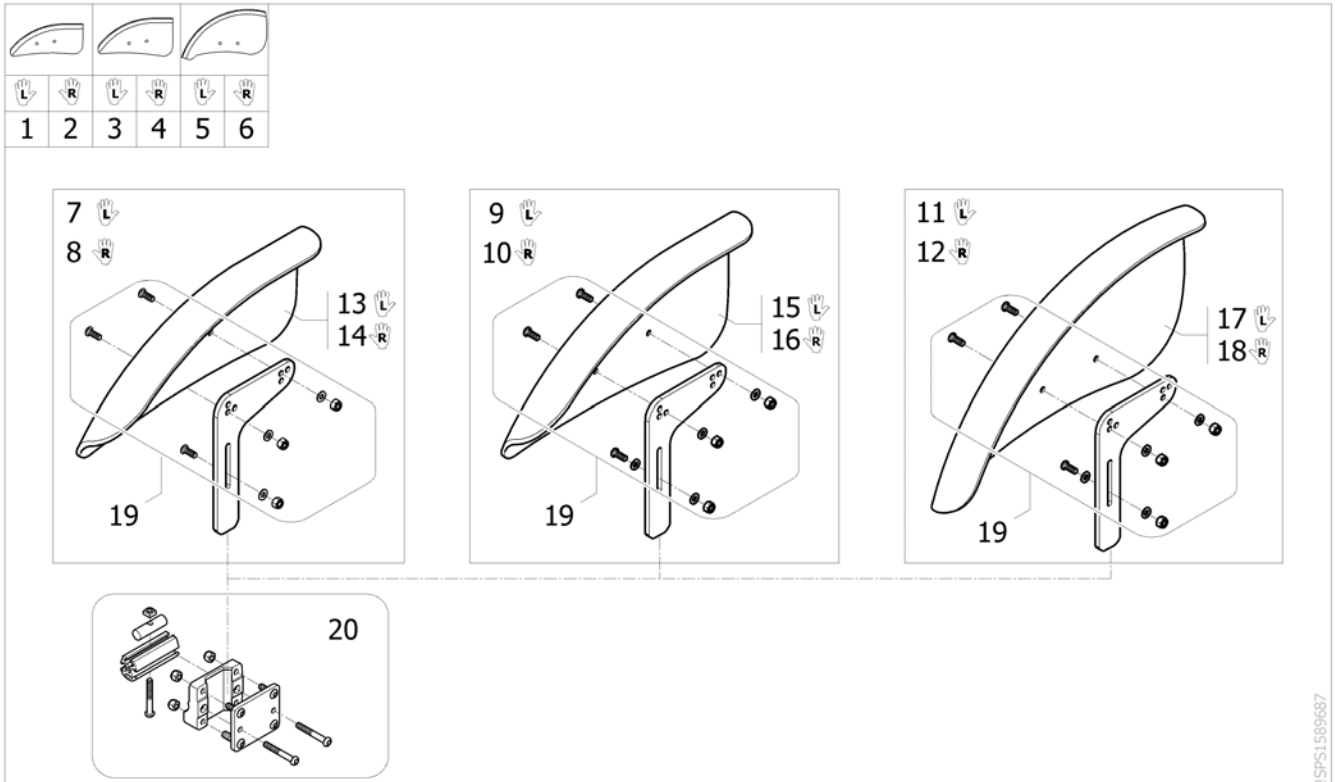


ISPS1599687

Pos.	Part number	Description	Valid from → until	Qty
1	SP1590444	Mudguard, medium, complete, Lh [(SJ_1330/SJ_1340)+(SJ_7200/SJ_7210/SJ_7220)] [SJ_1350+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1360+(SJ_7250/SJ_7260)]		1
2	SP1590445	Mudguard, medium, complete, Rh [(SJ_1330/SJ_1340)+(SJ_7200/SJ_7210/SJ_7220)] [SJ_1350+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1360+(SJ_7250/SJ_7260)]		1
3	SP1590446	Mudguard, large, complete, Lh [SJ_1320+(SJ_7200/SJ_7210/SJ_7220)] [(SJ_1330/SJ_1340)+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1350+(SJ_7250/SJ_7260)]		1
4	SP1590447	Mudguard, large, complete, Rh [SJ_1320+(SJ_7200/SJ_7210/SJ_7220)] [(SJ_1330/SJ_1340)+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1350+(SJ_7250/SJ_7260)]		1
5	SP1590448	Mudguard, extra large, complete, Lh [SJ_1300/SJ_1310] [SJ_1320+(SJ_7230/SJ_7240/SJ_7242/SJ_7244/SJ_7250/SJ_7260)] [(SJ_1330/SJ_1340)+(SJ_7250/SJ_7260)]		1
6	SP1590449	Mudguard, extra large, complete, Rh [SJ_1300/SJ_1310] [SJ_1320+(SJ_7230/SJ_7240/SJ_7242/SJ_7244/SJ_7250/SJ_7260)] [(SJ_1330/SJ_1340)+(SJ_7250/SJ_7260)]		1
7	SP1601477	Kit mudguard, medium, Lh, + L support [(SJ_1330/SJ_1340)+(SJ_7200/SJ_7210/SJ_7220)] [SJ_1350+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1360+(SJ_7250/SJ_7260)]		1
8	SP1601478	Kit mudguard, medium, Rh, + L support [(SJ_1330/SJ_1340)+(SJ_7200/SJ_7210/SJ_7220)] [SJ_1350+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1360+(SJ_7250/SJ_7260)]		1
9	SP1601479	Kit mudguard, large, Lh, + L support [SJ_1320+(SJ_7200/SJ_7210/SJ_7220)] [(SJ_1330/SJ_1340)+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1350+(SJ_7250/SJ_7260)]		1
10	SP1601480	Kit mudguard, large, Rh, + L support [SJ_1320+(SJ_7200/SJ_7210/SJ_7220)] [(SJ_1330/SJ_1340)+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1350+(SJ_7250/SJ_7260)]		1
11	SP1601481	Kit mudguard, extra large, Lh, + L support [SJ_1300/SJ_1310] [SJ_1320+(SJ_7230/SJ_7240/SJ_7242/SJ_7244/SJ_7250/SJ_7260)] [(SJ_1330/SJ_1340)+(SJ_7250/SJ_7260)]		1

Mudguard


Pos.	Part number	Description	Valid from → until	Qty
12	SP1601482	Kit mudguard, extra large, Rh, + L support [SJ_1300/SJ_1310] [SJ_1320+(SJ_7230/SJ_7240/SJ_7242/SJ_7244/SJ_7250/SJ_7260)] [(SJ_1330/SJ_1340)+(SJ_7250/SJ_7260)]		2
13	SP1586515	Mudguard, medium, Lh [(SJ_1330/SJ_1340)+(SJ_7200/SJ_7210/SJ_7220)] [SJ_1350+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1360+(SJ_7250/SJ_7260)]		1
14	SP1586516	Mudguard, medium, Rh [(SJ_1330/SJ_1340)+(SJ_7200/SJ_7210/SJ_7220)] [SJ_1350+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1360+(SJ_7250/SJ_7260)]		1
15	SP1586517	Mudguard, large, Lh [SJ_1320+(SJ_7200/SJ_7210/SJ_7220)] [(SJ_1330/SJ_1340)+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1350+(SJ_7250/SJ_7260)]		1
16	SP1586518	Mudguard, large, Rh [SJ_1320+(SJ_7200/SJ_7210/SJ_7220)] [(SJ_1330/SJ_1340)+(SJ_7230/SJ_7240/SJ_7242/SJ_7244)] [SJ_1350+(SJ_7250/SJ_7260)]		1
17	SP1584373	Mudguard, extra large, Lh [SJ_1300/SJ_1310] [SJ_1320+(SJ_7230/SJ_7240/SJ_7242/SJ_7244/SJ_7250/SJ_7260)] [(SJ_1330/SJ_1340)+(SJ_7250/SJ_7260)]		1
18	SP1584372	Mudguard, extra large, Rh [SJ_1300/SJ_1310] [SJ_1320+(SJ_7230/SJ_7240/SJ_7242/SJ_7244/SJ_7250/SJ_7260)] [(SJ_1330/SJ_1340)+(SJ_7250/SJ_7260)]		1
19	SP1601485	Kit fixation mudguard with L support		1
20	SP1601486	Mudguard support		2

Armrest n°1, Removable, height adjustable T-arm, desk armpad (280mm)

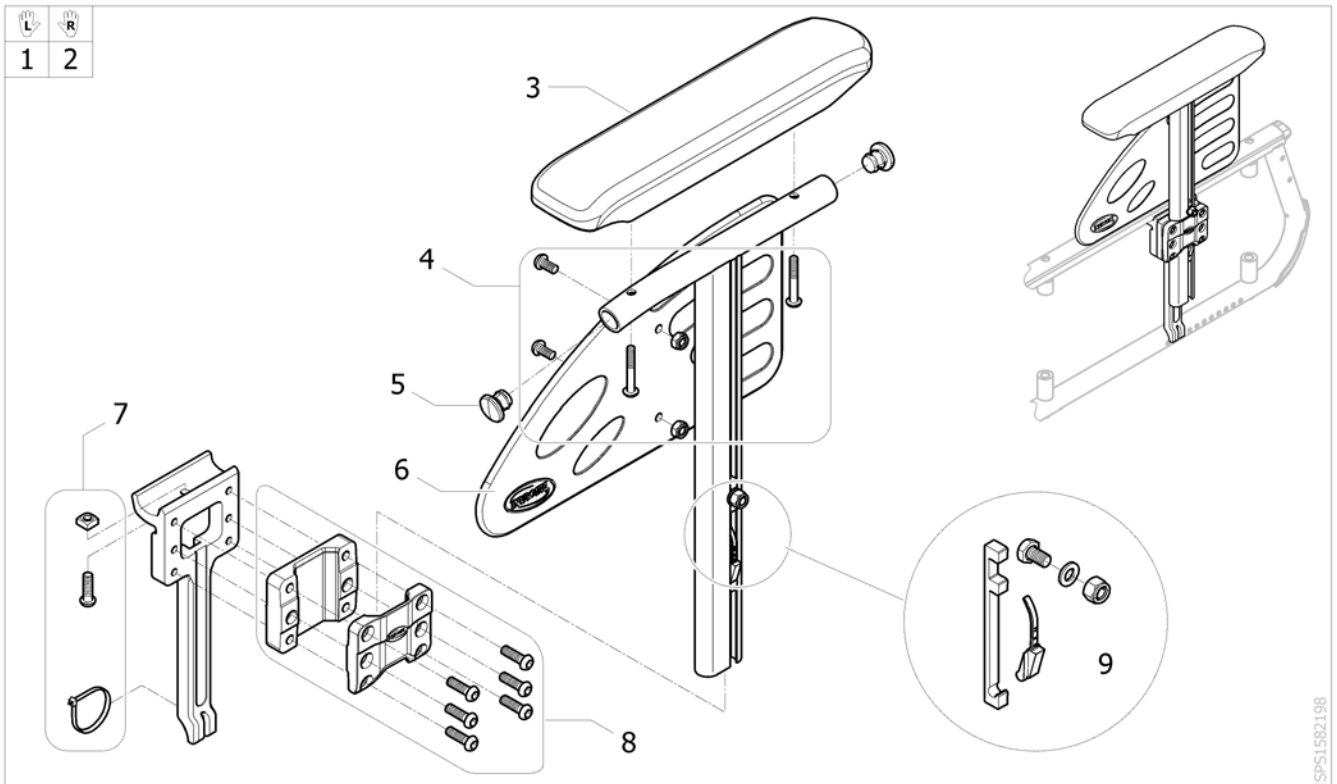

Pos.	Part number	Description	Valid from → until	Qty
1	SP1604616	Armrest 1 Lh, removable and height adjustable, desk armpad (280 mm) <i>[SJ_3910/SJ_4110]</i>		1
2	SP1604617	Armrest 1 Rh, removable and height adjustable, desk armpad (280 mm) <i>[SJ_3910/SJ_4010]</i>		1
3	1543220	Kit short armpad 280mm + 2 screws		2
4	SP1601494	Kit fixation skirtguard & armpad		2
5	0021-20003	Black 18 x 1.5 mm cylindrical cap		4
6	1425601	Skirtguard black for armrest 1		2
7	SP1589824	Kit attachment for reinforced support armrest 1		2
8	SP1589903	Armrest 1 support		2
9	SP1589904	Kit locking system for armrest 1		2


Hemi Dartex armpad multiposition standard

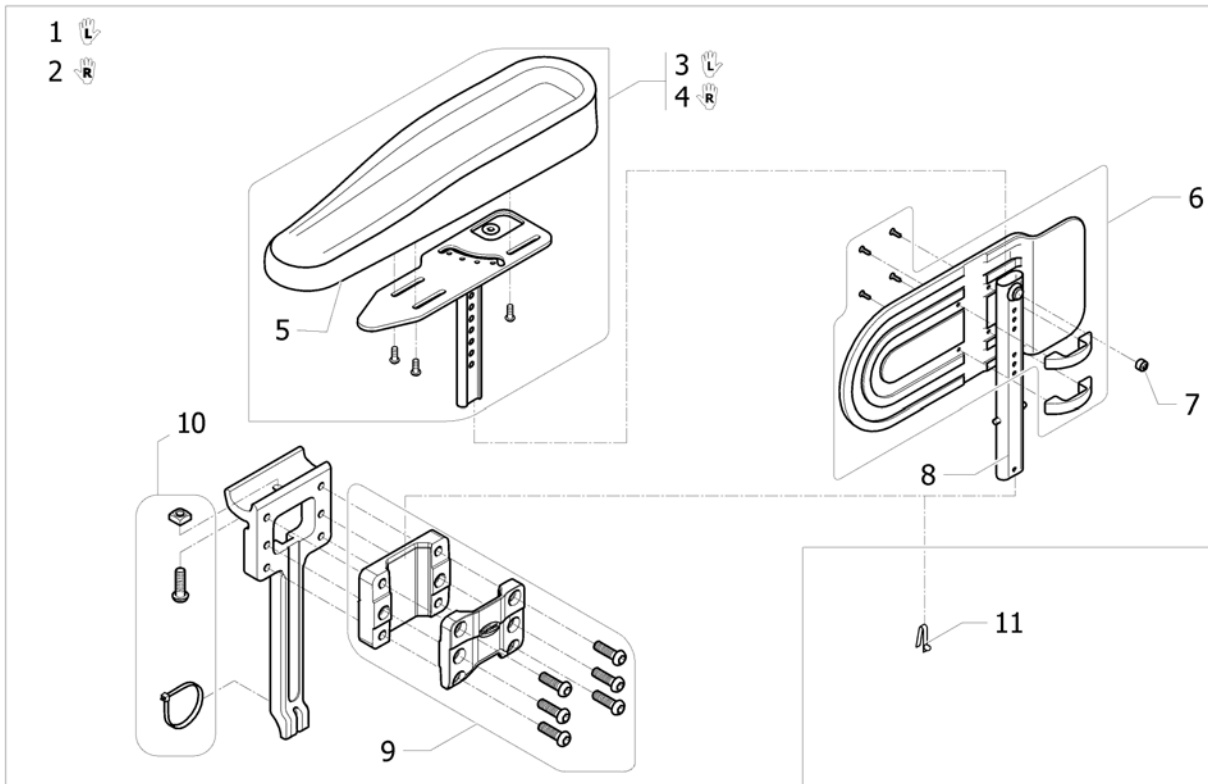

ISPS1589685

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583755	Lh hemi armrest N°4 multiposition Action 5 <i>[SJ_3960/SJ_4160]</i>		1
2	SP1583754	Rh hemi armrest N°4 multiposition Action 5 <i>[SJ_3960/SJ_4060]</i>		1
3	1493220	Lh hemi dartex armpad complete		1
4	1493219	Rh hemi dartex armpad complete		1
5	1493251	Hemi dartex armpad		2
6	1430425	Skirtguard Armrest N°4 Fixation Kit		2
7	5106533	Locking screw M12X10		2
8	1443058	Lowest Arm Rest Tube Support N°4		2
9	SP1589903	Armrest 1 support		2
10	SP1589824	Kit attachment for reinforced support armrest 1		2
11	0027-10003	Spring pin locked armrest		2

Armrest n°3, Swing away detachable & height adjustable, desk armpad (280mm)

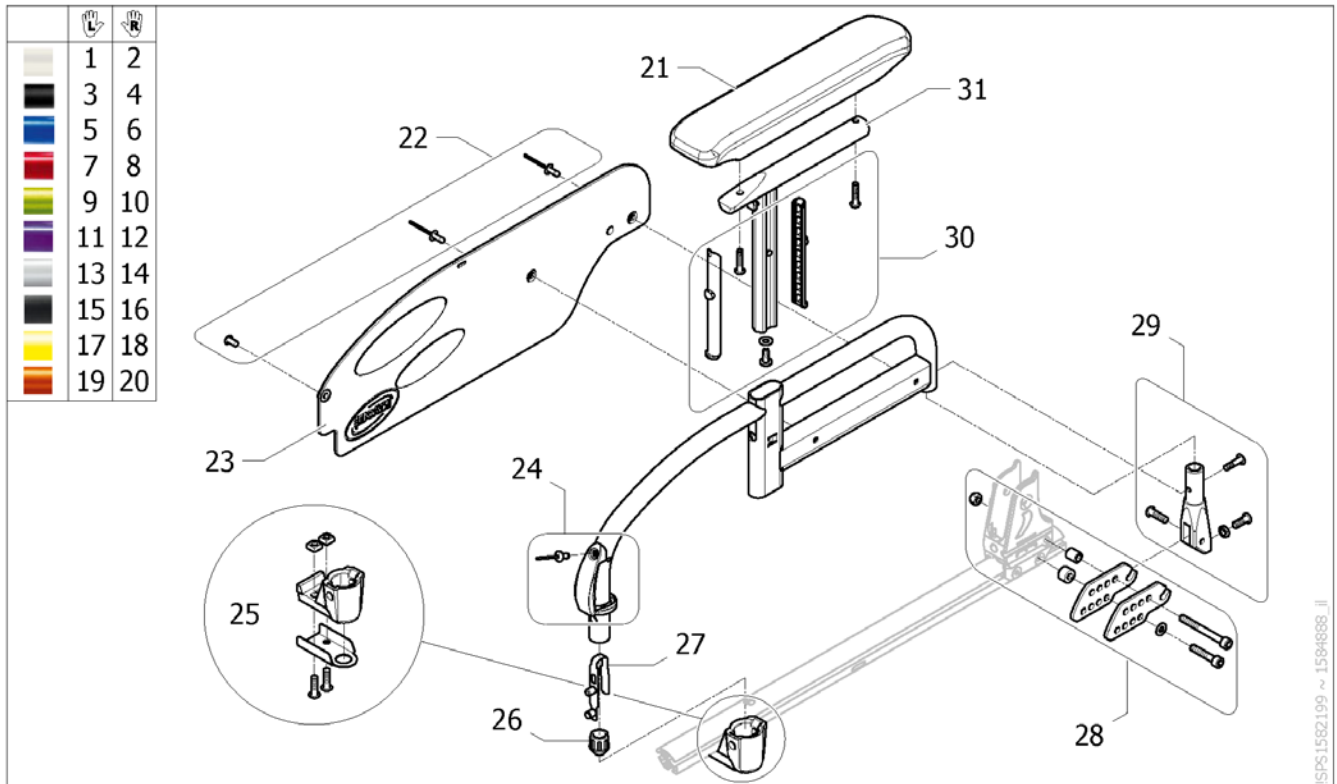


Pos.	Part number	Description	Valid from → until	Qty
1	SP1584889-0637	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Pearl White [[SJ_3930/SJ_4130)+SJ_6000]		2
2	SP1584888-0637	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Pearl White [[SJ_3930/SJ_4030)+SJ_6000]		2
3	SP1584889-0609	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Anthracite [[SJ_3930/SJ_4130)+SJ_6010]		2
4	SP1584888-0609	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Anthracite [[SJ_3930/SJ_4030)+SJ_6010]		2
5	SP1584889-0643	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Pop Blue [[SJ_3930/SJ_4130)+SJ_6020]		2
6	SP1584888-0643	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Pop Blue [[SJ_3930/SJ_4030)+SJ_6020]		2
7	SP1584889-0607	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Dark Red [[SJ_3930/SJ_4130)+SJ_6030]		2
8	SP1584888-0607	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Dark Red [[SJ_3930/SJ_4030)+SJ_6030]		2
9	SP1584889-0644	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Electric Green [[SJ_3930/SJ_4130)+SJ_6040]		2
10	SP1584888-0644	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Electric Green [[SJ_3930/SJ_4030)+SJ_6040]		2
11	SP1584889-0645	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Hypnotic Purple [[SJ_3930/SJ_1130)+SJ_6050]		2
12	SP1584888-0645	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Hypnotic Purple [[SJ_3930/SJ_4030)+SJ_6050]		2

Armrest n°3, Swing away detachable & height adjustable, desk armpad (280mm)

Pos.	Part number	Description	Valid from → until	Qty
13	SP1584889-0515	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Pearl Grey [(SJ_3930/SJ_4130)+SJ_6060]		2
14	SP1584888-0515	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Pearl Grey [(SJ_3930/SJ_4030)+SJ_6060]		2
15	SP1584889-0646	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Matt Black [(SJ_3930/SJ_4130)+SJ_6070]		2
16	SP1584888-0646	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Matt Black [(SJ_3930/SJ_4030)+SJ_6070]		2
17	SP1584889-0647	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Intense Yellow [(SJ_3930/SJ_4130)+SJ_6080]		2
18	SP1584888-0647	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Intense Yellow [(SJ_3930/SJ_4030)+SJ_6080]		2
19	SP1584889-0648	Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Vibrating Orange [(SJ_3930/SJ_4130)+SJ_6090]		2
20	SP1584888-0648	Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Vibrating Orange [(SJ_3930/SJ_4030)+SJ_6090]		2
21	SP1606082	Kit short armpad 280 mm + 2 screw for armrest 3		2
22	SP1601924	Fixing skirtguard kit armrest 3		2
23	1426725	Skirtguard black for armrest 3		2
24	SP1601925	Unlocking button kit armrest 3		2
25	SP1601926	Locking kit armrest3		2
26	0572-03405	Short conical cap		2
27	1018225	Double stud spring		2
28	SP1601927	Armpad support kit armrest 3		2
29	SP1601928	Articulation & crew kit armrest 3		2
30	SP1601929	Plastic insert & screw kit armrest 3		2
31	SP1601930	T support kit, short armpad, armrest 3		2


Front wheels

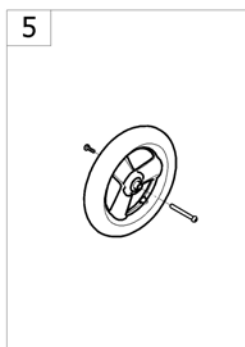

-<SPACED>ACTION550

Pos.	Part number	Description	Valid from → until	Qty
1	-	Forks for swing away frame <i>[SJ_0200]</i>		1
2	-	Forks for fixed frame <i>[SJ_0210]</i>		1
3	-	Forks support for swing away frame <i>[SJ_0200]</i>		1
4	-	Forks support for fixed frame <i>[SJ_0210]</i>		1
5	-	Front wheels		1

Forks for swing away frame

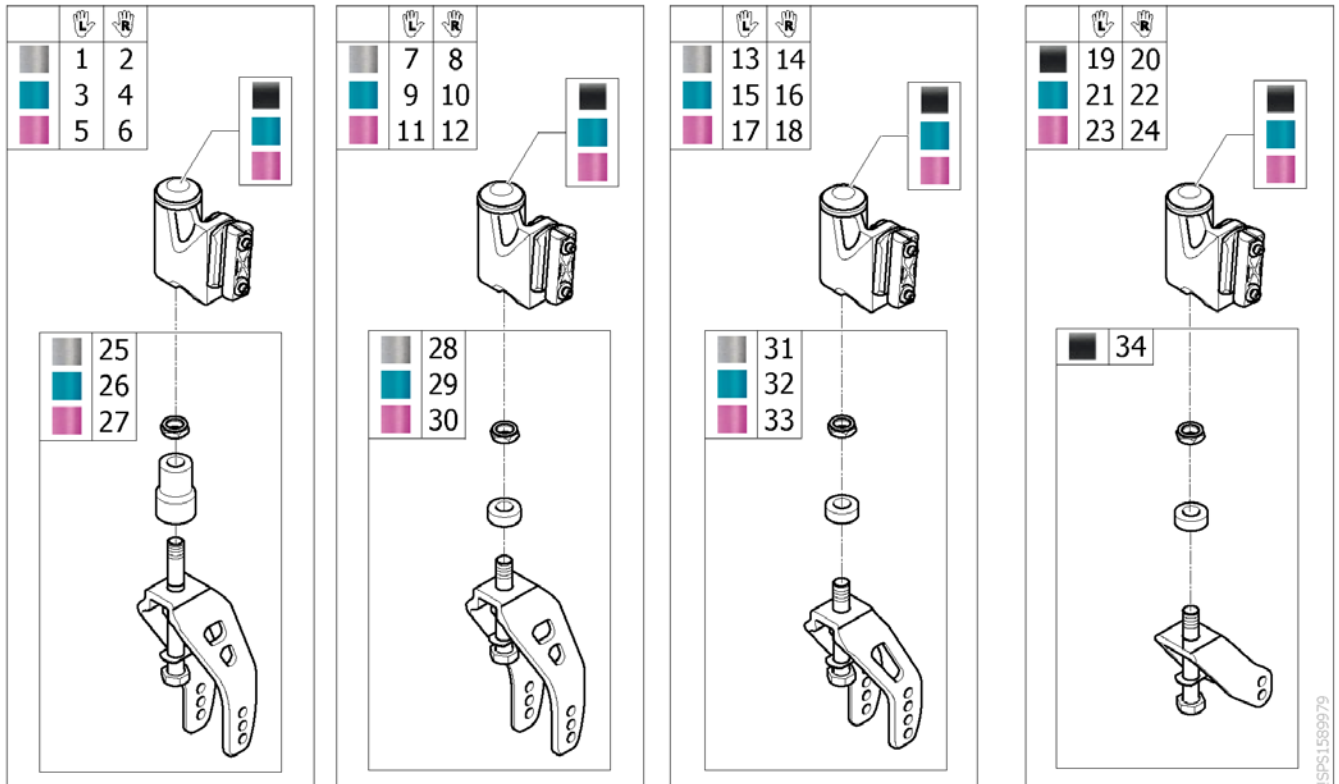


Pos.	Part number	Description	Valid from → until	Qty
1	SP1583759-0005	Kit fork medium, high, Lh, without accent, swing away frame [SJ_0200+SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6100]		1
2	SP1586262-0005	Kit fork medium, high, Rh, without accent, swing away frame [SJ_0200+SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6100]		1
3	SP1583759-0641	Kit fork medium, high, Lh, accent Blue, swing away frame [SJ_0200+SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6110]		1
4	SP1586262-0641	Kit fork medium, high, Rh, accent Blue, swing away frame [SJ_0200+SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6110]		1
5	SP1583759-0642	Kit fork medium, high, Lh, accent Purple, swing away frame [SJ_0200+SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6200]		1
6	SP1586262-0642	Kit fork medium, high, Rh, accent Purple, swing away frame [SJ_0200+SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6200]		1
7	SP1583757-0005	Kit fork medium, low, Lh, without accent, swing away frame [SJ_0200+SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
8	SP1586259-0005	Kit fork medium, low, Rh, without accent, swing away frame [SJ_0200+SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
9	SP1583757-0641	Kit fork medium, low, Lh, accent Blue, swing away frame [SJ_0200+SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
10	SP1586259-0641	Kit fork medium, low, Rh, accent Blue, swing away frame [SJ_0200+SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
11	SP1583757-0642	Kit fork medium, low, Lh, accent Purple, swing away frame [SJ_0200+SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
12	SP1586259-0642	Kit fork medium, low, Rh, accent Purple, swing away frame [SJ_0200+SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
13	SP1583758-0005	Kit fork short, Lh, without accent, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6100] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1

Forks for swing away frame

Pos.	Part number	Description	Valid from → until	Qty
14	SP1586260-0005	Kit fork short, Rh, without accent, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6100] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
15	SP1583758-0641	Kit fork short, Lh, accent Blue, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6110] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
16	SP1586260-0641	Kit fork short, Rh, accent Blue, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6110] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
17	SP1583758-0642	Kit fork short, Lh, accent Purple, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6200] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
18	SP1586260-0642	Kit fork short, Rh, accent Purple, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6200] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
19	SP1583761-0005	Kit fork extra short, Lh, without accent, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6100]		1
20	SP1586270-0005	Kit fork extra short, Rh, without accent, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6100]		1
21	SP1583761-0641	Kit fork extra short, Lh, accent Blue, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6110]		1
22	SP1586270-0641	Kit fork extra short, Rh, accent Blue, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6110]		1
23	SP1583761-0642	Kit fork extra short, Lh, accent Purple, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6200]		1
24	SP1586270-0642	Kit fork extra short, Rh, accent Purple, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6200]		1
25	SP1584027-0005	Kit fork medium, high, without accent [SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6100]		2
26	SP1584027-0641	Kit fork medium, high, accent Blue [SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6110]		2
27	SP1584027-0642	Kit fork medium, high, accent Purple [SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6200]		2
28	SP1584025-0005	Kit fork medium, low, without accent [SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		2
29	SP1584025-0641	Kit fork medium, low, accent Blue [SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		2
30	SP1584025-0642	Kit fork medium, low, accent Purple [SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		2
31	SP1584026-0005	Kit fork short, low, without accent [(SJ_1220/SJ_1230)+SJ_6100] [(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		2
32	SP1584026-0641	Kit fork short, low, accent Blue [(SJ_1220/SJ_1230)+SJ_6110] [(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		2
33	SP1584026-0642	Kit fork short, low, accent Purple [(SJ_1220/SJ_1230)+SJ_6200] [(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		2
34	SP1584029-0601	Kit fork extra short, low, black [SJ_1200/SJ_1210]		2

Forks for fixed frame

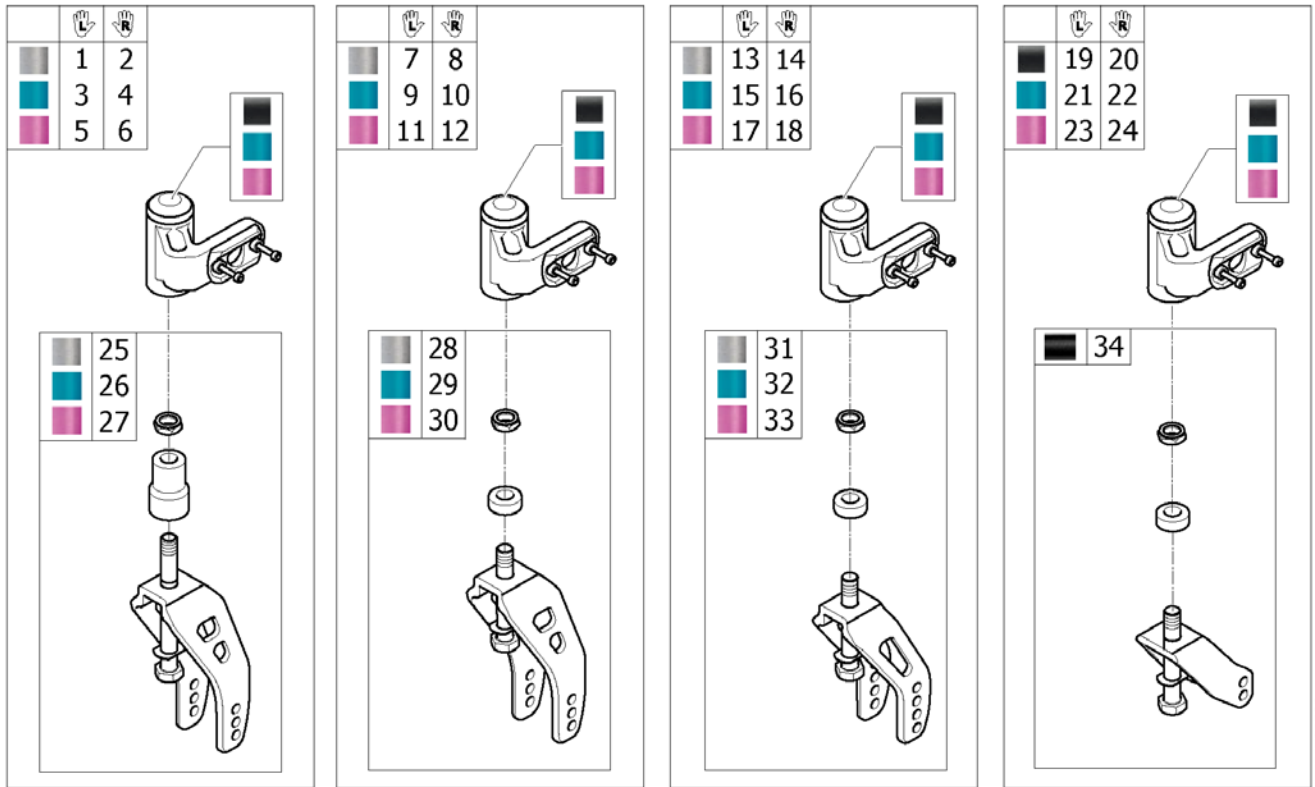


ESPS1580980

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583854-0005	Kit fork medium, high, Lh, without accent, fixed frame [SJ_0210+SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6100]		1
2	SP1586290-0005	Kit fork medium, high, Rh, without accent, fixed frame [SJ_0210+SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6100]		1
3	SP1583854-0641	Kit fork medium, high, Lh, accent Blue, fixed frame [SJ_0210+SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6110]		1
4	SP1586290-0641	Kit fork medium, high, Rh, accent Blue, fixed frame [SJ_0210+SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6110]		1
5	SP1583854-0642	Kit fork medium, high, Lh, accent Purple, fixed frame [SJ_0210+SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6200]		1
6	SP1586290-0642	Kit fork medium, high, Rh, accent Purple, fixed frame [SJ_0210+SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6200]		1
7	SP1583852-0005	Kit fork medium, low, Lh, without accent, fixed frame [SJ_0210+SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
8	SP1586286-0005	Kit fork medium, low, Rh, without accent, fixed frame [SJ_0210+SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
9	SP1583852-0641	Kit fork medium, low, Lh, accent Blue, fixed frame [SJ_0210+SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
10	SP1586286-0641	Kit fork medium, low, Rh, accent Blue, fixed frame [SJ_0210+SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
11	SP1583852-0642	Kit fork medium, low, Lh, accent Purple, fixed frame [SJ_0210+SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
12	SP1586286-0642	Kit fork medium, low, Rh, accent Purple, fixed frame [SJ_0210+SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		1
13	SP1583853-0005	Kit fork short, Lh, without accent, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6100] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1

Forks for fixed frame

Pos.	Part number	Description	Valid from → until	Qty
14	SP1586288-0005	Kit fork short, Rh, without accent, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6100] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
15	SP1583853-0641	Kit fork short, Lh, accent Blue, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6110] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
16	SP1586288-0641	Kit fork short, Rh, accent Blue, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6110] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
17	SP1583853-0642	Kit fork short, Lh, accent Purple, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6200] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
18	SP1586288-0642	Kit fork short, Rh, accent Purple, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6200] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		1
19	SP1583762-0005	Kit fork extra short, Lh, without accent, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6100]		1
20	SP1586292-0005	Kit fork extra short, Rh, without accent, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6100]		1
21	SP1583762-0641	Kit fork extra short, Lh, accent Blue, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6110]		1
22	SP1586292-0641	Kit fork extra short, Rh, accent Blue, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6110]		1
23	SP1583762-0642	Kit fork extra short, Lh, accent Purple, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6200]		1
24	SP1586292-0642	Kit fork extra short, Rh, accent Purple, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6200]		1
25	SP1584027-0005	Kit fork medium, high, without accent [SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6100]		2
26	SP1584027-0641	Kit fork medium, high, accent Blue [SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6110]		2
27	SP1584027-0642	Kit fork medium, high, accent Purple [SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6200]		2
28	SP1584025-0005	Kit fork medium, low, without accent [SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		2
29	SP1584025-0641	Kit fork medium, low, accent Blue [SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		2
30	SP1584025-0642	Kit fork medium, low, accent Purple [SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)]		2
31	SP1584026-0005	Kit fork short, low, without accent [(SJ_1220/SJ_1230)+SJ_6100] [(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		2
32	SP1584026-0641	Kit fork short, low, accent Blue [(SJ_1220/SJ_1230)+SJ_6110] [(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		2
33	SP1584026-0642	Kit fork short, low, accent Purple [(SJ_1220/SJ_1230)+SJ_6200] [(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)]		2
34	SP1584029-0601	Kit fork extra short, low, black [SJ_1200/SJ_1210]		2


Forks support for swing away frame

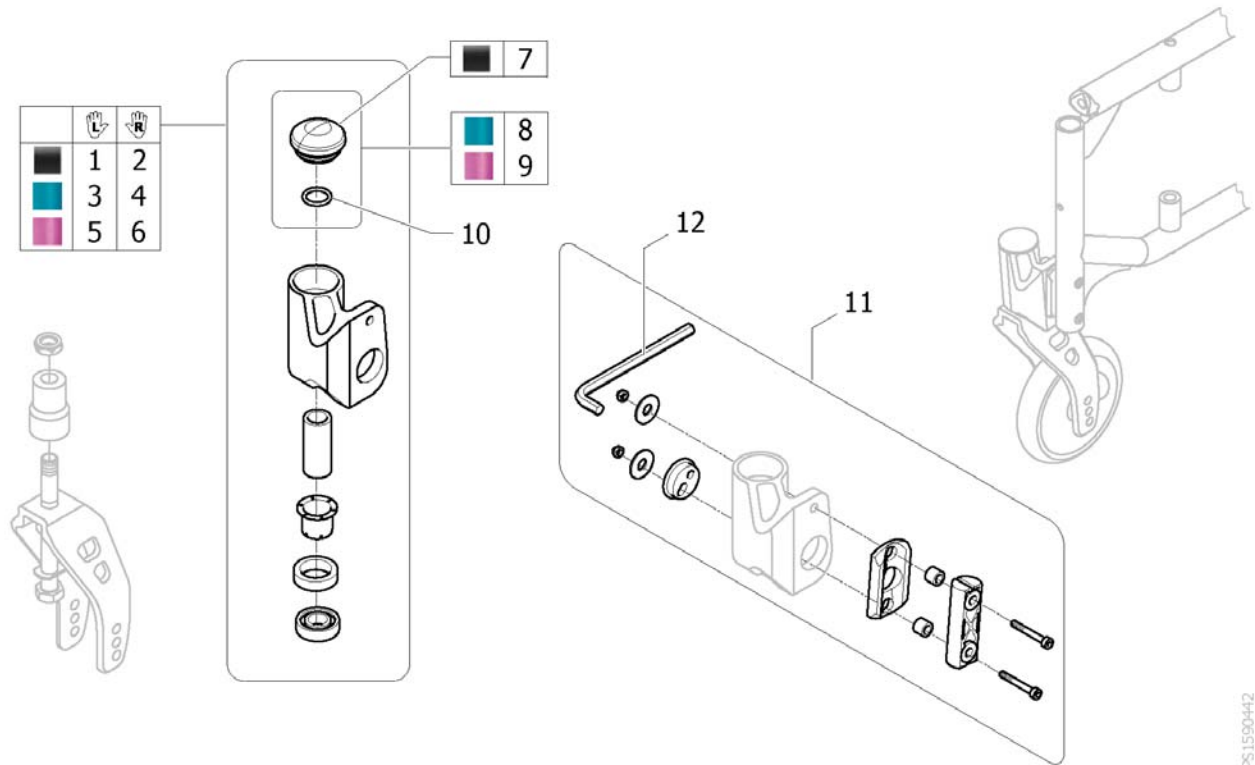

ISPS1590442

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584023-0005	Kit fork support, Lh, without accent, swing away frame [SJ_0200+SJ_6100]		1
2	SP1586261-0005	Kit fork support, Rh, without accent, swing away frame [SJ_0200+SJ_6100]		1
3	SP1584023-0641	Kit fork support, Lh, accent Blue, swing away frame [SJ_0200+SJ_6110]		1
4	SP1586261-0641	Kit fork support, Rh, accent Blue, swing away frame [SJ_0200+SJ_6110]		1
5	SP1584023-0642	Kit fork support, Lh, accent Purple, swing away frame [SJ_0200+SJ_6200]		1
6	SP1586261-0642	Kit fork support, Rh, accent Purple, swing away frame [SJ_0200+SJ_6200]		1
7	SP1589193	Castor housing cap, Black Replace SP1585342-0005	2015-12-23 →	2
8	SP1585342-0641	Castor housing cap, accent Blue [SJ_6110]		2
9	SP1585342-0642	Castor housing cap, accent Purple [SJ_6200]		2
10	SP1483872	O-ring 15,6 x 2,4 NITRILE 70 SH		2
11	SP1584024	Kit adjustment fork support, swing away frame		2
12	SP1586506	Short Metric Hex Key (5mm)		1


Forks support for fixed frame

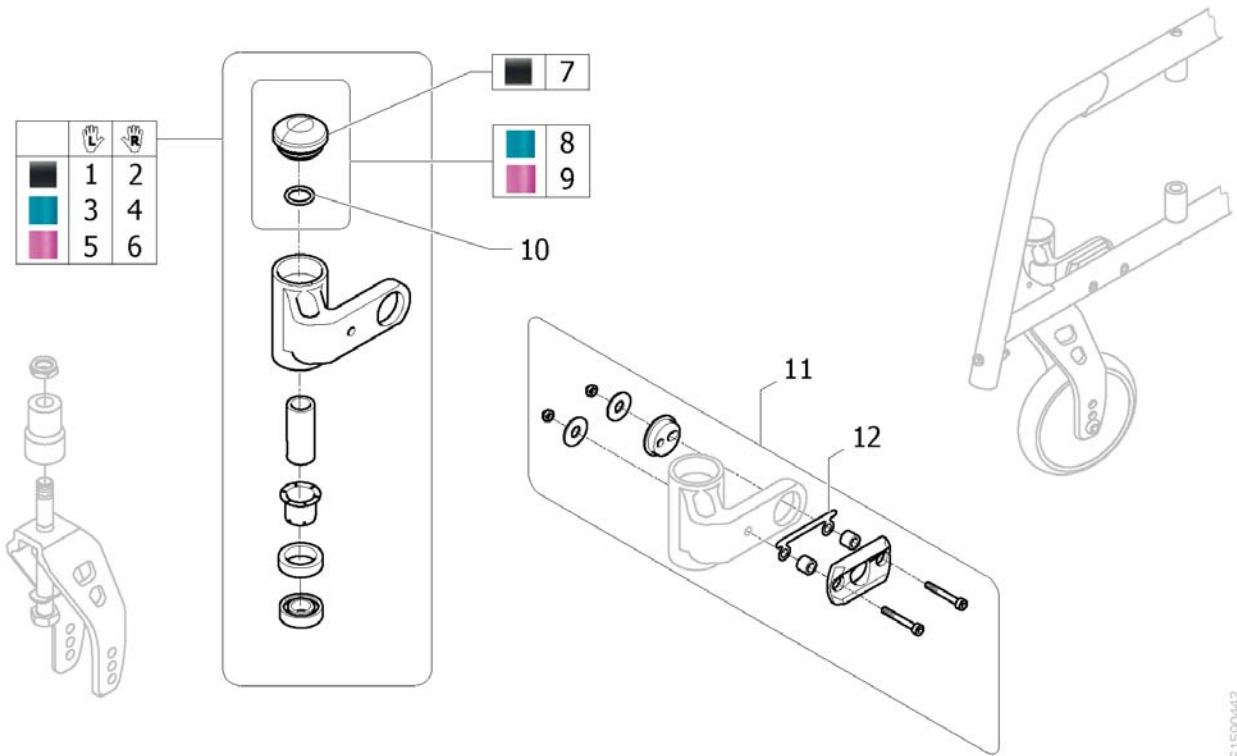

ISPS1590443

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586271-0005	Kit fork support, Lh, without accent, fixed frame <i>[SJ_0210+SJ_6100]</i>		1
2	SP1586285-0005	Kit fork support, Rh, without accent, fixed frame <i>[SJ_0210+SJ_6100]</i>		1
3	SP1586271-0641	Kit fork support, Lh, accent Blue, fixed frame <i>[SJ_0210+SJ_6110]</i>		1
4	SP1586285-0641	Kit fork support, Rh, accent Blue, fixed frame <i>[SJ_0210+SJ_6110]</i>		1
5	SP1586271-0642	Kit fork support, Lh, accent Purple, fixed frame <i>[SJ_0210+SJ_6200]</i>		1
6	SP1586285-0642	Kit fork support, Rh, accent Purple, fixed frame <i>[SJ_0210+SJ_6200]</i>		1
7	SP1589193	Castor housing cap, Black <i>Replace SP1585342-0005</i>	2015-12-23 →	2
8	SP1585342-0641	Castor housing cap, accent Blue <i>[SJ_6110]</i>		2
9	SP1585342-0642	Castor housing cap, accent Purple <i>[SJ_6200]</i>		2
10	SP1483872	O-ring 15,6 x 2,4 NITRILE 70 SH		2
11	SP1586287	Kit adjustment fork support, fixed frame		2
12	SP1604465	Kit adjusting part front wheel A5 (pair)		1


Front wheels

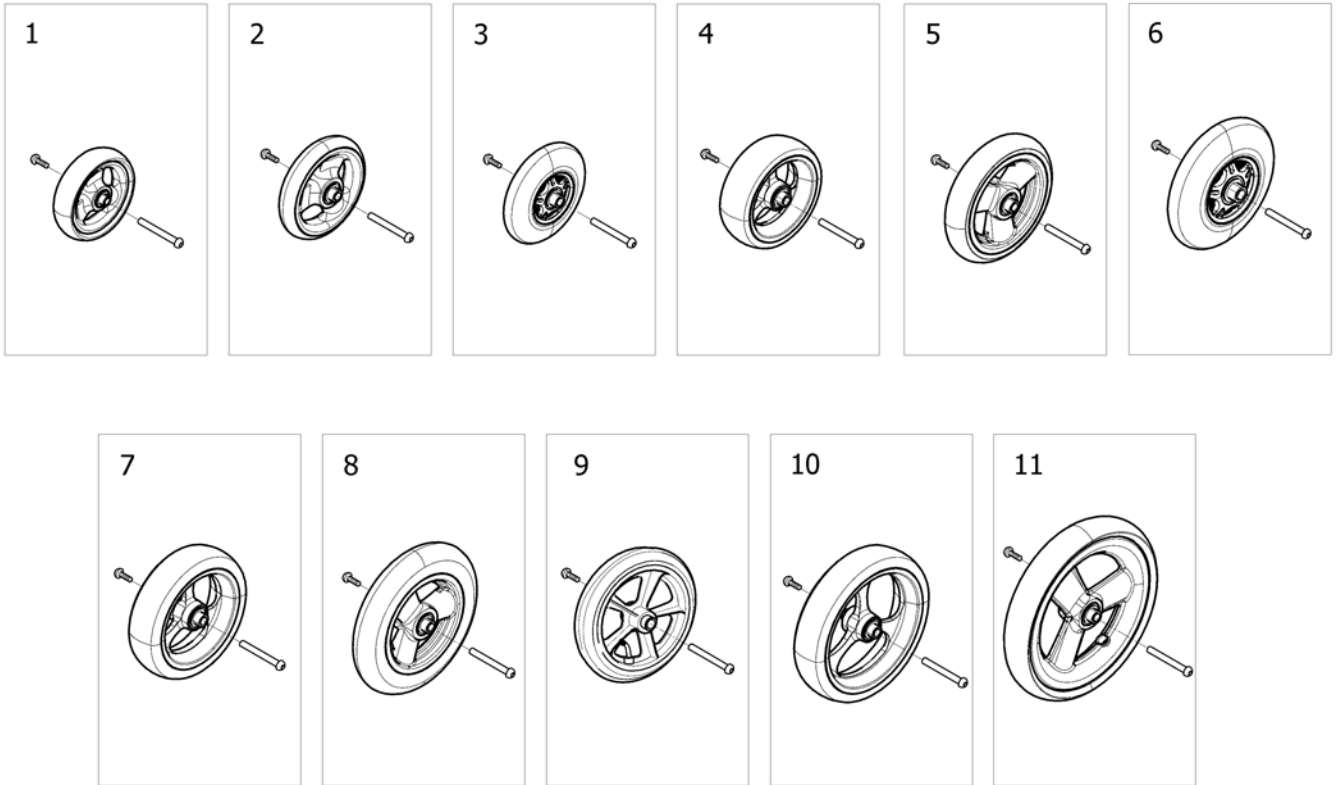

ISP51589961

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584356	Front wheel, 4", softroll for fork W40mm [SJ_7000]		2
2	SP1584357	Front wheel, 4" x 1", solid for fork W40mm [SJ_7020]		2
3	SP1584358	Front wheel, 4", solid lights for fork W40mm [SJ_7030]		2
4	SP1584359	Front wheel, 4", alu softroll for fork W40mm [SJ_7040]		2
5	SP1584354	Front wheel, 5" x 1", solid for fork W40mm [SJ_7055]		2
6	SP1584355	Front wheel, 5", solid lights for fork W40mm [SJ_7060]		2
7	SP1584360	Front wheel, 5", alu softroll for fork W40mm [SJ_7070]		2
8	SP1584238	Front wheel, 6" x 1", solid for fork W40mm [SJ_7080]		2
9	SP1584239	Front wheel, 6" x 1", pneumatic for fork W40mm [SJ_7090]		2
10	SP1584361	Front wheel, 6", alu softroll for fork W40mm [SJ_7100]		2
11	SP1584236	Front wheel, 7" x 1", solid for fork W40mm [SJ_7110]		2

Rear Wheels

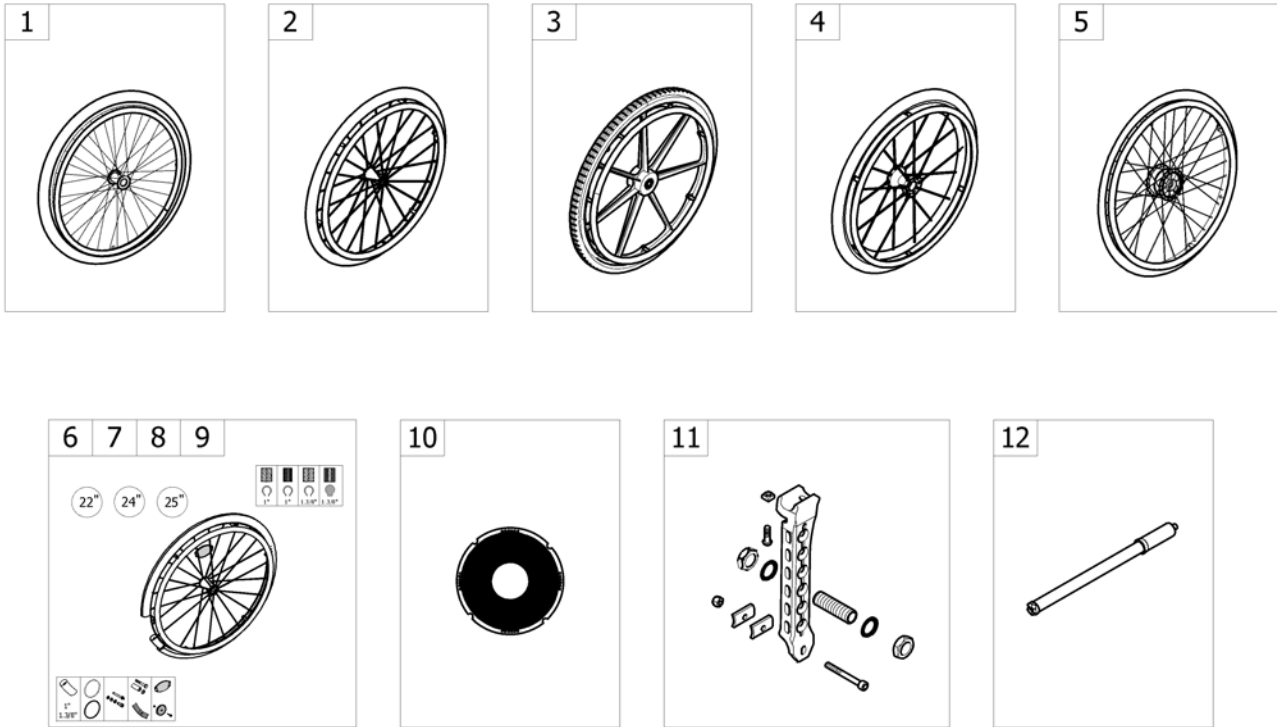

-<SPACTION555


Pos.	Part number	Description	Valid from → until	Qty
1	-	Rear wheel 22"/24", cross spoked [SJ_7200/SJ_7230]		1
2	-	Rear wheel Lightweight 22"/24"/25" [SJ_7210/SJ_7240/SJ_7250]		1
3	-	Rear wheel Composite 22"-24" [SJ_7220/SJ_7242]		1
4	-	Rear wheel Spinergy 24"-25" [SJ_7244/SJ_7260]		1
5	-	Rear wheel hub brake 22"-24" [(SJ_7200/SJ_7230)+SJ_8160]		1
6	-	Standard & hub brake wheels accessories [SJ_7200/SJ_7230]		1
7	-	Lightweight wheels accessories [SJ_7210/SJ_7240/SJ_7250]		1
8	-	Composite wheels accessories [SJ_7220/SJ_7242]		1
9	-	Spinergy wheels accessories [SJ_7244/SJ_7260]		1
10	-	Spoke protector [SJ_8000]		1
11	-	Rear wheel support		1
12	-	Pump [SJ_8020/SJ_8030]		1

Rear wheel 22"/24 ", cross spoked

	1"	1.3/8"	1.3/8"
22"	1	2	3
24"	4	5	6

	1"	1.3/8"	1.3/8"
22"	7	8	9
24"	10	11	12

	1"	1.3/8"	1.3/8"
22"	-	-	-
24"	13	14	15

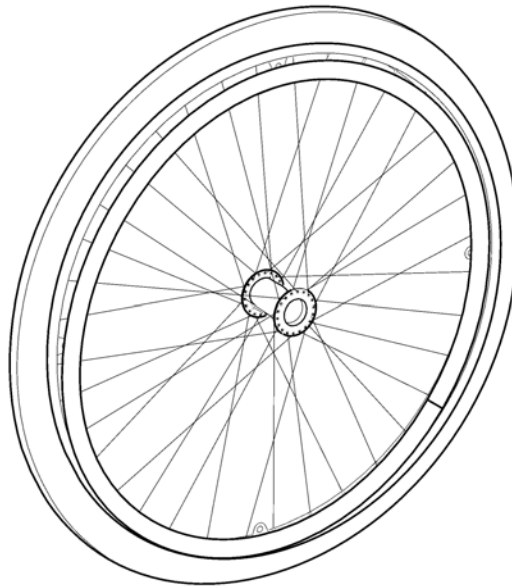


ISPS1590798

Pos.	Part number	Description	Valid from → until	Qty
2	1498493	Rear wheel, 22", HD, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7200+SJ_7305+SJ_7400]</i>		2
3	SP1588643	Rear wheel, 22", HD, solid tyre 1.3/8", aluminium handrim <i>[SJ_7200+SJ_7300+SJ_7400]</i>		2
4	SP1588711	Rear wheel, 24", spoked, pneumatic tyre HP 1", aluminium handrim <i>[SJ_7230+SJ_7310+SJ_7400]</i>		2
5	1498486	Rear wheel, 24", HD, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7230+SJ_7305+SJ_7400]</i>		2
6	1498467	Rear wheel, 24", HD, solid tyre 1.3/8", aluminium handrim <i>[SJ_7230+SJ_7300+SJ_7400]</i>		2
8	1498494	Rear wheel, 22", HD, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7200+SJ_7305+SJ_7420]</i>		2
9	SP1588644	Rear wheel, 22", HD, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7200+SJ_7300+SJ_7420]</i>		2
10	SP1588742	Rear wheel, 24", spoked, pneumatic tyre HP 1", anti-slip handrim <i>[SJ_7230+SJ_7310+SJ_7420]</i>		2
11	1498492	Rear wheel, 24", HD, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7230+SJ_7305+SJ_7420]</i>		2
12	1498478	Rear wheel, 24", HD, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7230+SJ_7300+SJ_7420]</i>		2
13	SP1588744	Rear wheel, 24", spoked, pneumatic tyre HP 1", stainless handrim <i>[SJ_7230+SJ_7310+SJ_7410] ONLY FOR EUROPE</i>		2
14	SP1588651	Rear wheel, 24", HD, pneumatic tyre 1.3/8", stainless handrim <i>[SJ_7230+SJ_7305+SJ_7410] ONLY FOR EUROPE</i>		2
15	SP1588649	Rear wheel, 24", HD, solid tyre 1.3/8", stainless handrim <i>[SJ_7230+SJ_7300+SJ_7410] ONLY FOR EUROPE</i>		2

Rear wheel Lightweight 22"/24"/25"

	1"	1"	1.3/8"	1.3/8"
22"	1	-	2	3
24"	4	5	6	7
25"	8	9	-	-



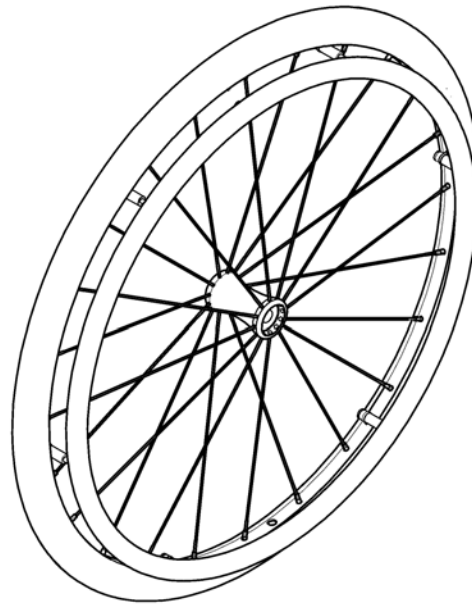

	1"	1"	1.3/8"	1.3/8"
22"	10	-	11	12
24"	13	14	15	16
25"	17	18	-	-








ISPS1590800 ~ sps1588755

Pos.	Part number	Description	Valid from → until	Qty
2	SP1588890	Rear wheel, 22", lightweight, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7210+SJ_7305+SJ_7400]</i>		2
3	SP1588888	Rear wheel, 22", lightweight, solid tyre 1.3/8", aluminium handrim <i>[SJ_7210+SJ_7300+SJ_7400]</i>		2
4	SP1588754	Rear wheel, 24", lightweight, pneumatic tyre HP 1", aluminium handrim <i>[SJ_7240+SJ_7310+SJ_7400]</i>		2
5	SP1588756	Rear wheel, 24", lightweight, pneumatic tyre Evo 1", aluminium handrim <i>[SJ_7240+SJ_7320+SJ_7400]</i>		2
6	SP1588752	Rear wheel, 24", lightweight, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7240+SJ_7305+SJ_7400]</i>		2
7	SP1588750	Rear wheel, 24", lightweight, solid tyre 1.3/8", aluminium handrim <i>[SJ_7240+SJ_7300+SJ_7400]</i>		2
9	SP1588748	Rear wheel, 25", lightweight, pneumatic tyre Evo 1", aluminium handrim <i>[SJ_7250+SJ_7320+SJ_7400]</i>		2
11	SP1588895	Rear wheel, 22", lightweight, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7210+SJ_7305+SJ_7420]</i>		2
12	SP1588889	Rear wheel, 22", lightweight, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7210+SJ_7300+SJ_7420]</i>		2
13	SP1588755	Rear wheel, 24", lightweight, pneumatic tyre HP 1", anti-slip handrim <i>[SJ_7240+SJ_7310+SJ_7420]</i>		2
14	SP1588887	Rear wheel, 24", lightweight, pneumatic tyre Evo 1", anti-slip handrim <i>[SJ_7240+SJ_7320+SJ_7420]</i>		2
15	SP1588753	Rear wheel, 24", lightweight, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7240+SJ_7305+SJ_7420]</i>		2
16	SP1588751	Rear wheel, 24", lightweight, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7240+SJ_7300+SJ_7420]</i>		2
18	SP1588749	Rear wheel, 25", lightweight, pneumatic tyre Evo 1", anti-slip handrim <i>[SJ_7250+SJ_7320+SJ_7420]</i>		2

Rear wheel Composite 22"-24"

			
1.3/8"	1.3/8"	1.3/8"	1.3/8"
22"	1	2	3
24"	4	5	6



			
1.3/8"	1.3/8"	1.3/8"	1.3/8"
22"	7	8	9
24"	10	11	12

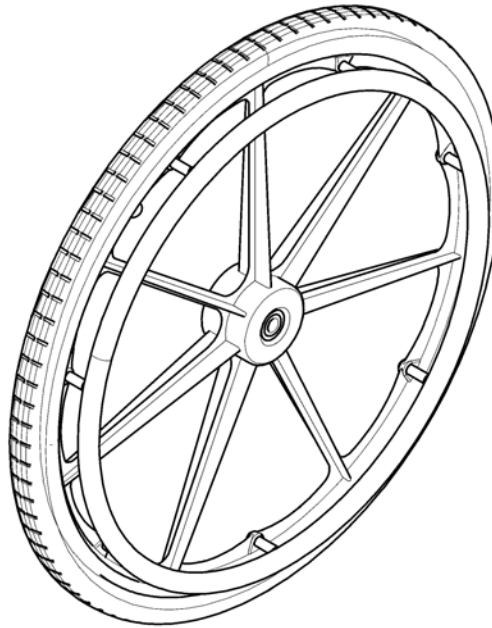


ISP51590799 ~ sp51588904

Pos.	Part number	Description	Valid from → until	Qty
1	SP1588902	Rear wheel, 22", composite, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7220+SJ_7305+SJ_7400]</i>		2
2	SP1588900	Rear wheel, 22", composite, solid tyre 1.3/8", aluminium handrim <i>[SJ_7220+SJ_7300+SJ_7400]</i>		2
3	SP1589156	Rear wheel, 22", composite, flat free insert, aluminium handrim <i>[SJ_7220+SJ_7330+SJ_7400]</i>		2
4	SP1588906	Rear wheel, 24", composite, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7242+SJ_7305+SJ_7400]</i>		2
5	SP1588904	Rear wheel, 24", composite, solid tyre 1.3/8", aluminium handrim <i>[SJ_7242+SJ_7300+SJ_7400]</i>		2
6	SP1589158	Rear wheel, 24", composite, flat free insert, aluminium handrim <i>[SJ_7242+SJ_7330+SJ_7400]</i>		2
7	SP1588903	Rear wheel, 22", composite, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7220+SJ_7305+SJ_7420]</i>		2
8	SP1588901	Rear wheel, 22", composite, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7220+SJ_7300+SJ_7420]</i>		2
9	SP1589157	Rear wheel, 22", composite, flat free insert, anti-slip handrim <i>[SJ_7220+SJ_7330+SJ_7420]</i>		2
10	SP1588923	Rear wheel, 24", composite, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7242+SJ_7305+SJ_7420]</i>		2
11	SP1588905	Rear wheel, 24", composite, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7242+SJ_7300+SJ_7420]</i>		2
12	SP1589159	Rear wheel, 24", composite, flat free insert, anti-slip handrim <i>[SJ_7242+SJ_7330+SJ_7420]</i>		2

Rear wheel Spinergy 24"-25"

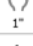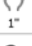		
		
24"	1	2
25"	3	4



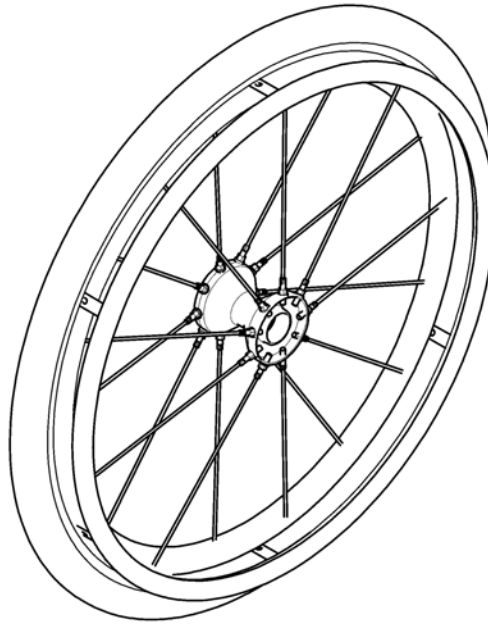

		
		
24"	5	6
25"	-	-











ISPS1590801 ~ sps1588928


Pos.	Part number	Description	Valid from → until	Qty
1	SP1588927	Rear wheel, 24", Spinergy, pneumatic tyre HP 1", aluminium handrim <i>[SJ_7244+SJ_7310+SJ_7400]</i>		2
2	SP1588929	Rear wheel, 24", Spinergy, pneumatic tyre Evo 1", aluminium handrim <i>[SJ_7244+SJ_7320+SJ_7400]</i>		2
4	SP1589151	Rear wheel, 25", Spinergy, pneumatic tyre Evo 1", aluminium handrim <i>[SJ_7260+SJ_7320+SJ_7400]</i>		2
5	SP1588928	Rear wheel, 24", Spinergy, pneumatic tyre HP 1", anti-slip handrim <i>[SJ_7244+SJ_7310+SJ_7420]</i>		2
6	SP1589149	Rear wheel, 24", Spinergy, pneumatic tyre Evo 1", anti-slip handrim <i>[SJ_7244+SJ_7320+SJ_7420]</i>		2

Rear wheel hub brake 22"-24"

			
22"	1	2	3
24"	4	5	6

			
22"	7	8	9
24"	10	11	12


			
22"	-	-	-
24"	13	14	15

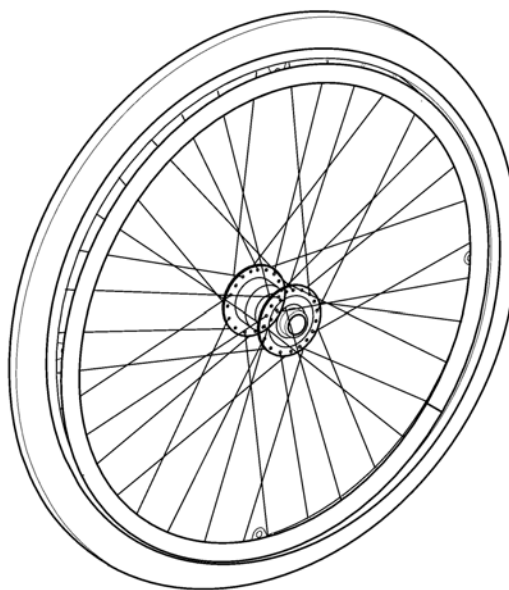

ISPS159080Z

Pos.	Part number	Description	Valid from → until	Qty
2	1498466	Rear wheel, 22", hub brake, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7200+SJ_7305+SJ_7400+SJ_8160]</i>		2
3	1498461	Rear wheel, 22", hub brake, solid tyre 1.3/8", aluminium handrim <i>[SJ_7200+SJ_7300+SJ_7400+SJ_8160]</i>		2
4	SP1588712	Rear wheel, 24", hub brake, pneumatic tyre HP 1", aluminium handrim <i>[SJ_7230+SJ_7310+SJ_7400+SJ_8160]</i>		2
5	1498428	Rear wheel, 24", hub brake, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7230+SJ_7305+SJ_7400+SJ_8160]</i>		2
6	1498427	Rear wheel, 24", hub brake, solid tyre 1.3/8", aluminium handrim <i>[SJ_7230+SJ_7300+SJ_7400+SJ_8160]</i>		2
8	1557823	Rear wheel, 22", hub brake, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7200+SJ_7305+SJ_7420+SJ_8160]</i>		2
9	1557801	Rear wheel, 22", hub brake, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7200+SJ_7300+SJ_7420+SJ_8160]</i>		2
10	SP1588743	Rear wheel, 24", hub brake, pneumatic tyre HP 1", anti-slip handrim <i>[SJ_7230+SJ_7310+SJ_7420+SJ_8160]</i>		2
11	1573481	Rear wheel, 24", hub brake, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7230+SJ_7305+SJ_7420+SJ_8160]</i>		2
12	1573482	Rear wheel, 24", hub brake, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7230+SJ_7300+SJ_7420+SJ_8160]</i>		2
13	SP1588745	Rear wheel, 24", hub brake, pneumatic tyre HP 1", stainless handrim <i>[SJ_7230+SJ_7310+SJ_7410+SJ_8160] ONLY FOR EUROPE</i>		2
14	SP1588652	Rear wheel, 24", hub brake, pneumatic tyre 1.3/8", stainless handrim <i>[SJ_7230+SJ_7305+SJ_7410+SJ_8160] ONLY FOR EUROPE</i>		2
15	SP1588650	Rear wheel, 24", hub brake, solid tyre 1.3/8", stainless handrim <i>[SJ_7230+SJ_7300+SJ_7410+SJ_8160] ONLY FOR EUROPE</i>		2

Standard & hub brake wheels accessories

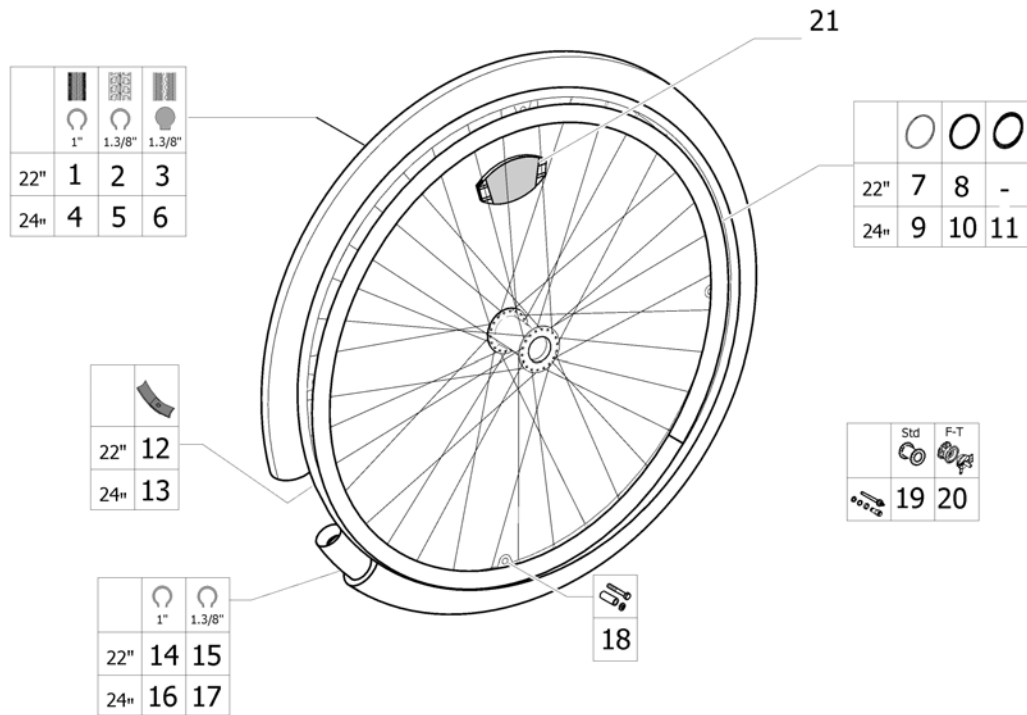


ISPS1590803

Pos.	Part number	Description	Valid from → until	Qty
2	0036-30035	Pneumatic tyre, 22" x 1.3/8", grey [SJ_7200+SJ_7305]		2
3	0036-10014	Solid tyre, 22" x 1.3/8", grey [SJ_7200+SJ_7300]		2
4	1026138	24" x 1. non marking pneumatic tyre Replace SP1585334	2015-09-02 →	2
5	0036-30001	Pneumatic tyre, 24" x 1.3/8", grey [SJ_7230+SJ_7305]		2
6	0036-10013	Solid tyre, 24" x 1.3/8", grey [SJ_7230+SJ_7300]		2
7	1505299	Kit aluminium handrim 22", 4 fixations [SJ_7200+SJ_7400]		2
8	1505920	Kit anti-slip handrim 22", 4 fixations [SJ_7200+SJ_7420]		2
9	1505291	Kit aluminium handrim 24", 4 fixations [SJ_7230+SJ_7400]		2
10	1505298	Kit anti-slip handrim 24", 4 fixations [SJ_7230+SJ_7420]		2
11	SP1591282	Kit stainless handrim 24", 4 fixations [SJ_7230+SJ_7410] ONLY FOR EUROPE		2
12	5750029	Rim tape 22" [SJ_7200]		2
13	0036-40003	Rim tape 24" [SJ_7230]		2
14	1061214	22" (501) Tube (Presta Valve) [SJ_7200+SJ_7310]		2
15	0036-20007	Inner tube, 22" x 1.3/8" [SJ_7200+SJ_7305]		2
16	1032240	Inner tube, 24" x 1" [SJ_7230+SJ_7310]		2
17	0036-20001	Inner tube, 24" x 1.3/8" [SJ_7230+SJ_7305]		2
18	1573626	Kit handrim fixation for spoked wheel, 4 pts		2
19	SP1591280	Kit quick release axle		2
20	SP1591281	Kit quick release axle for hub brake [SJ_8160]		2

Standard & hub brake wheels accessories

Pos.	Part number	Description	Valid from → until	Qty
21	N90003	Kit reflector for spoked wheel <i>[SJ_8600]</i>		2

Lightweight wheels accessories

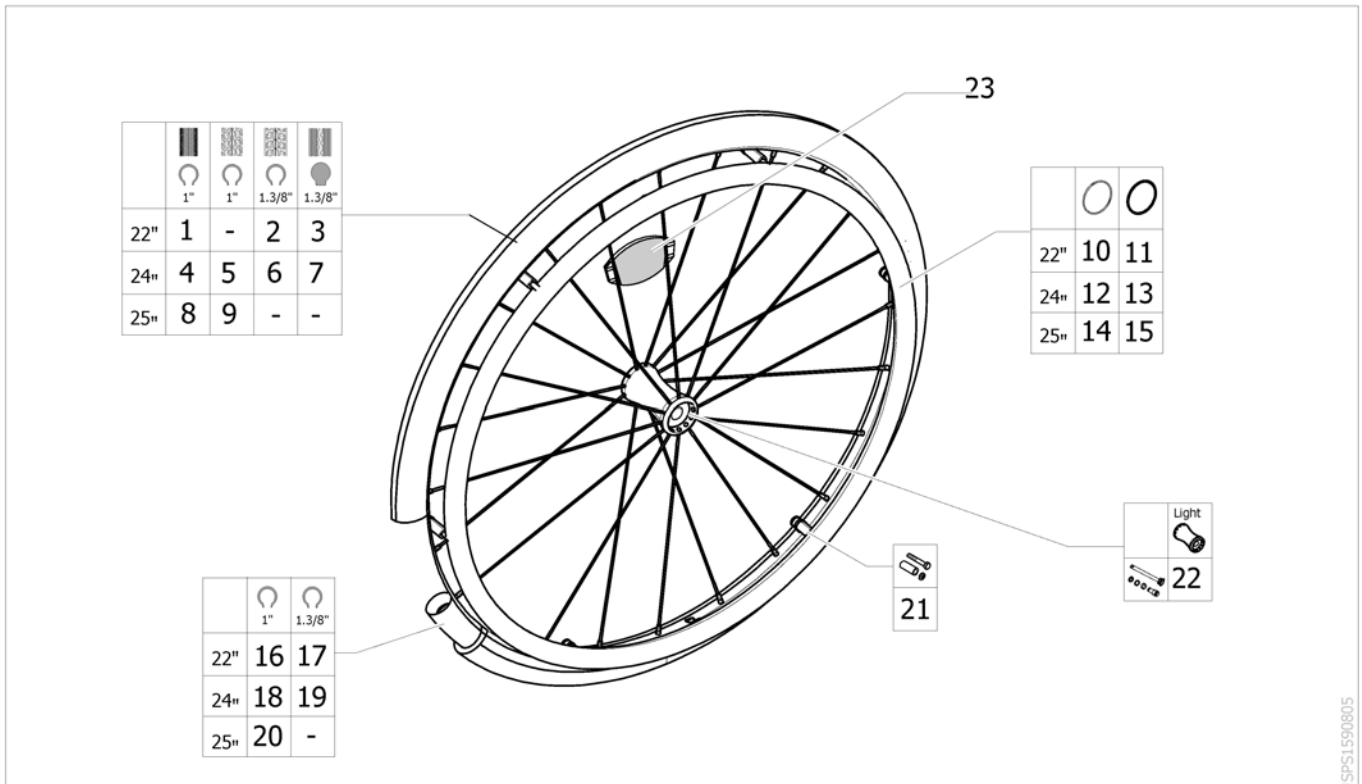


ISPS1590805

Pos.	Part number	Description	Valid from → until	Qty
2	0036-30035	Pneumatic tyre, 22" x 1.3/8", grey [SJ_7210+SJ_7305]		2
3	0036-10014	Solid tyre, 22" x 1.3/8", grey [SJ_7210+SJ_7300]		2
4	1026138	24" x 1. non marking pneumatic tyre Replace SP1585334	2015-09-02 →	2
5	SP1585331	Pneumatic tyre, 24" x 1", Marathon Plus Evo [SJ_7240+SJ_7320]		2
6	0036-30001	Pneumatic tyre, 24" x 1.3/8", grey [SJ_7240+SJ_7305]		2
7	0036-10013	Solid tyre, 24" x 1.3/8", grey [SJ_7240+SJ_7300]		2
9	SP1585332	Pneumatic tyre, 25" x 1", Marathon Plus Evo [SJ_7250+SJ_7320]		2
10	SP1601462	Kit aluminium handrim 22", 6 fixations, Lightweight [SJ_7210+SJ_7400]		2
11	SP1601463	Kit anti-slip handrim 22", 6 fixations, Lightweight [SJ_7210+SJ_7420]		2
12	SP1601464	Kit aluminium handrim 24", 6 fixations, Lightweight [SJ_7240+SJ_7400]		2
13	SP1601465	Kit anti-slip handrim 24", 6 fixations, Lightweight [SJ_7240+SJ_7420]		2
14	SP1601466	Kit aluminium handrim 25", 6 fixations, Lightweight [SJ_7250+SJ_7400]		2
15	SP1601467	Kit anti-slip handrim 25", 6 fixations, Lightweight [SJ_7250+SJ_7420]		2
16	1061214	22" (501) Tube (Presta Valve) [SJ_7210+SJ_7310]		2
17	0036-20007	Inner tube, 22" x 1.3/8" [SJ_7210+SJ_7305]		2
18	1032240	Inner tube, 24" x 1" [SJ_7240+SJ_7310/SJ_7320]		2
19	0036-20001	Inner tube, 24" x 1.3/8" [SJ_7240+SJ_7305]		2
20	SP1430886	Inner tube, 25", for High-Performance wheel [SJ_7250+SJ_7310/SJ_7320]		2

Lightweight wheels accessories

Pos.	Part number	Description	Valid from → until	Qty
21	SP1601472	Kit handrim fixation for Lightweight wheel, 6 pts		2
22	SP1601461-0005	Kit quick release axle with Grey cap <i>[SJ_6100]</i>		2
22	SP1601461-0641	Kit quick release axle with Blue cap <i>[SJ_6110]</i>		2
22	SP1601461-0642	Kit quick release axle with Purple cap <i>[SJ_6200]</i>		2
23	N90003	Kit reflector for spoked wheel <i>[SJ_8600]</i>		2


Composite wheels accessories

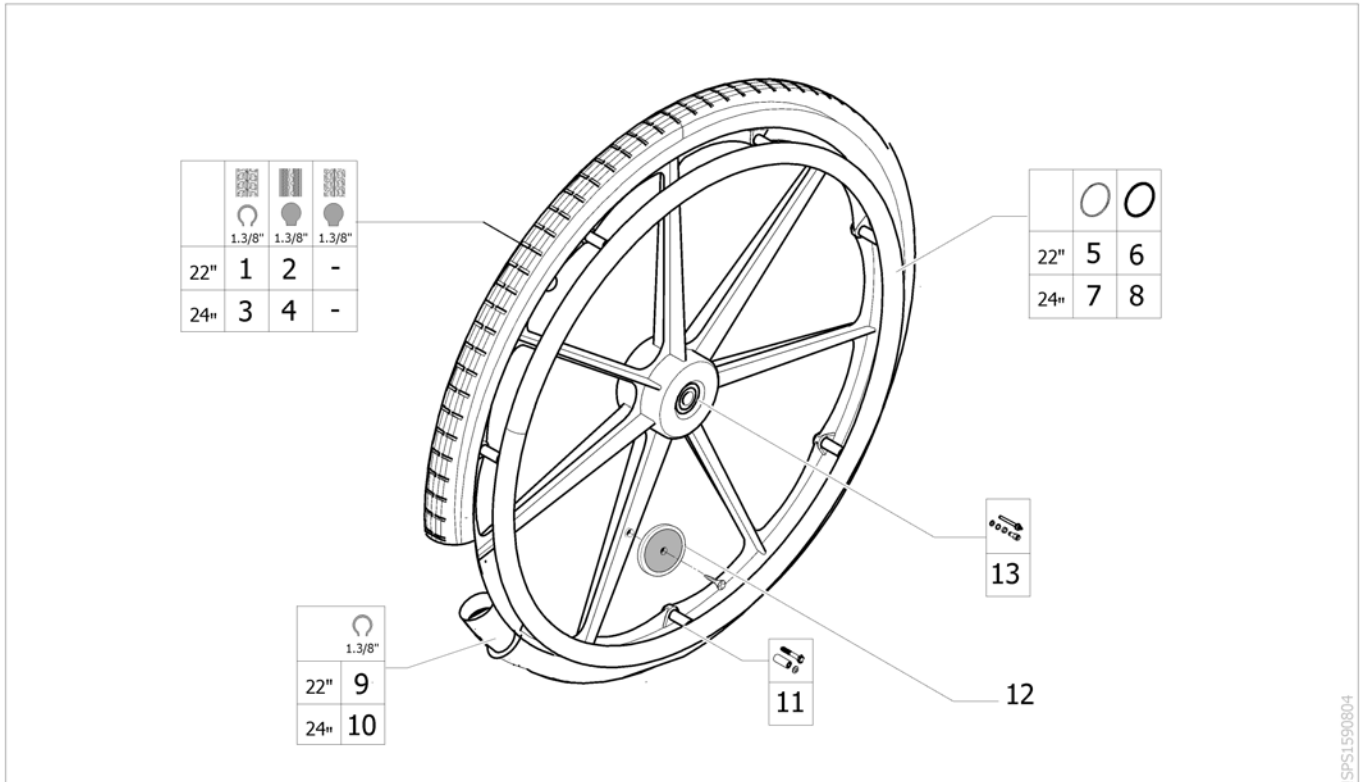

ISPS1590804

Pos.	Part number	Description	Valid from → until	Qty
1	0036-30035	Pneumatic tyre, 22" x 1.3/8", grey <i>[SJ_7220+SJ_7305]</i>		2
2	0036-10014	Solid tyre, 22" x 1.3/8", grey <i>[SJ_7220+SJ_7300]</i>		2
3	0036-30001	Pneumatic tyre, 24" x 1.3/8", grey <i>[SJ_7242+SJ_7305]</i>		2
4	0036-10013	Solid tyre, 24" x 1.3/8", grey <i>[SJ_7242+SJ_7300]</i>		2
5	SP1601839	Kit aluminium handrim 22", 7 fixations, composite wheel <i>[SJ_7220+SJ_7400]</i>		2
6	SP1601840	Kit anti-slip handrim 22", 7 fixations, composite wheel <i>[SJ_7220+SJ_7420]</i>		2
7	SP1601841	Kit aluminium handrim 24", 7 fixations, composite wheel <i>[SJ_7242+SJ_7400]</i>		2
8	SP1601842	Kit anti-slip handrim 24", 7 fixations, composite wheel <i>[SJ_7242+SJ_7420]</i>		2
9	0036-20007	Inner tube, 22" x 1.3/8" <i>[SJ_7220+SJ_7305]</i>		2
10	0036-20001	Inner tube, 24" x 1.3/8" <i>[SJ_7242+SJ_7305]</i>		2
11	SP1601484	Kit handrim fixation for composite wheel, 7 pts		2
12	9037-30001	Kit reflector for composite wheel <i>[(SJ_7220/SJ_7242)+SJ_8600]</i>		2
13	SP1591280	Kit quick release axle		2



Spinergy wheels accessories

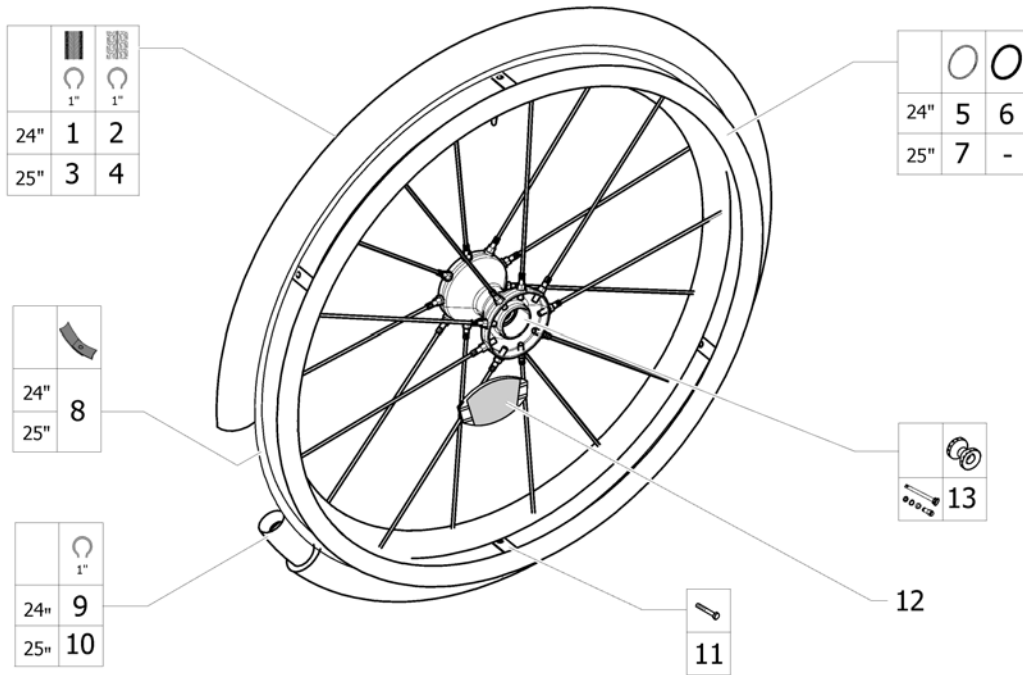

ISPS1590806

Pos.	Part number	Description	Valid from → until	Qty
1	1026138	24" x 1. non marking pneumatic tyre <i>Replace SP1585334</i>	2015-09-02 →	1
2	SP1585331	Pneumatic tyre, 24" x 1", Marathon Plus Evo <i>[SJ_7244+SJ_7320]</i>		2
4	SP1585332	Pneumatic tyre, 25" x 1", Marathon Plus Evo <i>[SJ_7260+SJ_7320]</i>		2
5	SP1601468	Kit aluminium handrim 24", 6 fixations, Spinergy <i>[SJ_7244+SJ_7400]</i>		2
6	SP1601469	Kit anti-slip handrim 24", 6 fixations Spinergy <i>[SJ_7244+SJ_7420]</i>		2
7	SP1601470	Kit aluminium handrim 25", 6 fixations, Spinergy <i>[SJ_7260+SJ_7400]</i>		2
8	0036-40003	Rim tape 24"		2
9	1032240	Inner tube, 24" x 1" <i>[SJ_7244+(SJ_7310/SJ_7320)]</i>		2
10	SP1430886	Inner tube, 25", for High-Performance wheel <i>[SJ_7260+(SJ_7310/SJ_7320)]</i>		2
11	SP1601502	Kit handrim fixation for Spinergy wheel, 6pts		2
12	N90003	Kit reflector for spoked wheel <i>[SJ_8600]</i>		2
13	SP1603952-0005	Kit quick release axle with Grey cap Wheel Spinergy <i>[SJ_6100]</i>		2
13	SP1603952-0641	Kit quick release axle with Blue cap Wheel Spinergy <i>[SJ_6100]</i>		2
13	SP1603952-0642	Kit quick release axle with Purple cap Wheel Spinergy <i>[SJ_6100]</i>		2


Spoke protector

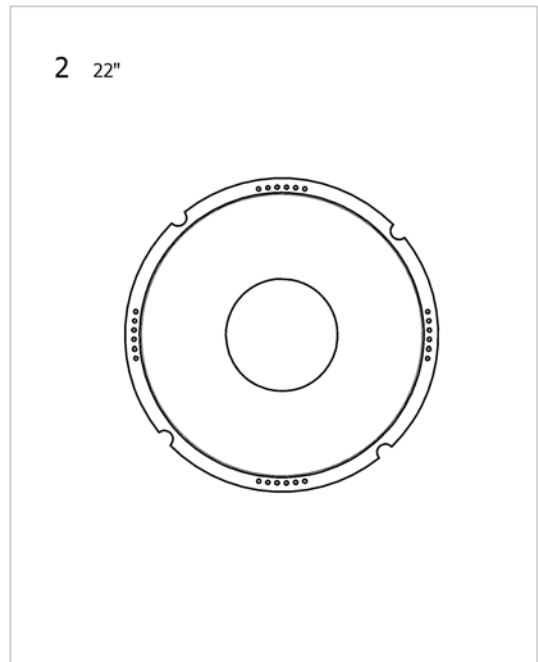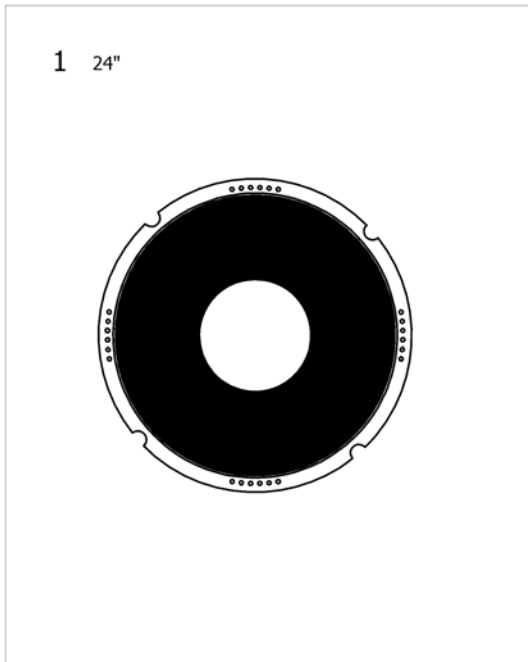

ISPS1590810

Pos.	Part number	Description	Valid from → until	Qty
1	0789-04400	Spoke guard 24" Black (pair) <i>[SJ_7230+SJ_8000]</i>		2
2	1556990	Spoke guard 22" Transparent (Pair) <i>[SJ_7200+SJ_8000]</i>		2


Rear wheel support

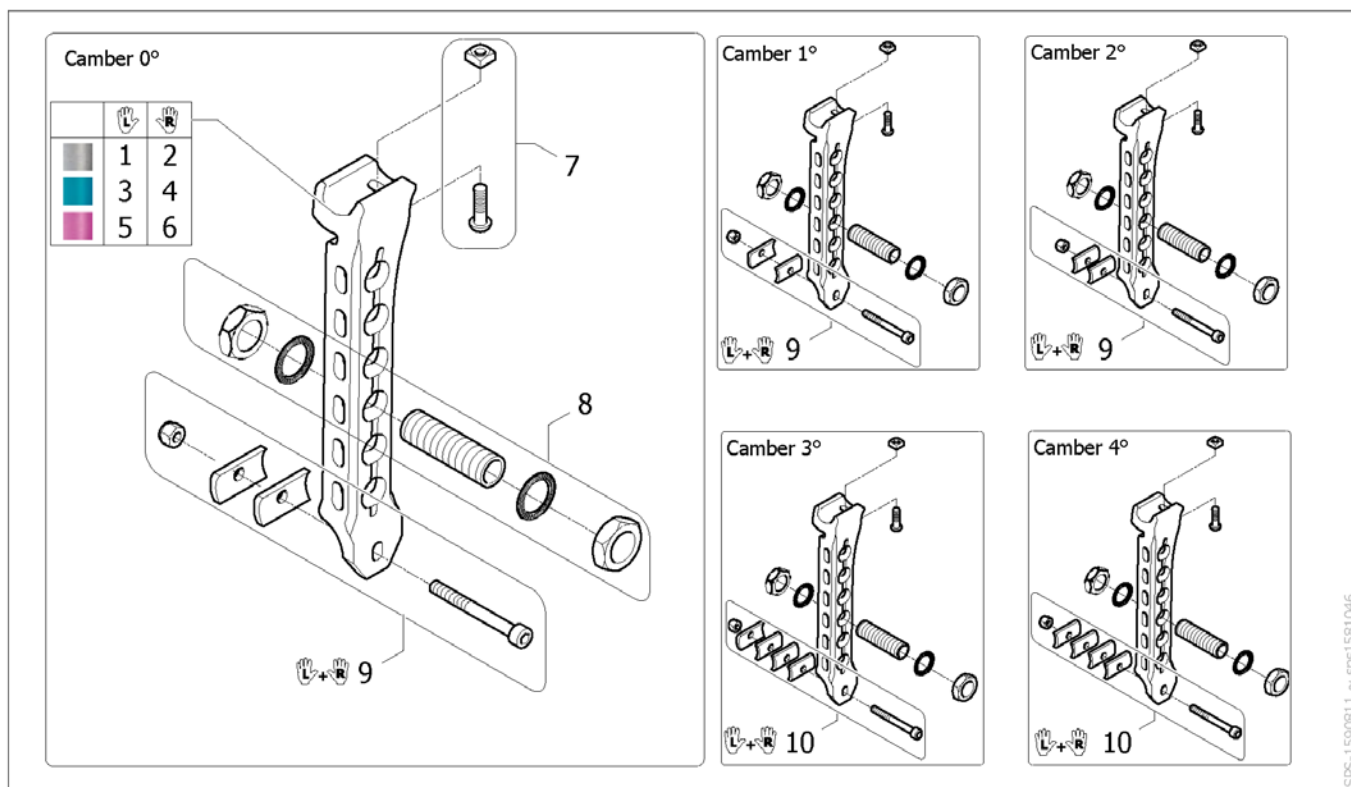

9SPS-1590811 ~ sps1581046

Pos.	Part number	Description	Valid from → until	Qty
1	SP1554604-0005	Kit rear wheel support, Lh, without accent <i>[SJ_6100]</i>		1
2	SP1555921-0005	Kit rear wheel support, Rh, without accent <i>[SJ_6100]</i>		1
3	SP1554604-0641	Kit rear wheel support, Lh, accent Blue <i>[SJ_6110]</i>		1
4	SP1555921-0641	Kit rear wheel support, Rh, accent Blue <i>[SJ_6110]</i>		1
5	SP1554604-0642	Kit rear wheel support, Lh, accent Purple <i>[SJ_6200]</i>		1
6	SP1555921-0642	Kit rear wheel support, Rh, accent Purple <i>[SJ_6200]</i>		1
7	SP1601541	Kit fixation rear wheel support		2
8	SP1601542	Kit rear wheel bearing		2
9	SP1584797	Kit camber 0° to 2° (pair) <i>[SJ_7700/SJ_7710/SJ_7720]</i>		1
10	SP1584798	Kit camber 3° to 4° (pair) <i>[SJ_7730/SJ_7740]</i>		1




Pump

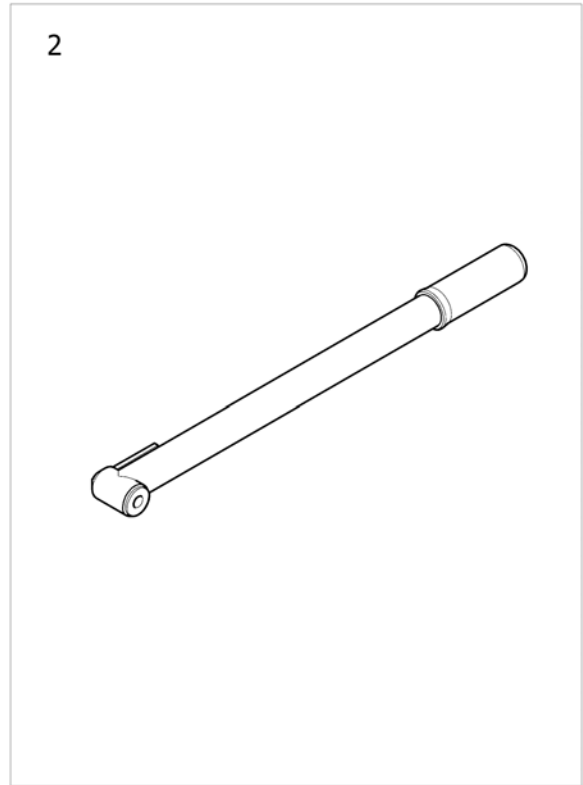

ISPS1590812

Pos.	Part number	Description	Valid from → until	Qty
1	SP1590818	Pump - Low pressure <i>[SJ_8020]</i>		1
2	SP1590819	Pump - High pressure <i>[SJ_8030]</i>		1


Brakes

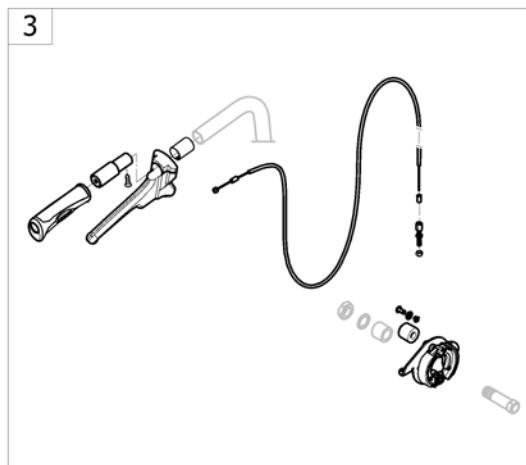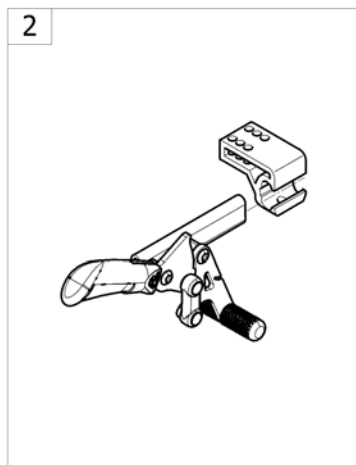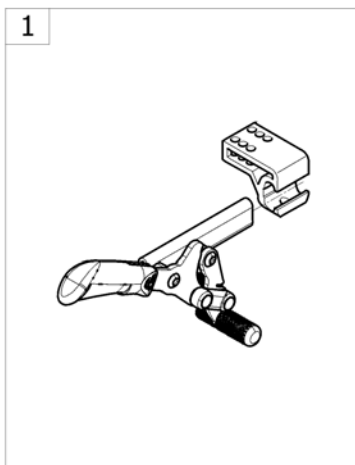

->SPOACTION565

Pos.	Part number	Description	Valid from → until	Qty
1	-	Push Brakes <i>[SJ_8100][SJ_8120]</i>		1
2	-	Pull Brakes <i>[SJ_8110][SJ_8130]</i>		1
3	-	Attendant Hub Brake <i>[SJ_8160]</i>		1


Push Brakes

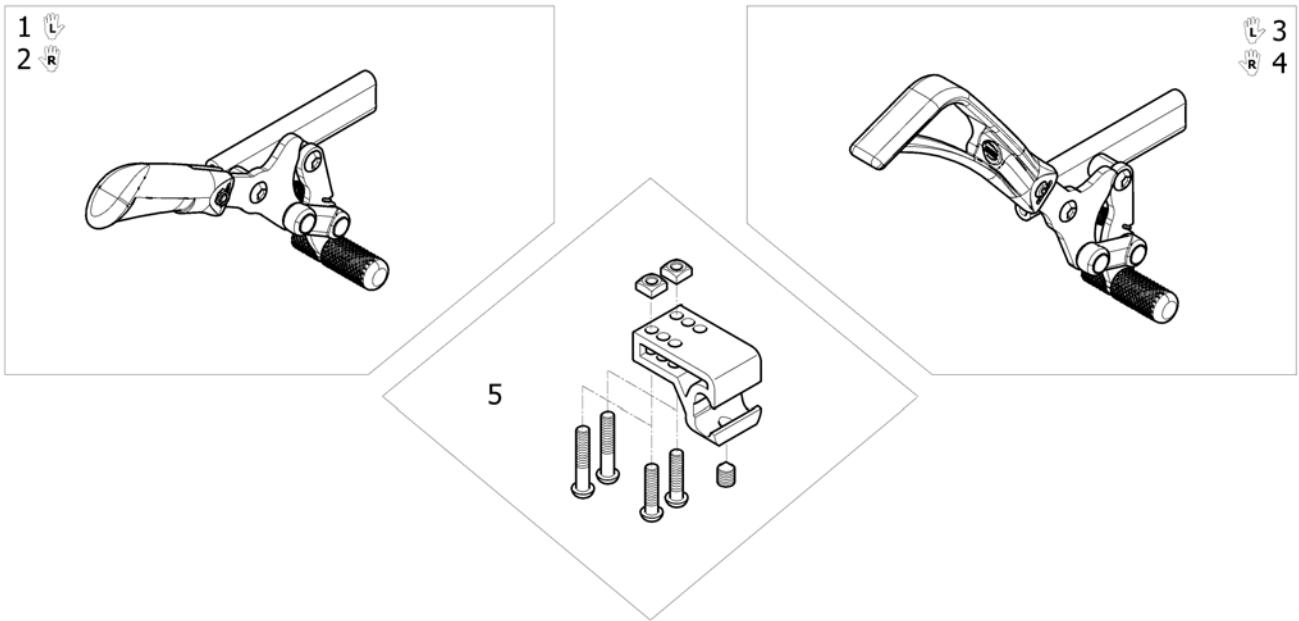

ISP51589977


Pos.	Part number	Description	Valid from → until	Qty
1	SP1581538	Push brake, Lh <i>[SJ_8100]</i>		1
2	SP1581537	Push brake, Rh <i>[SJ_8100]</i>		1
3	SP1581546	Push brake, Lh, extension handle <i>[SJ_8120]</i>		1
4	SP1581545	Push brake, Rh, extension handle <i>[SJ_8120]</i>		1
5	SP1589978	Kit brakes clamp		2

Pull Brakes

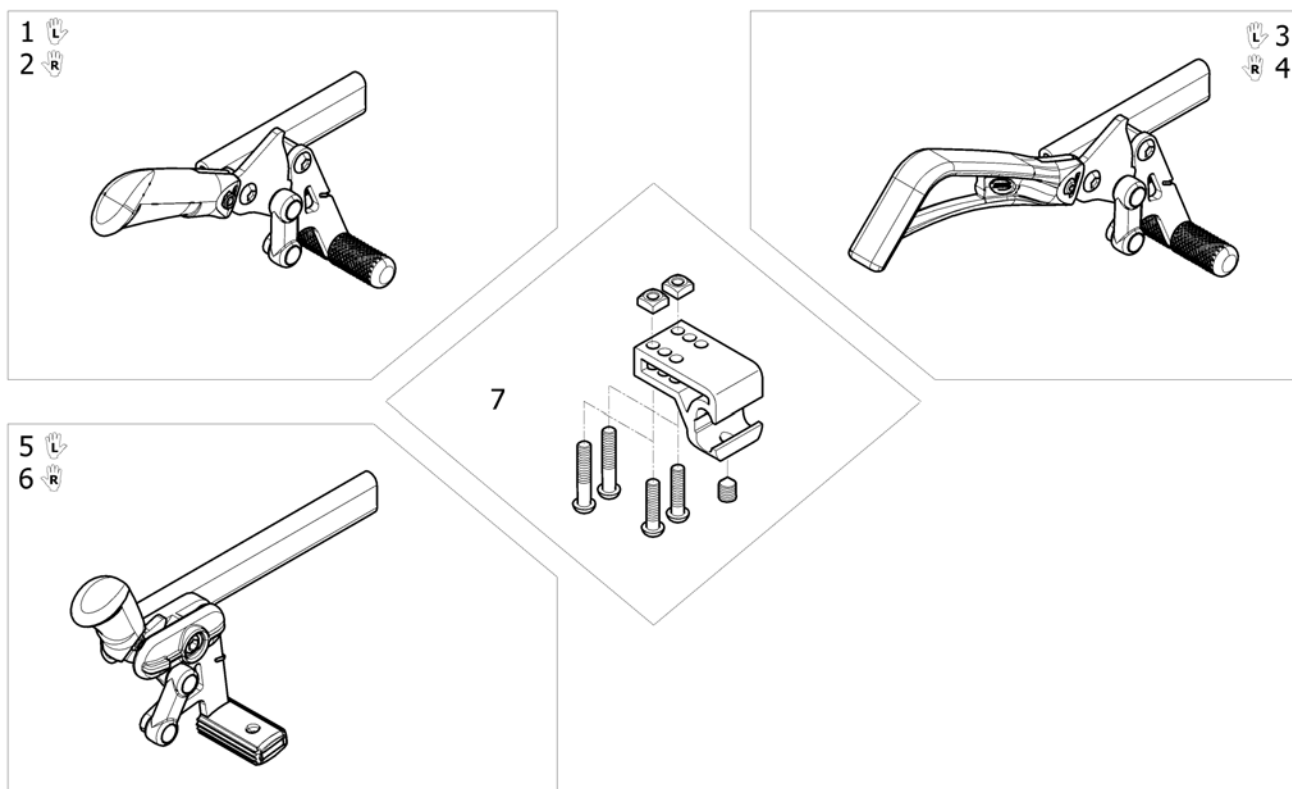





ISP51590815

Pos.	Part number	Description	Valid from → until	Qty
1	SP1581534	Pull brake, Lh <i>[SJ_8110]</i>		1
2	SP1581533	Pull brake, Rh <i>[SJ_8110]</i>		1
3	SP1581542	Pull brake, Lh, extension handle <i>[SJ_8130]</i>		1
4	SP1581541	Pull brake, Rh, extension handle <i>[SJ_8130]</i>		1
5	SP1559151	Left pull brake long support		1
6	SP1559150	Right pull brake long support		1
7	SP1589978	Kit brakes clamp		2

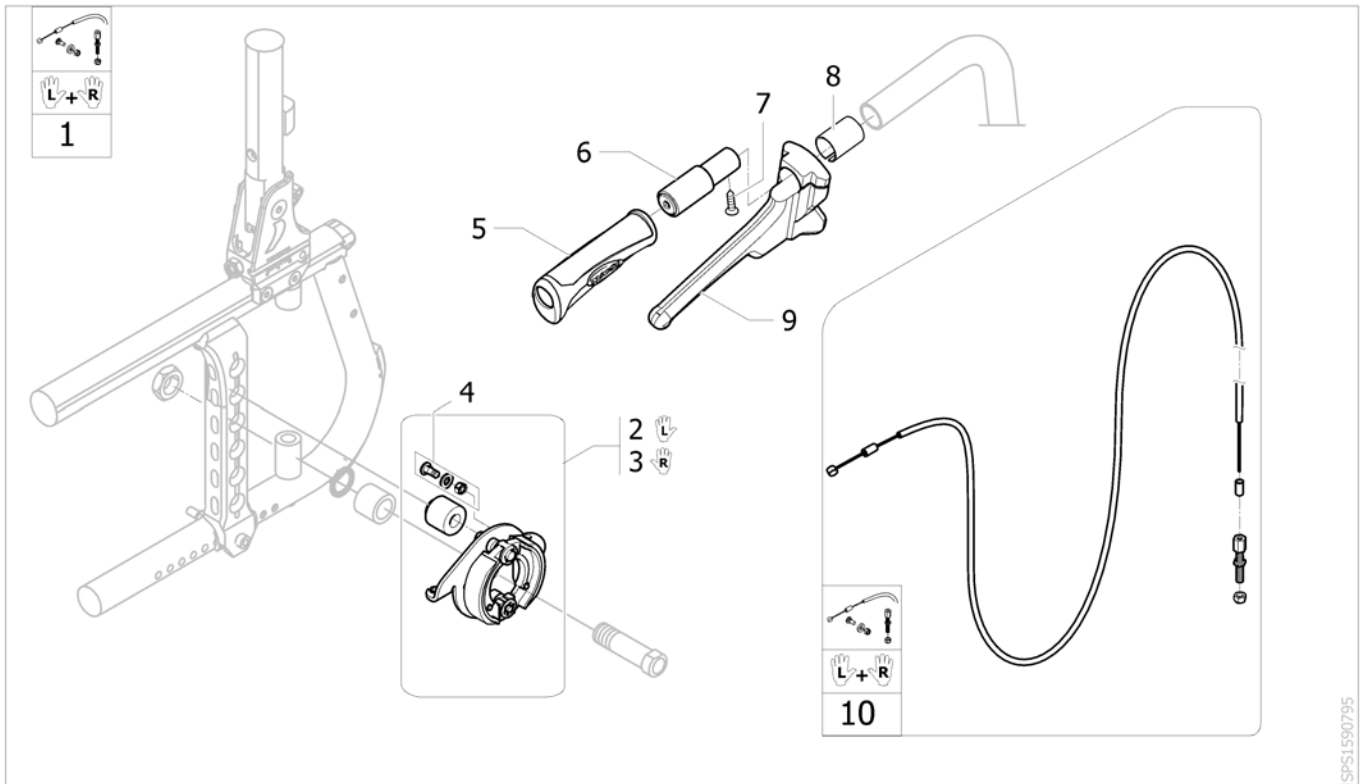

Pos.	Part number	Description	Valid from → until	Qty
1	SP1601543	Kit attendant hub brake complete (without wheels) [SJ_8160]		1
2	SP1601544	Kit hub brake plate Lh with spacer		1
3	SP1601545	Kit hub brake plate Rh with spacer		1
4	M30071	Cable Clamp Bolt c/wNut& Wash		2
5	1425473	Handle Invacare for backrest Ø22mm		2
6	1425530	Insert push handle		2
7	1428603	Screw Tfb 4,8x19 A2r Tx25 DIN7983		2
8	5221363	Fixed Backrest Hub Brake Adaptation		2
9	1420527	Black Brake Lever dia 22mm		2
10	1564821	Complete cable for attendant hub brake (pair)		1


Safety Options

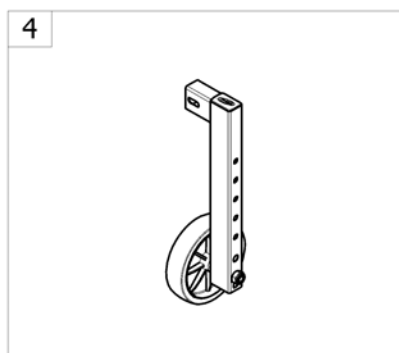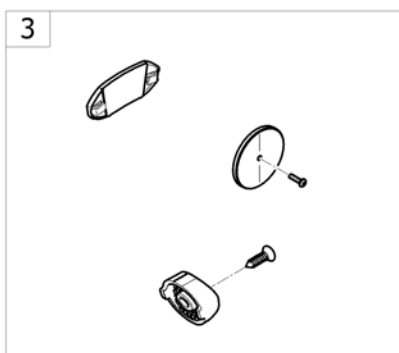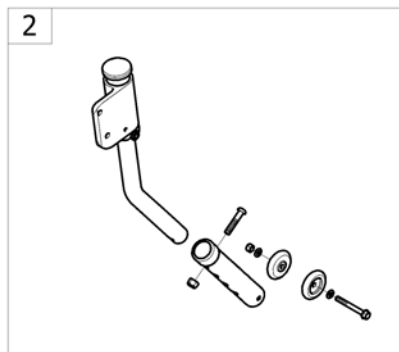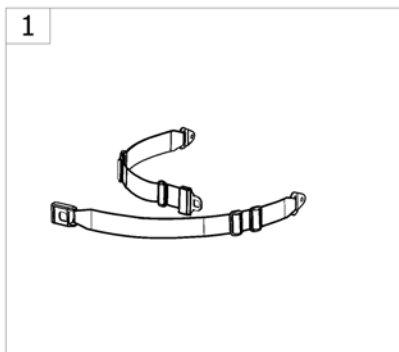

-<SPCACTIONS90

Pos.	Part number	Description	Valid from → until	Qty
1	-	Buckled belt [SJ_8300]		1
2	-	Buckled belt (UK) [SJ_8300] ONLY FOR UK		1
3	-	Anti-tipper [SJ_8500/SJ_8550]		1
4	-	Reflectors [SJ_8600]		1
5	-	Kit anti barrier [SJ_8900] ONLY FOR ITALY		1


Buckled belt

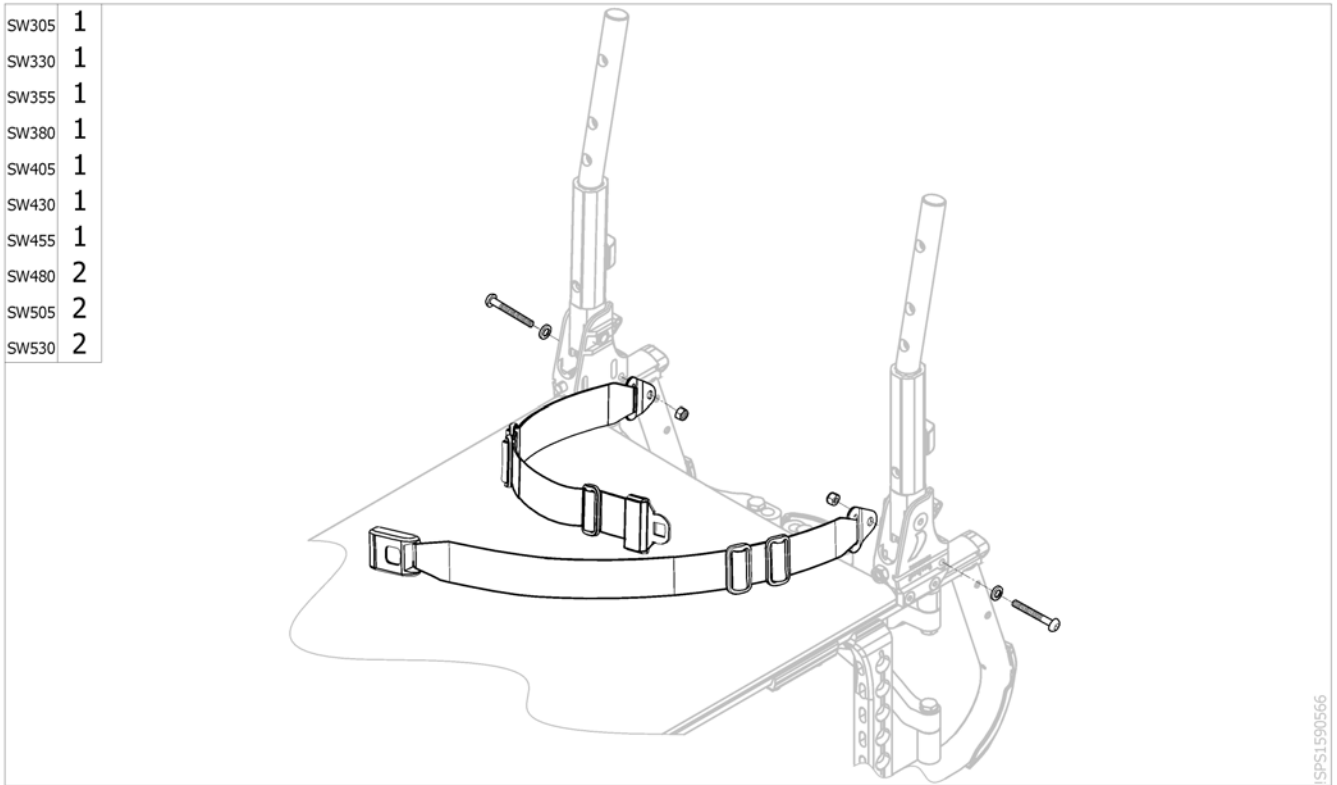

Pos.	Part number	Description	Valid from → until	Qty
1	SP1590426	Kit buckle belt metal SW305-455mm <i>[SJ_8300]</i>		1
2	SP1590427	Kit buckle belt metal SW480-530mm <i>[SJ_8300]</i>		1

Buckled belt (UK)



SW305	1
SW330	1
SW355	1
SW380	1
SW405	-
SW430	-
SW455	-
SW480	-
SW505	-
SW530	-

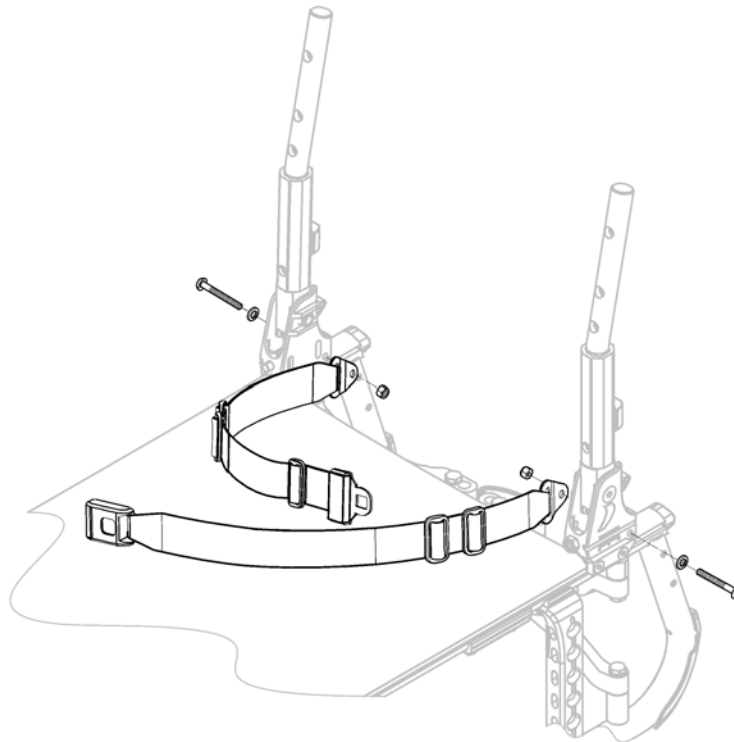

ISPS1606123

Pos.	Part number	Description	Valid from → until	Qty
------	-------------	-------------	--------------------	-----




Anti-tipper

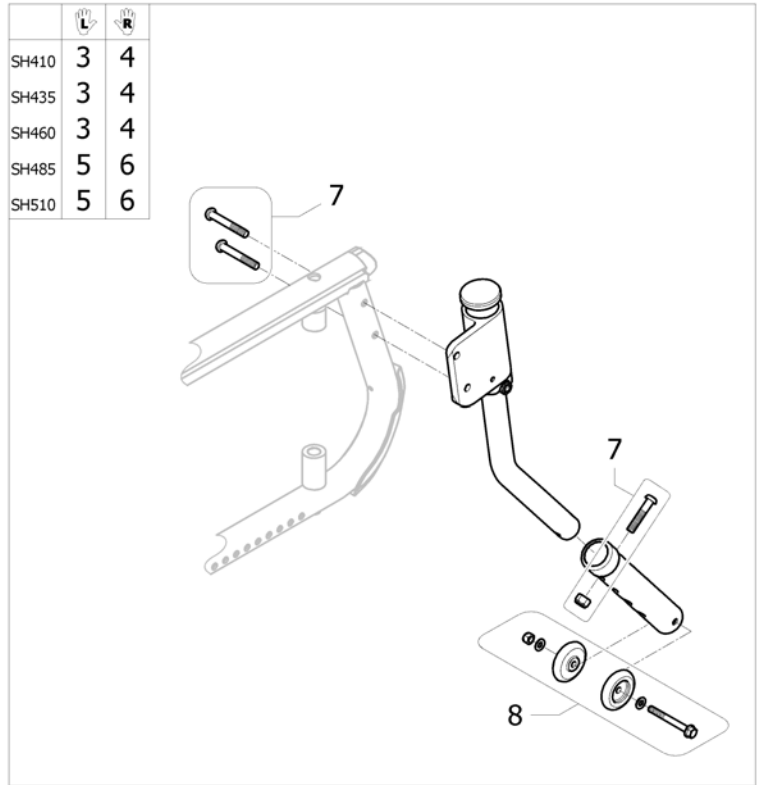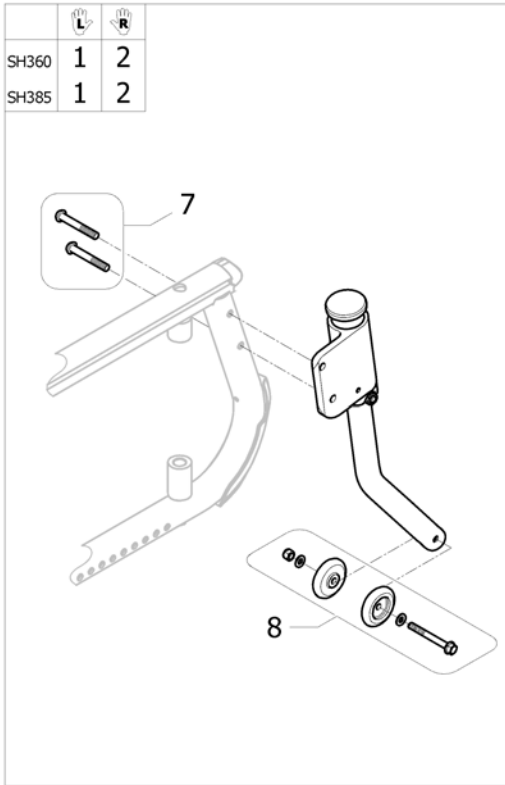

ESP51502180

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586894	Kit anti-tipper active, Lh, low [(SJ_1300/SJ_1310)+SJ_8550]		1
1	SP1586894-0004	Kit anti-tipper active, Lh, low Black Anod [(SJ_1300/SJ_1310)+SJ_8550]		1
2	SP1584735	Kit anti-tipper active, Rh, low [(SJ_1300/SJ_1310)+SJ_8500]		1
2	SP1584735-0004	Kit anti-tipper active, Rh, low Black anod [(SJ_1300/SJ_1310)+SJ_8500]		1
3	SP1586895	Kit anti-tipper active, Lh, medium [(SJ_1320/SJ_1330/SJ_1340)+SJ_8550]		1
3	SP1586895-0004	Kit anti-tipper active, Lh, medium Black Anod [(SJ_1320/SJ_1330/SJ_1340)+SJ_8550]		1
4	SP1584736	Kit anti-tipper active, Rh, medium [(SJ_1320/SJ_1330/SJ_1340)+SJ_8500]		1
4	SP1584736-0004	Kit anti-tipper active, Rh, medium Black Anod [(SJ_1320/SJ_1330/SJ_1340)+SJ_8500]		1
5	SP1586896	Kit anti-tipper active, Lh, high [(SJ_1350/SJ_1360)+SJ_8550]		1
5	SP1586896-0004	Kit anti-tipper active, Lh, high Black Anod [(SJ_1350/SJ_1360)+SJ_8550]		1
6	SP1584737	Kit anti-tipper active, Rh, high [(SJ_1350/SJ_1360)+SJ_8500]		1
6	SP1584737-0004	Kit anti-tipper active, Rh, high Black Anod [(SJ_1350/SJ_1360)+SJ_8500]		1
7	SP1589974	Kit screw for Anti-tipper		1
8	1548802	Kit wheels + attachment anti-tippers (washers out)		1


Reflectors

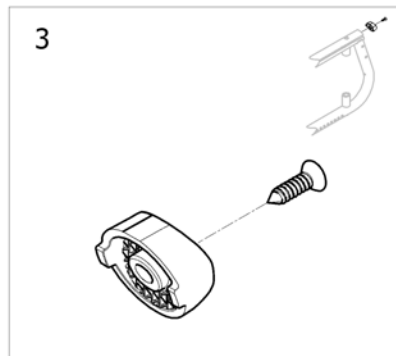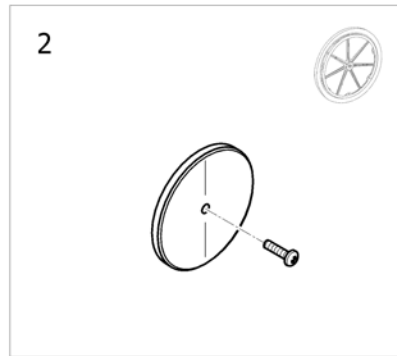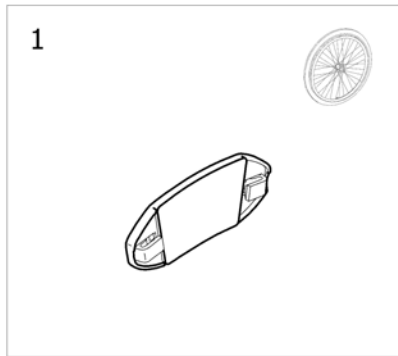

ISPS1582181





Pos.	Part number	Description	Valid from → until	Qty
1	N90003	Kit reflector for spoked wheel [[SJ_7200/SJ_7210/SJ_7230/SJ_7240/SJ_7244/SJ_7250/SJ_7260)+SJ_8600]		2
2	9037-30001	Kit reflector for composite wheel [SJ_7242+SJ_8600]		2
3	SP1589683	Kit rear cap reflector for frame		2

Kit anti barrier

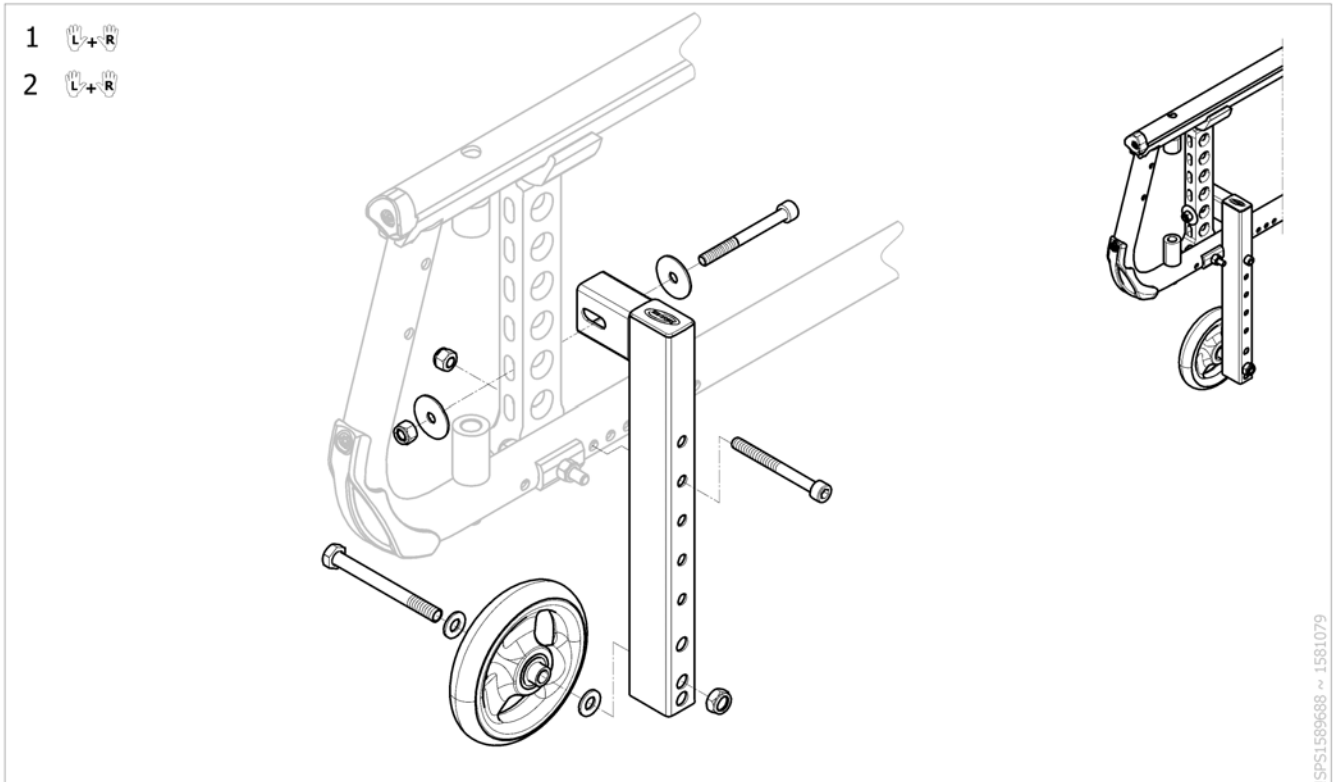

Pos.	Part number	Description	Valid from → until	Qty
1	SP1590532	Kit anti barrier low		1
2	SP1590533	Kit anti barrier high		1

Seating Options

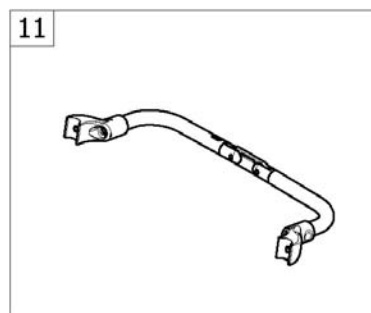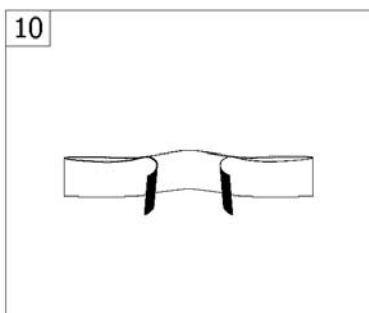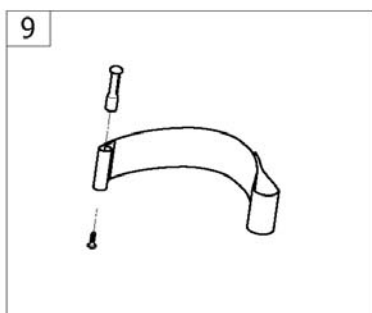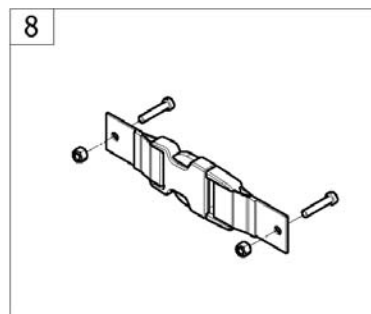

-<SPACATION>S92

Pos.	Part number	Description	Valid from → until	Qty
1	-	Cushion foam Nylon & anti-slip cover <i>[SJ_5000] ONLY FOR EUROPE</i>		1
2	-	Matrx Flo-Tech CONTOUR VISCO cushions <i>[SJ_5010] ONLY FOR EUROPE</i>		1
3	-	Matrx Flo-Tech IMAGE cushions <i>[SJ_5020] ONLY FOR EUROPE</i>		1
4	-	Matrx Flo-Tech LITE cushions <i>[SJ_5030] ONLY FOR EUROPE</i>		1
5	-	Matrx Flo-Tech XTRA cushions <i>[SJ_5040] ONLY FOR EUROPE</i>		1
6	-	Matrx VI Cushions <i>[SJ_5042] ONLY FOR EUROPE</i>		1
7	-	Matrx LIBRA Cushions <i>[SJ_5044] ONLY FOR EUROPE</i>		1
8	-	Clip to keep the chair folded <i>[SJ_5100]</i>		1
9	-	Heels strap <i>[SJ_5300]</i>		1
10	-	Calf strap <i>[SJ_5400]</i>		1
11	-	Folding backbrace <i>[SJ_5500]</i>		1

Cushion foam Nylon & anti-slip cover

	SD350	SD375	SD400	SD425	SD450	SD475	SD500
SW305	1	5	5	10	10	-	-
SW330	2	5	5	10	10	-	-
SW355	3	6	6	7	7	8	8
SW380	4	6	6	7	7	8	8
SW405	-	7	7	11	11	12	12
SW430	-	7	7	11	11	12	12
SW455	-	8	8	12	12	14	14
SW480	-	8	8	12	12	14	14
SW505	-	-	9	13	13	15	15
SW530	-	-	9	13	13	15	15



ISPS:1589738

Pos.	Part number	Description	Valid from → until	Qty
1	1500971	Essential Junior cushion (I), SW305 x SD275/325 mm <i>[SJ_5000]</i>		1
2	1500973	Essential Junior cushion (K), SW330 x SD275/325 mm <i>[SJ_5000]</i>		1
3	1500975	Essential Junior cushion (M), SW355 x SD275/325 mm <i>[SJ_5000]</i>		1
4	1500977	Essential Junior cushion (O), SW380 x SD275/325 mm <i>[SJ_5000]</i>		1
5	SP1564260	Cushion foam, 312 x 362mm <i>[SJ_5000]</i>		1
6	SP1564263	Cushion foam, 362 x 362mm <i>[SJ_5000]</i>		1
7	SP1564264	Cushion foam, 362 x 412mm <i>[SJ_5000]</i>		1
8	SP1564265	Cushion foam, 362 x 462mm <i>[SJ_5000]</i>		1
9	SP1564266	Cushion foam, 362 x 500mm <i>[SJ_5000]</i>		1
10	SP1564261	Cushion foam, 312 x 412mm <i>[SJ_5000]</i>		1
11	SP1564267	Cushion foam, 412 x 412mm <i>[SJ_5000]</i>		1
12	SP1564268	Cushion foam, 412 x 462mm <i>[SJ_5000]</i>		1
13	SP1564269	Cushion foam, 412 x 500mm <i>[SJ_5000]</i>		1
14	SP1564270	Cushion foam, 462 x 462mm <i>[SJ_5000]</i>		1
15	SP1564271	Cushion foam, 462 x 500mm <i>[SJ_5000]</i>		1

Matrx Flo-Tech CONTOUR VISCO cushions

	SD350	SD375	SD400	SD425	SD450	SD475	SD500
SW305	-	-	-	-	-	-	-
SW330	-	-	-	-	-	-	-
SW355	-	-	-	-	-	-	-
SW380	-	-	1	1	7	7	-
SW405	-	-	2	2	8	8	-
SW430	-	-	3	3	9	9	-
SW455	-	-	4	4	10	10	-
SW480	-	-	5	5	11	11	-
SW505	-	-	6	6	12	12	-
SW530	-	-	-	-	-	-	-



ISPS:1559739

Pos.	Part number	Description	Valid from → until	Qty
1	1559590	Cushion Contour Visco, SW380 x SD405 x H90mm MUW:100Kg (FCSB18/18/LMF/SGL) <i>[SJ_5010]</i>		1
2	1559659	Cushion Contour Visco, SW405 x SD405 x H90mm MUW:100Kg (FLSB16/16/LMF/SGL) <i>[SJ_5010]</i>		1
3	1559730	Cushion Contour Visco, SW430 x SD405 x H90mm MUW:100Kg (FLSB17/16/LMF/SGL) <i>[SJ_5010]</i>		1
4	1501114	Cushion Contour Visco, SW455 x SD405 x H90mm MUW:100Kg (FCSB18/16/LMF/SGL) <i>[SJ_5010]</i>		1
5	1501116	Cushion Contour Visco, SW480 x SD405 x H90mm MUW:100Kg (FCSB19/16/LMF/SGL) <i>[SJ_5010]</i>		1
6	1501118	Cushion Contour Visco, SW505 x SD405 x H90mm MUW:125Kg (FCSB20/16/LMF/SGL) <i>[SJ_5010]</i>		1
7	1559657	Cushion Contour Visco, SW380 x SD455 x H90mm MUW:100Kg (FLSB15/18/LMF/SGL) <i>[SJ_5010]</i>		1
8	1526594	Cushion Contour Visco, SW405 x SD455 x H90mm MUW:100Kg (FCSB16/18/LMF/SGL) <i>[SJ_5010]</i>		1
9	1526595	Cushion Contour Visco, SW430 x SD455 x H90mm MUW:100Kg (FCSB17/18/LMF/SGL) <i>[SJ_5010]</i>		1
10	1526608	Cushion Contour Visco, SW455 x SD455 x H90mm MUW:100Kg (FCSB18/18/LMF/SGL) <i>[SJ_5010]</i>		1
11	1525020	Cushion Contour Visco, SW480 x SD455 x H90mm MUW:125Kg (FCSB19/18/LMF/SGL) <i>[SJ_5010]</i>		1
12	1525021	Cushion Contour Visco, SW505 x SD455 x H90mm MUW:125Kg (FCSB20/18/LMF/SGL) <i>[SJ_5010]</i>		1

Matrix Flo-Tech IMAGE cushions

	SD350	SD375	SD400	SD425	SD450	SD475	SD500
SW305	-	-	-	-	-	-	-
SW330	-	-	-	-	-	-	-
SW355	-	-	-	-	-	-	-
SW380	-	-	1	1	7	7	-
SW405	-	-	2	2	8	8	-
SW430	-	-	3	3	9	9	-
SW455	-	-	4	4	10	10	-
SW480	-	-	5	5	11	11	-
SW505	-	-	6	6	12	12	-
SW530	-	-	-	-	-	-	-



ISPS:1587340

Pos.	Part number	Description	Valid from → until	Qty
1	1575728	Cushion Image, SW380 x SD405 x H78mm, MUW-140kg (FISB15/16) [SJ_5020]		1
2	1511196	Cushion Image, SW405 x SD405 x H78mm, MUW-140kg (FISB16/16) [SJ_5020]		1
3	1511197	Cushion Image, SW430 x SD405 x H78mm, MUW-140kg (FISB17/16) [SJ_5020]		1
4	1511198	Cushion Image, SW455 x SD405 x H78mm, MUW-140kg (FISB18/16) [SJ_5020]		1
5	1511199	Cushion Image, SW480 x SD405 x H78mm, MUW-140kg (FISB19/16) [SJ_5020]		1
6	1526417	Cushion Image, SW505 x SD405 x H78mm, MUW-140kg (FISB20/16) [SJ_5020]		1
7	1575729	Cushion Image, SW380 x SD455 x H78mm, MUW-140kg (FISB15/18) [SJ_5020]		1
8	1526412	Cushion Image, SW405 x SD455 x H78mm, MUW-140kg (FISB16/18) [SJ_5020]		1
9	1526413	Cushion Image, SW430 x SD455 x H78mm, MUW-140kg (FISB17/18) [SJ_5020]		1
11	1526415	Cushion Image, SW480 x SD455 x H78mm, MUW-140kg (FISB19/18) [SJ_5020]		1
10	1526414	Cushion Image, SW455 x SD455 x H78mm, MUW-140kg (FISB18/18) [SJ_5020]		1
12	1526416	Cushion Image, SW505 x SD455 x H78mm, MUW-140kg (FISB20/18) [SJ_5020]		1

Matrix Flo-Tech LITE cushions

	SD350	SD375	SD400	SD425	SD450	SD475	SD500
SW305	-	-	-	-	-	-	-
SW330	-	-	-	-	-	-	-
SW355	-	-	-	-	-	-	-
SW380	-	-	1	1	7	7	-
SW405	-	-	2	2	8	8	-
SW430	-	-	3	3	9	9	-
SW455	-	-	4	4	10	10	-
SW480	-	-	5	5	11	11	-
SW505	-	-	6	6	12	12	-
SW530	-	-	-	-	-	-	-



ISPS:1589741

Pos.	Part number	Description	Valid from → until	Qty
8	1575610	Cushion Lite, SW405 x SD450mm <i>[SJ_5030]</i>		1
1	1575607	Cushion Lite, SW380 x SD400mm <i>[SJ_5030]</i>		1
2	1575609	Cushion Lite, SW405 x SD400mm <i>[SJ_5030]</i>		1
3	1575611	Cushion Lite, SW430 x SD400mm <i>[SJ_5030]</i>		1
4	1575613	Cushion Lite, SW455 x SD400mm <i>[SJ_5030]</i>		1
5	1575615	Cushion Lite, SW480 x SD400mm <i>[SJ_5030]</i>		1
6	1575617	Cushion Lite, SW505 x SD400mm <i>[SJ_5030]</i>		1
7	1575608	Cushion Lite, SW380 x SD450mm <i>[SJ_5030]</i>		1
9	1575612	Cushion Lite, SW430 x SD450mm <i>[SJ_5030]</i>		1
10	1575614	Cushion Lite, SW455 x SD450mm <i>[SJ_5030]</i>		1
11	1575616	Cushion Lite, SW480 x SD450mm <i>[SJ_5030]</i>		1
12	1575618	Cushion Lite, SW505 x SD450mm <i>[SJ_5030]</i>		1

Matrix Flo-Tech XTRA cushions

	SD350	SD375	SD400	SD425	SD450	SD475	SD500
SW305	-	-	-	-	-	-	-
SW330	-	-	-	-	-	-	-
SW355	-	-	-	-	-	-	-
SW380	-	-	1	1	7	7	-
SW405	-	-	2	2	8	8	-
SW430	-	-	3	3	9	9	-
SW455	-	-	4	4	10	10	-
SW480	-	-	5	5	11	11	-
SW505	-	-	6	6	12	12	-
SW530	-	-	-	-	-	-	-



ISPS:15589742

Pos.	Part number	Description	Valid from → until	Qty
1	1494430	Cushion Solution Xtra, SW380 x SD405mm <i>[SJ_5040]</i>		1
2	1487145	Cushion Solution Xtra, SW405 x SD405mm <i>[SJ_5040]</i>		1
3	1494441	Cushion Solution Xtra, SW430 x SD405mm <i>[SJ_5040]</i>		1
4	1487147	Cushion Solution Xtra, SW455 x SD405mm <i>[SJ_5040]</i>		1
5	1494451	Cushion Solution Xtra, SW480 x SD405mm <i>[SJ_5040]</i>		1
6	1552058	Cushion Soutlion Xtra, SW505 x SD405mm <i>[SJ_5040]</i>		1
7	1494432	Cushion Solution Xtra, SW380 x SD455mm <i>[SJ_5040]</i>		1
8	1551914	Cushion Solution Xtra, SW405 x SD455mm <i>[SJ_5040]</i>		1
9	1551922	Cushion Solution Xtra, SW430 x SD455mm <i>[SJ_5040]</i>		1
10	1551925	Cushion Solution Xtra, SW455 x SD455mm <i>[SJ_5040]</i>		1
11	1552055	Cushion Solution Xtra, SW480 x SD455mm <i>[SJ_5040]</i>		1
12	1552060	Cushion Solution Xtra, SW505 x SD455mm <i>[SJ_5040]</i>		1

Matrx VI Cushions

	SD350	SD375	SD400	SD425	SD450	SD475	SD500
SW305	-	-	-	-	-	-	-
SW330	-	-	-	-	-	-	-
SW355	-	-	-	-	-	-	-
SW380	-	-	1	-	5	-	9
SW405	-	-	1	-	5	-	9
SW430	-	-	2	-	6	-	10
SW455	-	-	3	-	7	-	11
SW480	-	-	4	-	8	-	12
SW505	-	-	4	-	8	-	12
SW530	-	-	-	-	-	-	-



ISPS:1603975

Pos.	Part number	Description	Valid from → until	Qty
1	1579786	Matrx Flo-Tech VI cushions 410x410 mm		1
2	1579829	Matrx Flo-Tech VI cushions 430x410 mm		1
3	1579920	Matrx Flo-Tech VI cushions 460x410 mm		1
4	1579923	Matrx Flo-Tech VI cushions 510x410 mm		1
5	1579787	Matrx Flo-Tech VI cushions 410x460 mm		1
6	1579918	Matrx Flo-Tech VI cushions 430x460 mm		1
7	1579921	Matrx Flo-Tech VI cushions 460x460 mm		1
8	1579924	Matrx Flo-Tech VI cushions 510x460 mm		1
9	1579788	Matrx Flo-Tech VI cushions 410x510 mm		1
10	1579919	Matrx Flo-Tech VI cushions 430x510 mm		1
11	1579922	Matrx Flo-Tech VI cushions 460x510 mm		1
12	1579925	Matrx Flo-Tech VI cushions 510x510 mm		1

Matrx LIBRA Cushions


	SD350	SD375	SD400	SD425	SD450	SD475	SD500
SW305	-	-	-	-	-	-	-
SW330	-	-	-	-	-	-	-
SW355	-	-	-	-	-	-	-
SW380	-	-	1	-	7	-	13
SW405	-	-	2	-	8	-	14
SW430	-	-	3	-	9	-	15
SW455	-	-	4	-	10	-	16
SW480	-	-	5	-	11	-	17
SW505	-	-	6	-	12	-	17
SW530	-	-	-	-	-	-	-



SPS1603976

Pos.	Part number	Description	Valid from → until	Qty
1	LC1516	Matrx Libra cushions 380x410 mm		1
2	LC1616	Matrx Libra cushions 410x410 mm		1
3	LC1716	Matrx Libra cushions 430x410 mm		1
4	LC1816	Matrx Libra cushions 460x410 mm		1
5	LC1916	Matrx Libra cushions 480x410 mm		1
6	LC2016	Matrx Libra cushions 510x410 mm		1
7	LC1518	Matrx Libra cushions 380x460 mm		1
8	LC1618	Matrx Libra cushions 410x460 mm		1
9	LC1718	Matrx Libra cushions 430x460 mm		1
10	LC1818	Matrx Libra cushions 460x460 mm		1
11	LC1918	Matrx Libra cushions 480x460 mm		1
12	LC2018	Matrx Libra cushions 510x460 mm		1
13	LC1520	Matrx Libra cushions 380x510 mm		1
14	LC1620	Matrx Libra cushions 410x510 mm		1
15	LC1720	Matrx Libra cushions 430x510 mm		1
16	LC1820	Matrx Libra cushions 380x510 mm		1
17	LC1920	Matrx Libra cushions 480x510 mm		1
18	LC2020	Matrx Libra cushions 510x510 mm		1


Clip to keep the chair folded

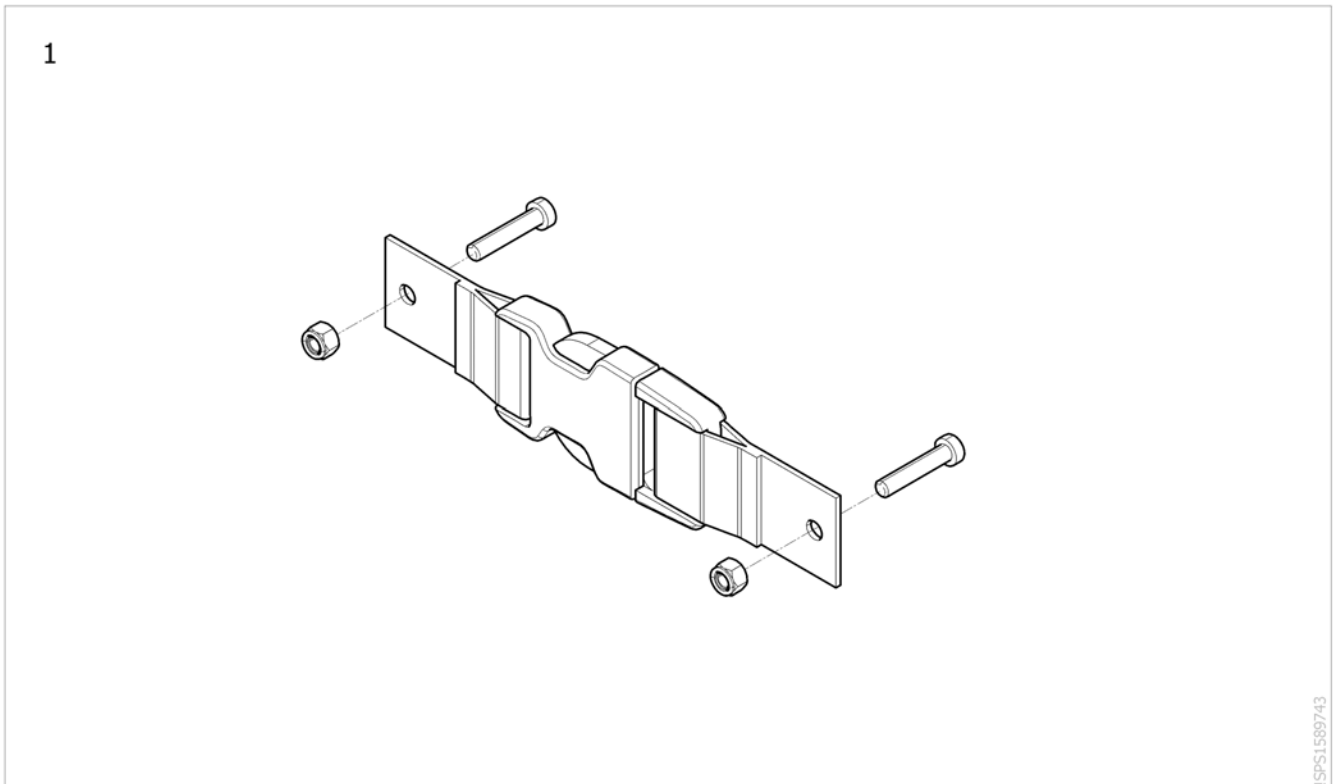

Pos.	Part number	Description	Valid from → until	Qty
1	SP1590820	Kit clip to keep the chair folded <i>[SJ_5100]</i>		1

Heels strap


SW305	1
SW330	1
SW355	1
SW380	2
SW405	2
SW430	2
SW455	2
SW480	2
SW505	2
SW530	2

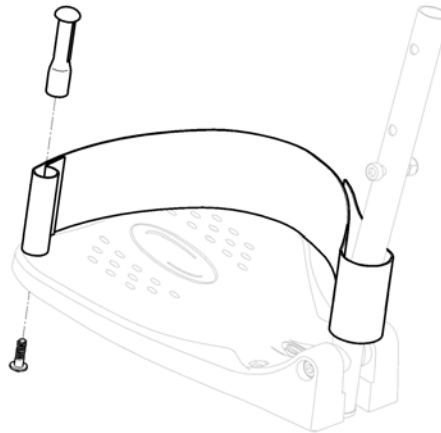

ISPS1589744


Pos.	Part number	Description	Valid from → until	Qty
1	0587-05201	Kit heelstrap, adjustable, small <i>[SJ_5300]</i>		2
2	1492315	Kit heelstrap, adjustable, medium <i>[SJ_5300]</i>		2

Calf strap

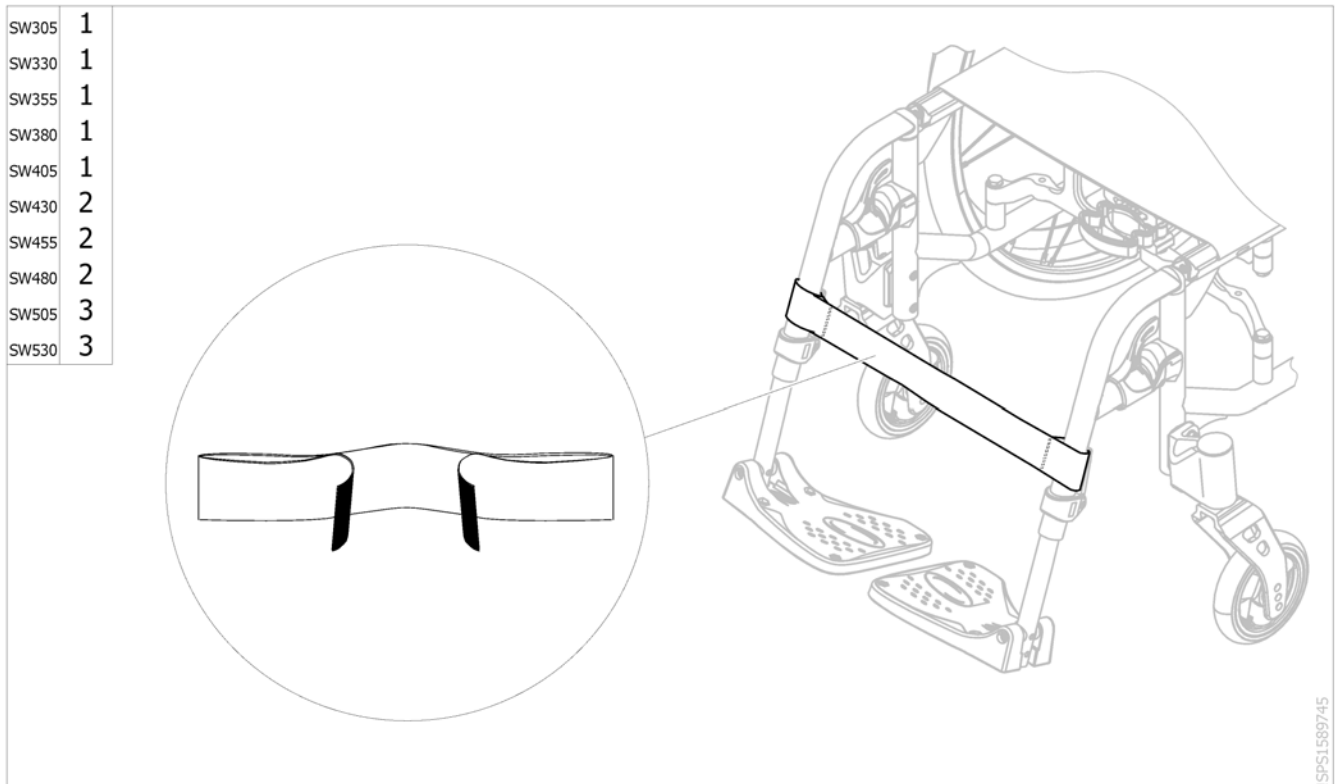




Pos.	Part number	Description	Valid from → until	Qty
1	0572-05794	Calf strap adjustable, small, SW305-405mm <i>[SJ_5400]</i>		1
2	1433580	Calf strap adjustable, medium <i>[SJ_5400]</i>		1
3	1497574	Calf strap adjustable, large, SW505-605mm <i>[SJ_5400]</i>		1

Folding backbrace

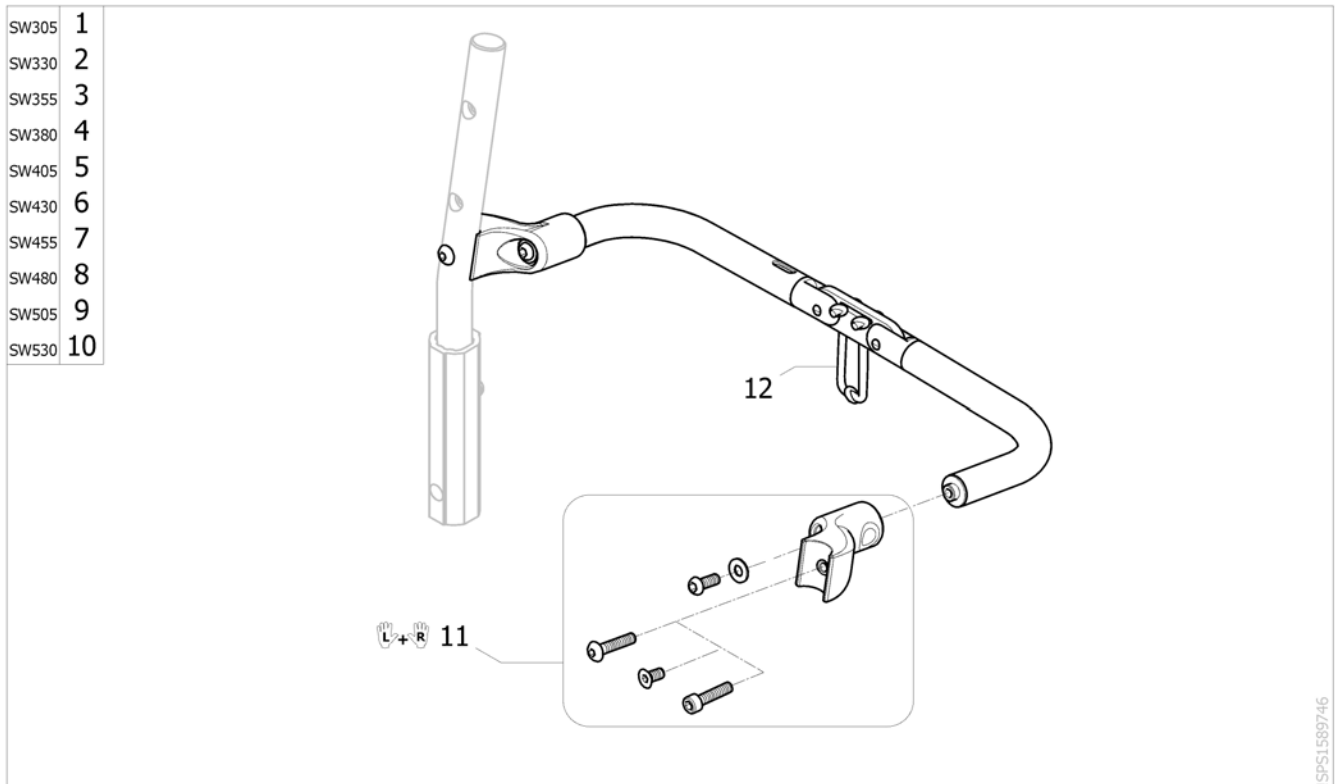

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583662	Kit folding backbrace, SW305mm [SJ_5500]		1
2	SP1583663	Kit folding backbrace, SW330mm [SJ_5500]		1
3	SP1583664	Kit folding backbrace, SW355mm [SJ_5500]		1
4	SP1583665	Kit folding backbrace, SW380mm [SJ_5500]		1
5	SP1583666	Kit folding backbrace, SW405mm [SJ_5500]		1
6	SP1583667	Kit folding backbrace, SW430mm [SJ_5500]		1
7	SP1583668	Kit folding backbrace, SW455mm [SJ_5500]		1
8	SP1583669	Kit folding backbrace, SW480mm [SJ_5500]		1
9	SP1583724	Kit folding backbrace, SW505mm [SJ_5500]		1
10	SP1583725	Kit folding backbrace, SW530mm [SJ_5500]		1
11	SP1589801	Kit fixation for backbrace (pair)		1
12	SP1601900	Cord for backbrace & angle/fold down backrest attachment		1


Other Options & Accessories

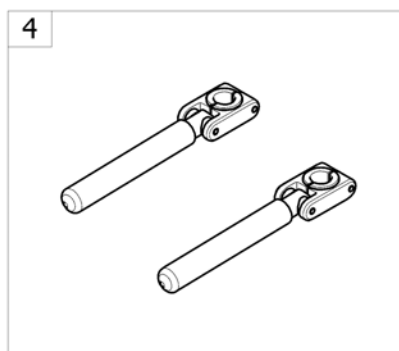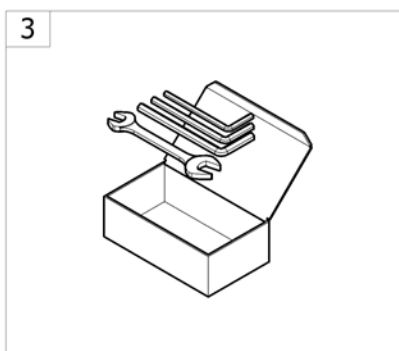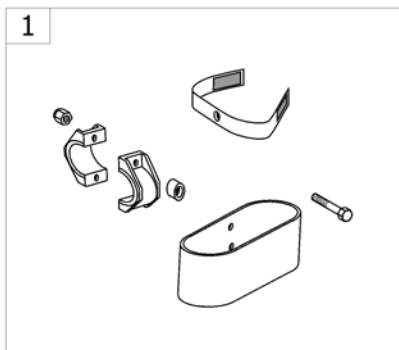

->SPCACTION598


Pos.	Part number	Description	Valid from → until	Qty
1	-	Cane holder <i>[SJ_9000]</i>		1
2	-	Tipping aid <i>[SJ_9100/SJ_9200]</i>		1
3	-	Tool Kit <i>[SJ_9300]</i>		1
4	-	Case holder <i>[SJ_9400]</i>		1

Cane holder

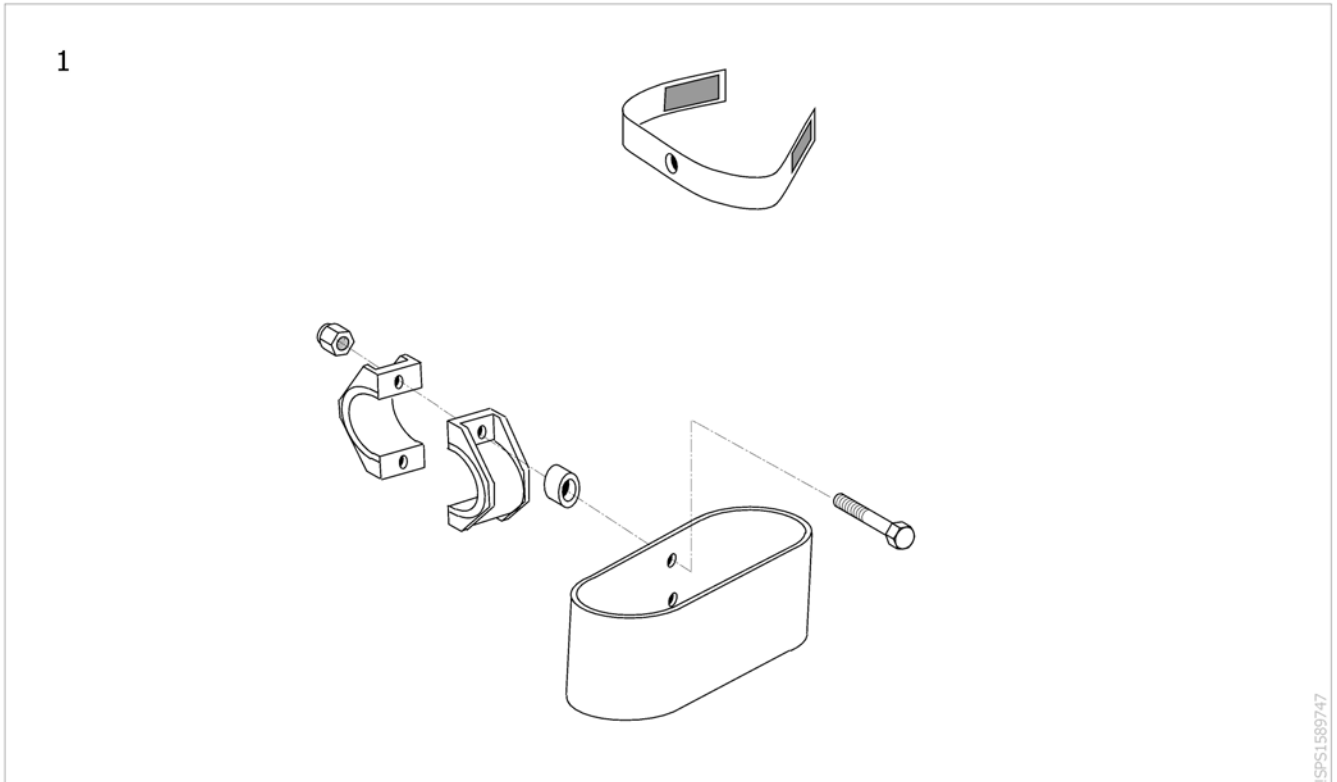

Pos.	Part number	Description	Valid from → until	Qty
1	0572-02056	Cane holder [SJ_9000]		1



Tipping aid

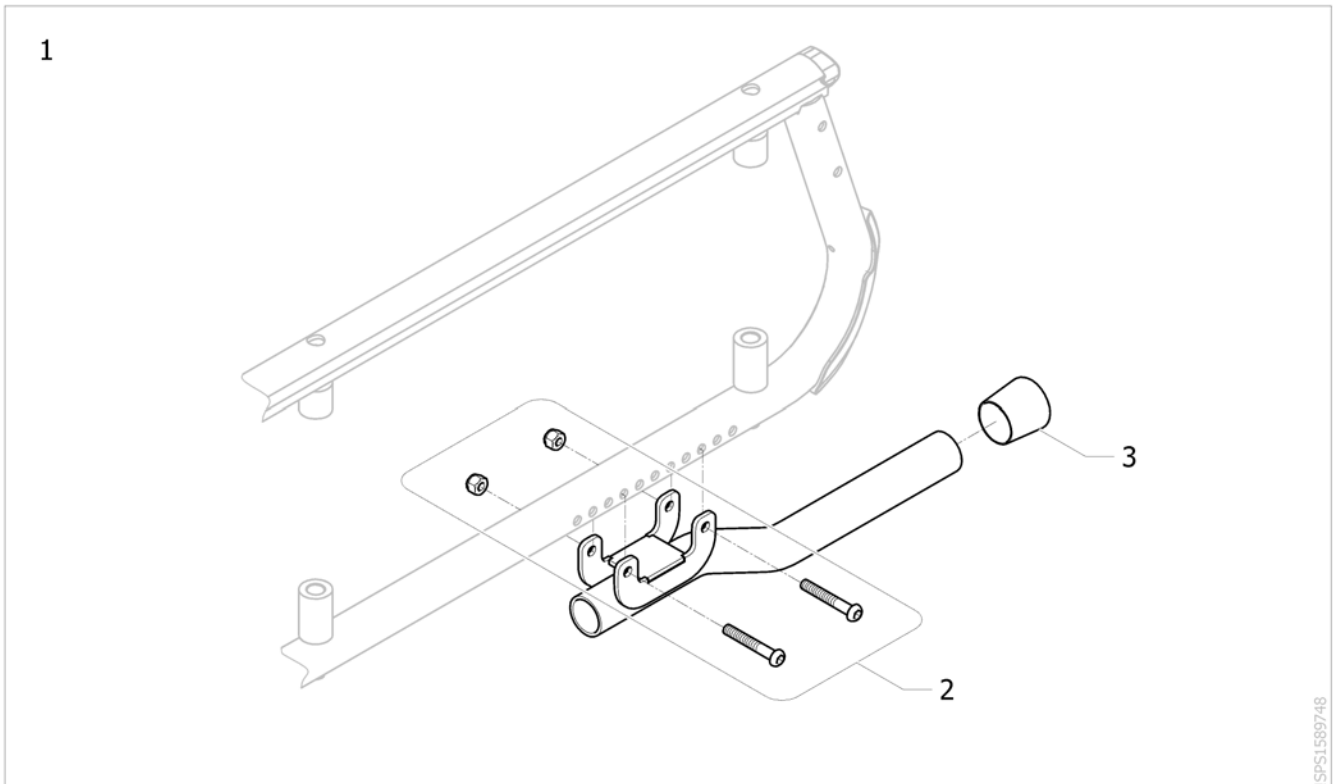

ISPS1589748

Pos.	Part number	Description	Valid from → until	Qty
1	SP1581062	Tipping aid complete <i>[SJ_9100/SJ_9200]</i>		1
2	SP1589693	Kit fixation tipping aid		1
3	SP1589694	Plastic cap for tipping aid		1


Tool Kit

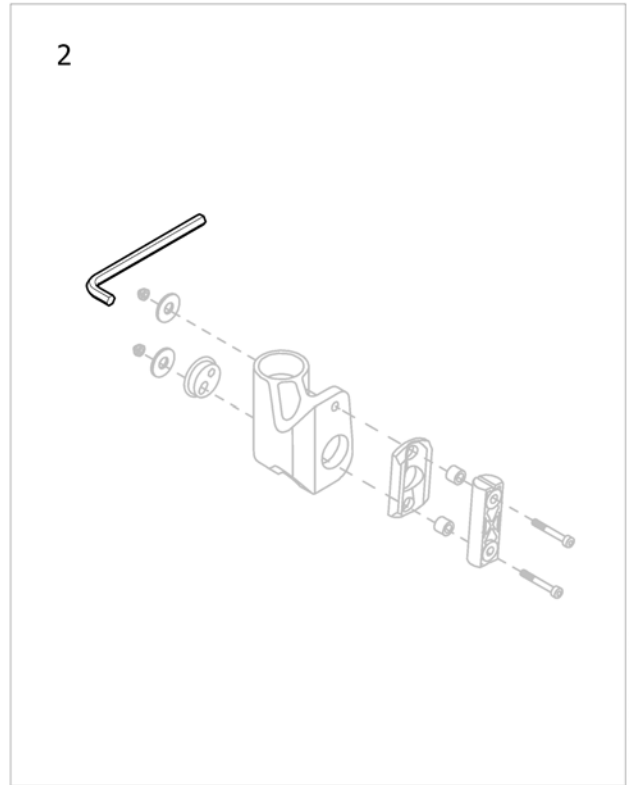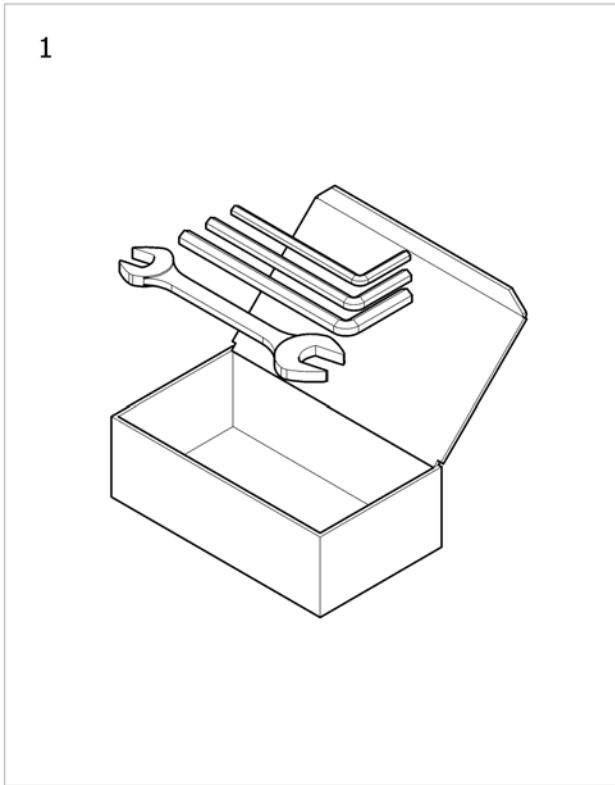

ESPS1604323 ~ 1603974

Pos.	Part number	Description	Valid from → until	Qty
1	SP1603974	Tool Kit		1
2	SP1586506	Short Metric Hex Key (5mm)		1

Case holder

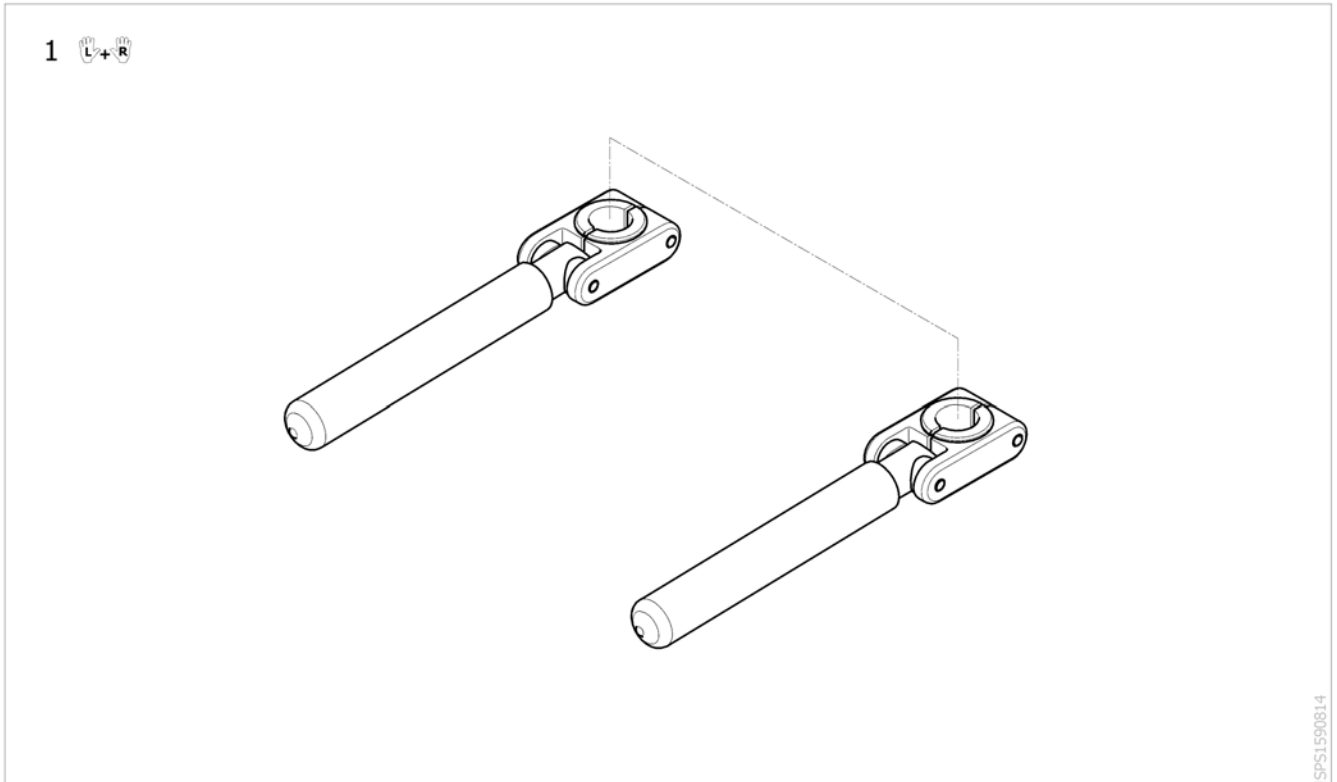

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587821	Case holder (pair) [SJ_9400]		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk