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**USER MANUAL (EN – p16)**

**BEDIENUNGSANLEITUNG (DE – p21)**

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## **MOBIRAMP ALU SERIES**

**TYPE MRT**

## GENERAL

### Technische specificaties / Caractéristiques techniques / Technical specifications / Technische Daten / Tekniska specifikationer

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USER MANUAL (EN)

**Important !**

Read the full user manual before using your new ramp for the first time.

**1. Usage:**

**1.1. General:**

- The Mobiramp Alu series mobile ramps are designed to bridge heights and steps for wheelchair users and people with reduced mobility.
- Use the ramp only for the above purposes

**1.2. Before the first usage:**

- The ramp must be assembled with 2 persons
- Carefully remove the packaging. All packaging material is recyclable.
- If the packaging shows damage, carefully check the ramp for possible damage. If damage is found, DO NOT use the ramp until the manufacturer has given permission.
- Attach the supplied handles to the ramps with the supplied bolts (see mounting drawings: Fig 1).
- Attach the guides to the sides of the ramps (see mounting drawings: Fig 2).
- Mount the trolley (see mounting drawings: Fig 3).
- Place the ramps on the trolley (see mounting drawings: Fig 4).
- Carefully check that the ramps are properly anchored in the slots of the trolley.

## ENGLISH

### 1.3. Before each usage:

- Carefully remove any impurities such as sand, ice, snow, oil or other products that may affect the non-slip properties of the slope surface.
- Check the ramp for any damage that may reduce its load-bearing capacity. Do not use the ramp if the supporting structure is damaged.

### 1.4. During the usage:

- The ramp must be set up with 2 persons.
- Always lay the ramp with the anti-slip side up.
- The TOP side is indicated on the side of the ramp. This is the highest point of the ramp when it is ready for use.
- Place the trolley about 2m away from the threshold or stairs to be bridged.
- Apply the brakes to the wheels.
- Start with setting up the ramp-part that will be at its lowest point after installation.
- Lift this ramp up a few centimetres from the trolley with the handles attached to both sides.
- Fold the ramp open while holding both handles. This ramp will always slide into the slots of the trolley at its highest point.
- Place the lower part of this ramp on the ground for the threshold or step.
- Carefully check that this ramp is correctly positioned

- Then start setting up the ramp-part that will be at the highest point after installation.

## ENGLISH

- Raise this ramp a few centimetres from the trolley with the handles attached to both sides.
- Fold this ramp open while holding both handles. The ramp will always slide into the slots of the trolley at its lowest point.
- Place this ramp with the curved end on the edge of the threshold or step. Make sure that the curved end of this slope is sufficiently over the edge to avoid sliding.
- Carefully check that this slope has also been correctly positioned.
  
- Make sure that the beginning and end of the entire ramp is always in full length contact with the surface..

### 1.5. After de usage:

- The storage of the ramp must be done with 2 persons.
- Carefully remove any impurities such as sand, ice, snow, oil or other products that may affect the non-slip properties of the slope surface.
  
- Start by folding the ramp that is at the highest point during placement.
- Lift this ramp slightly up on the TOP side and fold it closed with the handles attached to both sides. At its lowest point, the ramp will always slide into the slots of the trolley.
- Carefully check that this ramp has been folded tightly and is finally anchored in the slots of the trolley.
  
- Then start to fold up the ramp that is at its lowest point during installation.
- Lightly lift this ramp up at the lower end and fold it up with the handles attached to both sides. At its highest point, the ramp will always slide into the slots of the trolley.
- Carefully check that this ramp has been folded tightly and is finally anchored in the slots of the trolley.
  
- Release the brakes on the wheels.
- Always transport the ramp mounted on the trolley.
- Check the ramp for any damage that may reduce its load-bearing capacity. Do not use the ramp if the support structure is damaged.
- Place the ramp in a dry protected area

# ENGLISH

## 2. Maintenance:

- Only clean the ramp with a soft cloth, sponge or soft brush. Do not use hard brushes to avoid scratches.
- Use a normal all-purpose cleaner. Do not use aggressive cleaning products!
- Remove all ice from the tread by applying warm water to the surface.
- Always remove any sand, oil or other products that reduce the anti-slip properties of the ramp.

## 3. Service and warranty:

- The ramp has a standard warranty of 2 years.
- The ramp is designed for a practical use of about 5 years (under normal usage and maintenance as a good family man).
- When inspecting the ramp, pay particular attention to the following parts:
  - Tread (assembly, damage)
  - Deflection of the ramp unloaded and under load
  - Handle (assembly, damage)
  - Hinge (mounting, damage)
  - Trolley (mounting, damage)
  - Other parts (assembly, damage)
- If damage is found during inspection or in case of doubt, contact your dealer immediately. DO NOT use the ramp if you have any doubt about its correct operation.

## 4. Warnings:

- Read the complete user manual before using the ramp for the first time.
- The ramp is intended for wheelchair users only and may not be used for any other purpose.
- The maximum load of the ramp is 300kg (user and attendant). Do not jump on the ramp or move with shocks.
- Only the wheelchair user, his companion and luggage may be transported on the ramp.
- Drive slowly up and down the ramp without taking a run-up.
- Make sure that the ramp is in full contact with the ground both at the beginning and the end of the ramp.
- The surface on which the ramp is used must be flat and stable.
- Always drive in the middle of the ramp.
- Avoid collisions with the ramp when it is carried.

## ENGLISH

- The ramp must not be used for gradients greater than 20% (max 20cm height difference per meter length).
- The ramp must be stored in a dry, protected area. Place the ramp upright so that passers-by cannot fall over the ramp.
- Always move the ramp away from the body to avoid injuries.
- DO NOT use the ramp if it is damaged !

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