

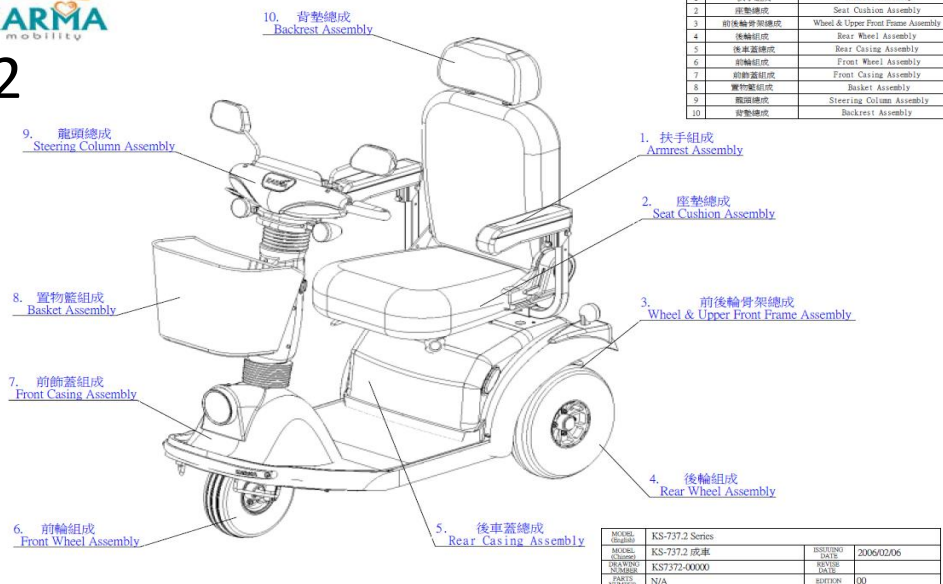
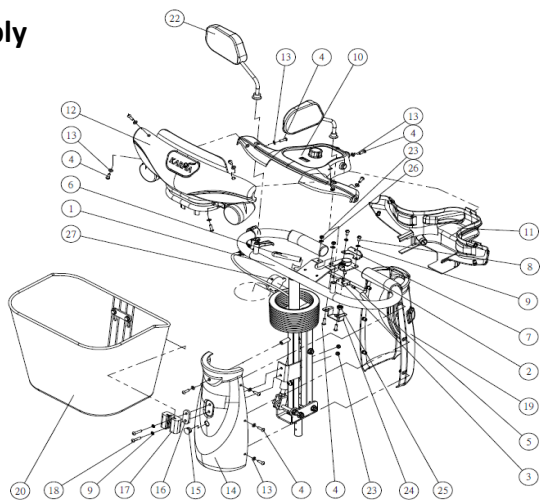






KS-737.2

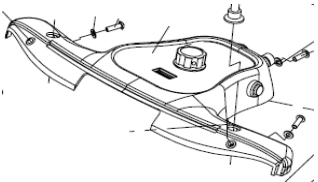


9. Steering Column Assembly


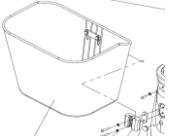


SAP no.	Description		Item on the view	Reference
156500000025	Steering Upper Casing assembly/ Silver		10	
156500000024	Steering Upper Casing assembly/ Red			
157700000052	Speed Dial Knob		n/a	
110707030008	Horn Button		n/a	
110707030007	Headlight/tallight button		n/a	
110707030009	Turn Signal Button		n/a	
110200000008	Rear Mirror		22	
151200000031	Basket assembly		20	

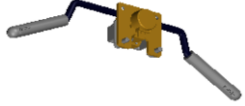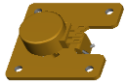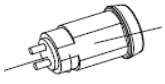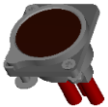
157700000001	Pulled Acceleration Lever Assy		3	
157700000013	5KVR ASSEMBLY		3.1	
110707030025	Key Switch S236-1(25KD) 1LOCK*2KEY		11.1	
110704010001	Charging Socket 3PIN 1A 100-240V 24V		14.1	
110704000018	Charger 6A 100V-240V 24V		n/a	
110705000004	Charger cable European Type		n/a	

9.1 Nameplate casing assembly

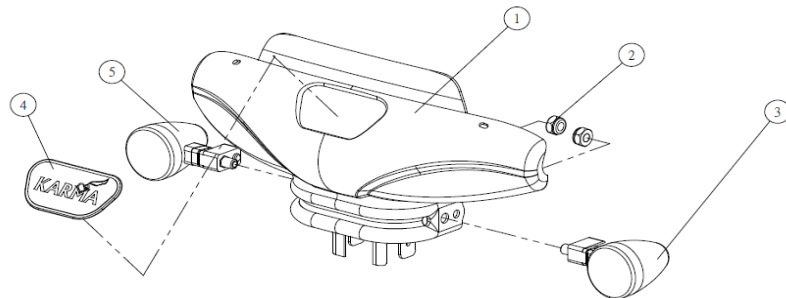

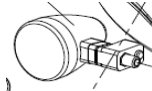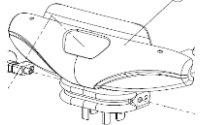
SAP no.	Description		Item on the view	Reference
110407020000	Logo Label Green		4	
110706020013	Circular Lamp Yellow 24V 5W Front/Left		3	
110706020012	Circular Lamp Yellow 24V 5W Front/Right		5	
166500000047	Front/Top STEERING CASING /Silver		1	
166500000039	Front/Top STEERING CASING Gem /Red		1	

7. Front casing assembly

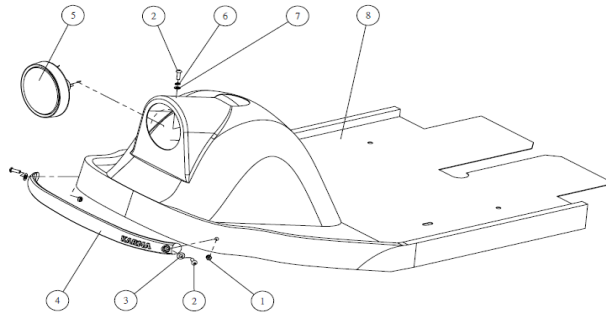


SAP no.	Description		Item on the view	Reference
166500000078	Front casing		8	
166500000069	Front casing			
110706010004	Round Headlight assembly		5	
110300000011	Bumper Black		4	

5. Rear Casing assembly

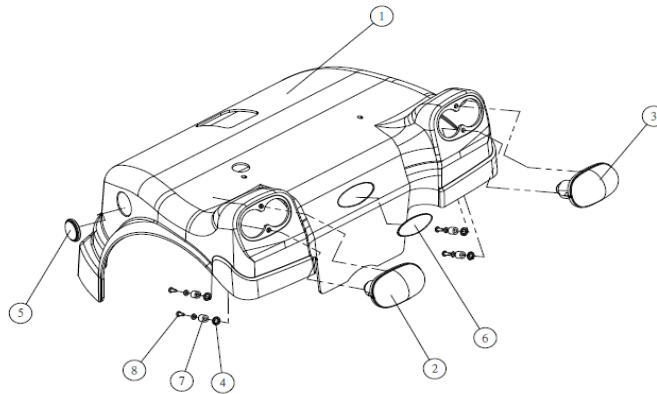


SAP no.	Description		Item on the view	Reference
166500000100	Rear casing /Silver		1	
166500000098	Rear casing /Red			
110706020025	Right Turn Signal		3	
110706020024	Left Turn Signal		2	
110308000005	Round Reflector yellow		5	

2. Seat Assembly

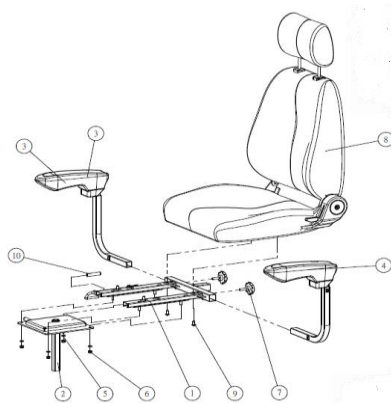

SAP no.	Description		Item on the view	Reference
157300000449	Driver's seat assembly PU 18"		8	
155600000023	Aremrest assembly Left		4	
155600000024	Aremrest assembly Right		3	
157300000038	Seat base frame assembly		2	
167300000005	Single seat slide frame assembly		1	
110303010008	Armpad		3.1	


6. Castor Assembly

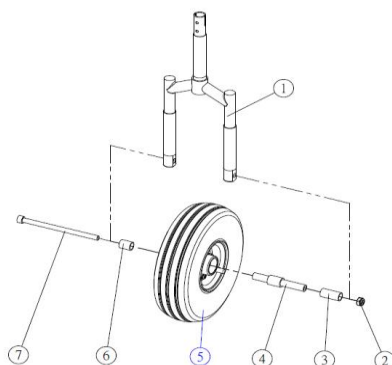

SAP no.	Description		Item on the view	Reference
166000000028	Castor fork		1	
152000000004	Castor assembly		5	


110305030011	10" Tire Assy American valve		5.1	
--------------	------------------------------	--	----------	--



4. Rear Weel Assembly

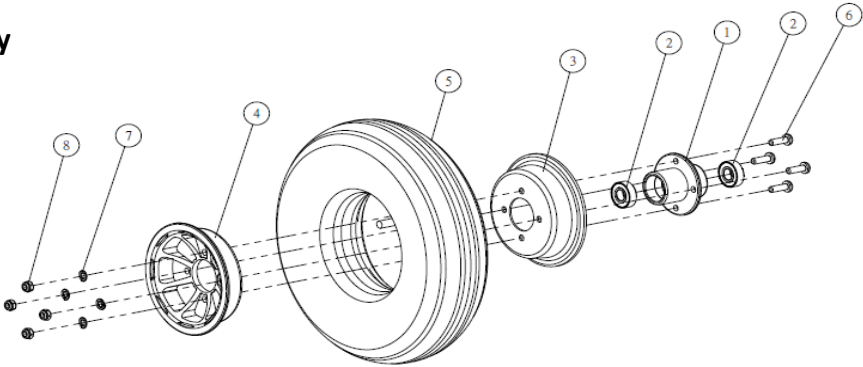


SAP no.	Description		Item on the view	Reference
152000000002	Rear wheel assembly		1~8	
110305030014	12" Tire Assy American valve		5	


3. Frame Assy



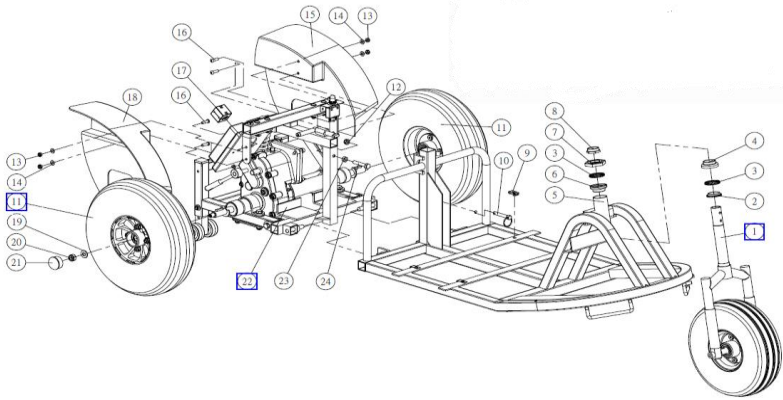


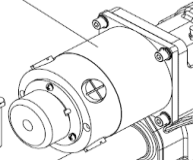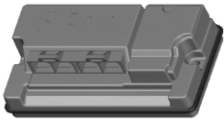SAP no.	Description		Item on the view	Reference
110700000001	Motor 550W		22.1	
157600000113	Differential Gear 20:1 Φ17		22.2	
110700000511	Power Module 120A		22.3	

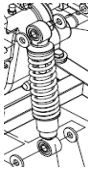110210000005	Rear Shock absorber 210L 35mm		15	
--------------	-------------------------------	--	---------	--

Battery

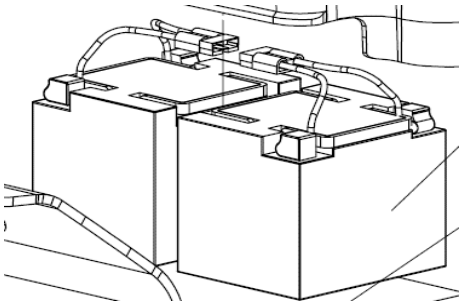

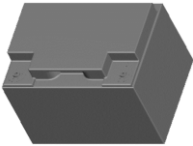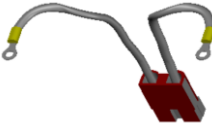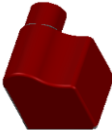SAP no.	Description		Item on the view	Reference
110703000018	Battery 12V 50AH		n/a	
110700000000	Battery Connector		n/a	
110306010033	Battery Terminal insulator / Red		n/a	
110306010034	Battery Terminal insulator / Black		n/a	