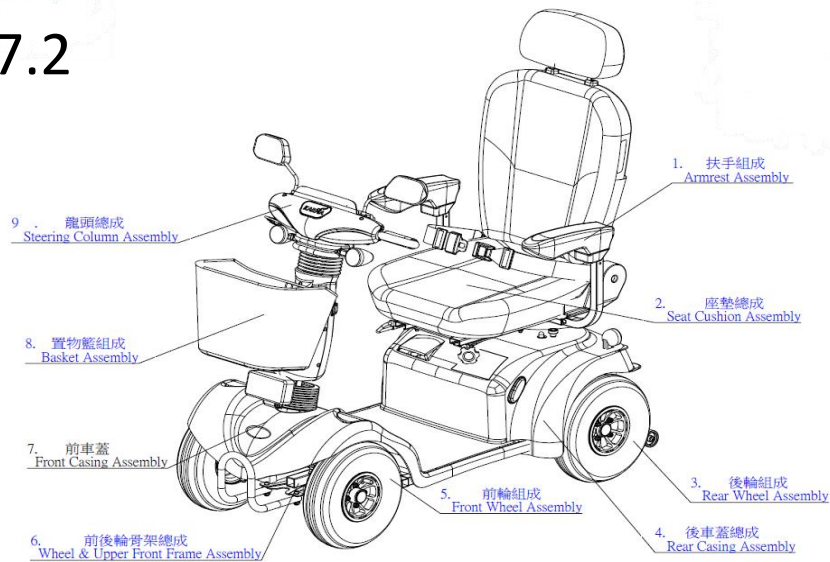
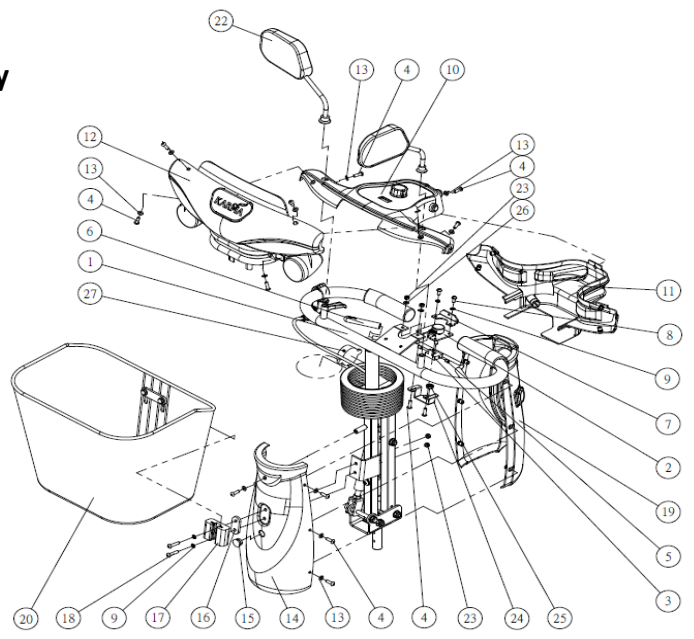


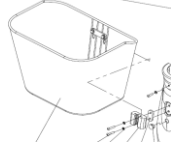
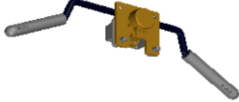
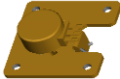
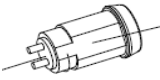
KS-747.2



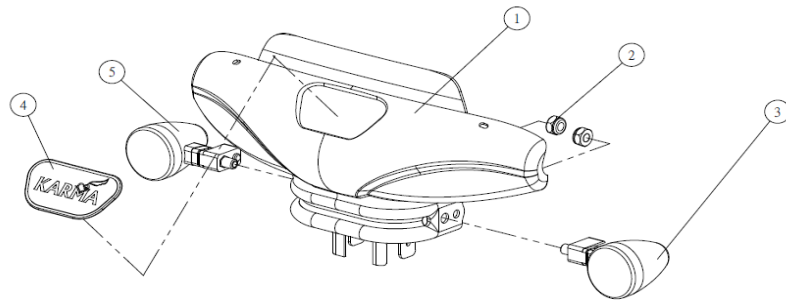
9. Double-pivot Steering Column Assembly

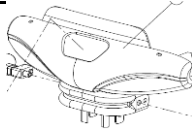
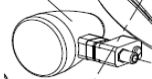


SAP no.	Description		Item on the view	Reference
156500000027	Steering Upper Casing assembly/ Silver		10	
110700000003	Speed Dial Knob		n/a	
110707030008	Horn Button		n/a	
110707030007	Headlight/tallight button		n/a	
110707030009	Turn Signal Button		n/a	
110200000008	Rear Mirror		22	

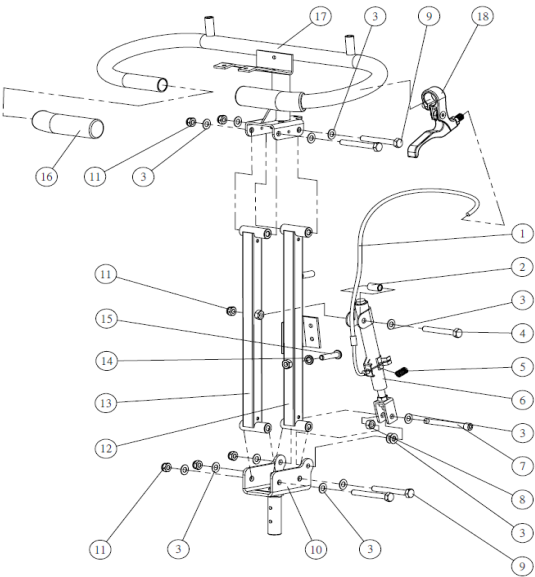
151200000031	Basket assembly		20	
157700000001	Pulled Acceleration Lever Assy		3	
157700000013	5KVR ASSEMBLY		3.1	
110707030025	Key Switch S236-1(25KD) 1LOCK*2KEY		11.1	
110704010001	Charging Socket 3PIN 1A 100-240V 24V		14.1	
110704000018	Charger 6A 100V-240V 24V		n/a	
110705000004	Charger cable European Type		n/a	

9.1 Nameplate casing assembly



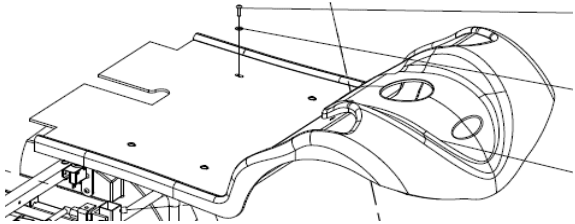
SAP no.	Description		Item on the view	Reference
166500000048	Front/Top STEERING CASING /Silver		1	
110706020013	Circular Lamp Yellow 24V 5W Front/Left		3	
110407020000	Logo Label Green		4	
110706020012	Circular Lamp Yellow 24V 5W Front/Right		5	

9.2 Double-pivot Steering Column Assembly



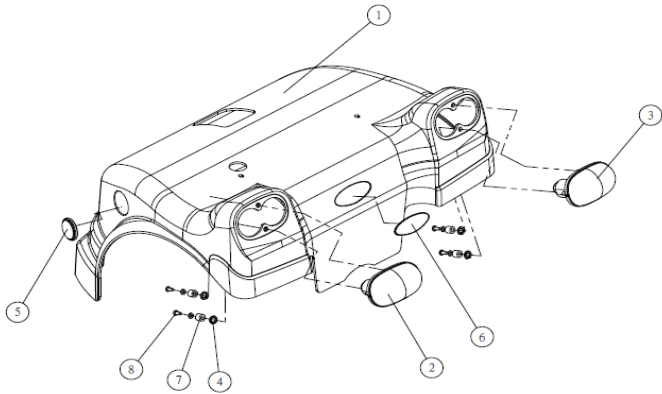
SAP no.	Description		Item on the view	Reference
110200000099	Steering Adj. Cylinder		6	

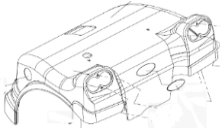
7. Front casing assembly

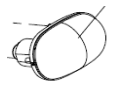


SAP no.	Description		Item on the view	Reference
166500000054	Front casing		n/a	

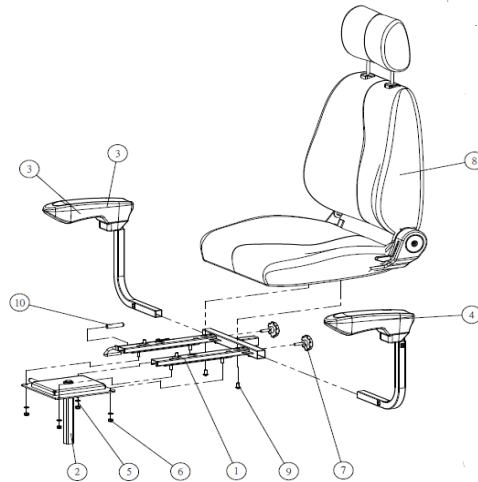
4. Rear Casing assembly

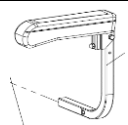
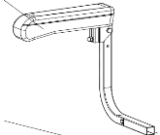
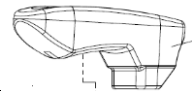
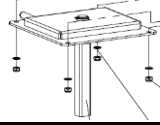
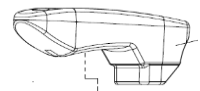
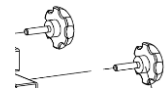


SAP no.	Description		Item on the view	Reference
166500000058	Rear casing /Silver		1	
110706020025	Right Turn Signal		3	

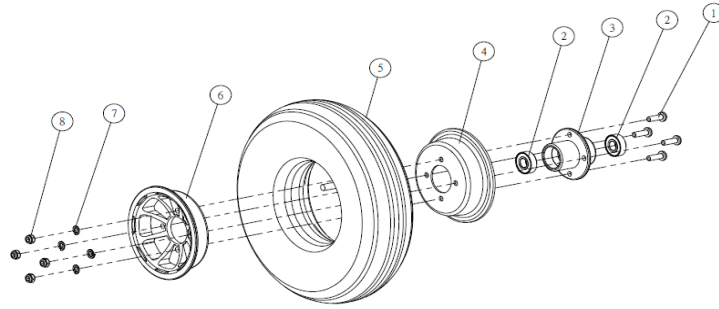
110706020024	Left Turn Signal		2	
110308000005	Round Reflector yellow		5	

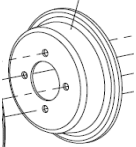
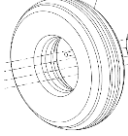
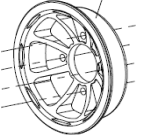
2. Seat Assembly



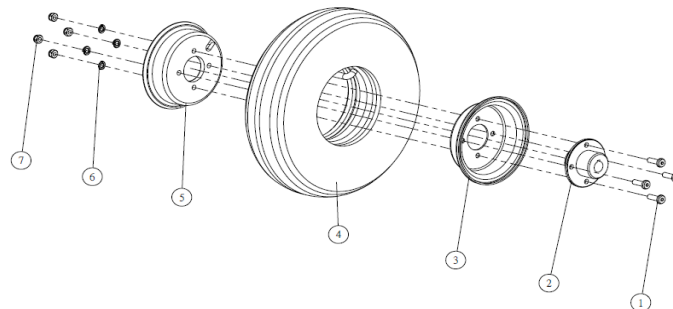
SAP no.	Description		Item on the view	Reference
157300000449	Driver's seat assembly PU 18"		8	
155600000023	Aremrest assembly Left		4	
155600000024	Aremrest assembly Right		3	
110303010008	Armpad		3.1	
157300000005	Seat base frame assembly		2	
167300000005	Single seat slide frame assembly		1	
110303010008	Armpad		3.1	
110806050009	Knob Bolt M8*32L		7	

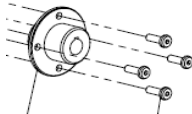
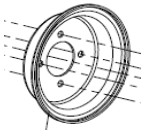
5. Front Wheel Assembly

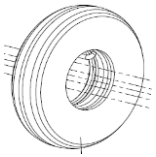
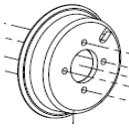


SAP no.	Description		Item on the view	Reference
156000000018	Wheel Hub & cap screws		1+3	
110201000013	Bearing		2	
110206020002	Inner Rim		4	
110305030014	12" Tire (Inner/outer tyre assembly)		5	
110105000006	Outer Rim		6	

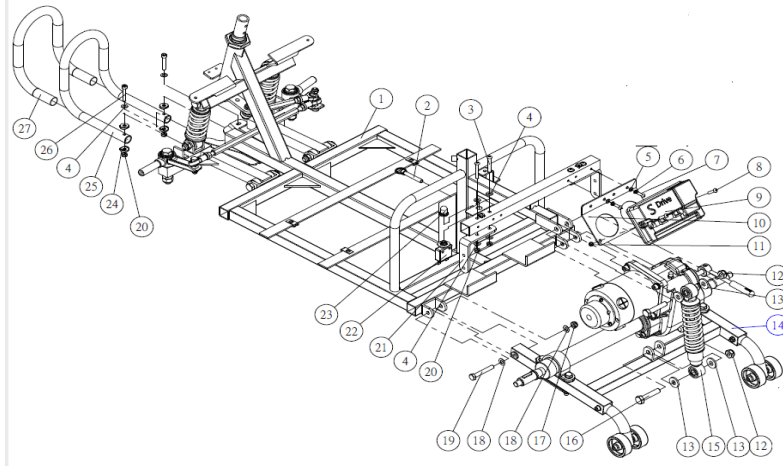
3. Rear Weel Assembly

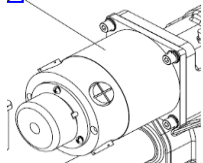
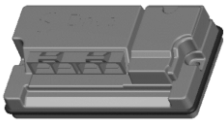
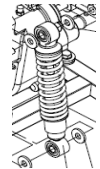
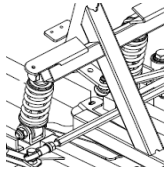
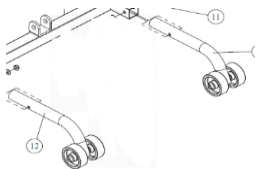


SAP no.	Description		Item on the view	Reference
156000000019	Wheel hub with cap screws		1+2	
110206020002	Inner Rim		3	

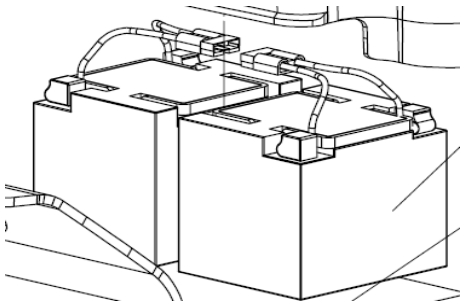
110305030014	12" Tire (Inner/outer tyre assembly)		4	
110105000006	Outer Rim		5	

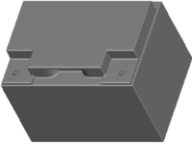
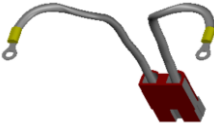
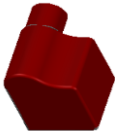
6. Frame Assy



SAP no.	Description		Item on the view	Reference
110700000001	Motor 550W		n/a	
157600000091	Differential 20:1 19MM WITH RING		n/a	
110700000002	S-Drive controller		9	
110210000005	Rear Shock absorber 210L 35mm		15	
110210000015	Shock Absorber Assy 130L 17mm		n/a	per 
151200000011	Anti-tipper assembly		n/a	

Battery



SAP no.	Description		Item on the view	Reference
110703000018	50AH Battery		n/a	per 
110700000000	Battery Connector		n/a	
110306010033	Battery Terminal insulator / Red		n/a	
110306010034	Battery Terminal insulator / Black		n/a	
110702030000	MOTOR CARBON BRUSH		n/a	
110702010008	ELECTRIC BRAKE (W/SDRIVE PLUG)		n/a	