SENSOR-rollaattori

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User Manual (EN)

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Bruksanvisning (SE)

Yleistä

1 Yleistä

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1 General information

Dear Customer,

This user manual contains a description of the medical device as well as important guidelines to ensure proper and safe use of the product. It is important to read this manual carefully before use.

It is especially important to read the safety requirements and follow them. REHASENSE continuously develops its products and reserves the right to change the specifications and features of the product without notice, unless the product is offered in a tender. If you have any questions, please contact your dealer or REHASENSE directly. Contact information can be found on the last page of this guide.

2 Intended use, indications and contraindications

Intended use: The rollator is designed as a tool to help the walking function of people who have mobility problems, for example standing, walking or sitting independently. Thanks to a walker, these people can easily stand, walk and rest, and they can carry out most of their daily tasks and routines without the help of others. Any other use is prohibited.

Indications: The device is specially designed for people who, due to a wide range of possible health problems, require additional stable support during walking.

Contraindications: The device must not be used by users who have serious problems with body balance or whose muscle strength is too low to be able to stand and walk, even with the additional support of a walker. The need and possibility of using a walker by the user should always be assessed and indicated by a doctor, physiotherapist or other visiting professional.

3 Delivery & General Product Description

The rollators are delivered in cardboard boxes and come folded for easier transport. The contents are: 1 complete, folded rollator and 1 user manual

The SENSOR is a four-wheeled, steel rollator with a foldable frame. The product is made of steel, so it is light and easy to store or transport. The rollator offers great maneuverability, stability and its push handles are height adjustable for good customization. The product consists of a powder coated, folding structural frame made of round pipes and a folding frame. It is also equipped with a plastic seat and a shopping cart that allows the user to rest and carry small items.

4 Safety requirements

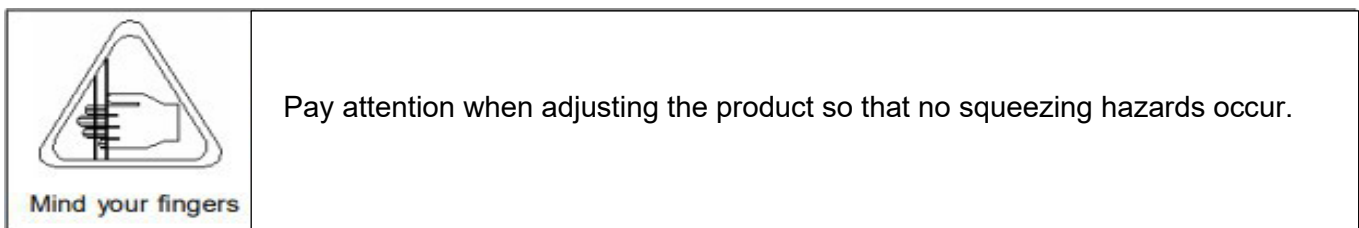
Make sure this user manual is read by all people using the device. The manufacturer assumes no responsibility for damage and/or personal injuries caused by the user manual not having been followed.

-  **Restrictions on use**
- The device should not be loaded with more than 150 kg.
 - The basket of the device has a load limitation of 10 kg.
 - The unit is designed for indoor and/or outdoor use on flat surfaces.
 - The device is not intended to be self-propelled while sitting
 - The brakes **MUST** be in the locked position **BEFORE** the seat is used or the rollator is left unattended.

Avoid accidents

- Use the device only for the purpose for which it is intended.
- Use the device only in good technical condition.
- Avoid making constructive changes to the device unless you have the manufacturer's written consent to such changes.
- During the various adjustments possible on the device, the user must take care not to catch their fingers or other body parts between two moving elements of the device
- The device must only be used on a stable surface.
- All wheels **MUST ALWAYS** be in contact with the floor during use. This will ensure that the rollator is properly balanced and should avoid accidents.
- When the rollator is used in a stationary position, the handbrakes **MUST** be locked.

IF A SERIOUS INCIDENT OCCURS IN CONNECTION WITH THE DEVICE, PLEASE CONTACT YOUR DEALER OR REHASENSE DIRECTLY (contact details at the end of the user manual). The immediate measures should be taken in accordance with the requirements of Regulation 2017/745 of the European Parliament and of the Council on medical devices of 5 April 2017.

5. Product information

6. USE

Initial check: Take the device out of the box and check (using the image above) that nothing is missing. If at this point you find that a part is missing or damaged, do not continue assembling the product. Contact your product supplier immediately.

For walking position: Hold the rollator by the handles, press down until you have all four wheels on the ground. The seat can be folded up to provide a larger aisle area and to increase accessibility.

Folding the rollator

1. Press the red lock button that connects the seat frame to the main frame.
2. Lift the seat frame to pull the main frame together and fold the rollator.

Adjusting the height of the handle

- Loosen the handle's 2 lock buttons.
- Make adjustments to the desired height.
- Tighten the 2 buttons.

- Both sides of the push handle must be adjusted to the same height.
- The correctly adjusted handle must be at the height of the user's hips.

Using the handbrakes

Pull both brakes upwards to brake

For parking.

Gently press down on the parking brake. A clear sound is heard when the brake is placed in the correct position. Always place both brakes in the parking position.

Pull upwards to release the brake.

- You should always apply the brakes when you do not plan to move the device.
- Never leave the device unattended without the brakes applied!
- Never leave the device on slopes without the brakes applied!

Brake/unlock the handbrakes

- To lock the brakes, slide the brake lever down until you hear the "click" sound.
- To unlock the brakes, pull up the brake lever until you feel the wheels are free to roll.

- When the rollator is used in a stationary position, the handbrakes **MUST** always be unlocked

Adjusting braking power

For desirable braking speed, adjust the brake cable with appropriate tools (not included). The cable can be adjusted in two places - under the brake lever or above the brake clamp. The brake lever must be in the neutral position. Loosen the locknut and turn the hexagon nipple counterclockwise to increase braking power. The distance between the brake clamp and the wheel should be 2 mm. Tighten the locknut again to secure the new alignment. Check the braking effect by applying the parking brake and check that the wheel cannot rotate.

- Be sure to tighten the locknuts after adjustment.

7. Cleaning

General:

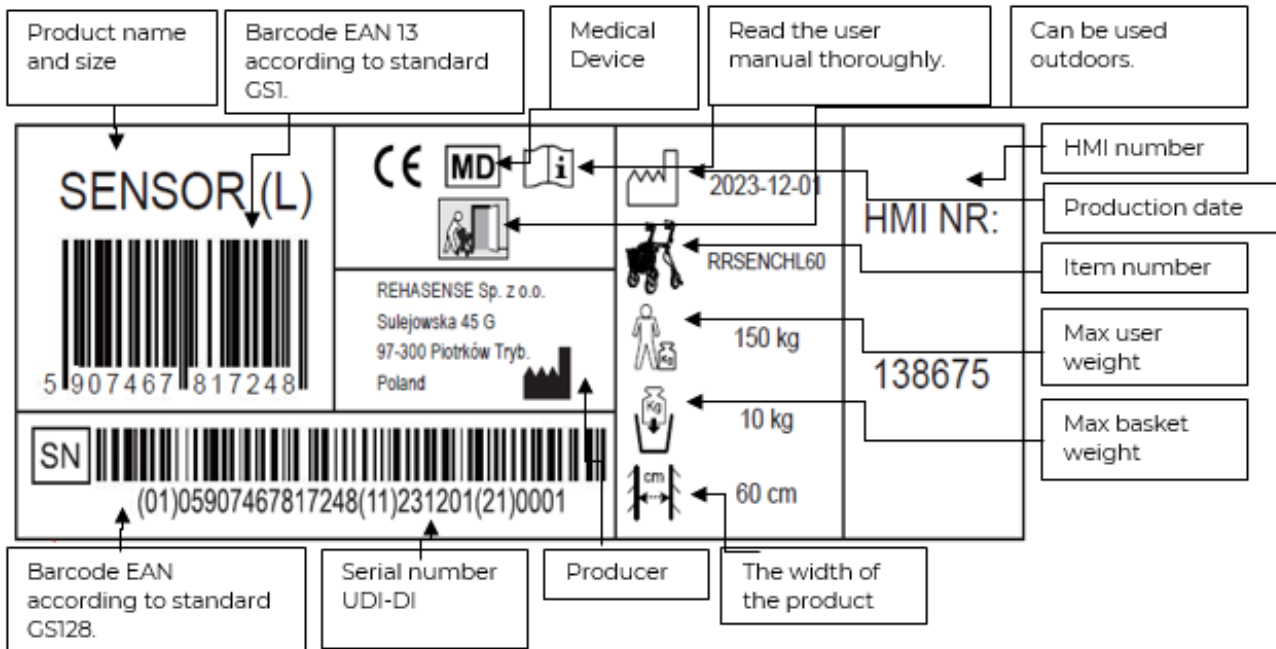
- Clean the device regularly and keep it clean.
- Lighter dirt and dust can be removed with a wet cloth.
- Larger portions of dirt can be removed with a non-corrosive, mild detergent and warm water.
- Do not use harsh abrasives or bleach-based cleaners, as this may cause damage or discoloration.
- Do not clean with solvents such as benzene or thinner. This can damage varnish and plastic parts.
- Chlorine-based cleaners are not allowed.
- Do not clean with detergents containing corrosive metals, substances or mixtures.
- After cleaning, wipe the rollator with a soft cloth.
- Can be washed in a washing hall of max 80° for max 10 min.

Wheel:

- Clean the wheels with warm water and a mild detergent. Wipe with a clean cloth.
- Lubricate with multifunction oil periodically or if the wheels start to creak.

8. Example label

The product label is placed on one of the lower pipes of the main frame:

9 Technical data

Model / Size	Seat height (mm)	Total Width (mm)	Length u. basket (mm)	Distance between rear wheels (mm)	Height adjustment (mm)	Distance between handles (mm)	Wheel diameter (mm)	Max weight in basket (kg)	Max user weight (kg)	Weight without basket (kg)
Sensor M	520	607	660	495	700-840	405	200	10	150	8
Sensor L	615	607	710	495	825-1020	405	200	10	150	8.3

10 Description of transport

The rollator is an easily foldable product that is convenient for transport. If it is necessary to transport it, we recommend folding it as indicated in the description above. The rollator must be transported in such a way that it is stable and that there is no risk of sudden and dangerous movements of the transported unit.

11 Maintenance

- Make sure that all parts of the rollator are safe at all times.
- Check all screws are securely fixed. If not, tighten if necessary.
- Replace broken, damaged or worn items immediately.
- Check that the brakes are working.
- Review checkpoints below when handing over to another user:

6. Checklist for inspection.

Checkpoints	Possible problem	Action	Notes	OK
Check that the wheels and forks are working properly, that they are free of defects and that they are securely secured.	The wheel rotates sluggishly.	<ul style="list-style-type: none"> Replace the wheel and locknut. 		
	The tread pattern is broken or damaged.	<ul style="list-style-type: none"> Replace the wheel Tighten the wheel bolts Replace the lock nut 	Use permanent locking adhesive if the locknut is missing.	
	The wheel bolts are loose.	<ul style="list-style-type: none"> Tighten the wheel bolts Replace the lock nut 	Use permanent locking adhesive if the locknut is missing.	
	The wheels do not align evenly in height.	<ul style="list-style-type: none"> Adjust the fork of the rear wheel. Tighten the screws. 		
	The air pressure is not right or the air goes out of the wheel. (Futura only with pneumatic wheels)	<ul style="list-style-type: none"> Repair the puncture Replace the wheel. 		
	The screws on the rear fork are loose.	<ul style="list-style-type: none"> Tighten the screws. 		
	The fork is not attached to the frame.	<ul style="list-style-type: none"> Tighten the fork to 60 Nm. 		
	The front fork is damaged.	<ul style="list-style-type: none"> Replace the fork. Tighten the fork to 60 Nm. 		
	The rear fork is damaged.	<ul style="list-style-type: none"> Replace the fork. 		
	The fork does not rotate smoothly and silently.	<ul style="list-style-type: none"> Replace the fork. Tighten the fork to 60 Nm. 		
Check that the brake system is working properly and is free from faults	The brake pad is worn.	<ul style="list-style-type: none"> Swap the brake globe. 		
	The braking globe does not return to its original position after braking.	<ul style="list-style-type: none"> Replace the brake pad or cable. 		
	The brake cable moves sluggishly in the cable housing.	<ul style="list-style-type: none"> Replace the brake cable. 		
	Poor brake effect.	Adjust the wire adjustment nuts or adjusting screws to achieve full braking effect.		
	The end parts of The wire caps are damaged.	<ul style="list-style-type: none"> Replace the cable cover. 		

12 Recycling

The rollator is suitable for recycling. Before it is disclosed to another user, it shall be subject to an inspection of the following items:

- Visual inspection of the frame
- The safety of nuts and bolts is sufficiently tightened
- The condition of the plastic elements
- Condition of wheels and ball bearings
- Efficiency and condition of the brakes

If one of the parts is damaged or broken, it must be replaced with a new one. It is recommended to replace the wheels when changing. All loose bolts and nuts must be tightened. The product must be thoroughly cleaned and disinfected before reuse.

WARNING! It is forbidden to reuse the rollator when the frame is bent or broken. The instructions for use must accompany the rollator at each shift.

13 How to deposit the product

The device must not be deposited with normal waste. It must be delivered to the recycling centre for reuse or recycling.

14 Right of complaint

- The warranty is not valid in case of improper intentional use of the product.
- The warranty excludes damage caused by improper operation, accidents and/or normal wear and tear.
- Any changes to the product will result in loss of warranty.
- Mechanical damage is not covered by this warranty.
- For repairs during the warranty period, please contact your local dealer

15 Life expectancy

The expected service life of the Product is ten (10) years if used in accordance with the safety instructions, maintenance and instructions for proper use contained in this manual. The actual lifetime of the product may vary depending on how much and how intensively it is used.

In case of technical problems, please contact your dealer. You can also contact Rehasense® directly.

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