VIKING

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Assembly and operating

instructions User manual (EN)

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DANSK

1 Generelle oplysninger

Kære kunde,

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
1 General information

Dear Customer,

This user manual describes a medical device and contains important advice that ensures the proper and safe use of the product. It is very important that you read this user manual carefully before using the product. It is particularly important to familiarize yourself with the safety requirements and follow them.

REHASense constantly develops its products and reserves the right to change product specifications and properties without prior notice.

If you have any questions, please contact your REHASense sales representative directly. Contact information is presented on the last page of the user manual.

2 Intended use, recommendations, and contraindications

Intended use: The walker is intended to help people with limited mobility, for example, people who have issues with standing or walking without help. The walker helps them stand and walk without difficulty and perform most daily tasks and activities without assistance. Any other use is prohibited.

Recommendations: The walker is intended specifically for people who, due to a wide range of possible health problems, need additional, stable support while walking.

Contraindications: It should not be used by people who have serious problems with maintaining balance or by people who cannot stand or walk due to insufficient muscle strength, even with additional support from a walker. The need and ability to use a walker must be assessed by a doctor or physiotherapist.

3 Delivery and general description

The walker is delivered in a cardboard box and is folded for ease of transport.

Contents:

- ✓ 1 walker, folded, including a seat and a bag
- ✓ 1 user manual

The "VIKING" walker has a powder-coated aluminum frame and 4 wheels. The lightweight aluminum frame makes it easy to fold, store, and transport the walker. It is easy to maneuver and stable. The height of the push handles is adjustable. The walker comes with a pre-assembled seat and shopping bag.

4 Safety requirements

Make sure that this user manual has been read by everyone who will be using the walker. The manufacturer is not liable for damage and/or injuries caused by failure to read and follow the instructions.


Restrictions of use

- The user's weight cannot exceed 150 kg.
- The walker's bag cannot carry more than 5 kg.
- The walker is intended for use on flat, even surfaces.
- The walker cannot be pushed while the user is sitting on it.
- The brakes **MUST** be in the locked position before sitting on the seat.


Risk of accidents

- Use the walker only for its intended purpose.
- Only use the walker if it is in perfect technical condition.
- Do not make design changes without written consent from the supplier.
- Be careful not to pinch fingers or other body parts between the two moving parts of the device while you are adjusting the device.
- The walker may only be used on stable surfaces.
- All wheels **MUST ALWAYS** be in contact with the floor during use. This will ensure the walker is properly balanced and prevent accidents.
- When the walker is used in a stationary position, the brakes **MUST** be locked.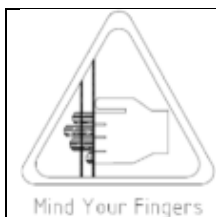

Pay attention to places where you see this symbol on the walker. Fingers or other body parts of the user and/or assistant may become trapped when folding, unfolding, or adjusting the walker. Be especially careful when adjusting the walker. Maintain a distance between the parts of the walker to avoid pinching fingers or other body parts.

5 Product description

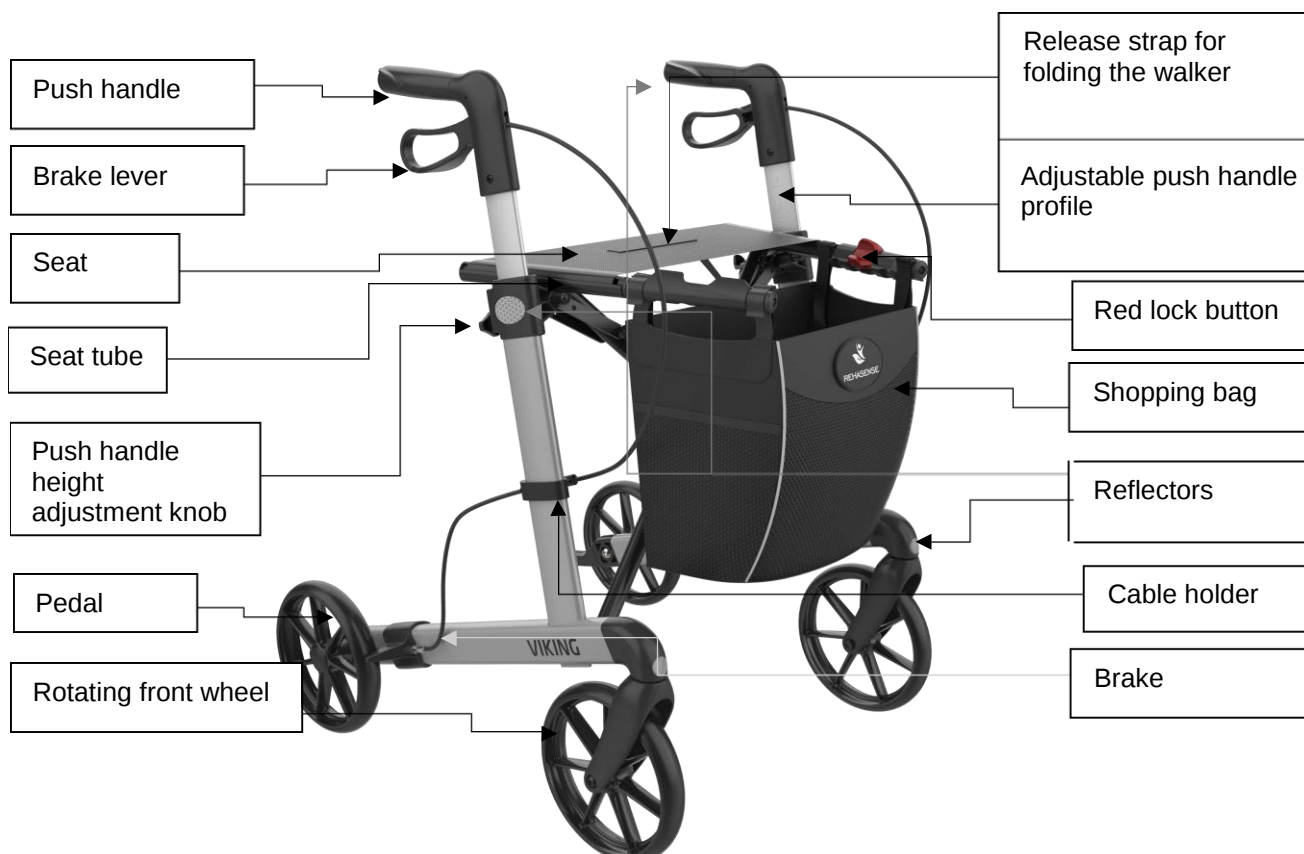6 Use

Inspection after delivery

After unpacking the walker, first, check if anything is missing and if all the parts shown in the picture above are there. If you find that any parts are missing, please contact your sales representative immediately.

Preparing for use

- Place the walker's wheels on the floor. Press the red button and grab the tubes by the seat. Push them in until you hear the locking mechanism click.
- The walker is now unfolded and secured against folding.
- Before use, check that the wheel mounting bolts are properly attached, that the walker is stable, and that no parts are moving.
- Before using the walker, check the brake.

Folding the walker

- To fold the walker, pull up the strap located in the center of the seat.
- The locking mechanism is then released, and the walker can be folded.

Adjusting the height of push handles

- To adjust the height of the push handles, loosen the knob.
- Then, adjust the handles to the desired height and tighten the knob again.

Please note that both handles must have the same height. When properly adjusted, the handles should be approximately at hip height when the user is standing with their arms relaxed.

To adjust the height of the push handles - loosen the two push handle locking buttons.

- Adjust the height.
- Lock the two buttons.

Using the brakes

- TO STOP THE WALKER, simply pull the brake up to the handle.
- TO STAY IN PLACE, hold the brake in the UP position.
- TO MOVE, release the brake. The handle will lower automatically, and the brake will be released.

- *You MUST always lock the brake if you do not intend to use the walker.*
- *Never leave the walker unattended unless the brakes are locked!*
- *Never leave the walker on a slope unless the brakes are locked!*

Locking/releasing the brake

- To lock the brakes, press the brake lever down until you hear a click.
- To release the brake, pull the brake lever up until you feel the wheels move freely.

- *When the walker is stationary, the brakes MUST be locked.*

Accessories

The walker has various accessories available (for example, a backrest, tray, release brake, one-handed brake, etc.). Please contact your sales representative for further information.

7 Cleaning

General:


- Clean the walker regularly and keep it well maintained.
- Lesser amounts of dirt and dust can be removed with a damp cloth.
- More persistent dirt can be removed with a mild detergent and warm water.
- Do not use harsh abrasives or bleaches, which may damage the walker and cause discoloration.
- Always wipe the walker with a soft cloth after cleaning.
- The walker can withstand autoclaving at a maximum of 80°C for a maximum of 10 minutes.

Wheels:

- Clean the wheels with warm water and a mild detergent. Dry with a clean cloth.
- Lubricate with universal oil regularly or if the wheels start to squeak.

8 Label

The product label is located on one of the lower tubes of the main frame. It contains important information about the product, such as its name, serial number, manufacturing date, product number, maximum user weight, maximum bag load, walker width, size, and manufacturer's information. The label should not be removed.

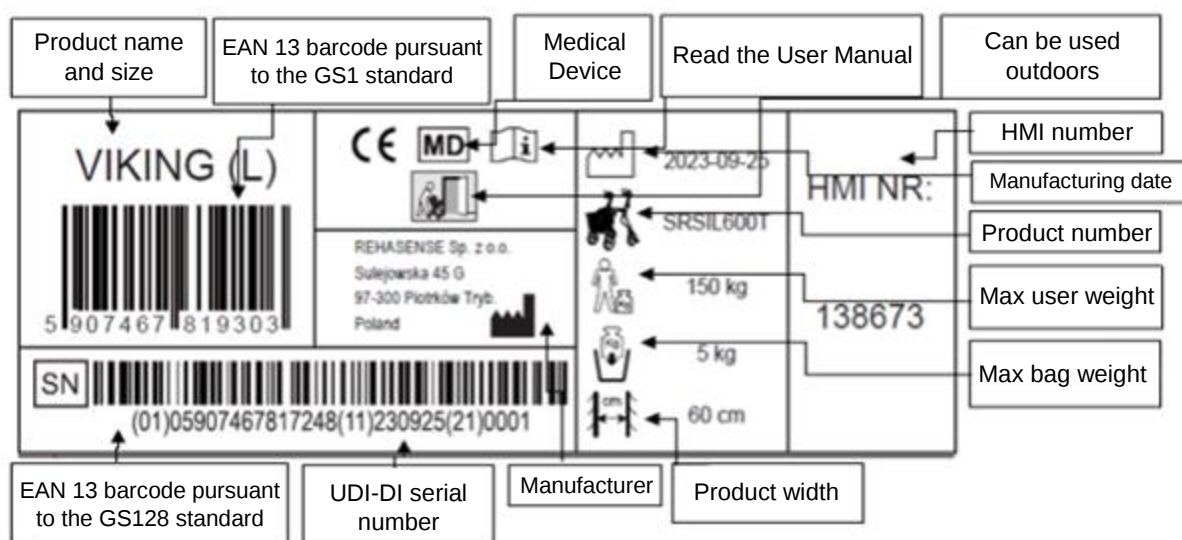9 Technical data:

VIKING	
Overall width unfolded/folded	60 cm / 24 cm
Seat width	60 cm
Length	68 cm
Seat height	L/62 cm - M/55 cm - S/50 cm
Push handle height (adjustable)	L/74-102 cm - M/66-86 cm - S/63-79 cm
Max load	150 kg
Max shopping bag load	5 kg
Total weight without bag (Large)	6.2 kg & bag 0.3 kg
Total weight without bag (Medium)	5.9 kg & bag 0.3 kg
Brakes	2 x handbrakes with a lock function

10 Troubleshooting

Problem description	Possible causes	Solution
The walker is difficult to push or steer	There is dirt between the wheels and the fork	Clean the space between the wheel and the fork. Alternatively, remove the wheel. Lubricate moving parts with grease.
The brakes are not working as they should, or one brake is working better than the other.	The brake pads are worn, or the brakes are not adjusted properly.	Try adjusting the brake cable or replacing the brake pads. If this does not solve the problem, contact your sales representative.
The seat does not lock in the correct position.	Check if the locking mechanism engages or is damaged. See if anything is in the way or if something is stuck.	Check or replace the locking mechanism. If this does not solve the problem, contact your sales representative.

11 Transport

The walker is easy to fold and transport. If transport is necessary, we recommend folding the walker as described above. The walker should not be able to move during transport.

The walker's design allows for easy unfolding and folding. Please ensure that the walker is in a stable position during transport to avoid bending.

12 Maintenance

- Make sure each component of the walker is tightened.
- Check that all screws and bolts are tightened. If not, tighten as needed.
- Replace damaged or worn components immediately.

13 Reuse

The walker can be reused. Before giving the walker to another user, it **MUST** be checked by a professional at the seller's office.

Perform the following checks:

- Visual inspection of the frame
- Check that all nuts and bolts are tightened
- Condition of plastic elements
- Condition of wheels and ball bearings
- Brake efficiency and condition

If any of the elements are damaged or cracked, they should be replaced with new ones. The wheels should be replaced when necessary. All loose bolts and nuts should be tightened. The product **MUST** be thoroughly cleaned and disinfected before reuse.

ATTENTION! It is prohibited to use the walker if the frame is bent or broken.

Always provide the user manual with a walker that is passed on to another person.

14 Utilization

The walker cannot be disposed of with general waste. It must be taken to a recycling center for reuse or recycling.

15 Warranty

The warranty does not apply if the product was used incorrectly.

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
The warranty does not cover damage caused by incorrect use or maintenance, accidents, and/or normal wear and tear.

Any modifications to the product will void the warranty. The warranty does not cover mechanical damage. If you need repairs within the warranty period, please contact your local sales representative.

16 Expected life of the product

The expected life of the product is ten (10) years when used in accordance with the safety, maintenance, and proper use guidelines contained in this manual. The actual life of the product may vary depending on how often and intensively it is used.

If technical problems occur, please contact the sales representative. You can also contact Rehasense directly.

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