

Snow Comfort



Thank you for choosing our Snow Comfort sled from the Classics series.

The Snow Comfort is a **special sled designed for children, teenagers and adults**. The functional, sporty design incorporates a **tubular railing and push handle system with solid side protection**. **Raised back support with adjustable headrest and sliding fitting** ensure a comfortable seating. Moreover, the seat itself is **spring-loaded**. **Three adjustable straps** are distributed over the entire sled in the foot, waist and thigh areas.

Two independently functioning metal brakes with steering and braking effect for ideal navigation. **Comfortable non-slip seat and integrated footrests** provide excellent support. May also be operated and guided by another person on the separate, **foldable rear platform**.

Double wall plastic undercarriage with **internal stainless steel runners** for outstanding gliding. Made of **high-quality plastic resistant to cold and light**.

Assembly instructions provided below. All you need is a **screwdriver**.

Loading-bearing capacity

Seat: 1 person from 6 years of age, max. 80 kg (in sitting position)

Trailer: 1 person from 15 years of age, max. 80 kg (in standing position)

If you have any questions, please do not hesitate to contact us at service@khw-geschwenda.de

Sled details



height-adjustable headrest



right and left front brake



Rear platform for riding companion



Brake for riding companion



Safety straps for body and feet



Hinged rear platform

Assembly

- Remove both self-tapping screws on the end of the support tubes of the rear seat.
- Guide the push handle (pre-assembled with the headrest) through the holes provided in the rear seat into the ends of the support tube and lock it with self-tapping screws.
- Loosen the star grip nut from the holding rail of the push handle.
- Swivel the arm brackets up into the holding rail and fasten them with the screw and the star grip nut.
- Attach the rear platform with the supplied spring clip.



Warning notices

Read the instructions carefully before use and keep them in a safe place. Failure to follow the instructions may affect the safety of your child.

- Only use the sled for the purpose specified by the manufacturer.
- Assembly may only be carried out by an adult.
- Before use, make sure that the steering and brakes are operational, and that all screws have been carefully tightened.
- Straps must be used when doing toboggan runs. During descents, straps must be secured to the seat part.
- Before riding the sled, your riding companion must check the braking system so that the seating rider is aware of its handling and operation.
- Do not modify the product in any way.
- Self-locking nuts must be replaced with new ones after disassembly.
- Folding sleds may only to be assembled or brought into the position of use by responsible adults.
- Do not use the product if damage has been detected.
- Only use original spare parts from the manufacturer. Available spare parts can be found at www.khw-ersatzteile.de.
- Clean the sled with environmentally friendly agents without any aggressive or corrosive properties.
- Store the sled in an airy and dry place.

Warranty against defects in materials or workmanship:
24 months from date of purchase

Safety regulations for parents and children

Your children should have a great and absolutely safe time in the snow with KHW sleds. This requires proper use of the sled.

- Please observe the minimum riding age, maximum load-bearing capacity and the number of seats.
- Attention. To be used only under adult supervision.
- The Snow Comfort is not suitable for use by children alone, as they are unable to control, steer and brake the sled.
- Attention. Children may only get on the sled under adult supervision. Risk of strangulation through the cord.
- Proactively teach children how to steer, brake and coordinate.
- Check the steering and brakes before using the sled.
- The Snow Comfort can only be used as a whole unit – dismantling it put all users at risk.
- Wear suitable protective clothing including helmet, sturdy shoes, gloves, and protectors if necessary. Do not drive with loosely hanging garments, as there is a risk of strangulation.
- Ride only on snow; avoid jumping over ramps or driving through ice gullies.
- Be considerate of others.
- The toboggan run should not be too steep and remain free of obstacles including stones, trees, etc.
- The toboggan run should never cross or end on a busy path or road.
- It should be noted that high speeds, as well as hard and icy surfaces, severely affect the steering and braking system.

- Do not allow the sled to be pulled by powered vehicles.
- Use the side rails as side handles.
- When sledding, do not place your feet in or on the sled, but spread them forward sideways so that you can brake and steer with your feet. Exception: sleds with steering and brakes.
- With sleds featuring steering and brakes, do not spread your feet sideways forward, but place them in or on the bob/sled so that you can brake and steer the vehicle safely with the side brakes or the steering unit.
- After using the sled, remove snow and ice residue before putting it away.
- Assembly instructions should be kept for future reference in the usage and safety regulations.

Use the Snow Comfort sled at your own risk and under the responsibility of your riding companion.





KHW Kunststoff- und Holzverarbeitungswerk GmbH
Alte Lage 1a · 99331 Geratal OT Geschwenda

Telefon +49 3 62 05 - 7 49-0
Telefax +49 3 62 05 - 7 49 34

info@khw-geschwenda.de
www.khw-geschwenda.de

Version 1 © 2022



#gemeinsamspurenhinterlassen

 **Klimaneutral**
Druckprodukt
ClimatePartner.com/12518-1907-1001